## Gyro Cup 2026

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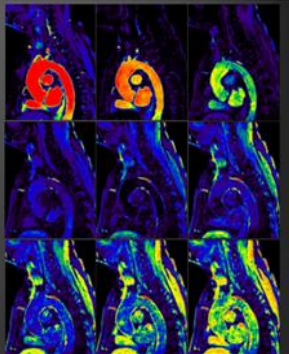
## BRIDGE

(Bright and dark blood images with multi-shot gradient echo -cho planar imaging)

<b>Geometry</b> FOV = 280 × 280 ACQ voxel size = 1.56 × 1.73 × 3 Recon voxel size = 0.7 × 0.7 × 1.5 SENSE (P,S) = (2,1) fold-over direction = AP slice orientation = sagittal	<b>Contrast</b> 3D TFEPI, multishot Contrast enhancement = no TFE factor = 6 shot interval = 2 beats profile order = low-high turbo direction = Z EPI factor = 7 TR (ac1) = shortest (12) TE (ac1) = shortest (5.2) FA = 15 Water-fat shift = minimum Fat suppression = ProSet (1331) T1E prepulse = invert shaved = yes, delay = shortest T2 prep = yes echo time = 50 refocusing pulses = 4
<b>Motion</b> PPU trigger delay = shortest Number of heart phases = maximum gate NSA = 2, SMART = yes	<b>Scan time = 2:58</b> (Heart Rate : 75)

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
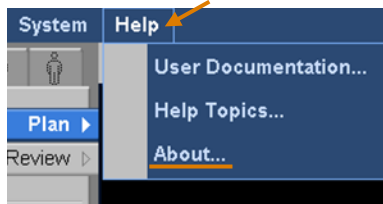
<リリース5以前>

Nominal Main Magnetic Field (B0)	1.5T						
Maximum Gradient of the static Magnetic Field	<a href="#">View details...</a>						
Main Operation Frequency for 1H	63.87 MHz						
Frequency range 1H	63.57 MHz - 64.18 MHz						
Frequency range Multi Nuclei	Option not available						
Maximum Gradient Output	<table border="1"><tr><td>20.0 cm</td><td>40.0 cm</td><td>60.0 cm</td></tr><tr><td>91.0 T/s</td><td>119.0 T/s</td><td>180.0 T/s</td></tr></table>	20.0 cm	40.0 cm	60.0 cm	91.0 T/s	119.0 T/s	180.0 T/s
20.0 cm	40.0 cm	60.0 cm					
91.0 T/s	119.0 T/s	180.0 T/s					
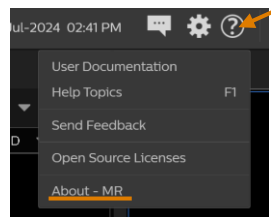
<リリース11以降>


Nominal Main Magnetic Field (B0)	00000						
Maximum Gradient of the static Magnetic Field	1.5T						
Main Operation Frequency for 1H	Information not available						
Frequency range 1H	63.87 MHz - 64.18 MHz						
Frequency range Multi Nuclei	Option not available						
Maximum Gradient Output	<table border="1"><tr><td>20.0 cm</td><td>40.0 cm</td><td>60.0 cm</td></tr><tr><td>91.0 T/s</td><td>119.0 T/s</td><td>180.0 T/s</td></tr></table>	20.0 cm	40.0 cm	60.0 cm	91.0 T/s	119.0 T/s	180.0 T/s
20.0 cm	40.0 cm	60.0 cm					
91.0 T/s	119.0 T/s	180.0 T/s					

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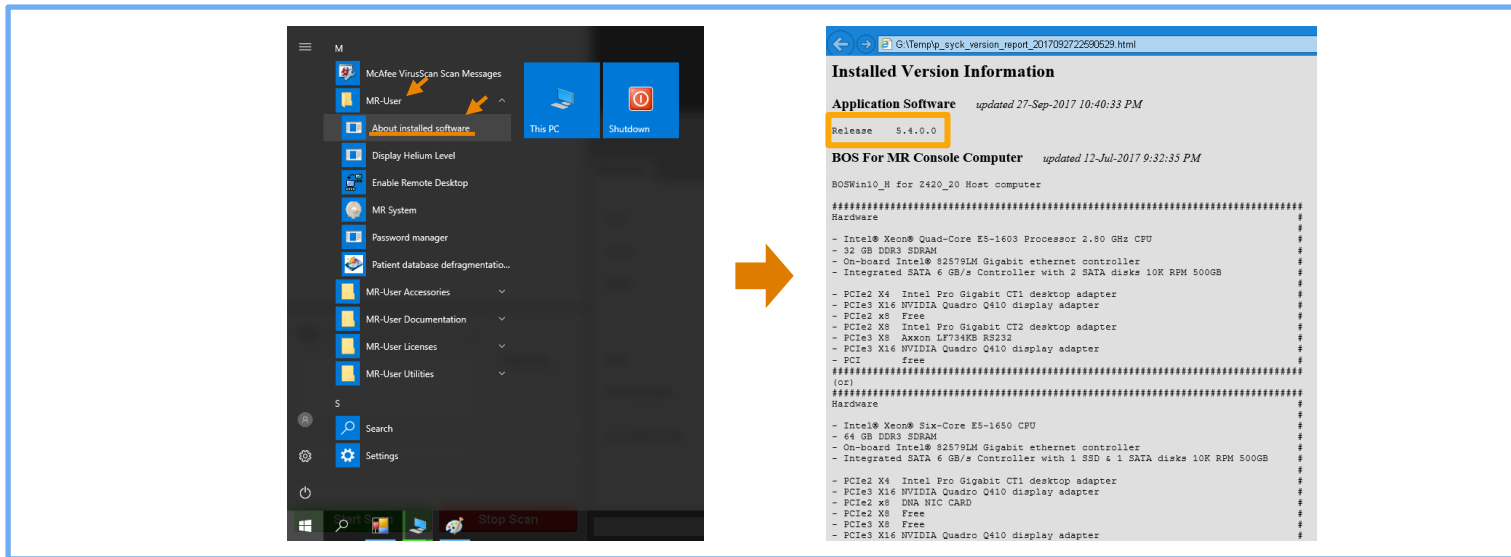The image shows a sequence of steps to check the software release. On the left, the Windows Start menu is open, and the 'About installed software' option is highlighted. An orange arrow points from this option to the right, where a web browser window displays the 'Installed Version Information' page. The browser address bar shows 'G:\Temp\ip\_sycl\_version\_report\_2017092722690529.html'. The page content includes:

- Application Software** updated 27-Sep-2017 10:40:33 PM  
Release 5.4.0.0
- BOS For MR Console Computer** updated 12-Jul-2017 9:32:35 PM

The BOS section contains hardware details for two computer models: BOSWin10\_H for 2420\_20 and BOSWin10\_H for 2420\_20. The hardware list includes Intel Xeon processors, DDR3 SDRAM, Gigabit ethernet controllers, SATA controllers, and various display adapters (Intel Pro Gigabit CT1, NVIDIA Quadro Q410).

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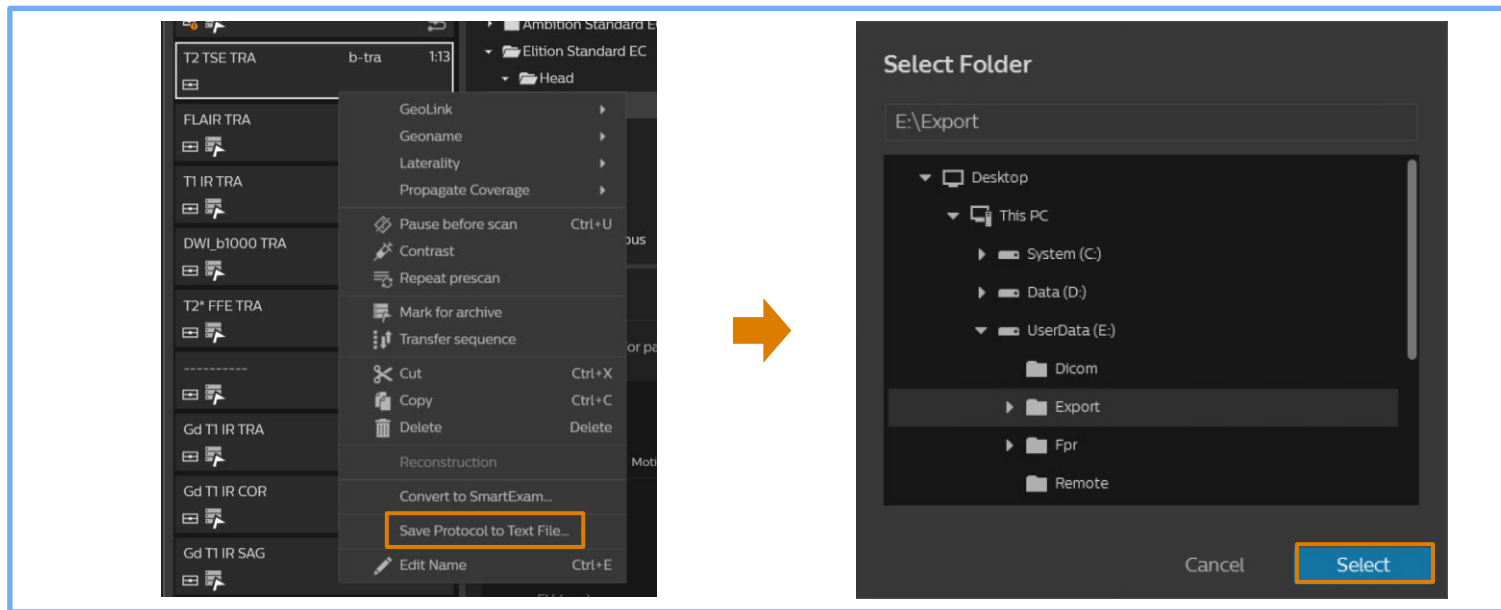
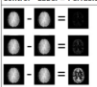
Brain methods and more > Perfusion Imaging > Arterial Spin Labeling (ASL)

### Arterial Spin Labeling (ASL)

Arterial Spin Labeling (ASL) is a method to obtain brain perfusion imaging without the use of contrast agent. ASL uses magnetically labeled blood in the arterial blood stream as an endogenous trace. The ASL sequence consists of a labeling and a control part. Perfusion images are automatically generated by subtracting the label images from the control images.

Control - Label = Perfusion (example of 3 dynamics)

Perfusion estimation relies on the signal changes (1%-2%).


#### Applications

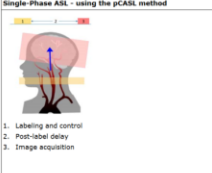- Cerebral perfusion studies.
- MR Angiography in the brain, see [MR Angiography](#) [DS](#) [DS/ASL/ASL](#)

#### Basic ASL-sequence

The ASL sequence is a dynamic sequence. Depending on the clinical question, you can perform ASL as single-phase or as multi-phase sequence using different ASL techniques.

##### Single-Phase ASL - using the pCASL method


1. Labeling and control
2. Post-label delay
3. Image acquisition

1	2	3	4	Readout
Pre-saturation	Labeling/Control	Post-labeling		
			ASL/CL	ASL/CL

1. Pre-saturation
2. Labeling/Control

##### Multi-Phase ASL - using the STAR method

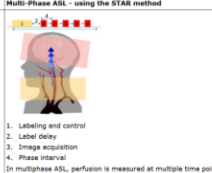1. Labeling and control
2. Label delay
3. Image acquisition
4. Phase interval

In multiphase ASL, perfusion is measured at multiple time points after labeling: the same area is imaged multiple times after labeling (or control). This results in multiple perfusion images.

1	2	3	4	Read out
Pre-saturation	Labeling	Post-labeling		
			ASL/CL	ASL/CL
			ASL/CL	ASL/CL
			ASL/CL	ASL/CL
			ASL/CL	ASL/CL