

# DICOM Conformance Statement

# Statement

Application Annex:  
3D-RA R6.8 on Interventional Workspot R1.8



*- This page is left intentionally empty -*

## 1. DICOM Conformance Statement Overview

For information about this section, Refer to HSDP-1068450 DICOM Conformance Statement Interventional Workspot R1.8.

**2. Contents**

- 1. DICOM Conformance Statement Overview ..... 3
- 2. Contents ..... 4
- 3. Introduction..... 5
  - 3.1. Revision History .....5
  - 3.2. Audience.....5
  - 3.3. Remarks .....5
  - 3.4. Definitions, Terms and Abbreviations .....6
  - 3.5. References.....6
- 4. Networking ..... 7
- 5. Media Interchange ..... 7
- 6. Support of Character Set ..... 7
- 7. Security ..... 7
- 8. Annexes of "3D-RA R6.8 Application " ..... 8
  - 8.1. IOD Contents .....8
    - 8.1.1. Acceptance Criteria .....8
    - 8.1.2. Created SOP Instance .....8
      - 8.1.2.1. List of Created SOP Classes .....9

### 3. Introduction

This DICOM Conformance Statement annex is applicable to the 3D-RA R6.8 later referred to as 3D-RA R6.8 Application. In general 3D-RA R6.8 Application allows for generating three-dimensional volumes from XA data and for viewing and analyzing XA images and derived data.

The following analysis packages are offered by 3D-RA R6.8 Application (licensing might be applicable):

- Advanced Vessel Analysis (AVA)
- Virtual Stenting

#### 3.1. Revision History

The revision history provides dates and differences of the different releases.

**Table 1: Revision History**

Document Version	Date of Issue	Description of change
01	10-Feb-2023	First release for 3D-RA R6.8 on Interventional Workspot R1.8
02	27-Feb-2024	Second release for 3D-RA R6.8 on Interventional Workspot R1.8 <ul style="list-style-type: none"> <li>• Updated support of below DICOM Conditional Modules with implemented attributes:                             <ul style="list-style-type: none"> <li>○ Contrast/Bolus Module</li> <li>○ General Reference Module</li> </ul> </li> </ul>

#### 3.2. Audience

This Conformance Statement is intended for:

- (Potential) customers
- System integrators of medical equipment
- Marketing staff interested in system functionality
- Software designers implementing DICOM interfaces

It is assumed that the reader is familiar with the DICOM standard.

#### 3.3. Remarks

The DICOM Conformance Statement is contained in chapter 4 through 8 and follows the contents and structuring requirements of DICOM PS 3.2.

This DICOM Conformance Statement by itself does not guarantee successful interoperability of Philips equipment with non-Philips equipment. The user (or user's agent) should be aware of the following issues:

- **Interoperability**  
 Interoperability refers to the ability of application functions, distributed over two or more systems, to work successfully together. The integration of medical devices into an IT environment may require application functions that are not specified within the scope of DICOM. Consequently, using only the information provided by this Conformance Statement does not guarantee interoperability of Philips equipment with non-Philips equipment.  
 It is the user's responsibility to analyze thoroughly the application requirements and to specify a solution that integrates Philips equipment with non-Philips equipment.
- **Validation**  
 Philips equipment has been carefully tested to ensure that the actual implementation of the DICOM interface corresponds with this Conformance Statement.

Where Philips equipment is linked to non-Philips equipment, the first step is to compare the relevant Conformance Statements. If the Conformance Statements indicate that successful information exchange should be possible, additional validation tests will be necessary to ensure the functionality, performance, accuracy and stability of image and image related data. It is the responsibility of the user (or user's agent) to specify the appropriate test suite and to carry out the additional validation tests.

- **New versions of the DICOM Standard**

The DICOM Standard will evolve in future to meet the user's growing requirements and to incorporate new features and technologies. Philips is actively involved in this evolution and plans to adapt its equipment to future versions of the DICOM Standard. In order to do so, Philips reserves the right to make changes to its products or to discontinue its delivery. The user should ensure that any non-Philips provider linking to Philips equipment also adapts to future versions of the DICOM Standard. If not, the incorporation of DICOM enhancements into Philips equipment may lead to loss of connectivity (in case of networking) and incompatibility (in case of media).

### 3.4. Definitions, Terms and Abbreviations

**Table 2: Definitions, Terms and Abbreviations**

Abbreviation/Term	Explanation
AVA	Advanced Vessel Analysis
DCS	DICOM Conformance statement
DICOM	Digital Imaging and Communications in Medicine
HSDP	HealthSuiteDigitalPlatform
IOD	Information Object Definition
IW	Interventional Workspot
NEMA	National Electrical Manufacturers Association
SOP	Service Object Pair
UID	Unique Identifier
VR	Value Representation
XA	X-Ray Angiography

### 3.5. References

[DICOM] Digital Imaging and Communications in Medicine, Parts 1 - 22 (NEMA PS 3.1- PS 3.22), National Electrical Manufacturers Association  
 1300 North 17th Street  
 Suite 900  
 Arlington, Virginia 22209  
 Internet: <https://www.dicomstandard.org/current>

#### **4. Networking**

For information about this section, Refer to HSDP-1068450 DICOM Conformance Statement Interventional Workspot R1.8.

#### **5. Media Interchange**

For information about this section, Refer to HSDP-1068450 DICOM Conformance Statement Interventional Workspot R1.8.

#### **6. Support of Character Set**

For information about this section, Refer to HSDP-1068450 DICOM Conformance Statement Interventional Workspot R1.8.

#### **7. Security**

For information about this section, Refer to HSDP-1068450 DICOM Conformance Statement Interventional Workspot R1.8.

## 8. Annexes of "3D-RA R6.8 Application "

### 8.1. IOD Contents

This section specifies each IOD accepted and / or created by 3D-RA R6.8 Application

**ACCEPTED** The applicable IOD is accepted for storage in the repository of the hosting platform and supported for import in 3D-RA R6.8 Application for viewing and analysis.

**CREATED** The 3D-RA R6.8 Application supports generation of derived data by using the applicable IOD and is able to store this data in the repository of the hosting platform

**Table 3: List of Created SOP Classes**

IOD		Supported	
Name	UID	Accepted	Created
CT Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.2	Yes	Yes
Secondary Capture Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.7	No	Yes
Multiframe True Color Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7.4	No	Yes
X-Ray Angiographic Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.12.1	Yes	Yes
X-Ray 3D Angiographic Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.13.1.1	Yes	Yes
Raw Data Storage SOP Class	1.2.840.10008.5.1.4.1.1.66	No	Yes

#### 8.1.1. Acceptance Criteria

This section specifies the acceptance criteria applied by 3D-RA R6.8 Application to which a dataset should adhere before it can be imported into the application. This can be criteria on the highest level (e.g. data from a certain manufacturer or system model) or certain DICOM attributes mandatory to be present into the dataset holding a specific value. In case one or more Philips private attributes are required, then a list of supported Philips system models will be mentioned.

**Table 4: Accepted System Models**

Manufacturer	Modality	System Model Name(s)
Philips	XA	Allura Azurion

**Table 5: Accepted Transfer Syntax per IOD**

For information about this Table, Refer to HSDP-1068450 DICOM Conformance Statement Interventional Workspot R1.8.

**Table 6: : Accepted Attribute Values**

Attribute Name	Attribute Number	Values/Comments
Not Applicable	Not Applicable	Not Applicable

#### 8.1.2. Created SOP Instance

This section specifies each IOD created by this application.

This section specifies each IOD created. It should specify the attribute name, tag, VR, and value. The value should specify the range and source (e.g. user input, Modality Worklist, automatically generated, etc.). For content items in templates, the range and source of the concept name and concept values should be specified. Whether the value is always present or not shall be specified.



Abbreviations used in the IOD tables for the column "Presence of Module" are:

- ALWAYS The module is always present
- CONDITIONAL The module is used under specified condition

Abbreviations used in the Module table for the column "Presence of Value" are:

- ALWAYS The attribute is always present with a value
- EMPTY The attribute is always present without any value (attribute sent zero length)
- VNAP The attribute is always present and its Value is Not Always Present (attribute sent zero length if no value is present)
- ANAP The attribute is present under specified condition – if present then it will always have a value

The abbreviations used in the Module table for the column "Source" are:

- AUTO The attribute value is generated automatically
- CONFIG The attribute value source is a configurable parameter
- COPY The attribute value source is another SOP instance
- FIXED The attribute value is hard-coded in the application
- IMPLICIT The attribute value source is a user-implicit setting
- MPPS The attribute value is the same as that use for Modality Performed Procedure Step
- MWL The attribute value source is a Modality Worklist
- USER The attribute value source is explicit user input

**8.1.2.1. List of Created SOP Classes**

**Table 7: List of Created SOP Classes**

SOP Class Name	SOP Class UID
CT Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.2
Secondary Capture Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.7
Multiframe True Color Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7.4
X-Ray Angiographic Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.12.1
X-Ray 3D Angiographic Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.13.1.1
Raw Data Storage SOP Class	1.2.840.10008.5.1.4.1.1.66

**8.1.2.1.1 CT Image Storage SOP Class**

**Table 8: IOD of Created X-Ray Angiographic Image Storage SOP Class Instances**

Information Entity	Module	Presence Of Module
Patient	Patient Module	ALWAYS
Study	General Study Module	ALWAYS
	Patient Study Module	ALWAYS
Series	General Series Module	ALWAYS
Frame of Reference	Frame of Reference Module	ALWAYS
Equipment	General Equipment Module	ALWAYS
Acquisition	General Acquisition Module	CONDITIONAL
Image	Image Plane Module	ALWAYS
	Contrast/Bolus Agent	CONDITIONAL
	General Image Module	ALWAYS
	General Reference Module	USER DEFINED

Information Entity	Module	Presence Of Module
	Image Pixel Module	ALWAYS
	CT Image Module	ALWAYS
	VOI LUT Module	ALWAYS
	SOP Common Module	ALWAYS

**Table 9: Patient Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Name	0010,0010	PN		VNAP	COPY	
Patient ID	0010,0020	LO		VNAP	COPY	
Issuer of Patient ID	0010,0021	LO		ANAP	COPY	
Patient's Birth Date	0010,0030	DA		VNAP	COPY	
Patient's Sex	0010,0040	CS		VNAP	COPY	
De-identification Method	0012,0063	LO		VNAP	COPY	

**Table 10: General Study Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Study Date	0008,0020	DA		ALWAYS	COPY	
Study Time	0008,0030	TM		ALWAYS	COPY	
Accession Number	0008,0050	SH		VNAP	COPY	
Referring Physician's Name	0008,0090	PN		VNAP	COPY	
Study Instance UID	0020,000D	UI		ALWAYS	COPY	
Study ID	0020,0010	SH		VNAP	COPY	

**Table 11: General Reference Module**

Attribute Name	Tag	VR	Value	Presence of Value	Source	Comment
Referenced Image Sequence	0008,1140	SQ		ANAP	AUTO	
>Referenced SOP Class UID	0008,1150	UI		ALWAYS	AUTO	
>Referenced SOP Instance UID	0008,1155	UI		ALWAYS	AUTO	
>Philips Imaging DD 001	2001,0010	UI		ANAP	AUTO	
>Image Reference	2001,10C1	LO		ANAP	AUTO	

**Table 12: Patient Study Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Weight	0010,1030	DS		ALWAYS	COPY	

**Table 13: General Series Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Date	0008,0021	DA		ALWAYS	AUTO	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Time	0008,0031	TM		ALWAYS	AUTO	
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Series Description	0008,103E	LO		ANAP	COPY	
Performing Physician's Name	0008,1050	PN		ANAP	COPY	
Patient Position	0018,5100	CS		ALWAYS	COPY	
Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
Series Number	0020,0011	IS		ALWAYS	COPY	
Laterality	0020,0060	CS		ANAP	COPY	
Performed Procedure Step Start Date	0040,0244	DA		ANAP	COPY	
Performed Procedure Step Start Time	0040,0245	TM		ANAP	COPY	
Performed Procedure Step ID	0040,0253	SH		ANAP	COPY	

Table 14: Frame of Reference Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Frame of Reference UID	0020,0052	UI		ALWAYS	COPY	
Position Reference Indicator	0020,1040	LO		VNAP	COPY	

Table 15: General Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Manufacturer	0008,0070	LO	Philips	ALWAYS	FIXED	
Institution Name	0008,0080	LO		VNAP	CONFIG	Configured hospital name
Station Name	0008,1010	SH		ANAP	AUTO	
Manufacturer's Model Name	0008,1090	LO	Interventional Workspot	ALWAYS	FIXED	
Device Serial Number	0018,1000	LO		ALWAYS	AUTO	Mac address of Hospital NIC
Software Versions	0018,1020	LO	1.8.x	ALWAYS	AUTO	Where "x" is the detailed application SW version.

Table 16: General Acquisition Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Acquisition Number	0020,0012	IS		ANAP	COPY	

Table 17: Contrast/Bolus Module

Attribute Name	Tag	VR	Value	Presence of Value	Source	Comment
Contrast/Bolus Agent	0018,0010	LO		VNAP	AUTO	

Contrast/Bolus Route	0018,1040	LO		ANAP	AUTO	
Contrast/Bolus Volume	0018,1041	DS		ANAP	AUTO	
Contrast Flow Rate	0018,1046	DS		ANAP	AUTO	
Contrast/Bolus Ingredient Concentration	0018,1049	DS		ANAP	AUTO	

**Table 18: Image Plane Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Slice Thickness	0018,0050	DS		ALWAYS	AUTO	
Image Position (Patient)	0020,0032	DS		ALWAYS	AUTO	
Image Orientation (Patient)	0020,0037	DS		ALWAYS	AUTO	
Slice Location	0020,1041	DS		ANAP	AUTO	
Pixel Spacing	0028,0030	DS		ALWAYS	AUTO	

**Table 19: General Image Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Image Type	0008,0008	CS		ALWAYS	COPY	
Content Date	0008,0023	DA		ANAP	AUTO	
Content Time	0008,0033	TM		ANAP	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	
Burned in Annotation	0028,0301	CS		VNAP	COPY	
Lossy Image Compression	0028,2110	CS		VNAP	COPY	

**Table 20: Image Pixel Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Rows	0028,0010	US		ALWAYS	AUTO	
Columns	0028,0011	US		ALWAYS	AUTO	
Photometric Interpretation	0028,0004	CS	MONOCHROME2	ALWAYS	AUTO	
Pixel Representation	0028,0103	US	0	ALWAYS	COPY	
Samples per Pixel	0028,0002	US	1	ALWAYS	FIXED	
Bits Allocated	0028,0100	US	16	ALWAYS	FIXED	
Bits Stored	0028,0101	US	16	ALWAYS	FIXED	
High Bit	0028,0102	US	15	ALWAYS	FIXED	
Pixel Data	7FE0,0010	OB/OW		ALWAYS	COPY	

Table 21: CT Image Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Image Type	0008,0008	CS	DERIVED, SECONDARY\AXIAL\3DRA_PROP Or DERIVED\SECONDARY\AXIAL\3DRA_ROLL	ALWAYS	COPY	
KVP	0018,0060	DS		VNAP	COPY	
Acquisition Number	0020,0012	IS		VNAP	COPY	
Samples per Pixel	0028,0002	US	1	ALWAYS	FIXED	
Photometric Interpretation	0028,0004	CS	MONOCHROME2	ALWAYS	FIXED	
Bits Allocated	0028,0100	US	16	ALWAYS	FIXED	
Bits Stored	0028,0101	US	16	ALWAYS	FIXED	
High Bit	0028,0102	US	15	ALWAYS	FIXED	
Rescale Intercept	0028,1052	DS		ALWAYS	COPY	
Rescale Slope	0028,1053	DS		ALWAYS	COPY	
Rescale Type	0028,1054	LO	US	ALWAYS	COPY	

Table 22: VOI LUT Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Window Center	0028,1050	DS	Value 1: 3D-RA R6.6/3D-RA R6.6 Window Width as set in histogram, converted to Hounsfield, XperCT Abdominal:60, XperCT Neuro:40	ALWAYS	COPY	
Window Width	0028,1051	DS		ALWAYS	COPY	

Table 23: SOP Common Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Specific Character Set	0008,0005	CS		ANAP	AUTO	As supported by hosting platform
Instance Creation Date	0008,0012	DA		ALWAYS	AUTO	
Instance Creation Time	0008,0013	TM		ALWAYS	AUTO	
SOP Class UID	0008,0016	UI	1.2.840.10008.5.1.4. 1.1.2	ALWAYS	FIXED	
SOP Instance UID	0008,0018	UI		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	

8.1.2.1.2 Secondary Capture Image Storage SOP Class

Table 24: IOD of Created Secondary Capture Image Storage SOP Class Instances

Information Entity	Module	Presence Of Module
Patient	Patient Module	ALWAYS
Study	General Study Module	ALWAYS
	Patient Study Module	ALWAYS
Series	General Series Module	ALWAYS
Equipment	General Equipment Module	ALWAYS
	SC Equipment Module	ALWAYS
Image	General Image Module	ALWAYS
	Image Pixel Module	ALWAYS
	SOP Common Module	ALWAYS

Table 25: Patient Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Name	0010,0010	PN		VNAP	COPY	
Patient ID	0010,0020	LO		VNAP	COPY	
Patient's Birth Date	0010,0030	DA		VNAP	COPY	
Patient's Sex	0010,0040	CS		VNAP	COPY	

Table 26: General Study Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Study Date	0008,0020	DA		ALWAYS	COPY	
Study Time	0008,0030	TM		ALWAYS	COPY	
Accession Number	0008,0050	SH		VNAP	COPY	
Referring Physician's Name	0008,0090	PN		VNAP	COPY	
Study Instance UID	0020,000D	UI		ALWAYS	COPY	
Study ID	0020,0010	SH		VNAP	COPY	

Table 27: Patient Study Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Weight	0010,1030	DS		ALWAYS	COPY	

Table 28: General Series Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Date	0008,0021	DA		ALWAYS	AUTO	
Series Time	0008,0031	TM		ALWAYS	AUTO	
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
Series Number	0020,0011	IS		ALWAYS	AUTO	
Related Series Sequence	0008,1250	SQ		VNAP	AUTO	
>Study Instance UID	0020,000D	UI		ALWAYS	AUTO	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
>Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
>Purpose of Reference Code Sequence	0040,A170	SQ		VNAP	AUTO	
Performed Procedure Step Start Date	0040,0244	DA		ANAP	COPY	
Performed Procedure Step Start Time	0040,0245	TM		ANAP	COPY	
Performed Procedure Step ID	0040,0253	SH		ANAP	COPY	

Table 29: General Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Manufacturer	0008,0070	LO	Philips	ALWAYS	FIXED	
Institution Name	0008,0080	LO		VNAP	CONFIG	Configured Hospital Name
Manufacturer's Model Name	0008,1090	LO	Interventional Workspot	ALWAYS	FIXED	
Station Name	0008,1010	SH		ALWAYS	AUTO	Hostname of IW PC
Software Versions	0018,1020	LO	1.8.x	ALWAYS	AUTO	where "x" is the detailed application SW version.

Table 30: SC Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Conversion Type	0008,0064	CS	WSD	ALWAYS	FIXED	

Table 31: General Image Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Instance Number	0020,0013	IS		ALWAYS	AUTO	
Patient Orientation	0020,0020	CS		VNAP	AUTO	

Table 32: Image Pixel Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Samples per Pixel	0028,0002	US		ALWAYS	AUTO	
Photometric Interpretation	0028,0004	CS		ALWAYS	AUTO	
Planar Configuration	0028,0006	US		ALWAYS	AUTO	
Rows	0028,0010	US		ALWAYS	AUTO	
Columns	0028,0011	US		ALWAYS	AUTO	
Bits Allocated	0028,0100	US	8	ALWAYS	AUTO	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Bits Stored	0028,0101	US	8	ALWAYS	AUTO	
High Bit	0028,0102	US	7	ALWAYS	AUTO	
Pixel Representation	0028,0103	US	0	ALWAYS	AUTO	
Pixel Data	7FE0,0010	OW/OB		ALWAYS	AUTO	

Table 33: SOP Common Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Specific Character Set	0008,0005	CS		ANAP	AUTO	As supported by hosting platform
SOP Class UID	0008,0016	UI	1.2.840.10008.5.1.4.1.1.7	ALWAYS	FIXED	
SOP Instance UID	0008,0018	UI		ALWAYS	AUTO	
Instance Creation Date	0008,0012	DA		ALWAYS	AUTO	
Instance Creation Time	0008,0013	TM		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	

8.1.2.1.3 Multi-frame True Color Secondary Capture Image Storage SOP Class

Table 34: IOD of Created Multi-frame True Color Secondary Capture Image Storage SOP Class Instances

Information Entity	Module	Presence Of Module
Patient	Patient Module	ALWAYS
Study	General Study Module	ALWAYS
	Patient Study Module	ALWAYS
Series	General Series Module	ALWAYS
Equipment	General Equipment Module	ALWAYS
	SC Equipment Module	ALWAYS
Image	General Image Module	ALWAYS
	Image Pixel Module	ALWAYS
	Cine Module	ALWAYS
	Multi-Frame Module	ALWAYS
	Multi-Frame Functional Groups Module	ALWAYS
	SC Multi-frame Image Module	ALWAYS
	SOP Common Module	ALWAYS

Table 35: Patient Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Name	0010,0010	PN		VNAP	COPY	
Patient ID	0010,0020	LO		VNAP	COPY	
Patient's Birth Date	0010,0030	DA		VNAP	COPY	
Patient's Sex	0010,0040	CS		VNAP	COPY	



Table 36: General Study Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Study Date	0008,0020	DA		ALWAYS	COPY	
Study Time	0008,0030	TM		ALWAYS	COPY	
Accession Number	0008,0050	SH		VNAP	COPY	
Referring Physician's Name	0008,0090	PN		VNAP	COPY	
Study Instance UID	0020,000D	UI		ALWAYS	COPY	
Study ID	0020,0010	SH		VNAP	COPY	

Table 37: Patient Study Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Weight	0010,1030	DS		ALWAYS	COPY	

Table 38: General Series Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Date	0008,0021	DA		ALWAYS	AUTO	
Series Time	0008,0031	TM		ALWAYS	AUTO	
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
Series Number	0020,0011	IS		ALWAYS	AUTO	
Related Series Sequence	0008,1250	SQ		VNAP	AUTO	
>Study Instance UID	0020,000D	UI		ALWAYS	AUTO	
>Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
>Purpose of Reference Code Sequence	0040,A170	SQ		VNAP	AUTO	
Performed Procedure Step Start Date	0040,0244	DA		ANAP	COPY	
Performed Procedure Step Start Time	0040,0245	TM		ANAP	COPY	
Performed Procedure Step ID	0040,0253	SH		ANAP	COPY	

Table 39: General Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Manufacturer	0008,0070	LO	Philips	ALWAYS	FIXED	
Institution Name	0008,0080	LO		VNAP	CONFIG	Configured Hospital Name
Station Name	0008,1010	SH		ALWAYS	AUTO	Hostname IW PC
Manufacturer's Model Name	0008,1090	LO	Interventional Workspot	ALWAYS	FIXED	
Device Serial Number	0018,1000			ANAP	AUTO	MAC address of hospital NIC

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Software Versions	0018,1020	LO	1.8.x	ALWAYS	AUTO	where “x” is the detailed application SW version.

**Table 40: SC Equipment Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Conversion Type	0008,0064	CS	WSD	ALWAYS	FIXED	

**Table 41: General Image Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Instance Number	0020,0013	IS		ALWAYS	AUTO	
Patient Orientation	0020,0020	CS		VNAP	AUTO	
Burned in Annotation	0028,0301	CS		ANAP	AUTO	

**Table 42: Image Pixel Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Samples per Pixel	0028,0002	US		ALWAYS	AUTO	
Photometric Interpretation	0028,0004	CS		ALWAYS	AUTO	
Planar Configuration	0028,0006	US		ALWAYS	AUTO	
Rows	0028,0010	US		ALWAYS	AUTO	
Columns	0028,0011	US		ALWAYS	AUTO	
Bits Allocated	0028,0100	US	8	ALWAYS	AUTO	
Bits Stored	0028,0101	US	8	ALWAYS	AUTO	
High Bit	0028,0102	US	7	ALWAYS	AUTO	
Pixel Representation	0028,0103	US	0	ALWAYS	AUTO	
Pixel Data	7FE0,0010	OW/OB		ALWAYS	AUTO	

**Table 43: Cine Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Frame Time	0018,1063	DS		ALWAYS	AUTO	

**Table 44: Multi-Frame Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Number of Frames	0028,0008	IS		ALWAYS	AUTO	
Frame Increment Pointer	0028,0009	AT		ALWAYS	AUTO	

**Table 45: Multi-Frame Functional Groups Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Instance Number	0020,0013	IS		ALWAYS	AUTO	
Number of Frames	0028,0008	IS		ALWAYS	AUTO	

**Table 46: SC Multi-Frame Image Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Frame Increment Pointer	0028,0009	AT		ALWAYS	AUTO	
Burned In Annotation	0028,0301	CS		ALWAYS	AUTO	

**Table 47: SOP Common Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Specific Character Set	0008,0005	CS		ANAP	AUTO	As supported by hosting platform
SOP Class UID	0008,0016	UI	1.2.840.10008.5.1.4.1.1.7.4	ALWAYS	FIXED	
SOP Instance UID	0008,0018	UI		ALWAYS	AUTO	
Instance Creation Date	0008,0012	DA		ALWAYS	AUTO	
Instance Creation Time	0008,0013	TM		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	

**8.1.2.1.4 X-Ray Angiographic Image Storage SOP Class**

**Table 48: IOD of Created X-Ray Angiographic Image Storage SOP Class Instances**

Information Entity	Module	Presence Of Module
Patient	Patient Module	ALWAYS
Study	General Study Module	ALWAYS
	Patient Study Module	ALWAYS
Series	General Series Module	ALWAYS
Equipment	General Equipment Module	ALWAYS
Acquisition	General Acquisition Module	ALWAYS
Image	General Image Module	ALWAYS
	Image Pixel Module	ALWAYS
	Cine Module	ALWAYS
	Multi-Frame Module	ALWAYS
	Display Shutter Module	ALWAYS
	X-Ray Image Module	ALWAYS
	X-Ray Acquisition Module	ALWAYS
	X-Ray Table Module	ALWAYS
	XA Positioner Module	ALWAYS
	DX Detector Module	ALWAYS
VOI LUT Module	ALWAYS	

Information Entity	Module	Presence Of Module
	SOP Common Module	ALWAYS

**Table 49: Patient Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Name	0010,0010	PN		VNAP	COPY	
Patient ID	0010,0020	LO		VNAP	COPY	
Patient's Birth Date	0010,0030	DA		VNAP	COPY	
Patient's Sex	0010,0040	CS		VNAP	COPY	

**Table 50: General Study Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Study Date	0008,0020	DA		ALWAYS	COPY	
Study Time	0008,0030	TM		ALWAYS	COPY	
Accession Number	0008,0050	SH		VNAP	COPY	
Referring Physician's Name	0008,0090	PN		VNAP	COPY	
Study Instance UID	0020,000D	UI		ALWAYS	COPY	
Study ID	0020,0010	SH		VNAP	COPY	

**Table 51: Patient Study Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Weight	0010,1030	DS		VNAP	COPY	

**Table 52: General Series Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Date	0008,0021	DA		ALWAYS	AUTO	
Series Time	0008,0031	TM		ALWAYS	AUTO	
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Series Description	0008,103E	LO		ANAP	COPY	
Performing Physician's Name	0008,1050	PN		ANAP	COPY	
Related Series Sequence	0008,1250	SQ		VNAP	AUTO	
>Study Instance UID	0020,000D	UI		ALWAYS	AUTO	
>Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
>Purpose of Reference Code Sequence	0040,A170	SQ		VNAP	AUTO	
Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
Series Number	0020,0011	IS		VNAP	AUTO	
Laterality	0020,0060	CS		ANAP	COPY	
Performed Procedure Step Start Date	0040,0244	DA		ANAP	COPY	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Performed Procedure Step Start Time	0040,0245	TM		ANAP	COPY	
Performed Procedure Step ID	0040,0253	SH		ANAP	COPY	

Table 53: General Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Manufacturer	0008,0070	LO	Philips	ALWAYS	FIXED	
Institution Name	0008,0080	LO		VNAP	CONFIG	Configured hospital name
Station Name	0008,1010	SH		ANAP	AUTO	
Manufacturer's Model Name	0008,1090	LO	Interventional Workspot	ALWAYS	FIXED	
Device Serial Number	0018,1000	LO		ALWAYS	AUTO	MAC address of hospital NIC
Software Versions	0018,1020	LO	1.8.x	ALWAYS	AUTO	where "x" is the detailed application SW version.

Table 54: General Acquisition Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Acquisition Date	0008,0022	DA		ALWAYS	COPY	
Acquisition Time	0008,0032	TM		ALWAYS	COPY	

Table 55: General Image Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Image Type	0008,0008	CS		ALWAYS	COPY	
Content Date	0008,0023	DA		ALWAYS	AUTO	
Content Time	0008,0033	TM		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	COPY	
Patient Orientation	0020,0020	CS		ANAP	COPY	
Lossy Image Compression	0028,2110	CS		ANAP	COPY	
Icon Image Sequence	0088,0200	SQ		ALWAYS	AUTO	
>Samples per Pixel	0028,0002	US		ALWAYS	AUTO	
>Photometric Interpretation	0028,0004	CS		ALWAYS	AUTO	
>Rows	0028,0010	US		ALWAYS	AUTO	
>Columns	0028,0011	US		ALWAYS	AUTO	
>Bits Allocated	0028,0100	US		ALWAYS	AUTO	
>Bits Stored	0028,0101	US		ALWAYS	AUTO	
>High Bit	0028,0102	US		ALWAYS	AUTO	
>Pixel Representation	0028,0103	US		ALWAYS	AUTO	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
>Pixel Data	7FE0,0010	OW/OB		ALWAYS	AUTO	

Table 56: Image Pixel Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Samples per Pixel	0028,0002	US		ALWAYS	COPY	
Photometric Interpretation	0028,0004	CS		ALWAYS	COPY	
Rows	0028,0010	US		ALWAYS	COPY	
Columns	0028,0011	US		ALWAYS	COPY	
Bits Allocated	0028,0100	US		ALWAYS	COPY	
Bits Stored	0028,0101	US		ALWAYS	COPY	
High Bit	0028,0102	US		ALWAYS	COPY	
Pixel Representation	0028,0103	US		ALWAYS	COPY	
Pixel Data	7FE0,0010	OB/OW		ALWAYS	COPY	

Table 57: Cine Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Cine Rate	0018,0040	IS		ALWAYS	COPY	
Frame Time	0018,1063	DS		ALWAYS	COPY	

Table 58: Multi-Frame Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Number of Frames	0028,0008	IS		ALWAYS	COPY	
Frame Increment Pointer	0028,0009	AT		ALWAYS	COPY	

Table 59: Display Shutter Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Shutter Shape	0018,1600	CS	RECTANGULAR	ALWAYS	COPY	
Shutter Left Vertical Edge	0018,1602	IS		ALWAYS	COPY	
Shutter Right Vertical Edge	0018,1604	IS		ALWAYS	COPY	
Shutter Upper Horizontal Edge	0018,1606	IS		ALWAYS	COPY	
Shutter Lower Horizontal Edge	0018,1608	IS		ALWAYS	COPY	

Table 60: X-Ray Image Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Image Type	0008,0008	CS		ALWAYS	COPY	
Samples per Pixel	0028,0002	US		ALWAYS	COPY	
Photometric Interpretation	0028,0004	CS		ALWAYS	COPY	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Frame Increment Pointer	0028,0009	AT		ALWAYS	COPY	
Bits Allocated	0028,0100	US		ALWAYS	COPY	
Bits Stored	0028,0101	US		ALWAYS	COPY	
High Bit	0028,0102	US		ALWAYS	COPY	
Pixel Representation	0028,0103	US		ALWAYS	COPY	
Pixel Intensity Relationship	0028,1040	CS		ALWAYS	COPY	
Lossy Image Compression	0028,2110	CS		ALWAYS	COPY	

**Table 61: X-Ray Acquisition Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
KVP	0018,0060	DS		VNAP	COPY	
Exposure Time	0018,1150	IS		VNAP	COPY	
Radiation Setting	0018,1155	CS		ALWAYS	COPY	
X-Ray Tube Current	0018,1151	IS		ANAP	COPY	
Imager Pixel Spacing	0018,1164	DS		ANAP	COPY	
Pixel Spacing	0028,0030	DS		ALWAYS	COPY	

**Table 62: X-Ray Table Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Table Motion	0018,1134	CS	STATIC	VNAP	COPY	
Table Angle	0018,1138	DS		ANAP	COPY	

**Table 63: XA Positioner Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Distance Source to Detector	0018,1110	DS		ALWAYS	COPY	
Distance Source to Patient	0018,1111	DS		ALWAYS	COPY	
Positioner Motion	0018,1500	CS		VNAP	COPY	
Positioner Primary Angle	0018,1510	DS		VNAP	AUTO	
Positioner Secondary Angle	0018,1511	DS		VNAP	AUTO	
Positioner Primary Angle Increment	0018,1520	DS		ANAP	AUTO	
Positioner Secondary Angle Increment	0018,1521	DS		ANAP	AUTO	

**Table 64: DX Detector Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Imager Pixel Spacing	0018,1164	DS		ALWAYS	COPY	
Pixel Spacing	0028,0030	DS		ALWAYS	COPY	

Table 65: VOI LUT Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Window Center	0028,1050	DS		ALWAYS	COPY	
Window Width	0028,1051	DS		ALWAYS	COPY	

Table 66: SOP Common Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Specific Character Set	0008,0005	CS		ANAP	AUTO	As supported by hosting platform
Instance Creation Date	0008,0012	DA		ALWAYS	AUTO	
Instance Creation Time	0008,0013	TM		ALWAYS	AUTO	
SOP Class UID	0008,0016	UI	1.2.840.10008.5.1.4.1.1.12.1	ALWAYS	FIXED	
SOP Instance UID	0008,0018	UI		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	

8.1.2.1.5 X-Ray 3D Angiographic Image Storage SOP Class

Table 67: IOD of Created X-Ray Angiographic Image Storage SOP Class Instances

Information Entity	Module	Presence Of Module
Patient	Patient Module	ALWAYS
Study	General Study Module	ALWAYS
	Patient Study Module	ALWAYS
Series	General Series Module	ALWAYS
	Enhanced Series Module	ALWAYS
Image	Frame Of Reference Module	ALWAYS
	General Equipment Module	ALWAYS
	Enhanced General Equipment Module	ALWAYS
	Image Pixel Module	ALWAYS
	Acquisition Context Module	ALWAYS
	Multi-frame Functional Groups Module	ALWAYS
	X-Ray 3D Image Module	ALWAYS
	X-Ray 3D Reconstruction Module	ALWAYS
	SOP Common Module	ALWAYS

Table 68: Patient Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Name	0010,0010	PN		VNAP	COPY	
Patient ID	0010,0020	LO		VNAP	COPY	
Patient's Birth Date	0010,0030	DA		VNAP	COPY	
Patient's Sex	0010,0040	CS		VNAP	COPY	



Table 69: General Study Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Study Date	0008,0020	DA		ALWAYS	COPY	
Study Time	0008,0030	TM		ALWAYS	COPY	
Accession Number	0008,0050	SH		VNAP	COPY	
Referring Physician's Name	0008,0090	PN		VNAP	COPY	
Study Instance UID	0020,000D	UI		ALWAYS	COPY	
Study ID	0020,0010	SH		VNAP	COPY	

Table 70: Patient Study Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Weight	0010,1030	DS		ALWAYS	COPY	

Table 71: General Series Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Date	0008,0021	DA		ALWAYS	AUTO	
Series Time	0008,0031	TM		ALWAYS	AUTO	
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Series Description	0008,103E	LO		ALWAYS	COPY	
Performing Physician's Name	0008,1050	PN		VNAP	COPY	
Related Series Sequence	0008,1250	SQ		VNAP	AUTO	
>Study Instance UID	0020,000D	UI		ALWAYS	AUTO	
>Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
>Purpose of Reference Code Sequence	0040,A170	SQ		VNAP	AUTO	
Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
Series Number	0020,0011	IS		ALWAYS	COPY	
Laterality	0020,0060	CS		ANAP	COPY	
Performed Procedure Step Start Date	0040,0244	DA		ANAP	COPY	
Performed Procedure Step Start Time	0040,0245	TM		ANAP	COPY	
Performed Procedure Step ID	0040,0253	SH		ANAP	COPY	

Table 72: Enhanced Series Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Number	0020,0011	IS		ALWAYS	AUTO	

Table 73: Frame of Reference Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Frame of Reference UID	0020,0052	UI		ALWAYS	COPY	
Position Reference Indicator	0020,1040	LO		VNAP	COPY	

Table 74: General Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Manufacturer	0008,0070	LO	Philips	ALWAYS	FIXED	
Institution Name	0008,0080	LO		VNAP	CONFIG	Configured hospital name
Manufacturer's Model Name	0008,1090	LO	Interventional Workspot	ALWAYS	FIXED	
Station Name	0008,1010	SH		ANAP	AUTO	
Device Serial Number	0018,1000	LO		ALWAYS	AUTO	MAC address of hospital NIC
Software Versions	0018,1020	LO	1.8.x	ALWAYS	AUTO	where "x" is the detailed application SW version.

Table 75: Enhanced General Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Manufacturer	0008,0070	LO	Philips	ALWAYS	FIXED	
Manufacturer's Model Name	0008,1090	LO	Interventional Workspot	ALWAYS	FIXED	
Device Serial Number	0018,1000	LO		ALWAYS	AUTO	MAC address of hospital NIC
Software Versions	0018,1020	LO	1.8.x	ALWAYS	AUTO	where "x" is the detailed application SW version.

Table 76: Image Pixel Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Samples per Pixel	0028,0002	US		ALWAYS	AUTO	
Photometric Interpretation	0028,0004	CS	MONOCHROME2	ALWAYS	AUTO	
Rows	0028,0010	US		ALWAYS	AUTO	
Columns	0028,0011	US		ALWAYS	AUTO	
Bits Allocated	0028,0100	US		ALWAYS	AUTO	
Bits Stored	0028,0101	US		ALWAYS	AUTO	
High Bit	0028,0102	US		ALWAYS	AUTO	
Pixel Representation	0028,0103	US		ALWAYS	AUTO	
Pixel Data	7FE0,0010	OB/OW		ALWAYS	AUTO	

Table 77: Acquisition Context Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Acquisition Context Sequence	0040,0555	SQ		VNAP	AUTO	

Table 78: Multi-Frame Functional Groups Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Content Date	0008,0023	DA		ALWAYS	AUTO	
Content Time	0008,0033	TM		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	
Number of Frames	0028,0008	IS		ALWAYS	AUTO	
Shared Functional Groups Sequence	5200,9229	SQ		VNAP	AUTO	
>Frame Anatomy Sequence	0020,9071	SQ		ALWAYS	AUTO	
>>Anatomic Region Sequence	0008,2218	SQ		ALWAYS	AUTO	
>>>Code Value	0008,0100	SH		ALWAYS	AUTO	
>>>Coding Scheme Designator	0008,0102	SH		ALWAYS	AUTO	
>>>Code Meaning	0008,0104	LO		ALWAYS	AUTO	
>>Frame Laterality	0020,9072	CS		ALWAYS	AUTO	
>Pixel Measures Sequence	0028,9110	SQ		ALWAYS	AUTO	
>>Slice Thickness	0018,0050	DS		ALWAYS	AUTO	
>>Pixel Spacing	0028,0030	DS		ALWAYS	AUTO	
>Frame VOI LUT Sequence	0028,9132	SQ		ALWAYS	AUTO	
>>Window Center	0028,1050	DS		ALWAYS	AUTO	
>>Window Width	0028,1051	DS		ALWAYS	AUTO	
Per-frame Functional Groups Sequence	5200,9230	SQ		ALWAYS	AUTO	
>X-Ray 3D Frame Type Sequence	0018,9504	SQ		ANAP	AUTO	
>>Frame Type	0008,9007	CS	DERIVED\PRIMARY\ VOLUME\NONE	ALWAYS	AUTO	
>>Reconstruction Index	0020,9536	US		ALWAYS	AUTO	
>>Pixel Presentation	0008,9205	CS	MONOCHROME	ALWAYS	AUTO	
>>Volumetric Properties	0008,9206	CS	VOLUME	ALWAYS	AUTO	
>>Volume Based Calculation Technique	0008,9207	CS	NONE	ALWAYS	AUTO	
>Frame Content Sequence	0020,9111	SQ		ALWAYS	AUTO	
>Plane Position Sequence	0020,9113	SQ		ALWAYS	AUTO	
>>Image Position (Patient)	0020,0032	DS		ANAP	AUTO	
>Plane Orientation Sequence	0020,9116	SQ		ALWAYS	AUTO	
>>Image Orientation (Patient)	0020,0037	DS		ANAP	AUTO	

Table 79: X-Ray 3D Image Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Image Type	0008,0008	CS	DERIVED\SECONDARY\ AXIAL\3DRA_PROP Or DERIVED\SECONDARY\ AXIAL\3DRA_ROLL	ALWAYS	AUTO	
Content Qualification	0018,9004	CS	PRODUCT	ALWAYS	AUTO	
Samples per Pixel	0028,0002	US		ALWAYS	AUTO	
Photometric Interpretation	0028,0004	CS	MONOCHROME2	ALWAYS	AUTO	
Bits Allocated	0028,0100	US		ALWAYS	AUTO	
Bits Stored	0028,0101	US		ALWAYS	AUTO	
High Bit	0028,0102	US		ALWAYS	AUTO	
Burned In Annotation	0028,0301	CS	NO	ALWAYS	AUTO	
Lossy Image Compression	0028,2110	CS	00	ALWAYS	AUTO	
Icon Image Sequence	0088,0200	SQ		ANAP	AUTO	
>Samples per Pixel	0028,0002	US		ALWAYS	AUTO	
>Photometric Interpretation	0028,0004	CS	MONOCHROME2	ALWAYS	AUTO	
>Rows	0028,0010	US		ALWAYS	AUTO	
>Columns	0028,0011	US		ALWAYS	AUTO	
>Bits Allocated	0028,0100	US		ALWAYS	AUTO	
>Bits Stored	0028,0101	US		ALWAYS	AUTO	
>High Bit	0028,0102	US		ALWAYS	AUTO	
>Pixel Presentation	0008,9205	CS	MONOCHROME	ALWAYS	AUTO	
>Pixel Data	7FE0,0010	OB		ALWAYS	AUTO	
Presentation LUT Shape	2050,0020	CS	IDENTITY	ALWAYS	AUTO	
Pixel Presentation	0008,9205	CS	MONOCHROME	ALWAYS	AUTO	
Volumetric Properties	0008,9206	CS	VOLUME	ALWAYS	AUTO	
Volume Based Calculation Technique	0008,9207	CS	NONE	ALWAYS	AUTO	

Table 80: X-Ray 3D Reconstruction Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
X-Ray 3D Reconstruction Sequence	0018,9530	SQ		ALWAYS	AUTO	
>Application Name	0018,9524	LO	Interventional Workspot	ALWAYS	FIXED	
>Application Version	0018,9525	LO	1.8.5	ALWAYS	FIXED	
>Application Manufacturer	0018,9526	LO	Philips	ALWAYS	FIXED	
>Algorithm Type	0018,9527	CS	FILTER_BACK_PROJ	ALWAYS	FIXED	
>Acquisition Index	0020,9518	US	1	ALWAYS	AUTO	

**Table 81: SOP Common Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Specific Character Set	0008,0005	CS		ANAP	AUTO	As supported by hosting platform
SOP Class UID	0008,0016	UI	1.2.840.10008.5.1.4.1.1.13.1.1	ALWAYS	FIXED	
SOP Instance UID	0008,0018	UI		ALWAYS	AUTO	
Instance Creation Date	0008,0012	DA		ALWAYS	AUTO	
Instance Creation Time	0008,0013	TM		ALWAYS	AUTO	
Original Specialized SOP Class UID	0008,001B	UI		ALWAYS	AUTO	
Content Qualification	0018,9004	CS		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ANAP	AUTO	

**8.1.2.1.6 Raw Data Storage SOP Class**

**Table 82: IOD of Created Raw Data Storage SOP Class Instances**

Information Entity	Module	Presence Of Module
Patient	Patient Module	ALWAYS
Study	General Study Module	ALWAYS
	Patient Study Module	ALWAYS
Series	General Series Module	ALWAYS
Frame of reference	Frame of reference Module	ALWAYS
Equipment	General Equipment Module	ALWAYS
Image	Acquisition Context Module	ALWAYS
	Raw Data Module	ALWAYS
	SOP Common Module	ALWAYS

**Table 83: Patient Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Name	0010,0010	PN		VNAP	COPY	
Patient ID	0010,0020	LO		VNAP	COPY	
Patient's Birth Date	0010,0030	DA		VNAP	COPY	
Patient's Sex	0010,0040	CS		VNAP	COPY	

**Table 84: General Study Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Study Date	0008,0020	DA		ALWAYS	COPY	
Study Time	0008,0030	TM		ALWAYS	COPY	
Accession Number	0008,0050	SH		VNAP	COPY	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Referring Physician's Name	0008,0090	PN		VNAP	COPY	
Study Instance UID	0020,000D	UI		ALWAYS	COPY	
Study ID	0020,0010	SH		VNAP	COPY	

Table 85: Patient Study Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Patient's Weight	0010,1030	DS		ALWAYS	COPY	

Table 86: General Series Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Series Date	0008,0021	DA		ALWAYS	AUTO	
Series Time	0008,0031	TM		ALWAYS	AUTO	
Modality	0008,0060	CS	XA	ALWAYS	FIXED	
Series Description	0008,103E	LO		ALWAYS	COPY	
Performing Physician Name	0008,1050	PN		ANAP	COPY	
Related Series Sequence	0008,1250	SQ		VNAP	AUTO	
>Study Instance UID	0020,000D	UI		ALWAYS	AUTO	
>Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
>Purpose of Reference Code Sequence	0040,A170	SQ		VNAP	AUTO	
Series Instance UID	0020,000E	UI		ALWAYS	AUTO	
Series Number	0020,0011	IS		VNAP	AUTO	
Laterality	0020,0060	CS		ANAP	AUTO	
Performed Procedure Step Start Date	0040,0244	DA		ANAP	COPY	
Performed Procedure Step Start Time	0040,0245	TM		ANAP	COPY	
Performed Procedure Step ID	0040,0253	SH		ANAP	COPY	

Table 87: Frame of Reference Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Frame of Reference UID	0020,0052	UI		ALWAYS	COPY	
Position Reference Indicator	0020,1040	LO		VNAP	COPY	

Table 88: General Equipment Module

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Manufacturer	0008,0070	LO	Philips	ALWAYS	FIXED	

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Institution Name	0008,0080	LO		VNAP	CONFIG	Configured hospital name
Station Name	0008,1010	SH		ANAP	AUTO	
Manufacturer's Model Name	0008,1090	LO	Interventional Workspot	ALWAYS	FIXED	
Device Serial Number	0018,1000	LO		ALWAYS	AUTO	Mac address of Hospital NIC
Software Versions	0018,1020	LO	1.8.x	ALWAYS	AUTO	Where "x" is the detailed application SW version.

**Table 89: Acquisition Context Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Acquisition Context Sequence	0040,0555	SQ		VNAP	AUTO	

**Table 90: Raw Data Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Content Date	0008,0023	DA		ALWAYS	AUTO	
Content Time	0008,0033	TM		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	
Creator Version UID	0008,9123	UI		ALWAYS	AUTO	

**Table 91: SOP Common Module**

Attribute Name	TAG	VR	Value	Presence of Value	Source	Comments
Specific Character Set	0008,0005	CS		ANAP	AUTO	As supported by hosting platform
SOP Class UID	0008,0016	UI	1.2.840.10008.5.1.4.1.1.66	ALWAYS	FIXED	
SOP Instance UID	0008,0018	UI		ALWAYS	AUTO	
Instance Creation Date	0008,0012	DA		ALWAYS	AUTO	
Instance Creation Time	0008,0013	TM		ALWAYS	AUTO	
Instance Number	0020,0013	IS		ALWAYS	AUTO	
Original Specialized SOP Class UID	0008,001B	UI		VNAP	COPY	

*- This part of the page is left intentionally empty -*

**Issued by:**

Philips Medical Systems Nederland B.V., a Philips Healthcare company  
P.O. Box 10.000  
5680 DA Best  
The Netherlands

Internet: <https://www.philips.com/DICOM>

Doc Id: HSDP-1068554

Date:27-Feb-2024

