



# Respironics V60/V60 Plus Ventilator User Manual



# Table of contents

---

<b>1. Warnings, cautions, and notes</b>	<b>1-1</b>
Definitions	1-1
General	1-1
Preparing for ventilation	1-4
Operation	1-7
Operation in high flow therapy (HFT)	1-7
Alarms and messages	1-8
Care and maintenance	1-8
First-time installation	1-9
Communications interface	1-9
Diagnostic mode	1-10
<b>2. Symbols</b>	<b>2-1</b>
<b>3. General information</b>	<b>3-1</b>
Intended use	3-1
Indications for use	3-1
Patient population	3-1
Contraindications	3-2
About CO <sub>2</sub> rebreathing	3-2
Potential side effects	3-2
General description	3-3
Physical description	3-4
Patient circuits, masks/patient interfaces, and accessories	3-4
Ventilator unit	3-6
About the backup battery	3-10
About the graphical user interface	3-12
Navigating the graphical user interface	3-13
Starting up the ventilator	3-15
Shutting down the ventilator	3-15
Training	3-15
<b>4. Principles of operation</b>	<b>4-1</b>
System operational overview	4-1
Pneumatic system operation	4-2
Breath delivery characteristics	4-2
Control variable	4-2
Triggering, cycling, and leak adaptation	4-2
Baseline pressure	4-2
Pressure rise time	4-3
Negative pressures	4-3
Oxygen concentration	4-3
Auto-Trak Sensitivity	4-3
Triggering	4-3
Cycling	4-3

Leak adaptation . . . . .	4-5
Auto-Trak+ (optional) . . . . .	4-6
High flow therapy . . . . .	4-6
Ventilation modes . . . . .	4-7
CPAP mode . . . . .	4-8
PCV mode . . . . .	4-9
S/T mode . . . . .	4-10
AVAPS mode . . . . .	4-11
PPV mode (optional) . . . . .	4-13
Oxygen mixing . . . . .	4-15
<b>5. Setting up the ventilator for use . . . . .</b>	<b>5-1</b>
Connecting oxygen . . . . .	5-1
Installing an oxygen analyzer/monitor . . . . .	5-1
Connecting to AC power . . . . .	5-2
Installing the patient circuit . . . . .	5-3
Installing the nebulizer . . . . .	5-4
Connecting external devices . . . . .	5-7
Before placing a patient on the ventilator . . . . .	5-8
Verify ventilator operation . . . . .	5-8
Running alarm tests . . . . .	5-9
Preparation . . . . .	5-9
High Inspiratory Pressure . . . . .	5-9
Low Tidal Volume . . . . .	5-10
Patient Disconnect . . . . .	5-10
Patient Circuit Occluded . . . . .	5-10
Using the ventilator for intra-hospital transport . . . . .	5-11
Storing the ventilator between patient use . . . . .	5-13
MRI safety information . . . . .	5-13
Security and Privacy Information . . . . .	5-13
<b>6. Operation . . . . .</b>	<b>6-1</b>
Changing the mode . . . . .	6-2
Changing control settings . . . . .	6-3
Making batch setting changes . . . . .	6-3
Changing individual ventilator settings . . . . .	6-4
Using the Ramp Time function . . . . .	6-5
Using the 100% O <sub>2</sub> function . . . . .	6-6
Using PPV . . . . .	6-7
About Max V and Max P alarms and alarm limits . . . . .	6-7
Guidelines for using PPV . . . . .	6-9
Changing alarm settings . . . . .	6-12
Selecting the mask and exhalation port . . . . .	6-12
Running the exhalation port test . . . . .	6-16
Procedure . . . . .	6-16
Troubleshooting . . . . .	6-17
Other functions: the Menu window . . . . .	6-17
Brightness . . . . .	6-17
Loudness . . . . .	6-17
Mask/Port . . . . .	6-18
Vent Info (ventilator information) . . . . .	6-18

	Screen Lock . . . . .	6-18
	Auto-Trak+ . . . . .	6-18
	Standby . . . . .	6-20
	Help function . . . . .	6-22
	Table of modes and control settings. . . . .	6-23
<b>7.</b>	<b>High flow therapy. . . . .</b>	<b>7-1</b>
	Circuit setup . . . . .	7-1
	High flow nasal cannula setup . . . . .	7-2
	Using the FEP Connect for high flow therapy . . . . .	7-2
	High flow nasal cannula setup . . . . .	7-2
	Changing from ventilation to high flow therapy (HFT) . . . . .	7-3
	Changing from high flow therapy (HFT) to ventilation . . . . .	7-3
	Connecting directly to a 22 mm circuit . . . . .	7-4
	Changing from an NIV mode to high flow therapy . . . . .	7-4
	Viewing and pausing the HFT graph. . . . .	7-5
	Changing from high flow therapy to an NIV mode . . . . .	7-6
	HFT alarms and messages . . . . .	7-6
<b>8.</b>	<b>Patient monitoring . . . . .</b>	<b>8-1</b>
	Display conventions . . . . .	8-1
	Table of monitored parameters . . . . .	8-2
	Scaling the waveform axes . . . . .	8-2
	Freezing and unfreezing waveforms . . . . .	8-3
<b>9.</b>	<b>Alarms, messages, and troubleshooting . . . . .</b>	<b>9-1</b>
	Responding to alarms. . . . .	9-1
	Setting alarm loudness. . . . .	9-4
	Silencing alarms . . . . .	9-5
	Resetting alarms . . . . .	9-5
	Manually resetting alarms . . . . .	9-5
	Clearing autoreset alarms from the Alarms list . . . . .	9-5
	Hiding/displaying alarm messages. . . . .	9-6
	Symptom-based troubleshooting . . . . .	9-6
	Alarms and other messages. . . . .	9-7
<b>10.</b>	<b>Care and maintenance . . . . .</b>	<b>10-1</b>
	Exterior and touchscreen cleaning. . . . .	10-2
	Approved cleaning agents . . . . .	10-2
	Cleaning instructions . . . . .	10-2
	Exterior and touchscreen disinfection . . . . .	10-3
	Approved disinfecting agents . . . . .	10-3
	Disinfection instructions . . . . .	10-3
	Bacteria filter, patient circuit, and other accessories. . . . .	10-4
	Preventive maintenance . . . . .	10-5
	Replacing the air inlet filter . . . . .	10-6
	Cleaning or replacing the cooling fan filter . . . . .	10-7
	Removing and replacing the battery . . . . .	10-8
	Disposal . . . . .	10-8
	Storage between patient use . . . . .	10-8

	Service and repairs . . . . .	10-9
	Repacking and shipping . . . . .	10-9
<b>11.</b>	<b>Technical specifications . . . . .</b>	<b>11-1</b>
	Control settings . . . . .	11-1
	Patient data . . . . .	11-3
	Alarms . . . . .	11-4
	Menu window settings . . . . .	11-4
	Diagnostic mode functions . . . . .	11-5
	Physical characteristics . . . . .	11-6
	Environmental specifications . . . . .	11-6
	Pneumatic specifications . . . . .	11-7
	Electrical specifications . . . . .	11-8
	Accessory requirements . . . . .	11-8
	Alarm-related specifications . . . . .	11-9
	Other specifications . . . . .	11-9
<b>A.</b>	<b>First-time installation . . . . .</b>	<b>A-1</b>
	Unpacking and inspection . . . . .	A-1
	Mounting the ventilator . . . . .	A-2
	Installing the battery . . . . .	A-4
	Installing oxygen inlet connector and AC power cord . . . . .	A-7
	Installing the oxygen manifold kit . . . . .	A-9
	Verifying ventilator operation and audible alarm . . . . .	A-9
	Configuration and screen calibration . . . . .	A-9
<b>B.</b>	<b>Communications interface . . . . .</b>	<b>B-1</b>
	RS-232 serial port . . . . .	B-2
	Supported communication protocols . . . . .	B-2
	Using Philips IntelliBridge or VueLink . . . . .	B-2
	Using Philips monitors and the IntelliBridge or VueLink	
	Open Interfaces . . . . .	B-2
	Data display . . . . .	B-3
	Remote alarm port . . . . .	B-5
<b>C.</b>	<b>Parts and accessories . . . . .</b>	<b>C-1</b>
	Patient Interfaces . . . . .	C-2
	Ventilation interfaces . . . . .	C-2
	HFT interfaces . . . . .	C-2
	O <sub>2</sub> analyzer/monitor . . . . .	C-2
	Patient breathing circuits . . . . .	C-3
	Humidifiers . . . . .	C-3
	Bacteria filters . . . . .	C-4
	Nebulizers . . . . .	C-4
	Operator maintenance parts . . . . .	C-4
	Other parts . . . . .	C-4

<b>D.</b>	<b>Regulatory compliance</b> .....	<b>D-1</b>
	Electromagnetic compatibility (EMC) .....	D-1
	Electromagnetic compatibility declaration .....	D-1
	Electromagnetic emissions .....	D-2
	Electromagnetic immunity .....	D-3
	RF immunity .....	D-6
	RFID reader separation distance .....	D-6
	Cables That May Affect IEC 60601-1-2 Compliance .....	D-7
	WEEE recycling directive .....	D-7
	Classification .....	D-8
	Safety .....	D-8
	Applied parts .....	D-9
	Accessible parts .....	D-9
	Detachable components .....	D-9
	Essential performance .....	D-9
<b>E.</b>	<b>Diagnostic mode</b> .....	<b>E-1</b>
	Entering the diagnostic mode .....	E-1
	System settings .....	E-3
	Language .....	E-4
	Date/Time .....	E-6
	Pressure Units .....	E-7
	Restore Default Settings .....	E-8
	Software Options .....	E-9
	Baud Rate .....	E-10
	Alarm Volume Escalation .....	E-11
	Service .....	E-12
	Significant Event Log .....	E-12
	Touchscreen calibration .....	E-14
	Exiting the diagnostic mode .....	E-15
	<b>Glossary</b> .....	<b>Glossary-1</b>
	<b>Index</b> .....	<b>Index-1</b>



# Chapter 1. Warnings, cautions, and notes

---

Before using the Respironics V60/V60 Plus Ventilator on a patient, familiarize yourself with this user manual, particularly the safety considerations listed. Be aware, however, that this manual is a reference only. It is not intended to supersede your institution's protocol regarding the safe use of assisted ventilation.

## Definitions

---

**WARNING:** Alerts the user to the possibility of injury, death, or other serious adverse reactions associated with the use or misuse of the device.

---

---

**CAUTION:** Alerts the user to the possibility of a problem with the device associated with its use or misuse, such as device malfunction, device failure, damage to the device, or damage to other property.

---

---

**NOTE:** Emphasizes information of particular importance.

---

## General

**WARNING:** Always have immediate access to an alternative means of ventilation. If there is a ventilator failure, immediately remove the ventilator from use and secure an alternative means of ventilation, such as a self-inflating, manually powered resuscitator and mask. Failure to do so can result in patient injury or death.

**WARNING:** Use the Respironics V60/V60 Plus Ventilator on spontaneously breathing patients only. It is an assist ventilator and is intended to augment the ventilation of a spontaneously breathing patient. It is not intended to provide the total ventilatory requirements of the patient.

**WARNING:** We do not recommend you use the Respironics V60/V60 Plus Ventilator on patients who require ventilation at predetermined tidal volumes. The ventilator provides continuous positive airway pressure (CPAP) and positive pressure ventilation (S/T, PCV, AVAPS, and PPV) and is indicated for assisted ventilation only. These modes do not provide ventilation with guaranteed tidal volume delivery.

## Warnings, cautions, and notes

- WARNING:** We do not recommend you use AVAPS on patients who require rapid and frequent IPAP adjustments to maintain a consistent tidal volume. AVAPS, a volume targeted mode, changes the IPAP setting in order to achieve the target tidal volume. During AVAPS setup, there may be a period of time before the target tidal volume is achieved. AVAPS is ideal for more stabilized patients.
- WARNING:** To reduce the risk of CO<sub>2</sub> rebreathing, make sure EPAP pressures and exhalation times are sufficient to clear all exhaled gas through the exhalation port. In noninvasive ventilation continuous air flow through the port flushes exhaled gases from the circuit. The ability to completely exhaust exhaled gas from the circuit depends on the EPAP setting and I:E ratio. Higher tidal volumes further increase the volume of CO<sub>2</sub> rebreathed by the patient. Note: this may occur if exhalation time is insufficient.
- WARNING:** To reduce the risk of CO<sub>2</sub> rebreathing, monitor the patient for changes in respiratory status at the start of ventilation and with each change in ventilator settings, circuit configuration, or patient condition. Pay attention to ventilator alarms that warn of increased CO<sub>2</sub> rebreathing risk.
- WARNING:** To ensure accuracy of oxygen administration and to monitor for the presence of contamination (incorrect gas connected), use an external oxygen monitor to verify the oxygen concentration in the delivered gas.
- WARNING:** To reduce the risk of fire, use the ventilator in well-ventilated areas away from flammable anesthetics. Do not use in a hyperbaric chamber or other similarly oxygen-enriched environments. Do not use near an open flame.
- WARNING:** To reduce the risk of electric shock from liquid entering the device, do not put a container filled with a liquid on the ventilator.
- WARNING:** To reduce patient risk of oxygen toxicity, keep free-flowing oxygen away from air inlet of ventilator.
- WARNING:** The nurse call/remote alarm should be considered a backup to the ventilator's primary alarm system.
- WARNING:** Set the alarm loudness above the ambient level. Setting the alarm loudness too low may prevent recognition of alarm conditions.
- WARNING:** Avoid blocking the alarm speakers beneath the ventilator.
- WARNING:** Do not leave the ventilator unattended when stationed on an incline.
- WARNING:** The V60/V60 Plus Ventilator may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the ventilator or shielding the location.
- WARNING:** This Equipment is designed to comply with IEC 60601-1-2. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. Harmful interference to other devices can be determined by turning this equipment ON and OFF. Try to correct the interference using one or more of the following:
- Reorient or relocate the receiving device
  - Increase the separation between the equipment

## Warnings, cautions, and notes

- **Connect the equipment to an outlet on a different circuit from that to which the other device(s) are connected and consult the Philips service technician for help**
- **Consult Philips for help**

**WARNING:** Use of non-approved accessories, transducers or cables may increase EMC emissions or decrease the EMC immunity performance of the equipment.

**WARNING:** Do not use the ventilator in an MRI environment. The V60/V60 Plus Ventilator is MR Unsafe. Keep it outside the MRI scan room (Zone IV). It represents a projectile hazard.

**CAUTION:** Federal law (USA) restricts this device to sale by or on the order of a physician.

**CAUTION:** The Respironics V60/V60 Plus Ventilator is designed to operate in the temperature range of 5 to 40 °C (41 to 104 °F). To minimize the risk of overheating the device, do not operate adjacent to heaters or other heat sources.

**NOTE:** The displays shown in this manual may not exactly match what you see on your own ventilator.

**NOTE:** Pressures are indicated on the ventilator in cmH<sub>2</sub>O. Millibars and hectopascals (hPa) are used by some institutions instead. Since 1 millibar equals 1 hPa, which equals 1.016 cmH<sub>2</sub>O, the units may be used interchangeably.

**NOTE:** The ventilator is *not* intended for use as an ambulance transport ventilator or as an Automatic Transport Ventilator as described by the American Hospital Association and referenced by the FDA. It *is* intended to allow the patient to be transported within the hospital setting using a cart to move the ventilator.

**NOTE:** When attachments or other components or subassemblies are added to the ventilator breathing system, the pressure gradient across the ventilator breathing system, measured with respect to the ventilator outlet, may increase.

**NOTE:** To ensure the correct performance of the ventilator and the accuracy of patient data, use only Respironics-approved accessories with the ventilator. See Appendix C, "Parts and accessories".

**NOTE:** This Respironics V60/V60 Plus Ventilator and its recommended accessories that have patient contact are not made with natural rubber latex.

**NOTE:** If an alarm persists for no apparent reason, discontinue ventilator use and contact Philips.

**NOTE:** If you detect any unexplained changes in the performance or visual displays of the ventilator, discontinue ventilator use and contact Philips.

**NOTE:** The Respironics V60/V60 Plus Ventilator does not support automatic record keeping.

## Warnings, cautions, and notes

- NOTE: All ventilator mode and alarm settings, alarm messages and significant events are retained and automatically logged, even when power is lost.
- NOTE: Any serious incident that has occurred in relation to this device should be reported to Philips and the competent authority of the country in which the user and/or patient is established.
- 

## Preparing for ventilation

- 
- WARNING:** Connect the ventilator only to an appropriate medical-grade oxygen source.
- WARNING:** To reduce the risk of hypoxia, connect only oxygen to the high-pressure connector at the rear of the ventilator.
- WARNING:** The ventilator is a high-flow device. Connect it only to a gas supply system that can provide adequate flow to all terminal outlets. Connecting the ventilator to an adequate gas supply system helps ensure that the ventilator and other connected devices perform to their specifications.
- WARNING:** To reduce the risk of fire, do not use a high-pressure oxygen hose that is worn or contaminated with combustible materials like grease or oil.
- WARNING:** The Respironics V60/V60 Plus Ventilator is designed to use ambient air and high pressure 100% oxygen. No other gases should be used.
- WARNING:** Do not use the ventilator with helium or helium mixtures. The ventilator is not intended to be used with helium or heliox. Connecting helium to the ventilator may affect ventilator performance, gas mixtures, and measurements due to the lower density of helium.
- WARNING:** Do not use the ventilator with nitric oxide.
- WARNING:** To prevent possible asphyxia and to reduce the risk of CO<sub>2</sub> rebreathing, take these precautions with respect to mask and exhalation port use:
- Use only an oro-nasal mask with an anti-asphyxia valve or a nasal mask for noninvasive ventilation.
  - Do not occlude the exhalation port.
  - Turn on the ventilator and verify that the port is operational before application. Pressurized gas from the ventilator should cause a continuous flow of air to exhaust from the leak port, flushing exhaled gas from the circuit.
  - Never leave the mask on the patient while the ventilator is not operating. When the ventilator is not operating, the exhalation port does not allow sufficient exhaust to eliminate CO<sub>2</sub> from the circuit. Substantial CO<sub>2</sub> rebreathing may occur.
- WARNING:** The patient's exhaled volume can differ from the measured exhaled volume due to leaks around the mask during noninvasive ventilation.
- WARNING:** To ensure normal air circulation and exchange, do not cover or block the ports on the ventilator. Do not block the air inlet panel on the right side of the ventilator.

## Warnings, cautions, and notes

- WARNING:** Do not cover or position the ventilator so as to adversely affect its operation or performance. For example, positioning the ventilator next to a curtain that blocks the flow of cooling air can cause the equipment to overheat. Use the V60/V60 Plus in an upright position that does not block the air inlet.
- WARNING:** Do not block the blower intake. Blocking the intake can impair ventilator performance and result in patient injury.
- WARNING:** To reduce the risk of the device overheating and possible burn injury, do not block the fan intake at the rear of the ventilator.
- WARNING:** To prevent possible patient injury and possible water damage to the ventilator, make sure the humidifier is set appropriately.
- WARNING:** When using a humidifier, always use either a circuit with a water trap or a heated wire circuit to minimize patient risk from condensate in the circuit.
- WARNING:** To prevent the possibility of inadequate humidification, pay close attention to the humidifier's functioning when operating the ventilator at an ambient temperature > 30 °C (86 °F). The ventilator warms the air delivered to the patient above ambient temperature, which may impair the humidifier's performance.
- WARNING:** To reduce the risk that the patient will aspirate condensed water from the breathing circuit, position any humidifier lower than both the ventilator and the patient.
- WARNING:** To prevent possible patient injury and equipment damage, do not turn the humidifier on until the gas flow has started and is regulated. Starting the heater or leaving it on without gas flow for prolonged periods may result in heat build-up, causing a bolus of hot air to be delivered to the patient. Circuit tubing may melt under these conditions. Turn the heater power switch off before stopping gas flow.
- WARNING:** To reduce the risk of fire, use only patient circuits intended for use in oxygen-enriched environments. Do not use antistatic or electrically conductive tubing.
- WARNING:** To prevent patient or ventilator contamination, always use a main flow bacteria filter on the patient gas outlet port. Filters not approved by Respironics may degrade system performance.
- WARNING:** During ventilation, patient exhalate is released into room air. Use of a patient circuit with a filter on its exhalation port is recommended.
- WARNING:** To reduce the risk of bacterial contamination or damage, handle bacteria filters with care.
- WARNING:** Avoid adding any components to the patient circuit that are not absolutely necessary. Additional components installed in the patient circuit can change the pressure gradient across the ventilator breathing system, increase the dead space, and adversely affect the ventilator performance.
- WARNING:** Any additional accessories in the patient circuit may substantially increase flow resistance and impair ventilation.
- WARNING:** Avoid adding resistive circuit components on the patient side of the proximal pressure line. Such components may defeat the disconnect alarm.

## Warnings, cautions, and notes

- WARNING:** To reduce the risk of strangulation from patient tubing, use a tubing support arm and secure the proximal pressure line with clips.
- WARNING:** To reduce the risk of electric shock, connect the ventilator to an AC supply mains with protective earth only.
- WARNING:** Do not use extension cords, adapters, or power cords with the ventilator that are not approved by Respironics.
- WARNING:** To prevent unintentional disconnection of the power cord, always use the correct, Philips-supplied power cord and lock it into place with the power cord retainer before you switch the ventilator on. The retainer is designed to hold the connector end of the Philips-supplied cord securely in place.
- WARNING:** The V60/V60 Plus Ventilator should not be positioned in a way that makes it difficult to disconnect from mains power if necessary. Disconnect from supply mains by removing the power cord from the wall outlet. The AC mains plug is used as disconnection device.
- WARNING:** To reduce the risk of electric shock, regularly inspect the AC power cord and verify that it is not frayed or cracked.
- WARNING:** To reduce the risk of strangulation, route the power cord to avoid entanglement.
- WARNING:** To reduce the risk of power failure to the ventilator, pay close attention to the battery's charge level. The battery's operation time is approximate and is affected by ventilator settings, discharge and recharge cycles, battery age, and ambient temperature.
- WARNING:** Always check the status of the oxygen cylinders before using the ventilator during transport.
- WARNING:** Provide external oxygen monitoring to minimize patient risk in case of O<sub>2</sub> supply loss or ventilator failure.
- WARNING:** To ensure the ventilator's safe operation, always verify ventilator operation as described in "Verify ventilator operation" on page 5-8 before using the ventilator on a patient. If the ventilator fails any tests, remove it from clinical use immediately. Do not use the ventilator until necessary repairs are completed and all tests have passed.
- WARNING:** To prevent possible patient injury due to nonannunciating alarms, verify the operation of any remote alarm device before use.
- WARNING:** To prevent possible patient injury, always return alarm settings to hospital-standard values after verifying ventilator operation.
- WARNING:** Manufacturer default settings are not appropriate for all patients. Prior to using the ventilator, verify that the current alarm settings or defaults are appropriate for each particular patient.
- CAUTION:** To prevent possible damage to the ventilator, ensure that the connection to the oxygen supply is clean and unlubricated, and that there is no water in the oxygen supply gas.
- CAUTION:** For 120 V equipment, grounding reliability can only be achieved when it is connected to an equivalent receptacle marked "hospital only" or "hospital grade."

## Warnings, cautions, and notes

- CAUTION:** Oxygen hose configurations using SIS connectors generate higher resistance to flow. Therefore, a supply pressure of 53 to 87 psig is recommended when adding supplemental O<sub>2</sub> accessories with SIS adapters such as the O<sub>2</sub> transport manifold.
- NOTE:** The V60/V60 Plus Ventilator is a single-limb device with substantial intentional and unintentional leak in the ventilator breathing system. Under those conditions, CO<sub>2</sub> cannot be measured accurately. Therefore, we do not recommend the use of CO<sub>2</sub> monitoring.
- 

## Operation

- WARNING:** To prevent possible patient injury, avoid setting alarm limits to extreme values, which can render the alarm system useless.
- WARNING:** PPV limits are not intended to be the primary ventilator alarms and should not be substituted for the alarms found in the Alarm Settings window.
- WARNING:** To prevent the delivery of excessive pressure or volume, set the PPV limits appropriately. Delivery of excessive pressure or volume can occur from a sudden increase in mask leak, inappropriate settings, or a plugged or kinked proximal pressure line. Conversely, insufficient treatment may result if limits are set too low.
- WARNING:** Nebulization or humidification can increase the resistance of breathing system filters. When using a nebulizer or humidifier, monitor the breathing system filter frequently for increased resistance and blockage.
- WARNING:** Using a jet nebulizer can cause inadvertent alarms and affect the accuracy of delivered FiO<sub>2</sub>. To reduce patient risk, use only an approved nebulizer.
- 

## Operation in high flow therapy (HFT)

- WARNING:** When transitioning from a high flow therapy interface to an NIV mask, ensure that an exhalation port is placed in the circuit and is unobstructed to reduce the risk of CO<sub>2</sub> rebreathing.
- WARNING:** When transitioning from ventilation to high flow therapy, remove the NIV mask and use only a Philips-approved high flow patient interface to minimize pressure build-up and patient discomfort.
- WARNING:** When transitioning from high flow therapy to ventilation, remove the high flow nasal cannula as it is restrictive and may defeat alarms such as patient disconnect. Using a high flow nasal cannula in an NIV mode may lead to hypercarbia due to the inability to provide pressure support.
- WARNING:** Patient alarms are not available during high flow therapy (HFT) as the therapy uses an open system. A high flow nasal cannula occupies only a portion of the nares and patients can breathe through their mouth, which prevents estimation of patient parameters such as tidal volume, respiratory rate, pressure, and minute ventilation. Provide external monitoring, including oximetry, to inform the clinician of a change in the patient's condition.
-

## Warnings, cautions, and notes

**WARNING:** During high flow therapy (HFT), verify that an occlusive patient interface is not being used. Occlusive patient interfaces include a cannula fully sealed within the nares, an NIV mask, or a direct connection to a tracheostomy tube or endotracheal tube. Remove any occlusive interface immediately as this may expose the patient to unintended high pressures.

---

## Alarms and messages

**WARNING:** If AC power fails and the backup battery is not installed or is depleted, an audible and visual alarm annunciates for at least 2 minutes. Immediately discontinue ventilator use and secure an alternative means of ventilation. As in most ventilators with passive exhalation ports, when power is lost, sufficient air is not provided through the circuit and exhaled air may be rebreathed.

---

## Care and maintenance

**WARNING:** To reduce the risk of electric shock, power down the ventilator and disconnect it from AC power before cleaning, disinfecting, or servicing it.

**WARNING:** Turn off the ventilator and disconnect it from the AC mains outlet before you perform decontamination or maintenance procedures. Failure to do so may result in electric shock.

**WARNING:** To prevent patient or ventilator contamination, inspect and replace the main flow bacteria filter between patients and at regular intervals (or as stated by the manufacturer).

**WARNING:** To prevent possible patient injury, inspect and verify the proper operation of the exhalation port regularly during use.

**WARNING:** To reduce the risk of fire, explosion, leakage, or other hazard, take these precautions with respect to the battery:

- Do not attempt to disassemble, open, drop, crush, bend or deform, insert foreign objects into, puncture, or shred the battery pack; modify or remanufacture it; immerse or expose it to water or other liquids; expose it to fire, excessive heat (including soldering irons); or put it in a microwave oven.
- Replace the battery only with another battery specified by the manufacturer.
- Follow all instructions for proper use of the battery.
- Do not short-circuit the battery or allow metallic or conductive objects to contact the battery connector housing.
- Use the battery with the Respironics V60/V60 Plus Ventilator only.

**WARNING:** Modification of the V60/V60 Plus Ventilator and associated equipment is not permitted and may compromise ventilator operation and patient safety. Service should only be performed by qualified service personnel.

**WARNING:** Only authorized service personnel should replace parts within the ventilator or perform other service activities. Unauthorized personnel without proper training are at risk of electric shock.

## Warnings, cautions, and notes

- WARNING:** This product consists of devices that may contain mercury, which must be recycled or disposed of in accordance with local, state, or federal laws. (Within this system, the backlight lamps in the monitor display contain mercury.)
- CAUTION:** Do not attempt to sterilize or autoclave the ventilator.
- CAUTION:** To prevent possible damage to the ventilator, use only those cleaning and disinfecting agents listed in this manual.
- CAUTION:** To prevent possible damage to the ventilator, do not drip or spray any liquids directly onto any surface including the front panel, touchscreen, navigation ring (legacy versions), and Accept button.
- CAUTION:** Never clean or disinfect the touchscreen with an abrasive brush or device, since this will cause irreparable damage.
- CAUTION:** To avoid introducing foreign matter into the ventilator and to ensure proper system performance, change the air inlet filter at regular intervals (or as stipulated by your institution).
- CAUTION:** To ensure proper system performance, use a Respiration-approved air inlet filter.
- CAUTION:** Because some environments cause a quicker collection of lint and dust than others, inspect the filters more often when needed. The air inlet filter should be replaced; the cooling fan filter should be cleaned.
- CAUTION:** To prevent possible damage to the ventilator, always ship it with the original packing material. If the original material is not available, contact Philips to order replacements.
- 

## First-time installation

- WARNING:** Never attempt to disconnect or connect the battery during operation.
- CAUTION:** To prevent possible damage to the ventilator, always secure it to its stand or securely place it on a flat, stable surface that is free of dirt and debris. Do not use the ventilator adjacent to, or stack it with, other equipment.
- 

## Communications interface

- WARNING:** Connect to the ventilator only items that are specified as part of or compatible with the ventilator system. Additional equipment connected to medical electrical equipment must comply with the respective IEC or ISO standards. Furthermore, all configurations shall comply with the requirements for medical electrical systems (see IEC 60601-1-1 or clause 16 of edition 3 of IEC 60601-1, respectively). Anybody connecting additional equipment to medical electrical equipment configures a medical system and is therefore responsible for ensuring that the system complies with the requirements for medical electrical systems. Also be aware that local laws may take priority over the above mentioned requirements. If in doubt, consult Philips.
- WARNING:** The USB port is not currently available for use. DO NOT connect or attempt to power any equipment from the USB port.
-

## Warnings, cautions, and notes

- WARNING:** It is the responsibility of the end user to validate the compatibility and use of information transmitted from the ventilator to the device to be connected to the ventilator.
- WARNING:** The data provided through the communications interface is for reference only. Decisions for patient care should be based on the clinician's observations of the patient.
- WARNING:** To prevent possible patient injury due to nonannunciating alarms, verify the operation of any remote alarm device before use.
- WARNING:** To ensure the functionality of the remote alarm, connect only Respiration-approved cables to the remote alarm port.
- CAUTION:** The remote alarm port is intended to connect only to an SELV (safety extra-low voltage and ungrounded system with basic insulation to ground), in accordance with IEC 60601-1. To prevent damage to the remote alarm, make sure the signal input does not exceed the maximum rating of 24 VAC or 36 VDC at 500 mA with a minimum current of 1 mA.
- 

## Diagnostic mode

- WARNING:** To prevent possible patient injury, do not enter the diagnostic mode while a patient is connected to the ventilator. Verify that the patient is disconnected before proceeding.
-

## Chapter 2. Symbols

Refer to these tables to interpret symbols used on the ventilator labels, backup battery labels, and packaging and on the ventilator screen. To interpret symbols pertaining to accessories, refer to their instructions for use.

Table 2-1: Symbols used on ventilator labels, battery, and packaging

Symbol	Description
	Warning: Risk of explosion. Do not use in the presence of flammable anesthetics.
	Attention, consult the accompanying documents.
	Read the user manual before using the ventilator. Symbol may be accompanied by the web address <a href="http://www.Philips.com/IFU">www.Philips.com/IFU</a> to indicate access to electronic IFUs.
	Electronic instructions for use. Indicates that relevant information for use of the product is available in electronic form.
	(Blue) It is mandatory for the operator to consult the accompanying documents.
	Protective earth (ground)
	Medical device
	Unique device identifier
	Distributor. Symbol accompanied by address
	Importer. Symbol accompanied by address

## Symbols

Table 2-1: Symbols used on ventilator labels, battery, and packaging

Symbol	Description
	Fragile
	Keep dry
	Stacking limit by number
	This end up
	Type B applied part, which is equipment that provides a particular degree of protection against electric shock, particularly in regard to allowable leakage current and of the protective earth connection
	Requires alternating current (AC)
<b>IPX1</b>	Degree of fluid ingress protection provided by the enclosure (drip-proof)
<b>IP21</b>	Degree of solid object protection and fluid ingress protection provided by the enclosure (drip-proof)
<b>Rx</b> ONLY	Caution: Federal law restricts this device to sale by or on the order of a physician
	Alarm and remote alarm
	Two states of control: ON and Shutdown
	Battery
<b>CE</b> 2797	European Conformity. Symbol is on rear panel of ventilator.

## Symbols

Table 2-1: Symbols used on ventilator labels, battery, and packaging

Symbol	Description
	<p>Brazilian Conformity. Certification by INMETRO (National Institute of Metrology, Standardization and Industrial Quality)/SGS (Societe Generale de Surveillance). One of these three symbols, depending upon available space.</p>
	<p>EurAsian Conformity mark - EAC</p>
	<p>Date of manufacture. Symbol accompanied by date.</p>
	<p>Country of Manufacture. Symbol accompanied by country code (and optionally manufacture date and/or manufacturer's name and address).</p>
	<p>Manufacturer. Symbol accompanied by manufacturer's name and address.</p>
	<p>EC representative</p>
	<p>Serial number</p>
	<p>Order number</p>
	<p>Lot or batch number</p>
	<p>Model number</p>

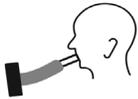
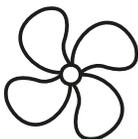
## Symbols

Table 2-1: Symbols used on ventilator labels, battery, and packaging

Symbol	Description
	Use by date
	RS-232 serial input/output
	USB port
	Oxygen
	(Yellow) Warning
	Ethernet connection
	Accept button on the top-right front of the ventilator
	Adjustment direction on the navigation ring (legacy versions only)
	ETL certification mark
	Do not disassemble. Refer to authorized service personnel.
	Product must be disposed of in accordance with the WEEE directive.

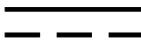
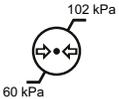
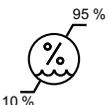
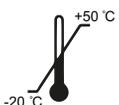
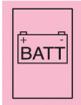
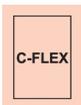
## Symbols

Table 2-1: Symbols used on ventilator labels, battery, and packaging

Symbol	Description
	Noninvasive ventilation (patient with mask)
	Invasive ventilation (intubated patient)
	The V60/V60 Plus Ventilator is MR Unsafe and presents a projectile hazard. Keep the ventilator outside MRI scan room (Zone IV).
	Do not block the cooling fan Inlet (at the rear of the ventilator).
	Used in conjunction with the blue “consult accompanying documents” symbol to indicate “Do not block the cooling fan Inlet.”
	No pushing. Do not push on the ventilator screen. Tipping hazard.
	Total mass (weight). See page 11-6 for details on physical characteristics of the ventilator, stand, and accessories.
 (On power cord)	Hospital-grade
	Packaging unit. Symbol accompanied by a number (indicating the number of pieces in the package)
	Recycle

## Symbols

Table 2-1: Symbols used on ventilator labels, battery, and packaging

Symbol	Description
	Recycle (Taiwan)
	RoHS (China). Administrative Measure on the Control of Pollution Caused by Electronic Information Products. Contains RoHS substances with 50 years environmentally friendly use period (EFUP).
	uR UL recognition symbol
	Direct current (DC). Symbol is on backup battery.
	Battery check.
	Rechargeable battery. Symbol is on backup battery.
	Lithium-ion battery. Battery must be recycled or disposed of properly. Symbol is on backup battery.
	Barometric pressure limitation. Indicates the acceptable upper and lower limits of barometric pressure for transport and storage.
	Humidity limitation. Indicates the acceptable upper and lower limits of relative humidity for transport and storage.
	Temperature limit. Indicates the maximum and minimum temperature limits at which the item shall be stored or transported.
	Battery included
	C-Flex feature

## Symbols

Table 2-1: Symbols used on ventilator labels, battery, and packaging

Symbol	Description
	AVAPS mode (included)
	PPV software option
	Auto-Trak+ software option
	High flow therapy <b>Note:</b> 3.00 software and above. HFT is optional for model V60 and included with model V60 Plus.

Table 2-2: Symbols used on graphical user interface

Symbol	Description
	Alarm (audible)
	Alarm is silenced
	High priority alarm
	Low priority alarm
	Alarm reset
	Informational message
	Alarm message is displayed. Touch to hide alarm messages.
	Alarm message is hidden. Touch to display alarm messages.

## Symbols

Table 2-2: Symbols used on graphical user interface (continued)

Symbol	Description
	Do not use an NIV mask during high flow therapy (3.00 software and above, and V60 Plus).
 	Increase and decrease (adjustment arrow) buttons. Adjusts a setting or selects a value.
	Accept button. Accepts set values.
	Cancel button. Cancels set values.
	+2:00 minutes button. Adds two minutes to 100% O <sub>2</sub> delivery.
	Ventilator is powered by AC power <i>and</i> the battery is installed.
	Ventilator is powered by AC power <i>and</i> the battery is not installed.
	Ventilator is powered by the battery. This symbol shows the approximate battery time remaining in hours and minutes, and it shows the capacity graphically.
	Help button. Touch to display onscreen help information.
	Vertical autoscale button. Autoscales the Y axis of the graphs to fit the data currently displayed.
	Pause button. Freezes waveforms in the Waveform window.

## Symbols

Table 2-2: Symbols used on graphical user interface (continued)

Symbol	Description
	Pause in progress
	Resume button. Resumes all waveform graphs from a paused state.
	Time base adjust button. Rescales the X axis of the graph display data at 3, 6, 12, and 24 second increments.
$\dot{V}_E$	Estimated minute ventilation
$V_T$	Estimated exhaled tidal volume
$T_I/T_{TOT}$	Duty cycle. Inspiratory time divided by total cycle time.
	No valid data to display
	Data is under range
	Data is over range
	Pressure, centimeters of water
	Flow, liters per minute. BTPS compensated.
	Volume, milliliters
	User-set Ramp Time. Ramp graphic fills in as Ramp Time progresses.

## Symbols

Table 2-2: Symbols used on graphical user interface (continued)

Symbol	Description
 A blue square icon with a white triangle pointing up and to the right. The text "OFF" is written in white above "mins" in a smaller font.	Ramp Time is OFF (no ramp time set).
 A blue square icon with a white number "3" and a small cluster of white dots to its left.	Intentional leak. The number corresponds to the leak symbol printed on Philips Respironics masks.

## Chapter 3. General information

---

This manual covers the Respironics V60 and V60 Plus Ventilator configurations. Both share the same platform. The V60 Plus Ventilator comes standard with High Flow Therapy (HFT). The V60 Ventilator can be field-upgraded with HFT, subject to local regulations. For a full list of features, modes, and options, see "General description" on page 3-3.

---

NOTE: The 3.00 software upgrade, which permits the activation of HFT, and the V60 Plus Ventilator are not available in all countries.

---

### Intended use

The Respironics V60/V60 Plus ventilator is an assist ventilator and is intended to augment patient breathing. It is intended for spontaneously breathing individuals who require mechanical ventilation: patients with respiratory failure, chronic respiratory insufficiency, or obstructive sleep apnea in a hospital or other institutional settings under the direction of a physician.

The Respironics V60/V60 Plus ventilator is intended to support pediatric patients (children and adolescents, weighing 20 Kg (44 lb) or greater), and adult patients. It is also intended for intubated patients meeting the same selection criteria as the noninvasive applications. The ventilator is intended to be used by qualified medical professionals, such as physicians, nurses, and respiratory therapists. The ventilator is intended to be used only with various combinations of Philips-recommended patient circuits, interfaces (masks), humidifiers, and other accessories.

### Indications for use

The Respironics V60/V60 Plus is an assist ventilator and is indicated for use to augment patient breathing. The ventilator is indicated for spontaneously breathing individuals who require mechanical ventilation: patients with respiratory failure, chronic respiratory insufficiency, or obstructive sleep apnea in a hospital or other institutional settings under the direction of a physician.

### Patient population

The Respironics V60/V60 Plus ventilator is intended to support pediatric patients (children and adolescents, weighing 20 kg (44 lb.) or greater), and adult patients. The ventilator is also intended for intubated patients meeting the same selection criteria as the noninvasive applications.

## General information

### Contraindications

The Respironics V60/V60 Plus Ventilator is contraindicated for patients with any of the following conditions:

- Lack of spontaneous respiratory drive
- Inability to maintain a patent airway or adequately clear secretions
- At risk for aspiration of gastric contents
- Acute sinusitis or otitis media
- Hypotension
- Untreated pertussis
- Epistaxis (nosebleed)

### About CO<sub>2</sub> rebreathing

As with mask ventilation in general, patient CO<sub>2</sub> rebreathing may occur under some circumstances. Follow these guidelines to minimize the potential for CO<sub>2</sub> rebreathing. If rebreathing is a significant concern for a particular patient and these guidelines are not sufficient to acceptably reduce the potential for CO<sub>2</sub> rebreathing, consider an alternative means of ventilation.

- Increase EPAP to decrease the potential for CO<sub>2</sub> rebreathing. Higher pressures produce more flow through the exhalation port, which helps to purge all CO<sub>2</sub> from the circuit to prevent rebreathing.
- Be aware that the potential for CO<sub>2</sub> rebreathing increases as inspiratory time increases. A longer inspiratory time decreases exhalation time, allowing less CO<sub>2</sub> to be purged from the circuit before the next cycle. In such circumstances, higher tidal volumes further increase the volume of CO<sub>2</sub> rebreathed by the patient.

### Potential side effects

Advise the patient to immediately report any unusual chest discomfort, shortness of breath, or severe headache. Other potential side effects of noninvasive positive pressure ventilation include: ear discomfort, conjunctivitis, skin abrasions due to mask/patient interface, and gastric distention (aerophagia). If skin irritation or breakdown develops from the use of the mask, refer to the accompanying mask instructions for appropriate action.

General description

The Respironics V60/V60 Plus Ventilator (Figure 3-1) is a microprocessor-controlled, bilevel positive airway pressure (BiPAP) ventilatory assist system that provides noninvasive positive pressure ventilation (NPPV) and invasive ventilatory support for spontaneously breathing adult and pediatric patients.

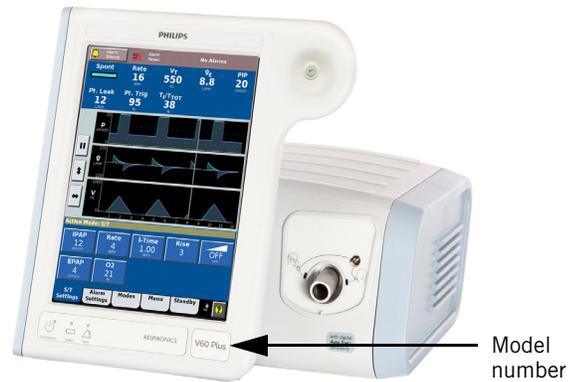


Figure 3-1: Respironics V60 Ventilator shown

**Ventilation modes.** The ventilator offers a range of conventional pressure modes, CPAP (continuous positive airway pressure), PCV (pressure-controlled ventilation), and S/T (spontaneous/timed). The volume-targeted AVAPS (average volume-assured pressure support) mode combines the attributes of pressure-controlled and volume-targeted ventilation. The optional PPV mode provides pressure ventilation in proportion to the patient’s efforts.

**Modes, therapies and features.** Table 3-1 shows which modes, therapies and features are included or optional for the V60 and V60 Plus models.

Table 3-1: V60 and V60 Plus comparison

Ventilator Model	Modes		Therapy	Features	
	AVAPS	PPV	HFT	Auto-Trak+	C-Flex
V60	Included	Optional	Optional	Optional	Included
V60 Plus	Included	Optional	Included	Optional	Included

**High flow therapy (HFT).** High flow therapy provides a set flow of mixed air and oxygen. Flow and O<sub>2</sub> percentage settings are selected by the clinician. HFT is available for 3.00 software and above, as well as for the V60 Plus.

**Auto-Trak Sensitivity** allows the ventilator to automatically compensate for intentional and unintentional leaks by maintaining a stable baseline and adjusting trigger and cycle thresholds for optimum patient-to-ventilator synchrony. The optional Auto-Trak+ feature lets you further adjust the level of Auto-Trak Sensitivity.

**User interface.** The ventilator’s 12.1-inch (31-cm) color touchscreen, Accept button, navigation ring (legacy versions only), and key panel let you easily access ventilator settings and monitored parameters.

## General information

**Monitoring.** The ventilator displays monitored parameters as numbers and as real-time waveforms (curves or scalars).

**Alarms.** The ventilator's operator-adjustable and nonadjustable alarms help ensure the patient's safety.

**Power and gas supplies.** The ventilator uses AC mains as its primary power source. An internal backup battery provides a secondary power source. For more information, see Table 11-10 on page 11-8.

The ventilator uses high-pressure oxygen. An integral blower pressurizes gas for delivery to the patient.

---

NOTE: Oxygen delivered through the compressed gas hose and blower is used as fresh gas.

---

**Mounting.** The ventilator can be mounted to a stand. When equipped with the optional cylinder holder, the stand can accommodate two E-size oxygen cylinders. An oxygen manifold kit is available, which allows two oxygen cylinders and one wall oxygen supply line to be used as inputs to the ventilator.

**Communications interface.** The ventilator can output data through the RS-232 serial port upon receiving a command from a host computer or bedside monitoring system. The ventilator is equipped with a remote alarm/nurse call connection to activate alarms remotely.

**Upgradability via Respi-Link remote diagnostic system.** The Respi-Link interface permits software upgrade and remote troubleshooting of the ventilator through the RS-232 port.

## Physical description

### **Patient circuits, masks/patient interfaces, and accessories**

Figure 3-2 shows the Respironics V60/V60 Plus Ventilator with its patient circuit and accessories. Appendix C provides ordering information for parts and accessories.



## General information

### Ventilator unit

Depending on the version of the ventilator you have, the Accept button on the upper-right front of the device may or may not be surrounded by a navigation ring (scrollable wheel).

Newer versions have a streamlined appearance and the Accept button, and look like this:



Legacy versions have both the navigation ring and Accept button, and look like this:



The photos and illustrations throughout this manual reflect the newer version of the ventilator (Accept button *without* navigation ring). For more information on the functionality differences, see "Navigating the graphical user interface" on page 3-13.

Figure 3-3 through Figure 3-5 show the controls, indicators, and other important parts of the ventilator unit.

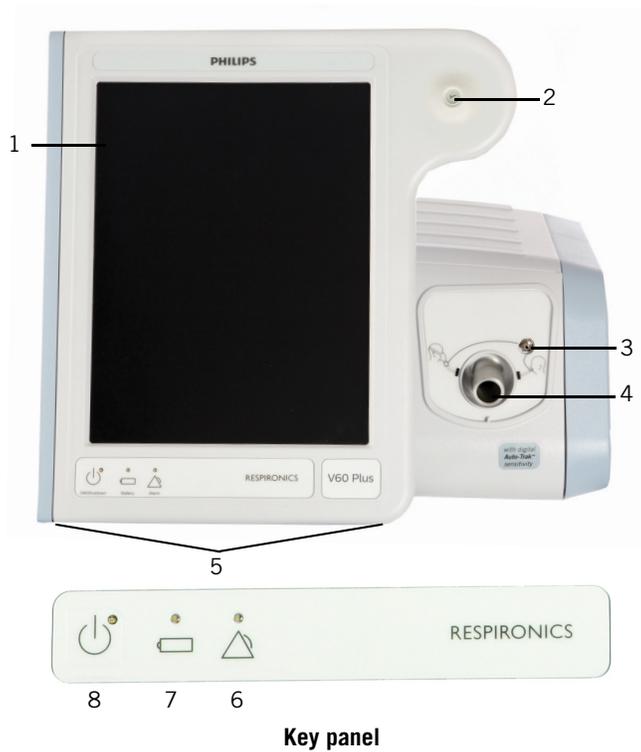


Figure 3-3: Front view

Number	Description
1	<b>Graphical user interface.</b> Color LCD (liquid crystal display) with touchscreen.
2	<b>Accept button.</b> Activates selections.
3	<b>Proximal pressure port.</b> Connection for tubing that monitors patient pressure in the patient circuit.
4	<b>Ventilator outlet (To patient) port.</b> Main connection for the patient circuit. Delivers air and oxygen in prescribed pressures to the patient.
5	<b>Alarm speakers</b> (beneath ventilator)
6	<b>Alarm LED.</b> Flashes during a high-priority alarm. On continuously during a ventilator inoperative condition.
7	<b>Battery (charged) LED.</b> Flashes when battery is charging. On continuously when battery is charged. Off when ventilator is running on battery, when a battery error or failure is detected, or when the ventilator is off and AC power is not connected.
8	<b>ON/Shutdown key with LED.</b> Turns on AC power and initiates ventilator shutdown. LED is continuously on when AC power is connected.

## General information



Figure 3-4: Side view

Number	Description
1	<b>Ventilation vents.</b> Allow intake of air for delivery to the patient.
2	<b>Air inlet filter</b> (under side panel). Filters the air for delivery to the patient.



Figure 3-5: Rear view

Number	Description
1	Backup battery (compartment under side panel).
2	Remote alarm/nurse call connector
3	Reserved for future use
4	Power cord retainer
5	Power cord
6	RS-232 serial connector (female DB-25). Connects to hospital information systems and other serial devices. Connects Respi-Link remote diagnostic system gateway for software updates.
7	Cooling fan filter
8	High-pressure oxygen inlet connector
9	Option labels

## General information

### About the backup battery

---

**WARNING:** To reduce the risk of power failure to the ventilator, pay close attention to the battery's charge level. The battery's operation time is approximate and is affected by ventilator settings, discharge and recharge cycles, battery age, and ambient temperature.

**NOTE:** The backup batteries are intended for short-term use only. They are not intended to be a primary power source.

**NOTE:** We recommend that the ventilator's batteries be fully charged before you ventilate a patient. If the batteries are not fully charged and AC power fails, always pay close attention to the level of battery charge.

**NOTE:** A new backup battery should be installed and charged within one year of the date of manufacture identified on the battery and on the shipping box.

**NOTE:** Storing a battery for an extended period of time after its manufacture date without being put into service, or at temperatures that exceed its limits, increases the risk of discharge whereby the battery is unable to be recharged by the ventilator.

**NOTE:** Batteries that are not installed in the ventilator should be stored at -20 to 25°C and charged at least once per year.

**NOTE:** Lithium Ion batteries are shipped with a charge level of under 30%, which may further impact the time until the battery is charged and discharged.

---

The internal backup battery protects the ventilator from low, or failure of, AC (mains) power. If AC power fails, the ventilator automatically switches to operation on backup battery with no interruption in ventilation. The battery powers the ventilator until AC power is again adequate or until the battery is depleted. For electrical specifications, see Table 11-10 on page 11-8.

As a safeguard, the ventilator provides a low battery alarm. It also has a capacitor-driven backup alarm that sounds for at least 2 minutes when battery power is completely lost.

The ventilator charges the battery whenever the ventilator is connected to AC, with or without the ventilator switched on. The Battery (charged) LED flashes to show that the battery is being charged.

Check the battery charge level before putting a patient on the ventilator and before unplugging the ventilator for transport or other purposes. The power source symbol at the bottom right-hand corner of the screen shows the power source in use and, if the ventilator is running on battery, the level of battery charge (Figure 3-6).


---

**NOTE:** The battery charge level displays within about a minute of power on.

---

If the battery is not fully charged, recharge it by connecting the ventilator to AC power for a minimum of 5 hours. Pressing the Help button shows you the approximate time remaining until the battery is fully charged. If the battery is not fully charged within 16 hours, or the ventilator displays a *Check Vent: Battery Failed* alarm, replace the battery.

**CAUTION:** Avoid allowing the ventilator battery to become completely discharged. Otherwise, the battery may become over-discharged and require long recharge times of up to 16 hours or more. The over-discharged condition may permanently damage the battery so that it is unable to recharge. To prevent the occurrence of a non-recoverable over-discharged battery, always keep the ventilator connected to an AC outlet.

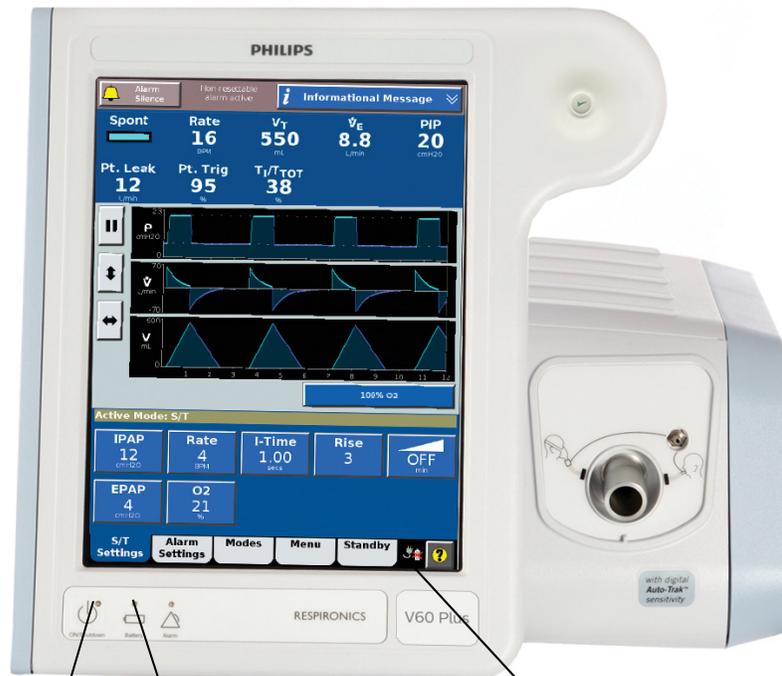

**ON/Shutdown LED**

*On continuously:* AC power is connected

**Battery (charged) LED**

*Flashes:* Battery is charging  
*On continuously:* Battery is charged (90 to 100%)  
*Off:* Ventilator is running on battery, or the ventilator is off and AC power is not connected

**Power source symbol**



The ventilator is powered by AC and the battery is installed.



The ventilator is powered by AC and the battery is not installed.



The ventilator is powered by the battery. This symbol shows the approximate battery time remaining in hours and minutes, and it shows the remaining capacity graphically.

Figure 3-6: Power indicators

## General information

### About the graphical user interface

Through the graphical user interface (Figure 3-7) you make ventilator settings and view ventilator and patient data. During ventilation, the upper screen displays alarms and patient data. The middle screen displays real-time waveforms and alarm and informational messages. The lower screen lets you access modes and other ventilator settings, display help information, and see the power status.

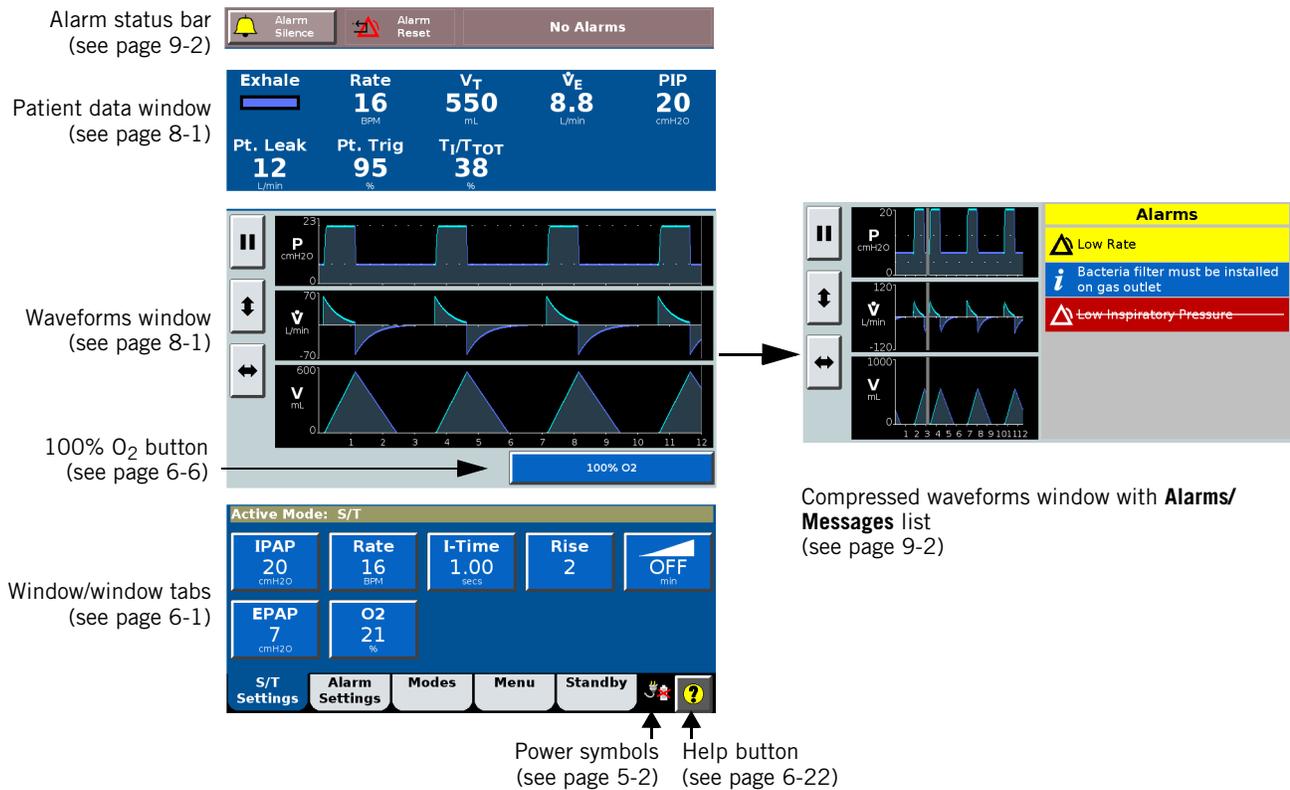


Figure 3-7: Parts of graphical user interface

### Navigating the graphical user interface

Select a function by touching the desired tab or button on the touchscreen.

If your ventilator has only the Accept button on the top-right front of the device (newer versions), you adjust values and navigate the graphical user interface by using the touchscreen.

If your ventilator has a navigation ring around the Accept button on the top-right front of the device (legacy versions), you can adjust values and navigate the graphical user interface by either rotating your finger on the navigation ring or by using the touchscreen.

Touchscreen navigation	Front panel equivalent
 Touch increase button (adjustment arrow). Press and hold for faster adjustments.*	On navigation ring (legacy versions), touch and rotate finger clockwise to increase value or move cursor forward
 Touch decrease button (adjustment arrow). Press and hold for faster adjustments.*	On navigation ring (legacy versions), touch and rotate finger counterclockwise to decrease value or move cursor backward
 Accept Touch Accept button to apply selection	 Press Accept (checkmark) button to apply selection

\* Available in Revision 2.30 software and above.


After making selections and adjusting values, accept those selections and apply the changes by either touching the Accept button in the user interface or by pressing the Accept button (checkmark) on the top-right front of the ventilator.

**To open a window,** touch the window tab.

**To cancel a function and close the window,** either select **Cancel** or touch another window tab.

**To adjust a parameter,** touch the arrow button or select the value with the navigation ring (legacy versions). Each touch changes the value in single increments or, for parameters with wide ranges, press and hold the arrow key to make faster changes. The slider flag moves along the setting range scale. Select Accept to apply.

## General information

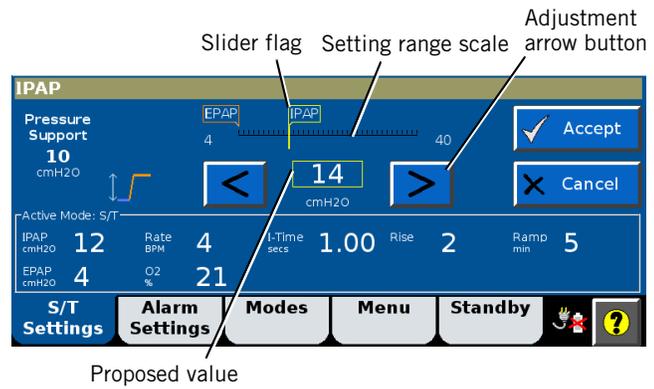

The navigation ring (on legacy versions) also lets you adjust the position of the cursor in the waveforms window while the screen is frozen. See "Freezing and unfreezing waveforms" on page 8-3 for more information.

## Starting up the ventilator

---

NOTE: Upon power-on the ventilator automatically runs a test of the backup audible alarm followed by the primary audible alarm. You should hear a high-pitched tone, followed by a beep. If you do not hear all of these sounds, discontinue use of the ventilator and have it serviced.

---

1. Power on the ventilator with the **ON/Shutdown** key.
2. Verify the ventilator operation, as described on page 5-8.

## Shutting down the ventilator

Shut down the ventilator as follows:

1. Press and release the **ON/Shutdown** key. The **Shutdown** window opens.
2. Select **Ventilator Shutdown**. The ventilator shuts down.



---

NOTE: Improper shutdown may cause a **Power has been restored** message the next time the ventilator is turned on.

NOTE: If the screen is blank and the dialogue box cannot be displayed, shut down the ventilator by pressing the **ON/Shutdown key**, then the Accept button on the top-right front of the ventilator.

---

## Training

Product training is available. Contact your local Philips sales representative or Philips Customer Support for assistance. Call 1-800-225-0230 for ordering and 1-800-722-9377 for service.

**General information**

## Chapter 4. Principles of operation

---

### System operational overview

The Respironics V60/V60 Plus Ventilator is a microprocessor-controlled pneumatic system that delivers a mixture of air and oxygen. It is powered by AC with battery backup to protect against power failure or unstable power and to facilitate intrahospital transport. The ventilator's pneumatics deliver gas and its electrical systems control pneumatics, monitor the patient, and distribute power.

The user provides inputs to the ventilator through a touchscreen, a key panel, a navigation ring (legacy versions), and an Accept button. These inputs become instructions for the pneumatics to deliver a precisely controlled gas mixture to the patient. Pressure and flow sensors provide feedback, which is used to adjust gas delivery to the patient. Monitored data based on sensor inputs is also displayed by the graphical user interface.

The ventilator's gas delivery and monitoring functions are cross-checked. This cross-checking helps prevent simultaneous failure of these two main functions and minimizes the possible hazards of system failure.

A comprehensive system of visual and audible alarms helps ensure the patient's safety. Clinical alarms can indicate an abnormal physiological condition. Technical alarms, triggered by the ventilator's self-tests, can indicate a hardware or software failure. In the case of some technical alarms, limited ventilation is provided to give the user time for corrective actions. When a condition is critical enough to possibly compromise safe ventilation, the ventilator is placed into the ventilator inoperative state, in which oxygen flow and blower operation are disabled.

The ventilator has several means to ensure that safe patient or respiratory pressures are maintained. The maximum working pressure is ensured by the high inspiratory pressure (HIP) alarm limit. If the set high pressure limit is reached, the ventilator cycles into exhalation.

## Principles of operation

### Pneumatic system operation

The ventilator uses ambient air and high-pressure oxygen (Figure 4-1). Air enters through an inlet filter. Oxygen enters through a high-pressure inlet, and a proportioning valve provides the operator-set concentration. The system mixes the air and oxygen, pressurizes it in the blower, and then regulates it to the user-set pressure. To do this, the ventilator compares the proximal (patient) pressure measurement with the ventilator outlet (machine) pressure, and adjusts the machine pressure to compensate for the pressure drop across the inspiratory filter, patient circuit, and humidifier. This helps ensure accurate and responsive pressure delivery and leak compensation.

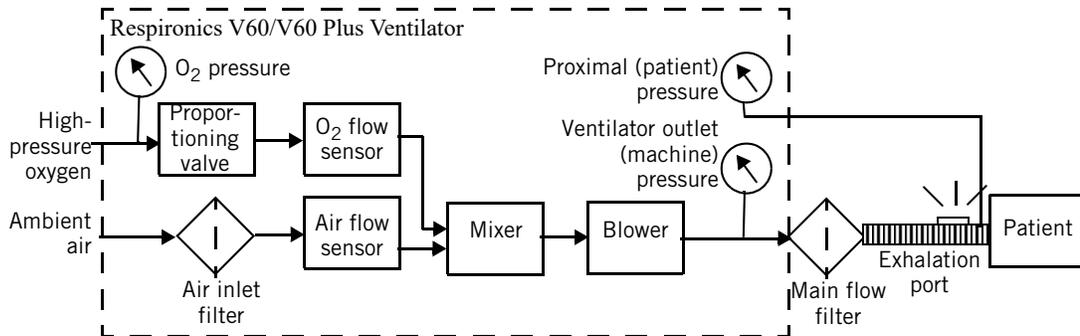


Figure 4-1: Respironics V60/V60 Plus Ventilator gas delivery system

The ventilator delivers gas to the patient through a main flow (inspiratory) bacteria filter, a single-limb patient breathing circuit, a humidification device (optional) and a patient interface such as a mask or ET tube. A pressure tap proximal to the patient is used to monitor patient pressure. The exhalation port continually exhausts gas from the circuit during inspiration and exhalation to minimize rebreathing and ensure CO<sub>2</sub> removal.

### Breath delivery characteristics

#### Control variable

Breaths delivered by the Respironics V60/V60 Plus Ventilator are pressure controlled. In the AVAPS mode, the ventilator's applied pressure is automatically adjusted over a period of time to maintain a target tidal volume.

#### Triggering, cycling, and leak adaptation

Unlike other ventilators, the Respironics V60/V60 Plus Ventilator does not require you to set triggering and cycling sensitivity or to adjust baseline flow. The ventilator's unique Auto-Trak Sensitivity algorithm adjusts these automatically; see "Auto-Trak Sensitivity" on page 4-3.

#### Baseline pressure

A positive baseline pressure (EPAP or CPAP) may be set for all breaths in all modes.

### Pressure rise time

The operator-set Rise Time defines the time required for inspiratory pressure to rise to the set (target) pressure.

### Negative pressures

There are no negative pressures generated during exhalation.

### Oxygen concentration

The Respironics V60/V60 Plus Ventilator incorporates an oxygen mixer. Oxygen concentration can be set in all modes.

## Auto-Trak Sensitivity

An important characteristic of the Respironics V60/V60 Plus Ventilator is its ability to recognize and compensate for intentional and unintentional leaks in the system, and to automatically adjust its triggering and cycling algorithms to maintain optimum performance in the presence of leaks. This is called Auto-Trak Sensitivity. The following subsections describe this function in detail.

### Triggering

Breaths are patient (flow) triggered in all modes, typically when patient effort causes a certain volume of gas to accumulate above baseline flow (volume method). An inspiration is also triggered when the patient inspiratory effort distorts the expiratory flow waveform sufficiently (shape signal method; see page 4-4).

### Cycling

Cycling to exhalation occurs in these cases:

- Patient expiratory effort distorts the inspiratory flow waveform sufficiently (shape signal method). See “Shape signal method of cycling and triggering.” on page 4-4.
- Patient flow reaches the spontaneous exhalation threshold (SET). See “SET method of cycling.” on page 4-4.
- After 3 seconds at the IPAP level (timed backup safety mechanism)
- When a flow reversal occurs, typically due to a mask or mouth leak

## Principles of operation

**Shape signal method of cycling and triggering.** The shape signal or “shadow trigger” method uses a mathematical model derived from the flow signal. A new flow signal (shape signal) is generated by offsetting the signal from the actual flow and delaying it (Figure 4-2). This intentional delay causes the flow shape signal to be slightly behind the patient’s flow signal. If there is a sudden change in patient flow, the patient’s flow signal crosses the shape signal; this results in a trigger or a cycle. As a result, a sudden decrease in expiratory flow from an inspiratory effort will cross the shape signal and create a signal for ventilator triggering.

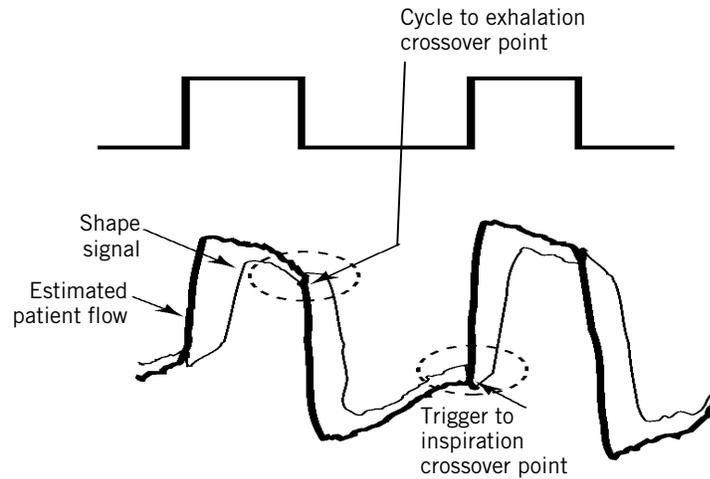


Figure 4-2: Shape signal

**SET method of cycling.** Patient flow reaches the spontaneous exhalation threshold (SET); see Figure 4-3. The SET represents the intersection of the flow waveform and a line of a given slope. SET is updated each breath.

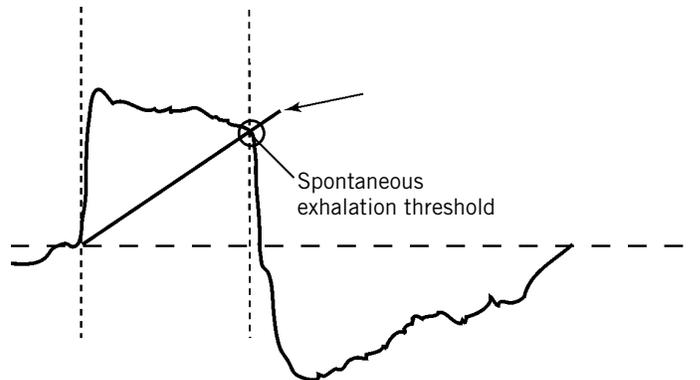


Figure 4-3: Spontaneous exhalation threshold (SET)

### Leak adaptation

Noninvasive ventilation in particular may involve considerable leakage around the mask or through the mouth. Some leakage is known or *intentional*: it is a characteristic of the mask/patient interface design. So that it can accurately adjust its baseline flow, the ventilator has you enter the intentional leakage value specific to the mask/patient interface (“Selecting the mask and exhalation port” on page 6-12). Other leakage is unpredictable or *unintentional*, and it changes as the patient’s breathing pattern changes.

To maintain prescribed pressures in the presence of leakage, the ventilator adjusts its baseline flow. Because the unintentional part of the leakage may constantly change, the ventilator recalculates the baseline flow each breath at the end of exhalation. The ventilator uses two main mechanisms to update its baseline flow: expiratory flow adjustment and tidal volume adjustment.

**Expiratory flow adjustment.** Every breath, at end-exhalation, the ventilator updates its flow baseline. At end-exhalation patient flow is assumed to be zero, so any difference between actual patient flow and the original baseline flow indicates a change in leakage. Figure 4-4 shows how the ventilator adjusts the baseline.

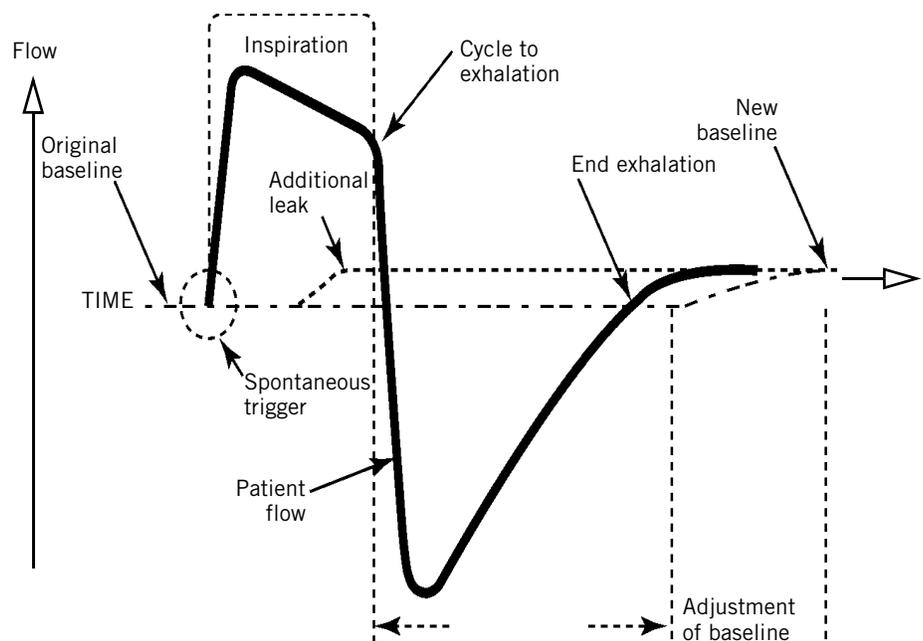


Figure 4-4: Expiratory flow adjustment

## Principles of operation

**Tidal volume adjustment.** Every breath, the ventilator compares the inspiratory and expiratory tidal volumes. Any difference is assumed to be due to an unintentional circuit leak. The ventilator adjusts the baseline to reduce this tidal volume difference for the next breath. Figure 4-5 shows how the ventilator adjusts the baseline.

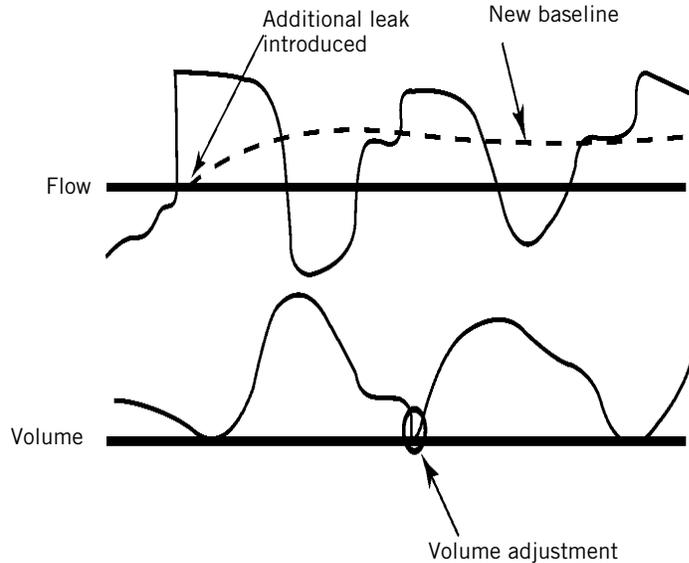


Figure 4-5: Tidal volume adjustment

## Auto-Trak+ (optional)

The Auto-Trak+ option for the Respirationics V60/V60 Plus Ventilator lets you further adjust the level of Auto-Trak Sensitivity, a feature that recognizes and compensates for intentional and unintentional leaks. This algorithm has multiple breath trigger and cycle thresholds. When you adjust Auto-Trak+ settings, you adjust these multiple trigger and/or cycle thresholds simultaneously, retaining all the auto-adaptive features of Auto-Trak Sensitivity.

The Normal Auto-Trak settings work well for most patients. Pediatric patients, however, may benefit from more sensitive trigger settings, while some adult patients may benefit from more or less sensitive cycle settings.

## High flow therapy

High flow therapy (HFT) enables delivery of a humidified gas mixture at an operator-set flow rate via a high flow nasal cannula interface or tracheal adapter. The principle mechanism of action for high flow therapy is delivering a known  $\text{FiO}_2$  at a flow rate equal to or greater than the patient's peak flow, thus minimizing dilution of the gas.

HFT provides blended gas to the patient at a targeted flow. Both  $\text{O}_2$  concentration and flow are set by the clinician. Heated humidification is recommended during high flow therapy.

HFT controls flow instead of pressure and is accessed only while in Standby mode. Patient alarms are not available during high flow therapy. This therapy is not considered a breath delivery mode.

HFT requires 3.00 software and above.

## Ventilation modes

The Respironics V60/V60 Plus Ventilator operates in the following ventilation modes:

- CPAP (continuous positive airway pressure) mode
- S/T (spontaneous/timed) mode
- PCV (pressure-controlled ventilation) mode
- AVAPS (average volume-assured pressure support) mode
- PPV (proportional pressure ventilation) mode (optional)

Table 4-1 summarizes the characteristics of these modes. Note that on the ventilator, the Timed breath indicator means the breath is ventilator triggered, while the Spont breath indicator means the breath is patient triggered.

*Table 4-1: Characteristics of Respironics V60/V60 Plus ventilation modes*

Mode	Timed breaths			Spont breaths		
	Trigger*	Limit†	Cycle‡	Trigger	Limit	Cycle
CPAP	N/A	N/A	N/A	Auto-Trak	Pressure	Auto-Trak
PCV	Time	Pressure	Time	Auto-Trak	Pressure	Time
S/T	Time	Pressure	Time	Auto-Trak	Pressure	Auto-Trak
AVAPS	Time	Pressure	Time	Auto-Trak	Pressure	Auto-Trak
PPV	Time	Pressure	Time	Auto-Trak	Pressure	Auto-Trak

\* A trigger variable starts inspiration.

† A limit variable can reach and maintain a preset level *before* inspiration ends but it does not end inspiration.

‡ A cycle variable is a measured parameter used to end inspiration.

## Principles of operation

### CPAP mode

In the CPAP (continuous positive airway pressure) mode, the ventilator functions as a demand flow system, with the patient triggering all breaths and determining their timing and size. The patient triggers and cycles based on the ventilator's Auto-Trak Sensitivity algorithms. The control settings active in the CPAP mode are shown in Figure 4-6. Figure 4-7 shows CPAP mode waveforms.

The C-Flex feature setting enhances traditional CPAP by reducing the pressure at the beginning of exhalation – a time when patients may be uncomfortable with CPAP – and returning it to the set CPAP level before the end of exhalation.

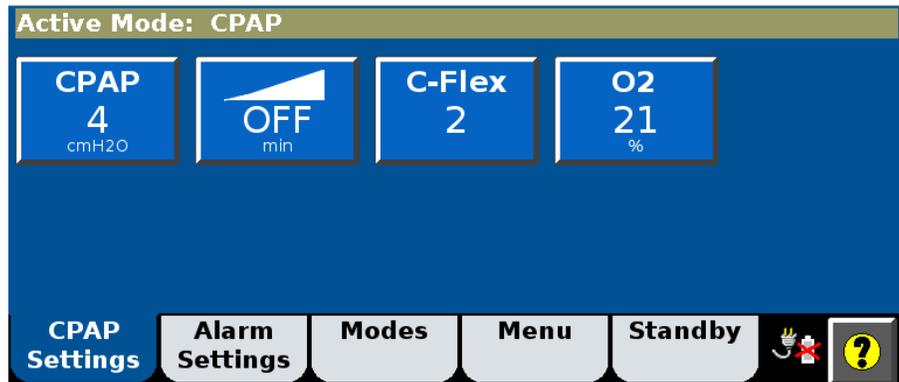


Figure 4-6: CPAP controls

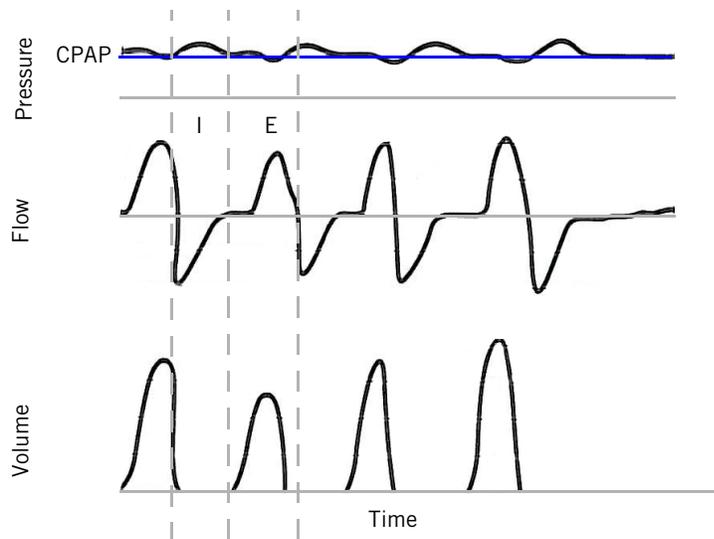


Figure 4-7: CPAP waveforms

**PCV mode**

The PCV (pressure-controlled ventilation) mode delivers pressure-controlled breaths, either triggered by the ventilator (Timed) or the patient (Spont). The control settings active in the PCV mode are shown in Figure 4-8. The IPAP setting defines the applied inspiratory pressure for all breaths. If the patient fails to trigger a breath through Auto-Trak within the interval determined by the rate setting, the ventilator triggers a mandatory breath. The I-Time setting is the cycle criterion for all breaths. Figure 4-9 shows a PCV mode pressure waveform.

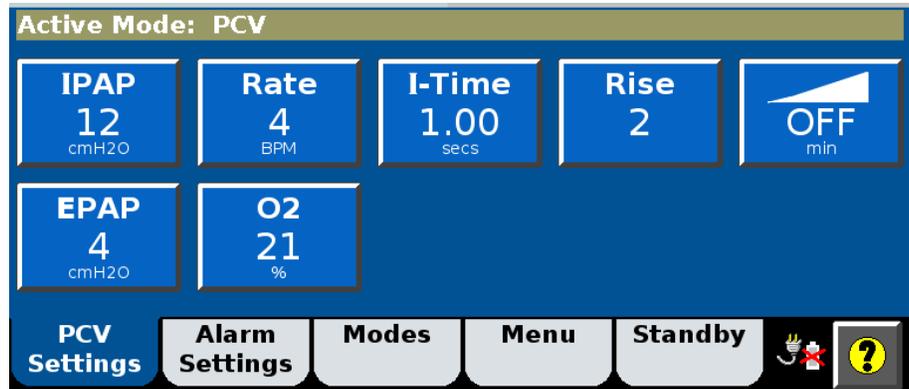


Figure 4-8: PCV controls

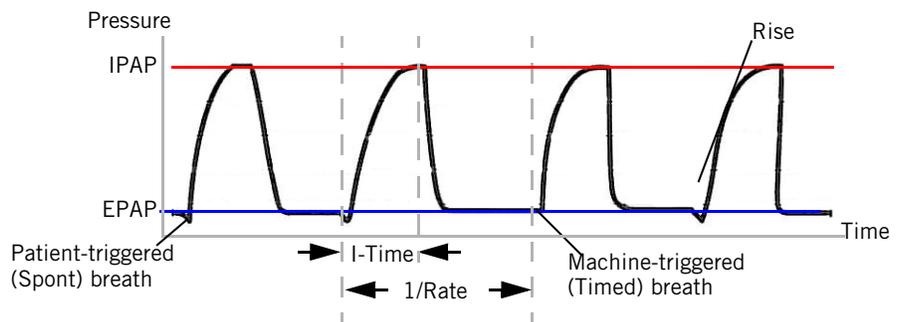


Figure 4-9: PCV pressure waveform

## Principles of operation

### S/T mode

The S/T (spontaneous/timed) mode guarantees breath delivery at the user-set rate. It delivers pressure-controlled, time-cycled mandatory and pressure-supported spontaneous breaths, all at the IPAP pressure level. If the patient fails to trigger a breath within the interval determined by the Rate setting, the ventilator triggers a mandatory breath with the set I-Time. The patient triggers and cycles based on the ventilator's Auto-Trak Sensitivity algorithms. The control settings active in the S/T mode are shown in Figure 4-10. Figure 4-11 shows an S/T mode pressure waveform.

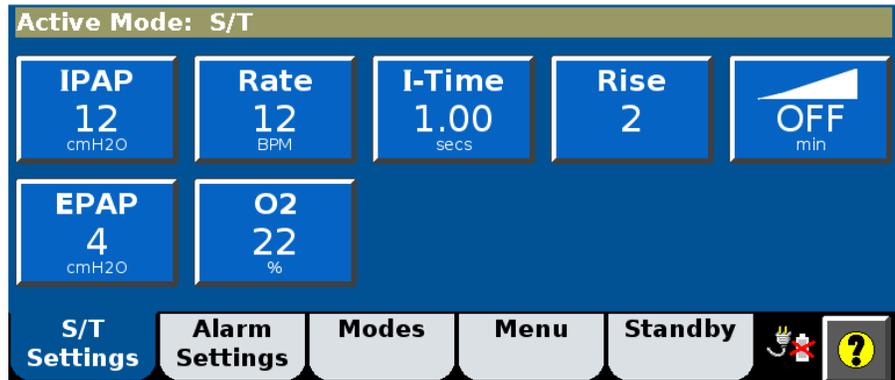


Figure 4-10: S/T controls

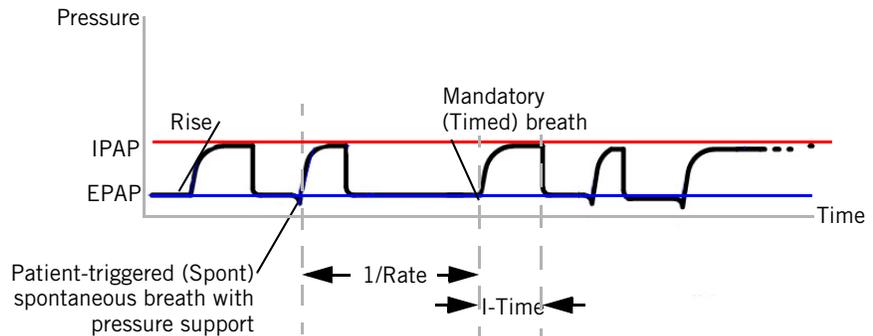


Figure 4-11: S/T pressure waveform

**AVAPS mode**

---

**NOTE:** When you adjust AVAPS minimum and maximum pressures, remember that IPAP is adjusted to meet the target value. If the calculated target pressure is outside of the minimum and maximum pressure range, the target volume will not be achieved.

---

Unlike most pressure modes, the AVAPS (average volume-assured pressure support) mode delivers a target tidal volume. It achieves the target volume by regulating the pressure applied following an initial pressure ramp-up. The AVAPS mode delivers time-cycled mandatory breaths and pressure-supported spontaneous breaths.

If the patient fails to trigger a breath within the interval determined by the Rate control, the ventilator triggers a mandatory breath with the set I-Time. Mandatory and spontaneous breaths are delivered at a pressure that is continually adjusted over a period of time to achieve the volume target,  $V_T$ . Min P and Max P define the minimum and maximum pressures that can be applied. The patient triggers and cycles based on the ventilator's Auto-Trak Sensitivity algorithms.

At start-up, AVAPS applies an inspiratory pressure equal to one of the following, whichever is greater:

- EPAP + (target volume / 60 ml/cmH<sub>2</sub>O)
- EPAP + 8 cmH<sub>2</sub>O
- Min P

The control settings active in the AVAPS mode are shown in Figure 4-12. Figure 4-13 shows AVAPS mode waveforms.

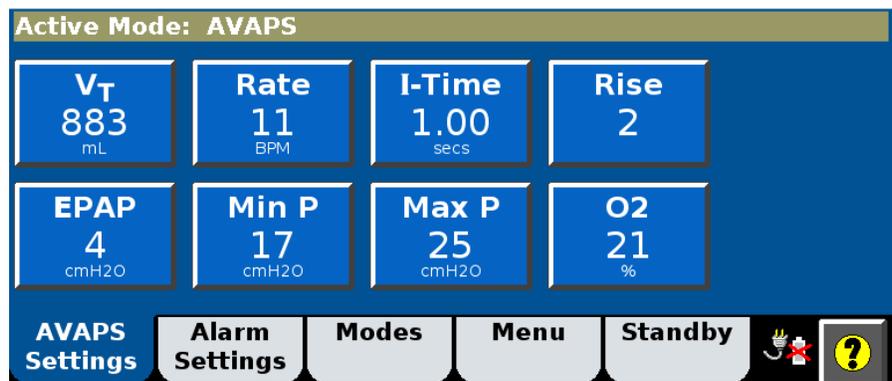


Figure 4-12: AVAPS controls

## Principles of operation

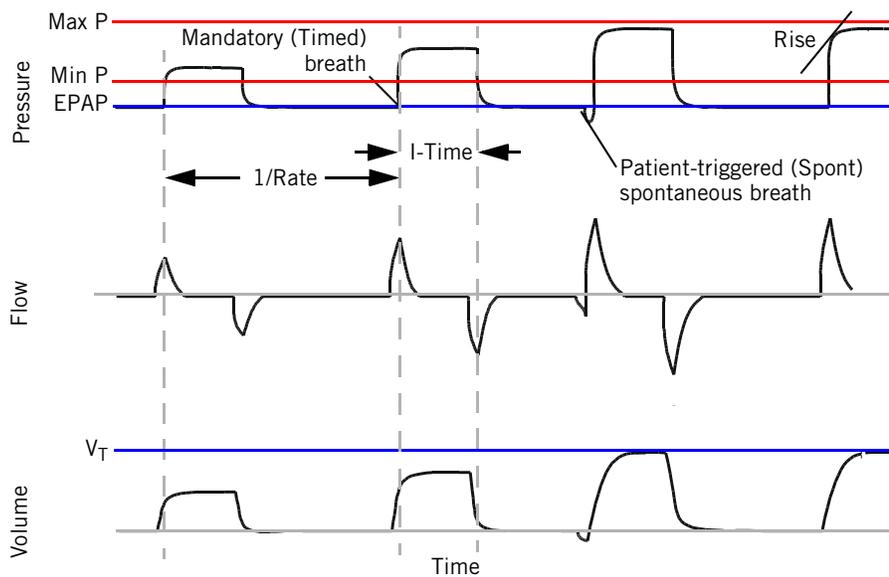


Figure 4-13: AVAPS waveforms

### PPV mode (optional)

The PPV (proportional pressure ventilation) mode provides patient-triggered breaths that deliver pressure in proportion to patient effort. Additionally a user-settable backup rate activates machine-triggered, pressure-limited, and time-cycled breaths in the case of apnea. In the PPV mode, patient effort determines the pressure, flow, and tidal volume delivered by the ventilator. The ventilator responds to patient effort, allowing the patient to determine when to start and end a breath. Additionally flow and pressure change based on the patient's efforts throughout inspiration.

**The physics behind PPV.** Two forces oppose ventilation, *resistance* and *elastance*.

Resistance is the impedance to air movement in the airways:

$$\text{Pressure/Flow} = \text{Resistance}$$

Airway resistance in healthy adults ranges from approximately 0.5 to 2.5 cmH<sub>2</sub>O/L/s.

Elastance is the elastic opposition to ventilation or the tendency of the lungs to resist inflation (elastance is the reciprocal of compliance):

$$\text{Pressure/Volume} = 1/\text{Compliance} = \text{Elastance}$$

The compliance of lungs and chest wall for a healthy adult is approximately 0.1 L/cmH<sub>2</sub>O, resulting in an elastance value of 10 cmH<sub>2</sub>O/L.

The inspiratory muscles, therefore, must generate force to overcome the resistance and elastance of the respiratory system. The proximal airway pressure is the net result of this contraction of these muscles: it is the force of the inspiratory muscle contraction minus both the pressure needed to generate air flow (overcome respiratory system resistance) and the pressure generated to inflate the lungs (overcome respiratory system elastance).

**How PPV works.** The delivery of a PPV breath is controlled by the maximum elastance (volume) assist (Max E), maximum resistance (flow) assist (Max R), and PPV % settings. The actual delivered assistance to overcome elastance is the product of PPV % and Max E. The actual delivered assistance to overcome resistance is the product of PPV % and Max R. In general, Max E should be set relative to the respiratory elastance and Max R should be set relative to the respiratory resistance, although you do not need to know the actual value of either to apply PPV. You adjust assist levels to optimize patient comfort. The resultant pressure support delivered in the PPV mode is the resistance assist times patient flow plus the elastance assist times the patient volume. Because the patient completely controls ventilatory output,<sup>1</sup> PPV may significantly improve patient-ventilator synchrony and ultimately, patient comfort.

The PPV backup rate ensures that the patient receives a minimum number of breaths per minute if the spontaneous breathing rate falls below the Rate

---

1 Marantz, S., Patrick, W., Webster, K., et al. "Response of ventilator-dependent patients to different levels of proportional assist." *Journal of Applied Physiology*, Vol. 80: 397-403, 1996.

## Principles of operation

setting. If the patient fails to trigger a breath within the interval determined by the Rate control, the ventilator triggers a Timed (backup) breath with the set I-Time, Rise, and IPAP settings.

The control settings active in the PPV mode are shown in Figure 4-14.

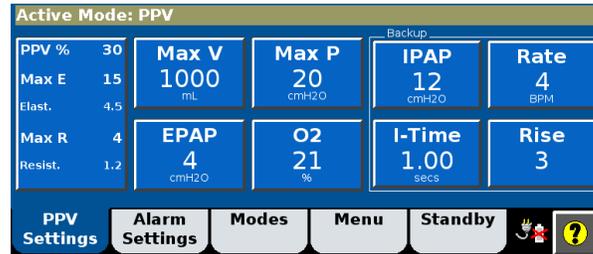


Figure 4-14: PPV controls

Figure 4-15 shows PPV mode waveforms. Note how volume and pressure increase as does the ventilatory demand of the patient. Max V (PPV maximum volume limit) and Max P (PPV maximum pressure limit) are used to prevent the delivery of excessive pressure or volume. More information about these limits is provided in "About Max V and Max P alarms and alarm limits" on page 6-7.


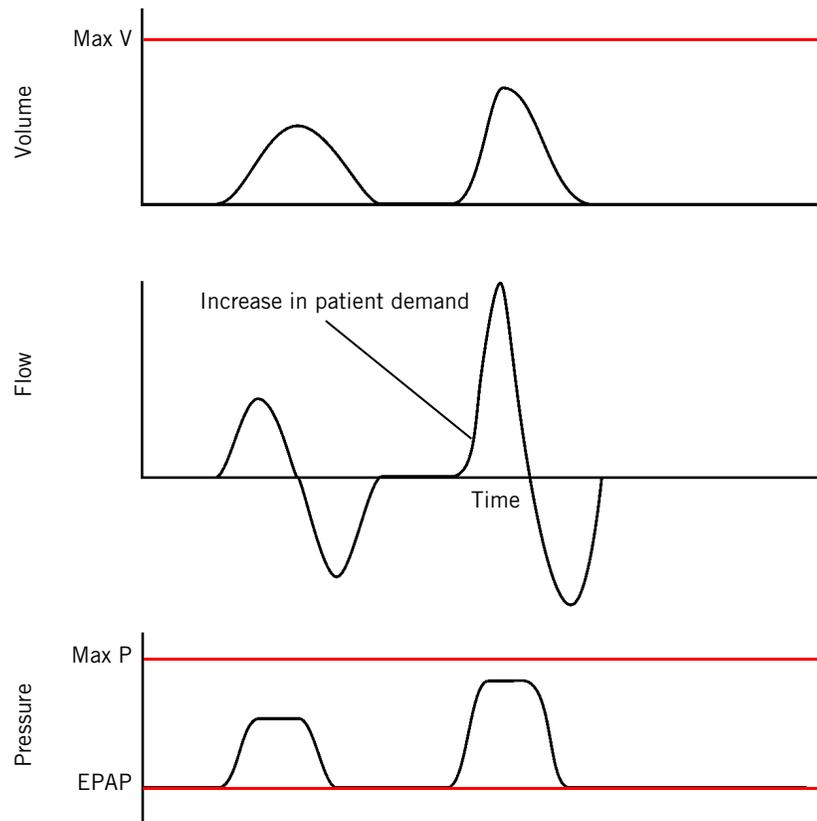


Figure 4-15: PPV waveforms

## Oxygen mixing

The ventilator's oxygen mixer regulates and proportions oxygen into the air from the blower according to the O<sub>2</sub> setting. The delivered oxygen accuracy is ±5% of the set value up to the maximum oxygen flow available. The ventilator can deliver up to 240 L/min of air/oxygen mix to assist in managing uncontrolled leaks during noninvasive ventilation.

Many hospital oxygen supply systems, however, cannot meet such high flow demands. Under extraordinary conditions (high O<sub>2</sub> setting plus high leak, and/or high patient demand) where demand exceeds available oxygen system flow, the ventilator provides additional air flow from the blower to ensure the target pressure is met. Under such conditions, the accuracy of delivered oxygen may be affected. Figure 4-16 shows the effect on the delivered oxygen concentration as the maximum oxygen system flow is exceeded. This graph assumes a continuous flow demand. Normally the higher "peak" flow is only needed during inspiration, so this is a worst case scenario.

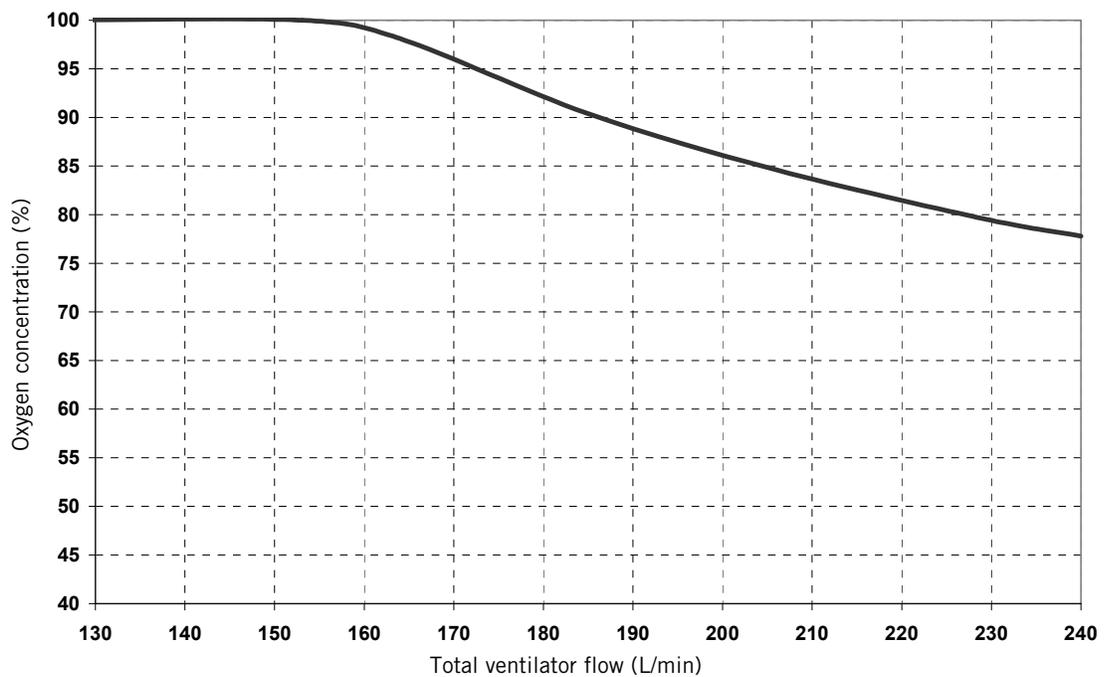

Assumptions: At an O<sub>2</sub> setting of 100% and an oxygen supply with a 50 psig inlet pressure capable of delivering up to 160 L/min.

Figure 4-16: O<sub>2</sub> concentration as a function of total ventilator flow

## Principles of operation

## Chapter 5. Setting up the ventilator for use

---

Set up the ventilator for each patient use as described in this chapter. For first-time installation, refer to Appendix A. For use with high flow therapy (HFT), set up the ventilator as described in this chapter, then refer to Chapter 7, High flow therapy.

### Connecting oxygen

- 
- WARNING:** Connect the ventilator only to an appropriate medical-grade oxygen source.
- WARNING:** To ensure accuracy of oxygen administration and to monitor for the presence of contamination (incorrect gas connected), use an external oxygen monitor to verify the oxygen concentration in the delivered gas.
- WARNING:** To reduce the risk of fire, do not use a high-pressure oxygen hose that is worn or contaminated with combustible materials like grease or oil.
- WARNING:** To reduce the risk of hypoxia, connect only oxygen to the high-pressure connector at the rear of the ventilator.
- WARNING:** To reduce patient risk of oxygen toxicity, keep free-flowing oxygen away from air inlet of ventilator.
- CAUTION:** To prevent possible damage to the ventilator, ensure that the connection to the oxygen supply is clean and unlubricated, and that there is no water in the oxygen supply gas.
- 

Connect the oxygen hose to an appropriate high-pressure oxygen source using country-specific O<sub>2</sub> connectors, as applicable.

Use of SIS connectors and supplemental oxygen accessories such as the O<sub>2</sub> manifold requires higher oxygen supply pressures. Consult Table 11-9 on page 11-7 for appropriate oxygen pressure ranges.

### Installing an oxygen analyzer/monitor

Install an Analytical Industries 2000M oxygen analyzer/monitor, and follow the manufacturer's instructions for setup, alarms, and calibration.

The Analytical Industries 2000M monitor includes user-settable high and low oxygen % alarms and is approved for use with the V60/V60 Plus Ventilator. Refer to the monitor instructions for use for detailed instructions on the proper setup and operation of the oxygen monitor.

---

**NOTE:** The V60/V60 Plus Ventilator also incorporates a loss of oxygen supply alarm to further protect the patient from low oxygen supply pressure conditions.

---

## Setting up the ventilator for use

### Connecting to AC power

- 
- WARNING:** To reduce the risk of electric shock, connect the ventilator to an AC supply mains with protective earth only.
- WARNING:** Do not use extension cords, adapters, or power cords with the ventilator that are not approved by Respirationics.
- WARNING:** To prevent unintentional disconnection of the power cord, always use the correct, Philips-supplied power cord and lock it into place with the power cord retainer before you switch the ventilator on. The retainer is designed to hold the connector end of the Philips-supplied cord securely in place.
- WARNING:** To reduce the risk of electric shock, regularly inspect the AC power cord and verify that it is not frayed or cracked.
- WARNING:** To reduce the risk of strangulation, route the power cord to avoid entanglement.
- CAUTION:** For 120 V equipment, grounding reliability can only be achieved when it is connected to an equivalent receptacle marked “hospital only” or “hospital grade.”
- 

Plug the power cord into a grounded outlet that supplies AC power between 100 and 240 V, 50/60 Hz.

Always check the reliability of the AC outlet. If you are using a 120 V outlet, make sure that it is hospital grade.

## Installing the patient circuit

- 
- WARNING:** To reduce the risk of strangulation from patient tubing, use a tubing support arm and secure the proximal pressure line with clips.
- WARNING:** To prevent possible patient injury and possible water damage to the ventilator, make sure the humidifier is set appropriately.
- WARNING:** To prevent possible patient injury and equipment damage, do not turn the humidifier on until the gas flow has started and is regulated. Starting the heater or leaving it on without gas flow for prolonged periods may result in heat build-up, causing a bolus of hot air to be delivered to the patient. Circuit tubing may melt under these conditions. Turn the heater power switch off before stopping gas flow.
- WARNING:** To reduce the risk that the patient will aspirate condensed water from the breathing circuit, position any humidifier lower than both the ventilator and the patient.
- WARNING:** To reduce the risk of fire, use only patient circuits intended for use in oxygen-enriched environments. Do not use antistatic or electrically conductive tubing.
- WARNING:** To prevent patient or ventilator contamination, always use a main flow bacteria filter on the patient gas outlet port. Filters not approved by Respironics may degrade system performance.
- WARNING:** During ventilation, patient exhalate is released into room air. Use of a patient circuit with a filter on its exhalation port is recommended.
- WARNING:** To reduce the risk of bacterial contamination or damage, handle bacteria filters with care.
- WARNING:** Avoid adding any components to the patient circuit that are not absolutely necessary. Additional components installed in the patient circuit can change the pressure gradient across the ventilator breathing system, increase the dead space, and adversely affect the ventilator performance.
- WARNING:** Any additional accessories in the patient circuit may substantially increase flow resistance and impair ventilation.
- WARNING:** Avoid adding resistive circuit components on the patient side of the proximal pressure line. Such components may defeat the disconnect alarm.
- WARNING:** Using a jet nebulizer can cause inadvertent alarms and affect the accuracy of delivered FiO<sub>2</sub>. To reduce patient risk, use only an approved nebulizer.
- NOTE:** Bacteria filter must be installed onto gas outlet.
- NOTE:** Resistive components can include but are not limited to HMEs, proximal flow sensors, a filter at the patient connection, or a narrow diameter circuit attached to a mask.
- NOTE:** Under extreme conditions and a missing, ruptured, or defective bacteria filter, the entire gas pathway can become contaminated with bodily fluids or exhaled gas.
- 

Install the patient circuit as shown in this section. For a list of compatible parts and accessories offered by Philips, see “Parts and accessories” on page C-1.

## Setting up the ventilator for use

Assemble the patient circuit, including the main flow (inspiratory) bacteria filter, proximal pressure line, oxygen sensor tee, and if desired, humidifier and nebulizer.

Figure 5-1 shows the circuit configuration for noninvasive ventilation or high flow therapy. For details on high flow therapy setup, see “High flow nasal cannula setup” on page 7-2.

Figure 5-2 and Figure 5-3 show circuit configurations for noninvasive and invasive ventilation. Follow the manufacturers’ instructions for use for the individual parts.

### Installing the nebulizer

For installation of the Aerogen nebulizer, follow the manufacturer’s instructions that came with the nebulizer, or visit:

[www.aerogen.com/nebulization-product-support](http://www.aerogen.com/nebulization-product-support)

# Setting up the ventilator for use

NOTE: This circuit setup is recommended for noninvasive ventilation. It is also recommended for high flow therapy when using the AC611 high flow nasal cannula with FEP Connector to block the exhalation port.

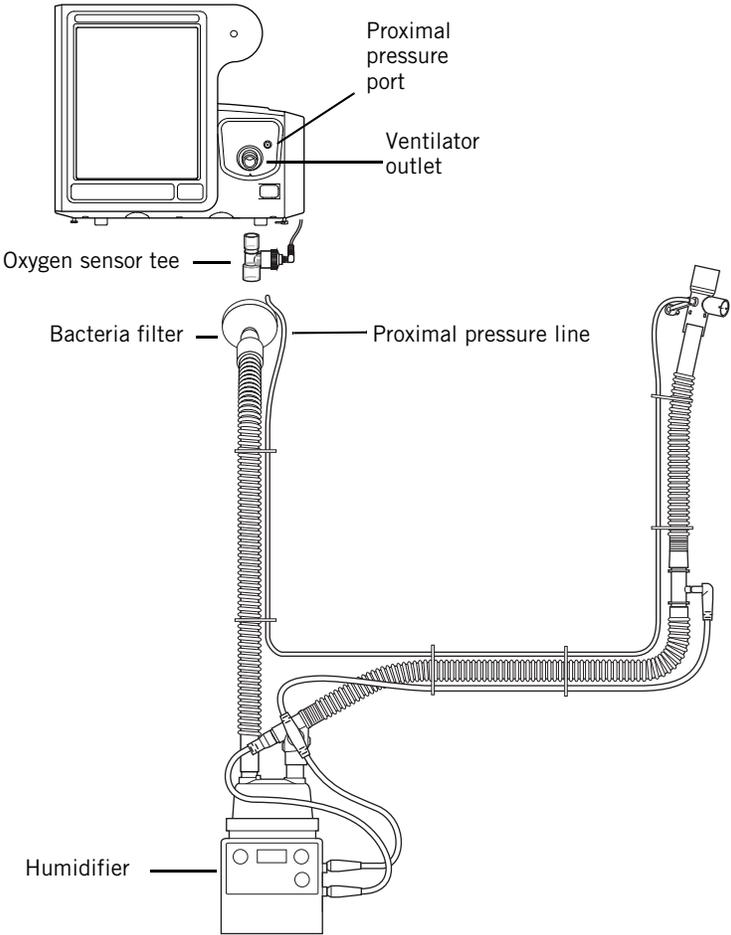



Figure 5-1: Patient circuit, with heated-wire and humidification for noninvasive ventilation or high flow therapy

## Setting up the ventilator for use

---

NOTE: This circuit setup is recommended for noninvasive ventilation.

---

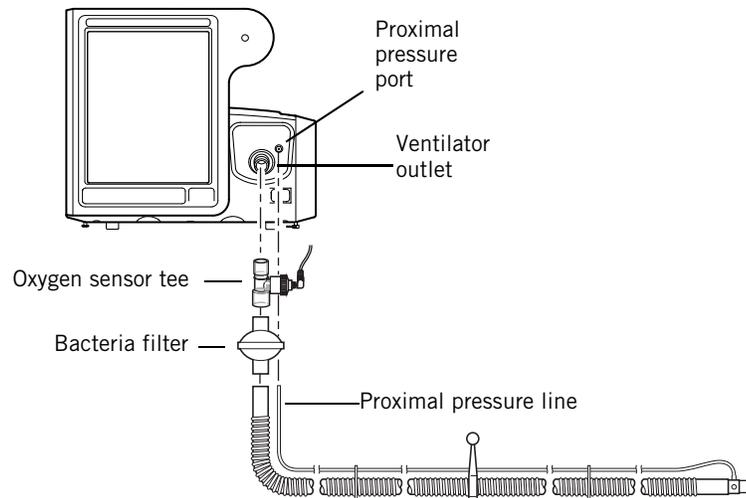


*Figure 5-2: Noninvasive patient circuit, without humidification*

## Setting up the ventilator for use

---

**NOTE:** This circuit setup is recommended for both noninvasive and invasive ventilation. Figure 5-3 shows an invasive circuit as an example.

---

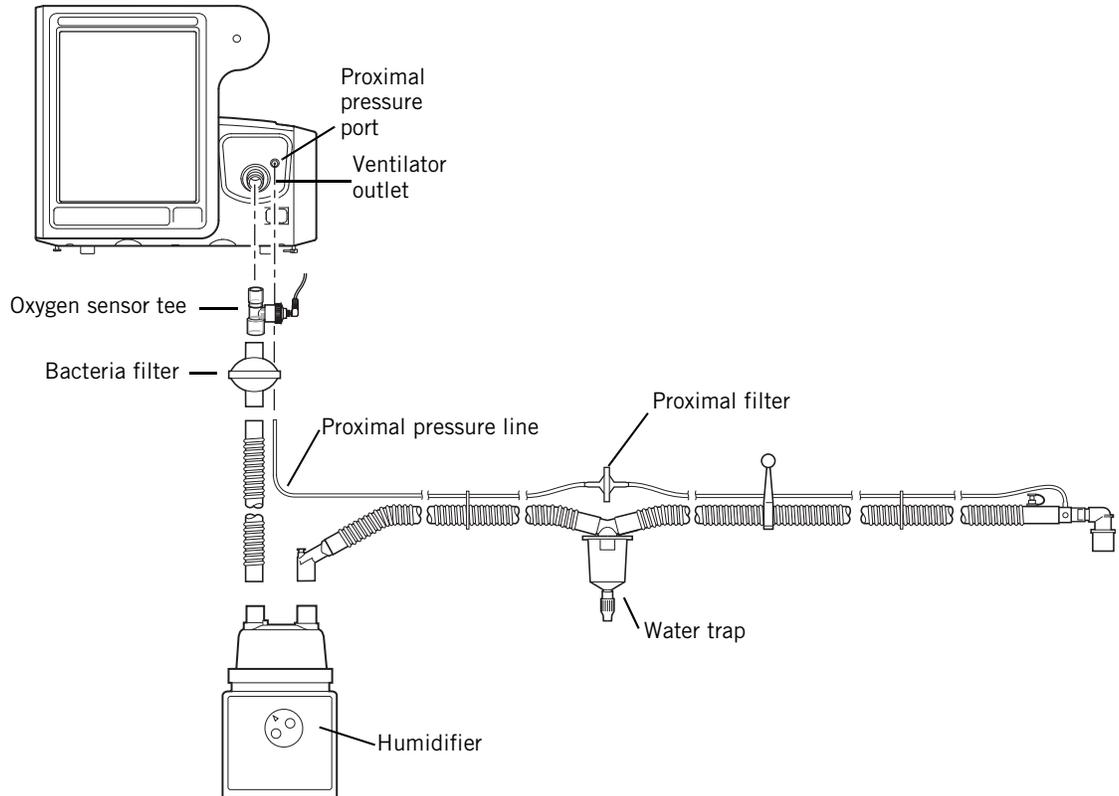

*Figure 5-3: Invasive patient circuit, with humidification*

### Connecting external devices

Connect the ventilator to a remote alarm (nurse call) device and a patient monitor or other external device, if applicable.

The Respironics V60/V60 Plus Ventilator can communicate with a Philips patient monitor using the IntelliBridge Open Interface. See “Using Philips monitors and the IntelliBridge or VueLink Open Interfaces” on page B-2. The ventilator also supports the VueLink Open Interface. VueLink has been replaced by IntelliBridge, but information is included in this manual for backwards compatibility. See “Data display” on page B-3.

For more information about connecting with non-Philips systems, contact your Philips representative.

## Setting up the ventilator for use

### Before placing a patient on the ventilator

- WARNING:** To ensure the ventilator's safe operation, always verify ventilator operation as described in "Verify ventilator operation" on page 5-8 before using the ventilator on a patient. If the ventilator fails any tests, remove it from clinical use immediately. Do not use the ventilator until necessary repairs are completed and all tests have passed.
- WARNING:** To reduce the risk of power failure to the ventilator, pay close attention to the battery's charge level. The battery's operation time is approximate and is affected by ventilator settings, discharge and recharge cycles, battery age, and ambient temperature.
- NOTE:** If the ventilator has a backup battery, the battery must be adequately charged to verify operation. Recharge as necessary before verifying operation. Based on the age and state of the battery, it may take up to 16 hours or more to fully charge.
- NOTE:** The backup batteries are intended for short-term use only. They are not intended to be a primary power source.
- NOTE:** We recommend that the ventilator's batteries be fully charged before you ventilate a patient. If the batteries are not fully charged and AC power fails, always pay close attention to the level of battery charge.
- 

### Verify ventilator operation

1. Ensure that the ventilator is connected to AC power.
2. Power on the ventilator. The ventilator automatically runs a test of the backup audible alarm followed by the primary audible alarm. Verify that you hear a high-pitched tone, followed by a beep.
3. Create a patient alarm, such as a disconnect alarm.
  - a. VERIFY that the proper alarm is annunciated (audio, visual, flashing, and alarm LED).
  - b. VERIFY that the audio volume is adequate for the environment in which it will be used.
  - c. VERIFY remote alarm setup, if applicable.
4. Resolve the alarm condition and manually reset the alarm.
5. Disconnect the ventilator from AC power while the ventilator is running and verify the following:
  - a. VERIFY that the ventilator switches over to battery power (battery symbol in lower-right corner of screen is displayed).  

  - b. VERIFY that the audible alarm sounds intermittently.
  - c. VERIFY that the yellow "Running on Internal Battery" alarm is displayed, and manually reset it.

## Setting up the ventilator for use

- d. VERIFY that the blue “Running on Internal Battery” message is displayed.
6. Install a fully-charged battery and reconnect the ventilator to AC power.
7. Repeat step 5 to verify that the ventilator switches over to battery power.

## Running alarm tests

The ventilator performs a self-check during start-up and continuously during operation. Alarm functionality is verified by this self-check. You may also want to run alarm tests, which demonstrate the alarms' operation. Follow these steps to perform the tests.

---

**WARNING:** To prevent possible patient injury, always return alarm settings to hospital-standard values after verifying ventilator operation.

---

### Preparation

1. Set the ventilator up as for normal ventilation, complete with breathing circuit (PN 582073 or the equivalent) and a 1-liter test lung assembly (PN 1021671).
2. Set the mode to S/T and make the following control settings: Rate: 4 BPM, IPAP: 10 cmH<sub>2</sub>O, EPAP: 6 cmH<sub>2</sub>O, I-Time: 1 sec, Rise: 1, Ramp: Off, O<sub>2</sub>: 21%.
3. Make the following alarm settings: Hi Rate: 90 BPM, Lo Rate: 1 BPM, Hi V<sub>T</sub>: 2000 mL, Lo V<sub>T</sub>: OFF, HIP: 50 cmH<sub>2</sub>O, LIP: OFF, Lo V<sub>E</sub>: OFF, LIP T: 5 secs.

### High Inspiratory Pressure

1. Lower the HIP alarm limit to 8 cmH<sub>2</sub>O.
2. VERIFY that the **High Inspiratory Pressure** alarm is activated, the ventilator cycles into exhalation, and pressure falls to 6 cmH<sub>2</sub>O (the EPAP level).
3. Raise the HIP alarm limit to 15 cmH<sub>2</sub>O.

## Setting up the ventilator for use

### Low Tidal Volume

1. Raise the Lo  $V_T$  alarm setting above the displayed, measured  $V_T$ .
2. VERIFY that the **Low Tidal Volume** alarm is activated.
3. Turn the Lo  $V_T$  alarm setting OFF.
4. VERIFY that the alarm resets.

### Patient Disconnect

1. Disconnect the test lung.
2. VERIFY that the **Patient Disconnect** alarm is activated.
3. Reconnect the test lung.
4. VERIFY that the alarm resets and that the ventilator automatically resumes ventilation.

### Patient Circuit Occluded

1. Disconnect the patient circuit (including bacteria filter) from the ventilator outlet, and block the ventilator outlet.
2. VERIFY that the **Patient Circuit Occluded** alarm is activated.
3. Unblock the outlet, and reconnect the circuit.
4. VERIFY that the alarm resets.


### Using the ventilator for intra-hospital transport

- 
- WARNING:** Always check the status of the oxygen cylinders before using the ventilator during transport.
- WARNING:** To reduce the risk of power failure to the ventilator, pay close attention to the battery's charge level. The battery's operation time is approximate and is affected by ventilator settings, discharge and recharge cycles, battery age, and ambient temperature.
- WARNING:** The V60/V60 Plus Ventilator requires a pressurized oxygen supply that provides a minimum flow of 175 SLPM. Do not use any devices such as valves, hoses, Grab n' Go regulators or other brands of combined cylinder/regulators that limit supply of oxygen flow below 175 SLPM.
- WARNING:** Do not leave the ventilator unattended when stationed on an incline.
- 


Do the following to conserve oxygen during transport with the ventilator.

- Make sure all cylinders are full (13,790 kPa/2000 psig or more).
- Make sure the cylinder regulators are turned off while the ventilator is connected to wall oxygen.
- Never turn the cylinder regulator on until you are ready to begin transport.
- Only turn one cylinder regulator on at a time. If you turn on both cylinders, they may become depleted simultaneously, leaving you with no backup oxygen.
- Whenever possible, reduce the O<sub>2</sub> setting before transport.
- Minimize all inadvertent leaks. Tighten masks prior to transport, and loosen up when patient is back on wall oxygen.
- Avoid using masks that have an exhalation port built into the mask when there is already an exhalation port in the circuit.
- Be aware that oxygen is more rapidly depleted at higher leak rates (see Figure 5-4).

## Setting up the ventilator for use

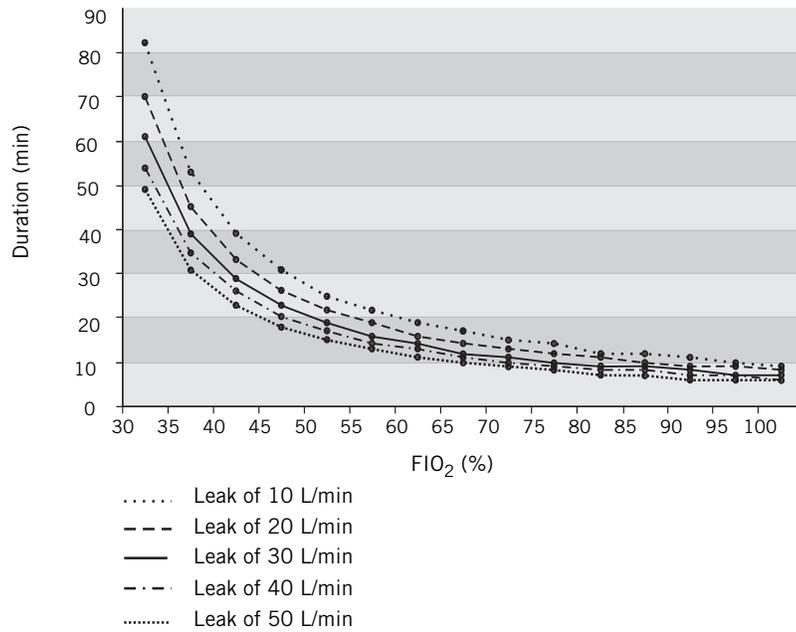

a.  $V_T = 500$  mL, Rate = 40 BPM, EPAP = 6 cmH<sub>2</sub>O, IPAP = 18 cmH<sub>2</sub>O

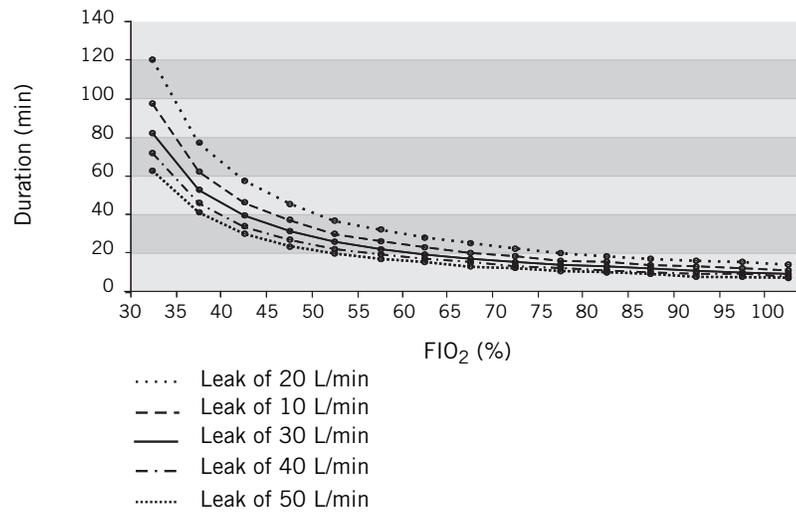

b.  $V_T = 500$  mL, Rate = 20 BPM, EPAP = 6 cmH<sub>2</sub>O, IPAP = 18 cmH<sub>2</sub>O

Figure 5-4: Duration of cylinder oxygen (13,790 kPa/2000 psig) at various leak rates

**Storing the ventilator  
between patient use**

See “Storage between patient use” on page 10-8 for information about storing the ventilator.

**MRI safety  
information**

---

**WARNING: The V60/V60 Plus Ventilator is MR Unsafe. Keep it outside the MRI scan room (Zone IV). It represents a projectile hazard.**

---

**Security and Privacy  
Information**

To develop a security strategy related to the use of the ventilator, refer to the Hospital Respiratory Care product security guide. Download it from: [www.philips.com/ifu](http://www.philips.com/ifu).

## Setting up the ventilator for use

## Chapter 6. Operation

---

---

**WARNING:** To ensure the ventilator's safe operation, always verify ventilator operation as described in "Verify ventilator operation" on page 5-8 before using the ventilator on a patient. If the ventilator fails any tests, remove it from clinical use immediately. Do not use the ventilator until necessary repairs are completed and all tests have passed.


**NOTE:** Before operation, prepare the ventilator as instructed in Chapter 5.

**NOTE:** Any mode and settings can be used for closed suctioning.

---

After power-on, the ventilator starts up in the mode and with the settings that were active before last power down. Check these settings and adjust as required. You must be familiar with using the touchscreen and navigation ring (legacy versions) to select, adjust, activate, and confirm parameters. For details, see "Before placing a patient on the ventilator" on page 5-8.

Access the ventilator setting windows from the tabs at the bottom of the screen.

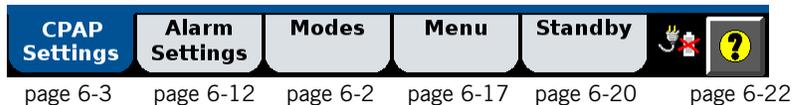


## Operation

### Changing the mode

The active ventilation mode is displayed in the bottom, left-hand corner of the screen. Change the mode as follows. For details on modes, see “Ventilation modes” on page 4-7.

1. Open the Modes window.
2. Select the desired mode.

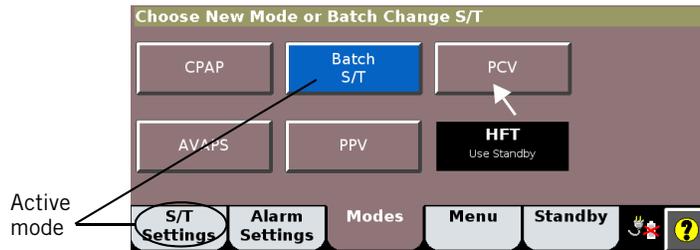

3. Adjust settings as desired (see “Changing individual ventilator settings” on page 6-4). Newly adjusted setting values are shown in yellow.



4. Select Activate Mode to apply.



## Changing control settings

Table 6-3 on page 6-23 is an alphabetical list of the control settings with their ranges. Table 11-2 on page 11-2 shows the control settings applicable to the different modes. For more information on control settings as they apply in the different ventilation modes, see “Ventilation modes” on page 4-7.

### Making batch setting changes

The ventilator allows you to make multiple changes (a “batch” of changes) at once.


---

NOTE: During a batch setting change, you cannot change the Ramp Time setting when a ramp is active.


---

This process applies to ventilation settings only, not to alarm settings.

1. Open the Modes window.
2. Select the active mode.

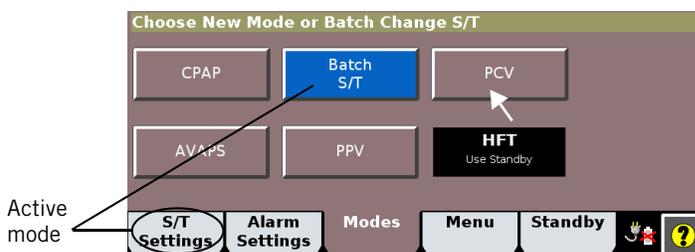

3. Adjust settings as desired (see “Changing individual ventilator settings” on page 6-4). Newly adjusted setting values are shown in yellow.

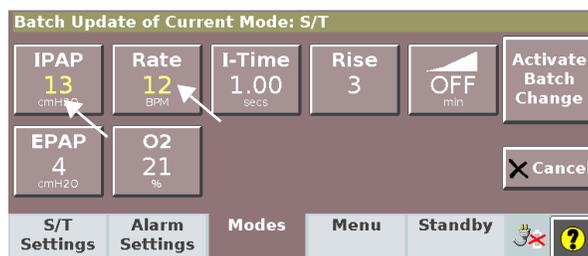

4. Select Activate Batch Change to apply.



# Operation


## Changing individual ventilator settings

You can make ventilator settings from the Settings window.

1. Open the Settings window.
2. Select the desired setting. As an example we will show the IPAP adjustment.



3. The setting window opens. Adjust the setting. Select Accept to apply.

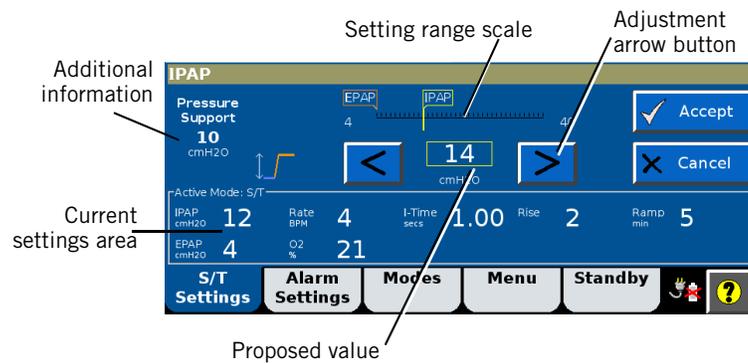

## Using the Ramp Time function

The Ramp Time function helps your patient adapt to ventilation by gradually increasing inspiratory and expiratory pressure (IPAP and EPAP/CPAP) from subtherapeutic to user-set pressures over a user-set interval. Table 6-3 on page 6-23 describes this function's principles of operation.

Follow these instructions to use the Ramp Time function:

1. Select the Ramp Time button in the Settings window.



The ramp starts. As the ramp progresses, the Ramp Time button graphic fills in.



2. To change the ramp interval or to end the ramp, select the Ramp Time button again. The Ramp in Progress window opens.



3. To end the ramp and apply the full IPAP and EPAP/CPAP immediately, select End Ramp.
4. To end the ramp and start a new one, select Start New Ramp. The Ramp Time setting window opens again so that you can set up a new ramp.

## Operation

### Using the 100% O<sub>2</sub> function

---

NOTE: The 100% O<sub>2</sub> feature is available in Revision 2.30 software and above.

---

The 100% O<sub>2</sub> function delivers 100% oxygen to the patient. It is available during Screen Lock status.

Follow these instructions to use the 100% O<sub>2</sub> function:

1. Select the **100% O<sub>2</sub>** button in the main GUI window.



2. The ventilator delivers 100% oxygen for two minutes. A countdown timer displays.




While 100% oxygen delivery is active, you can press the +2:00 button to add two minutes more. Press Cancel to stop.

## Using PPV

Follow these instructions to set up the ventilator in the PPV mode, referring to Figure 6-3. For principles of operation, see “PPV mode (optional)” on page 4-13.

1. Open the PPV Settings window.
2. Set EPAP, O<sub>2</sub>, alarm limits, and backup settings to appropriate values. The HIP alarm limit should be greater than Max P. See “Principles of operation” on page 4-1 for a detailed explanation of these settings.

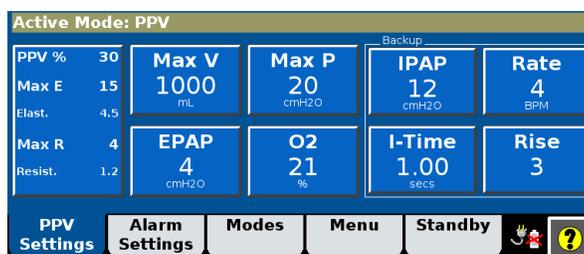

3. Set the Max V and Max P limits.
4. Set alarm limits to appropriate values. The HIP alarm limit should be greater than the Max P.

### About Max V and Max P alarms and alarm limits

Max V (PPV maximum volume limit) and Max P (PPV maximum pressure limit) are used to prevent the delivery of excessive pressure or volume.

---

**WARNING:** PPV limits are not intended to be the primary ventilator alarms and should not be substituted for the alarms found in the Alarm Settings window.

**WARNING:** To prevent the delivery of excessive pressure or volume, set the PPV limits appropriately. Delivery of excessive pressure or volume can occur from a sudden increase in mask leak, inappropriate settings, or a plugged or kinked proximal pressure line. Conversely, insufficient treatment may result if limits are set too low.

---

When the Max V (PPV maximum volume limit) is reached, the breath is terminated and a message is displayed. After the limit is reached in three consecutive breaths, the audible alarm sounds. A PPV waveform with Max V is shown in Figure 6-1.

## Operation

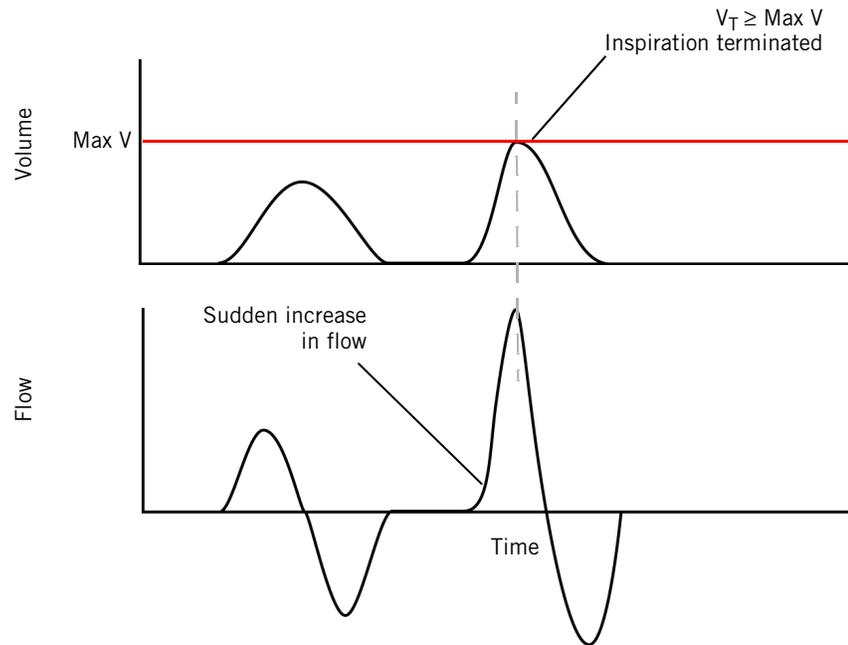


Figure 6-1: PPV waveform – Max V limit

When the Max P (PPV maximum pressure limit) is reached, pressure is limited but the breath is not terminated, and a message is displayed. After the limit is reached in three consecutive breaths, the audible alarm sounds. A PPV waveform with Max P is shown in Figure 6-2.

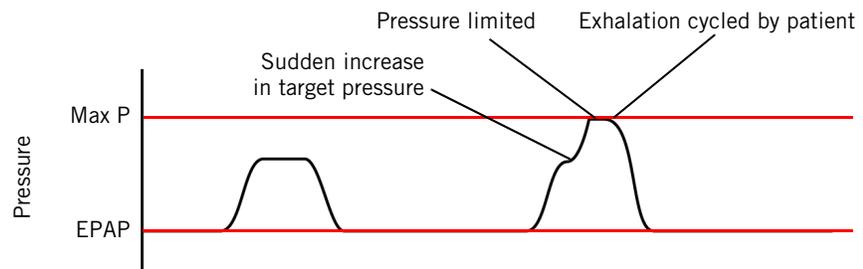


Figure 6-2: PPV waveform – Max P limit

Frequent annunciation of one or both alarms typically indicates improved patient status. It may, however, indicate that the patient is more actively breathing, possibly due to agitation or a change in the patient's level of sedation. It may also indicate an increase in leakage.

The  $V_T$  (estimated exhaled tidal volume) measurement may remain below the set Max V limit even though the inspired volume exceeds Max V. This results from variable leakage, which reduces the exhaled volume in relation to the inspired volume.

**Guidelines for using PPV**

---

NOTE: The guidelines below are based on recommendations by clinicians. They do not replace the clinical judgment of a physician and should not, on their own, be used for clinical decision making.

---

**Determining Max R and Max E settings**

It is recommended you set Max R (flow assist) and Max E (volume assist) to initial values and then titrate them based on the patient’s disease process:

- Obstructive disease (COPD, asthma): Focus on Max R. Overcoming increased resistance is typically the emphasis, not volume delivery.
- Restrictive disease (neuromuscular, chest-wall deformities, obesity hypoventilation): Focus on Max E. Maintaining sufficient volume is typically the emphasis, not overcoming increased resistance.
- Mixed disease processes affecting both resistance and elastance: Titrate both Max R and Max E settings.

**Suggested titration procedure** Follow this procedure to titrate settings to optimize patient comfort while avoiding overassisting. See also the flow chart in Figure 6-3.

---

NOTE: You may also need to adjust PPV % according to patient response, as you do for the other PPV settings described below. Mask leakage, especially a sudden increase, is interpreted as patient effort by the ventilator and assisted accordingly; this may necessitate lowering the PPV % setting. However, the best solution is to maintain a minimal leak.

---

1. Set EPAP, O<sub>2</sub>, alarm limits, and backup settings to appropriate values. The HIP alarm limit should be greater than Max P.

Suggested starting settings:

EPAP	4 cmH <sub>2</sub> O*
O <sub>2</sub>	Current setting or per prescription
Max P	25 cmH <sub>2</sub> O
Max V	1000 to 1500 mL
PPV %	80 to 100%
Max E	5 cmH <sub>2</sub> O/L
Max R	2 cmH <sub>2</sub> O/L/s
All other backup settings and alarms	Per usual protocol

\* Consider higher EPAP settings for COPD patients to treat autoPEEP as evidenced by missed triggers

## Operation

2. Adjust Max E:
  - a. Evaluate the patient. Check whether any of these conditions is true:
    - The patient says they are getting too much air, pressure, or volume
    - The patient is using accessory muscles to actively stop inspiration
    - The Max V or Max P limit is reached
    - The mask leak has suddenly increased
  - b. If none is true, increase Max E in increments of 2 cmH<sub>2</sub>O/L while continuing to evaluate the patient's response.
  - c. If any is true, decrease Max E by 2 cmH<sub>2</sub>O/L, and re-evaluate. Repeat to optimize patient comfort.
3. Repeat the process above adjusting Max R, increasing and decreasing in increments of 1 cmH<sub>2</sub>O/L/s to optimize patient comfort.
4. Repeat adjustment for Max E as needed.
5. Adjust PPV % downward as tolerated.

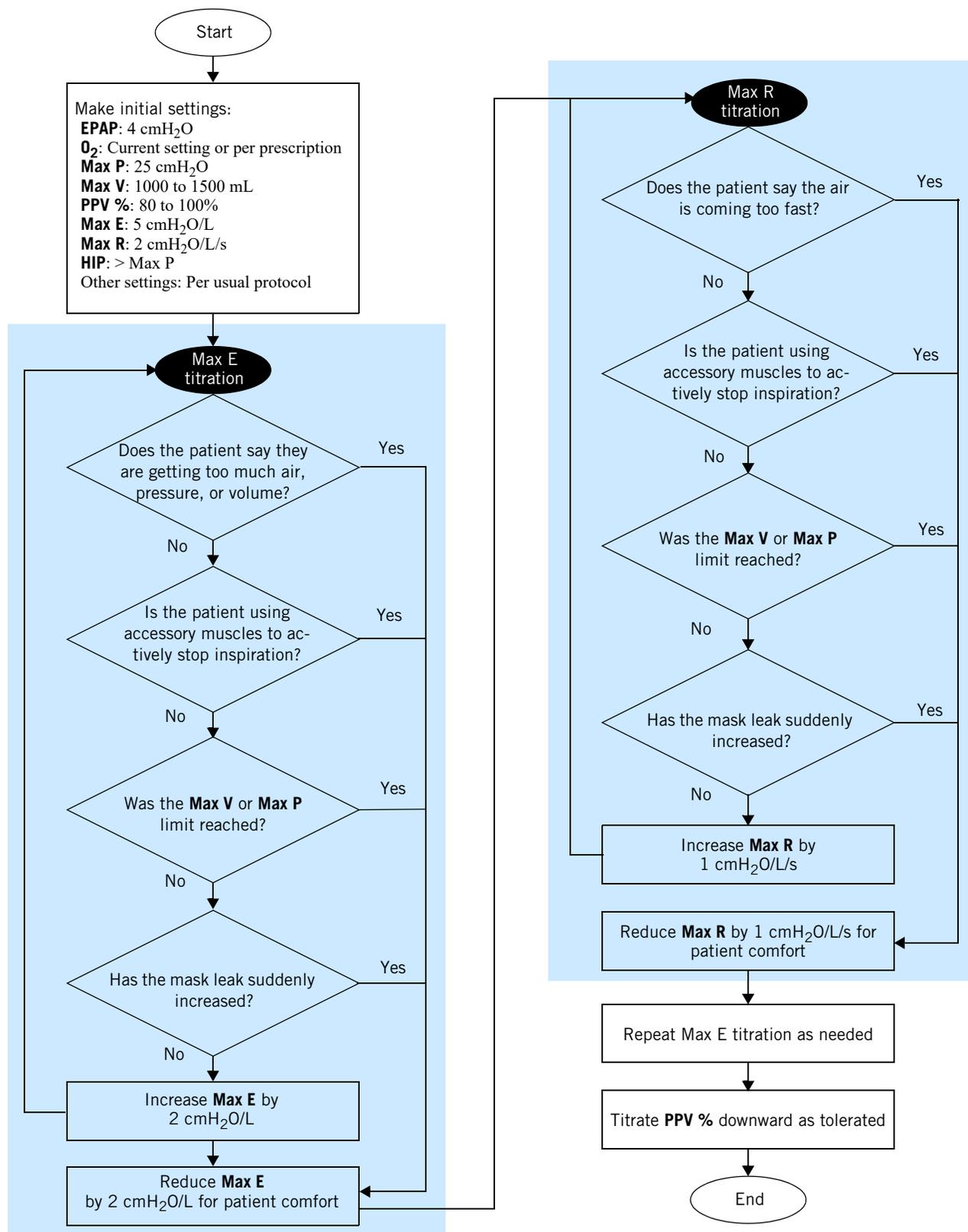


Figure 6-3: PPV initial setup

## Operation

### Changing alarm settings

---


**WARNING:** To prevent possible patient injury, avoid setting alarm limits to extreme values, which can render the alarm system useless.

---

Some ventilator alarm settings are operator adjustable. You can adjust these at any time. Table 6-4 on page 6-26 lists the alarm settings and their ranges.

Review and adjust the alarm settings as follows:

1. Open the Alarm Settings window.

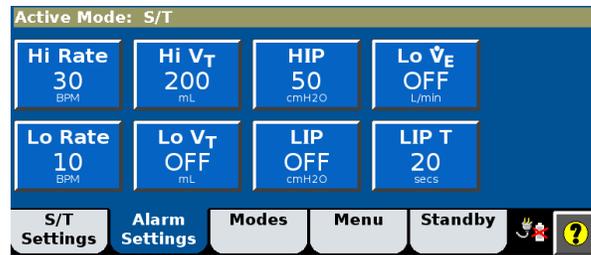

2. Select the desired setting, adjust it, and select Accept to apply.

The ventilator annunciates an alarm when a monitored value goes out of the range bounded by the alarm limits.

### Selecting the mask and exhalation port


To be able to display full leakage data plus accurate tidal and minute volumes, the ventilator must know the intentional leak characteristics of the specific mask/patient interface and exhalation port.

After power-on, the Messages list displays the current mask and port settings for 5 minutes.




Change these settings as follows:

1. Open the Menu window.
2. Select Mask/Port.

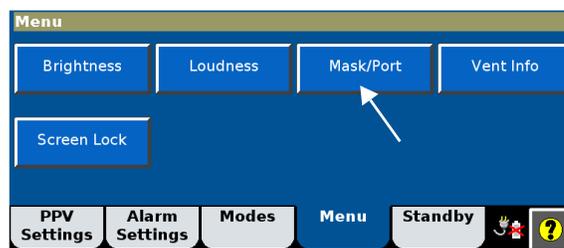


3. Select the desired mask/patient interface type (Table 6-1). Select Accept to apply.

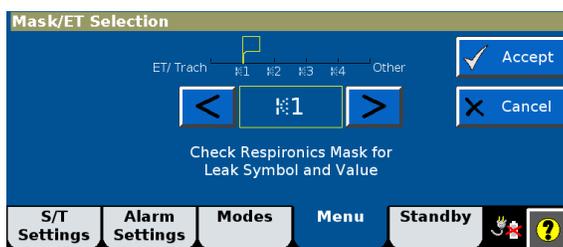

For information concerning mask/port leak characteristics, see the instructions provided with each mask/port. See Appendix C for a list of masks, circuits, and related components used with the ventilator.


Table 6-1: Mask/patient interface selections

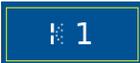Mask/patient interface type *	Description
 ET/Trach	ET or tracheostomy tube
 Leak 1	Mask with minimal intentional leak characteristics. Enter Leak 1 for any of these Philips Respironics masks: <ul style="list-style-type: none"> <li>• Contour Deluxe nasal mask</li> <li>• PerformaTrak mask</li> <li>• AF531, AF541 (EE)</li> </ul>
 Leak 2	Mask with medium intentional leak characteristics. Enter Leak 2 for this mask: <ul style="list-style-type: none"> <li>• Philips Respironics Performax oro-nasal mask [EE]</li> <li>• AF531, AF541 (EE)</li> </ul>
 Leak 3	AP111
 Leak 4	Reserved for future use
 Other	Mask not manufactured by Philips Respironics  NOTE: If you select Other, the ventilator displays Tot.Leak rather than Pt. Leak.

\* A leak symbol is printed on Respironics masks.

## Operation

4. Select the desired exhalation port type (Table 6-2). Select Accept to apply.

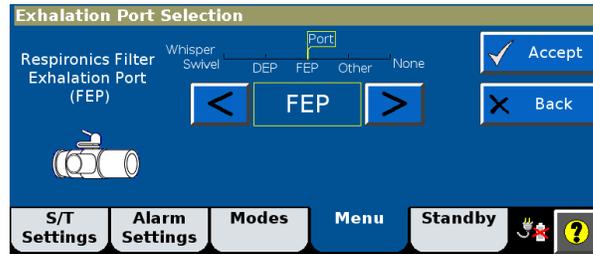




If you select an exhalation port that is not compatible with the selected mask, Not allowed with current mask is displayed.

---

**NOTE:** In ventilation modes, ET/tracheostomy tubes and most Philips Respironics masks require the use of an exhalation port. If you selected ET/Trach or Leak 1 as a mask/patient interface, you may not select None as an exhalation port.

---

Table 6-2: Exhalation port selections

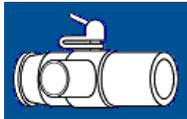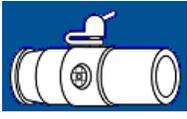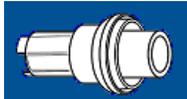Port type*	Exhalation port test recommended?
 <p>FEP Philips Respironics Filtered Exhalation Port</p>	No
 <p>DEP Philips Respironics Disposable Exhalation Port</p>	No
 <p>Whisper Swivel Philips Respironics Whisper Swivel</p>	No
 <p>PEV Philips Respironics Plateau Exhalation Valve</p>	Yes
 <p>Other Exhalation port not supplied by Philips Respironics.</p>	Yes
 <p>None No inline circuit exhalation port</p> <hr/> <p>NOTE: If you select None, refer to the manufacturer's instructions to make sure the mask selected contains an exhalation port.</p> <hr/>	No

\* Depending on your software version, your available exhalation port selections may vary.

- Run the exhalation port test if indicated in the table (see “Running the exhalation port test” on page 6-16 for instructions).

**CAUTION:** If you selected PEV or Other as an exhalation port, you must run an exhalation port test.

**NOTE:** If the exhalation port test is not run or if it fails, the intentional leak is unknown. Tot.Leak rather than Pt. Leak is displayed in the patient data window.

## Operation


### Running the exhalation port test

The exhalation port test is required and its window is automatically displayed when PEV or Other is selected.


#### Procedure

Run the test as follows:


1. Disconnect the patient circuit from the mask/patient interface.

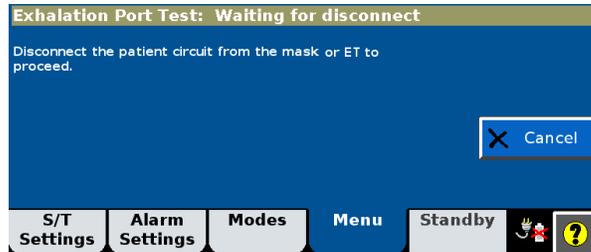


2. Occlude the circuit outlet. Select Start Test.

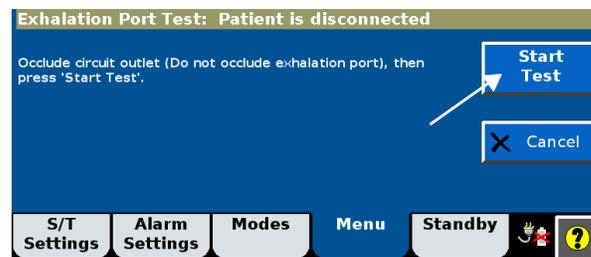

3. Wait while the test runs.

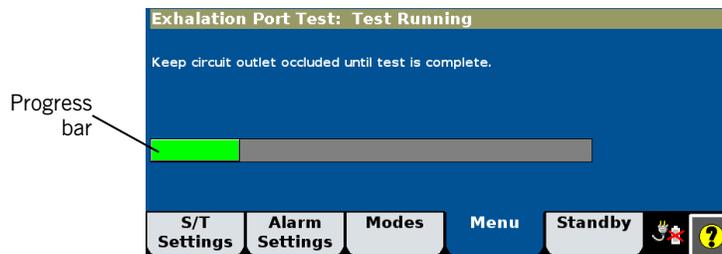

4. Verify that Test Passed is displayed.

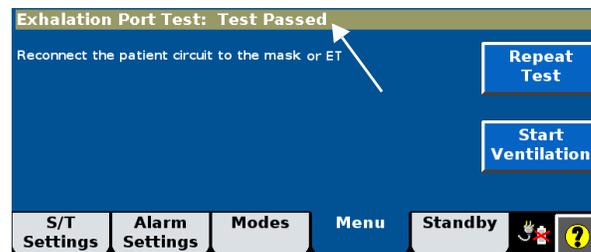


5. Reconnect the patient circuit to the mask/interface.
6. Select Start Ventilation to initiate ventilation.

### Troubleshooting

If Test Failed is displayed, check for leaks in the patient circuit, and install an exhalation device with lower leak characteristics. Repeat test. If the exhalation port test fails again, the intentional leak is unknown and Tot.Leak rather than Pt. Leak is displayed in the patient data window.

### Other functions: the Menu window

From the Menu window you can adjust user preferences.

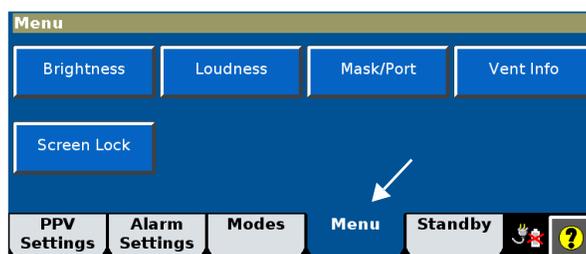

### Brightness

Use Brightness to adjust the screen for optimum daytime or nighttime viewing.

### Loudness

---


**WARNING:** Set the alarm loudness above the ambient level. Setting the alarm loudness too low may prevent recognition of alarm conditions.

**WARNING:** Avoid blocking the alarm speakers beneath the ventilator.

---

Use Loudness to adjust the volume of the alarm and touchscreen audible feedback. You will hear audible feedback as you go through the selections.

The Alarm Volume Escalation status is also displayed on this screen. See “Alarm Volume Escalation” on page E-11 for more information.

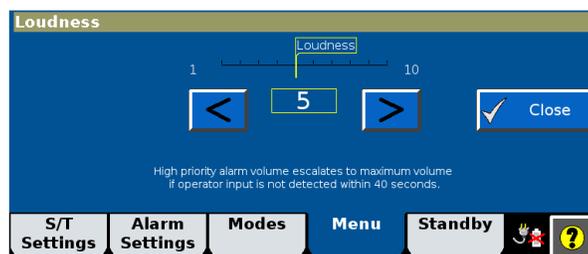


## Operation

### Mask/Port

See “Selecting the mask and exhalation port” on page 6-12.

### Vent Info (ventilator information)

The Ventilator Information window displays software version and other information specific to your ventilator.

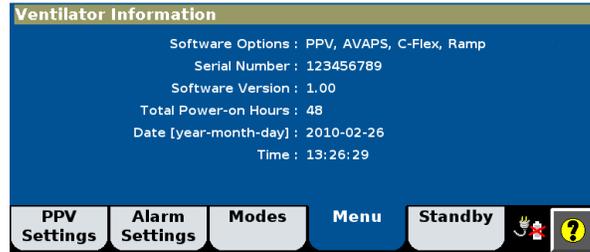

### Screen Lock

Screen Lock deactivates all buttons and tabs on the touchscreen except Alarm Silence, Alarm Reset, the Alarm/Message button, and Help. Tabs are grayed out as in this example.



This message bar is displayed at the top of the screen:



To unlock the screen, press the Accept button on the top-right front of the ventilator.

---

NOTE: If Screen Lock is active, the touchscreen remains locked even if an alarm becomes active.


---

### Auto-Trak+


The Normal Auto-Trak settings work well for most patients. Pediatric patients, however, may benefit from more sensitive trigger settings, while some adult patients may benefit from more or less sensitive cycle settings.

Changing Auto-Trak+ settings


1. Select Auto-Trak+ from the Menu window.

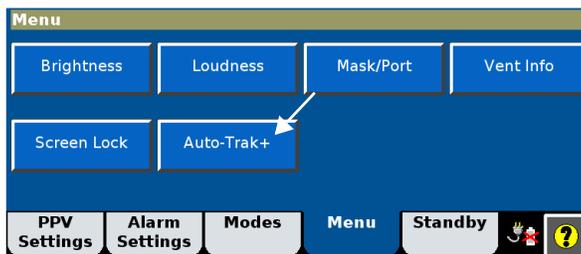

2. Select the desired adjustment. As an example, the E-Cycle adjustment is shown below.

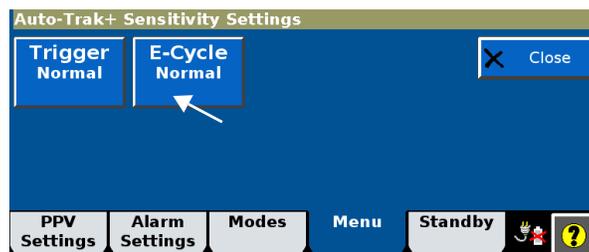

3. The setting window opens. Adjust the setting, referring to the pressure-time graphic which represents the effect on I-Time. Select Accept to apply.

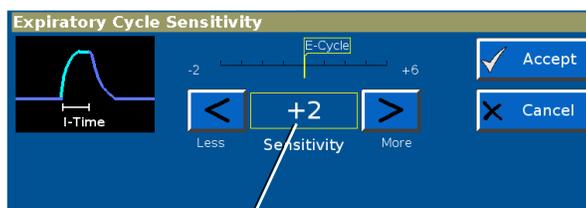


Proposed value

When Auto-Trak+ is active (when either Trigger or E-Cycle is set to a value other than Normal), the ventilator setting window displays Auto-Trak+.



Additionally, after power-on the Messages list displays the Auto-Trak+ settings for 5 minutes.

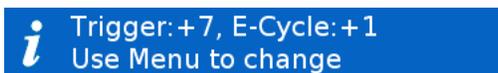


## Operation

### Standby

Standby lets you safely suspend ventilation to temporarily disconnect the patient from the ventilator or to set up the ventilator before connecting the patient. Alarms are disabled during standby.

You can also change ventilator settings and most menu functions during standby. The settings changes are effective when you exit standby. Enter standby as follows:

1. Select Standby. The Entering Standby window opens.

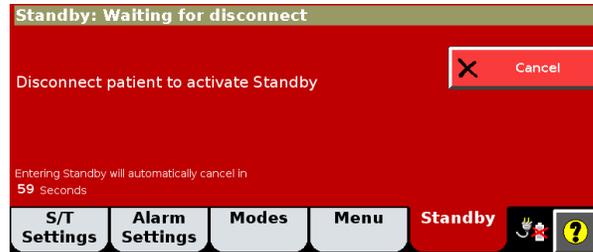


---

**NOTE:** Remove the mask/patient interface in order to enter standby. The ventilator will not enter standby with a patient connected. If the patient is not disconnected, the ventilator continues breath delivery while waiting for the patient to be disconnected. The standby mode request cancels in 60 seconds if the patient remains connected.

**NOTE:** Standby mode disables alarms and should be used when the patient is disconnected.

---

2. Disconnect the patient from the ventilator now. The ventilator enters standby and displays the Standby screen.

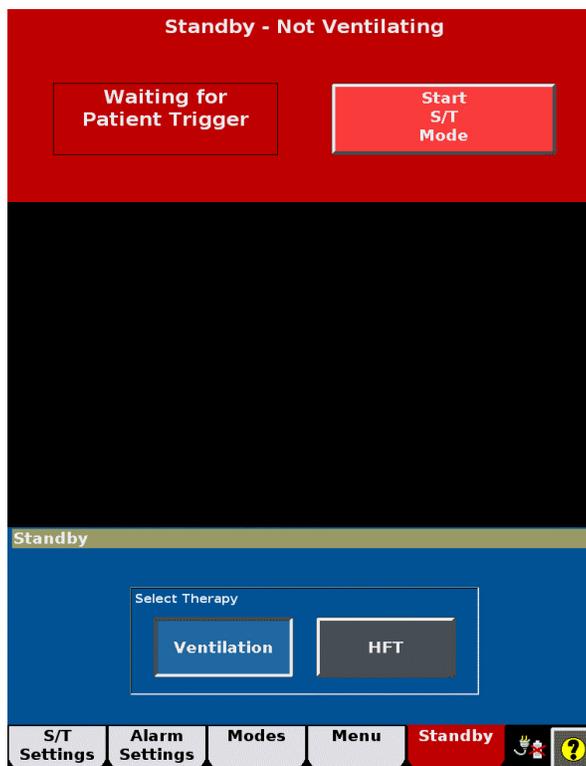

3. To resume ventilation, reconnect the patient. When the ventilator senses a patient breathing effort, ventilation automatically resumes in the previous mode.

---


NOTE: You can also manually resume ventilation with the Restart Mode button.

---


## Operation

### Help function

Select the help button to display additional information.



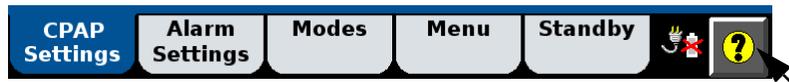


Help messages are displayed:

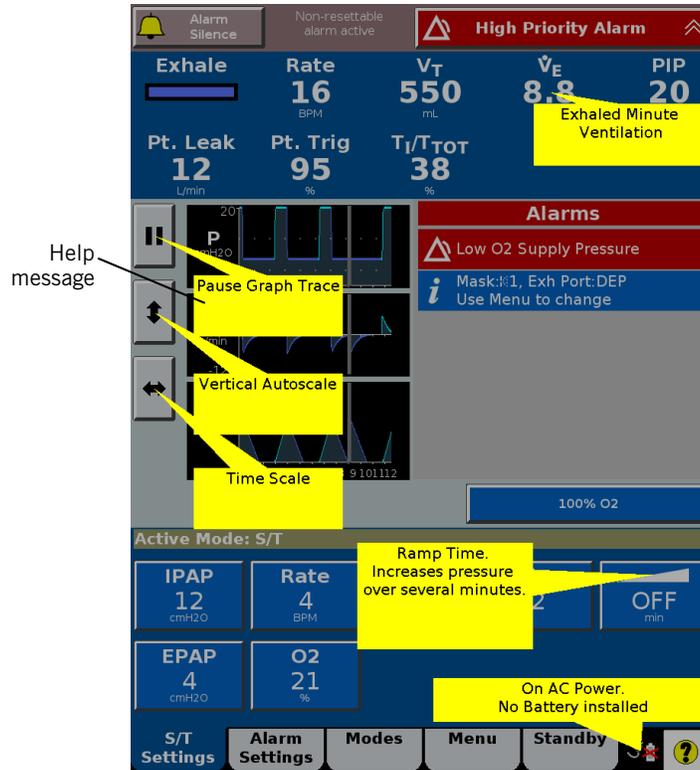

## Table of modes and control settings

Table 6-3: Modes and control settings with ranges

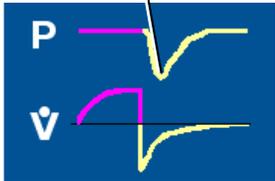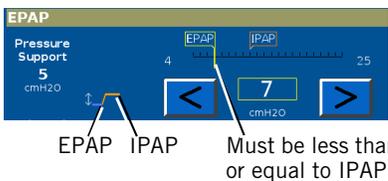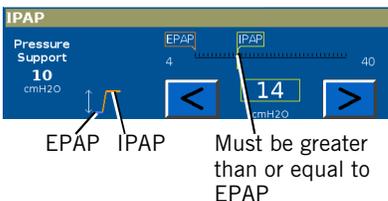Setting	Description	Range	
<b>Modes</b>			
Modes	Ventilation mode	AVAPS, CPAP, S/T, PCV Optional: PPV	
<b>Control settings</b>			
C-Flex	Enhances traditional CPAP by reducing the pressure at the beginning of exhalation—a time when patients may be uncomfortable with CPAP—and returning it to the set CPAP pressure before the end of exhalation. The amount of pressure relief is determined by the C-Flex setting and the expiratory flow. The higher the setting number (1, 2 or 3) and the greater the expiratory flow, the greater the pressure relief (during the active part of exhalation only). Applies in CPAP mode only.	<p>Pressure relief</p> 	OFF, 1 to 3
CPAP	Continuous positive airway pressure. The baseline pressure applied during the expiratory phase. Applies in CPAP mode only.		4 to 25 cmH <sub>2</sub> O
E-Cycle (optional)	Expiratory Cycle Sensitivity. Auto-Trak+ employs several algorithms to determine the point at which the ventilator cycles into exhalation. This setting adjusts all algorithms simultaneously. At the lowest setting (-2), inspiration terminates later, resulting in the longest inspiratory time. At the highest setting (+6), inspiration terminates earlier, resulting in the shortest inspiratory time. Normal is the Auto-Trak setting used when Auto-Trak+ is not enabled. Applies only when the optional Auto-Trak+ feature is installed.		-2, -1, Normal, +1 to +6
EPAP	Expiratory positive airway pressure. The application and maintenance of pressure above atmospheric at the airway throughout the expiratory phase of positive-pressure mechanical ventilation.		4 to 25 cmH <sub>2</sub> O
IPAP	Inspiratory positive airway pressure. The application and maintenance of pressure above atmospheric at the airway throughout the inspiration phase of positive-pressure mechanical ventilation.		4 to 40 cmH <sub>2</sub> O

## Operation

Table 6-3: Modes and control settings with ranges (continued)

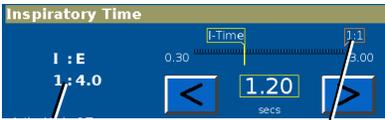
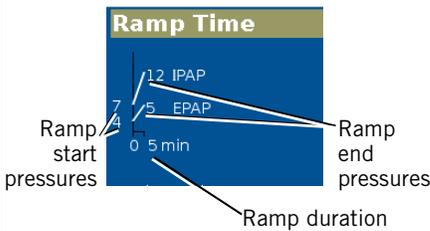
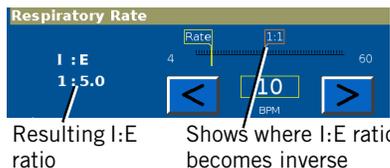
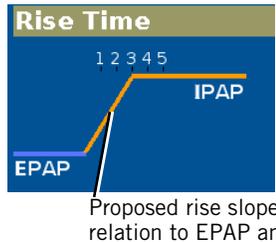
Setting	Description	Range
I-Time (Inspiratory Time)	Time to deliver the required gas. Inverse ratio ventilation is not allowed.	0.30 to 3.00 secs
	 <p>Resulting I:E ratio</p> <p>Shows where I:E ratio becomes inverse</p>	
Max E	The maximum elastance (volume assist) value used by the PPV mode to overcome the elastance of the patient's lungs. See also PPV % setting. Applies in PPV mode only.	0 to 100 cmH <sub>2</sub> O/L
Max P (AVAPS Maximum IPAP Pressure)	The maximum pressure to be applied.  NOTE: When you adjust the AVAPS minimum and maximum pressures, remember that IPAP is adjusted to meet the target value. If the calculated target pressure is outside of the minimum and maximum pressure range, the target volume will not be achieved.  Applies in AVAPS mode only.	6 to 40 cmH <sub>2</sub> O
Max P (PPV Maximum Pressure Limit)	The maximum pressure to be applied. When the limit is reached, the ventilator limits the pressure and displays a PPV Max P alarm message. If the condition persists for three consecutive PPV inspirations, an audible alarm also sounds. Applies in PPV mode only.  WARNING:PPV limits are not intended to be the primary ventilator alarms and should not be substituted for the alarms found in the Alarm Settings window.  WARNING:To prevent the delivery of excessive pressure or volume, set the PPV limits appropriately. Delivery of excessive pressure or volume can occur from a sudden increase in mask leak, inappropriate settings, or a plugged or kinked proximal pressure line. Conversely, insufficient treatment may result if limits are set too low.	5 to 40 cmH <sub>2</sub> O
Max R	The maximum resistance (flow assist) value used by the PPV mode to overcome pulmonary resistance. See also PPV % setting. Applies in PPV mode only.	0 to 50 cmH <sub>2</sub> O/L/s
Max V (PPV Maximum Volume Limit)	The maximum volume to be delivered. When the limit is reached, the ventilator terminates the breath and displays a PPV Max V alarm message. If the condition persists for three consecutive PPV inspirations, an audible alarm also sounds. Applies in PPV mode only.  WARNING:PPV limits are not intended to be the primary ventilator alarms and should not be substituted for the alarms found in the Alarm Settings window.  WARNING:To prevent the delivery of excessive pressure or volume, set the PPV limits appropriately. Delivery of excessive pressure or volume can occur from a sudden increase in mask leak, inappropriate settings, or a plugged or kinked proximal pressure line. Conversely, insufficient treatment may result if limits are set too low.	200 to 3500 mL

Table 6-3: Modes and control settings with ranges (continued)

Setting	Description	Range
Min P (AVAPS Minimum IPAP Pressure)	The minimum pressure to be applied.  NOTE: When you adjust the AVAPS minimum and maximum pressures, remember that IPAP is adjusted to meet the target value. If the calculated target pressure is outside of the minimum and maximum pressure range, the target volume will not be achieved.  Applies in AVAPS mode only.	5 to 30 cmH <sub>2</sub> O
O <sub>2</sub>	Oxygen concentration to be delivered.	21 to 100%
PPV %	Percentage of PPV assist or gain. This gain is applied to the Max E and Max R settings, yielding the applied Elastance and Resistance assist values. Applies in PPV mode only.	0 to 100%
Ramp Time	An interval during which time the ventilator linearly increases pressure, helping to reduce patient anxiety.  $\text{Initial CPAP/EPAP} = \frac{\text{CPAP/EPAP} + 4 \text{ cmH}_2\text{O}}{2}$ $\text{Initial IPAP} = \text{Initial EPAP} + \frac{(\text{IPAP} - \text{EPAP})}{2}$	
Rate (Respiratory Rate)	Respiratory frequency or number of breaths per minute. Inverse ratio ventilation is not allowed.	
Rise (Rise Time)	Speed with which inspiratory pressure rises to the set (target) pressure. If the Rise Time is insufficient to reach the target IPAP pressure, adjust the Rise Time or I-Time setting.	

## Operation

Table 6-3: Modes and control settings with ranges (continued)

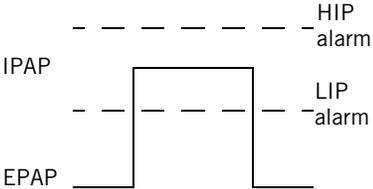
Setting	Description	Range
Trigger (optional)	Trigger Sensitivity. Auto-Trak+ employs several algorithms to determine the point at which the inspiration begins. The larger the value, the more sensitive the trigger (that is, the patient can trigger inspiration with less effort). Normal is the Auto-Trak setting used when Auto-Trak+ is not enabled. Applies only when the optional Auto-Trak+ feature is installed.	Normal, +1 to +7
$V_T$ (AVAPS Target Tidal Volume)	Target tidal volume to be delivered during inspiration. The ventilator meets this target by adjusting the inspiratory pressure with each breath. Applies in AVAPS mode only.	200 to 2000 mL

Table 6-4 lists the settable alarms. (For a complete list of non-settable alarms, including Patient Disconnect, Occlusion, Pressure Regulation High, and other alarms, refer to Table 9-3 on page 9-7.)

Table 6-4: Alarm settings

Setting	Description	Range
Hi Rate (High Rate Alarm)	High total breath rate.	5 to 90 BPM
Lo Rate (Low Rate Alarm)	Low total breath rate.	1 to 89 BPM
	NOTE: In non-CPAP modes, the Low Rate Alarm is essentially off if set below the Respiratory Rate setting.	
Hi $V_T$ (High Tidal Volume Alarm)	High exhaled tidal volume.	200 to 3500 mL
Lo $V_T$ (Low Tidal Volume Alarm)	Low exhaled tidal volume.	OFF to 1500 mL
HIP (High Inspiratory Pressure Alarm)	High pressure at the patient airway.	5 to 50 cmH <sub>2</sub> O
LIP (Low Inspiratory Pressure Alarm)	Low pressure at the patient airway.	OFF to 40 cmH <sub>2</sub> O

Table 6-4: Alarm settings (continued)

Setting	Description	Range
	<p>NOTE: In the S/T and PCV modes, the LIP alarm should be set 3-5 cmH<sub>2</sub>O below the IPAP level. When set in this manner, the alarm works in conjunction with the LIP T alarm to indicate if there is a failure to trigger between the two pressure levels. It will also alert the clinician to pressure degradation due to excessive leaks. See figure below.</p> 	
LIP T (Low Inspiratory Pressure Delay Time)	The interval from the detection of low inspiratory pressure until the alarm becomes active.	5 to 60 secs
Lo $\dot{V}_E$ (Low Minute Ventilation Alarm)	Low expiratory minute volume.	OFF to 99.0 L/min

**Operation**

## Chapter 7. High flow therapy

---

The high flow therapy (HFT) feature is available for 3.00 software and above, as well as V60 Plus. HFT is accessed from the **Standby** mode. For more information, see “Standby” on page 6-20.

For principles of operation, see “High flow therapy” on page 4-6.

---

**WARNING:** When transitioning from a high flow therapy interface to an NIV mask, ensure that an exhalation port is placed in the circuit and is unobstructed to reduce the risk of CO<sub>2</sub> rebreathing.

**WARNING:** When transitioning from ventilation to high flow therapy, remove the NIV mask and use only a Philips-approved high flow patient interface to minimize pressure build-up and patient discomfort.

**WARNING:** When transitioning from high flow therapy to ventilation, remove the high flow nasal cannula as it is restrictive and may defeat alarms such as patient disconnect. Using a high flow nasal cannula in an NIV mode may lead to hypercarbia due to the inability to provide pressure support.

**WARNING:** Patient alarms are not available during high flow therapy (HFT) as the therapy uses an open system. A high flow nasal cannula occupies only a portion of the nares and patients can breathe through their mouth, which prevents estimation of patient parameters such as tidal volume, respiratory rate, pressure, and minute ventilation. Provide external monitoring, including oximetry, to inform the clinician of a change in the patient's condition.

**WARNING:** During high flow therapy (HFT), verify that an occlusive patient interface is not being used. Occlusive patient interfaces include a cannula fully sealed within the nares, an NIV mask, or a direct connection to a tracheostomy tube or endotracheal tube. Remove any occlusive interface immediately as this may expose the patient to unintended high pressures.

**NOTE:** High flow therapy (HFT) is accessed only from the Standby window. Standby mode cannot be entered if a high flow nasal cannula is connected to the circuit.

---

### Circuit setup

See “Installing the patient circuit” on page 5-3 for circuit configuration.

## High flow therapy

### High flow nasal cannula setup

Use either the AC611 high flow nasal cannula with FEP Connector (Figure 7-1) or the AC611 high flow nasal cannula 22 mm (Figure 7-3), which connects directly to the patient circuit.

#### Using the FEP Connect for high flow therapy


---

**NOTE:** This section applies only if you are using the AC611 high flow nasal cannula with filter exhalation port (FEP) connector (“FEP Connect”) to administer high flow therapy.

---

#### High flow nasal cannula setup

1. Insert the AC611 high flow nasal cannula with FEP Connector into the FEP, making sure the perforations in the port are completely blocked.

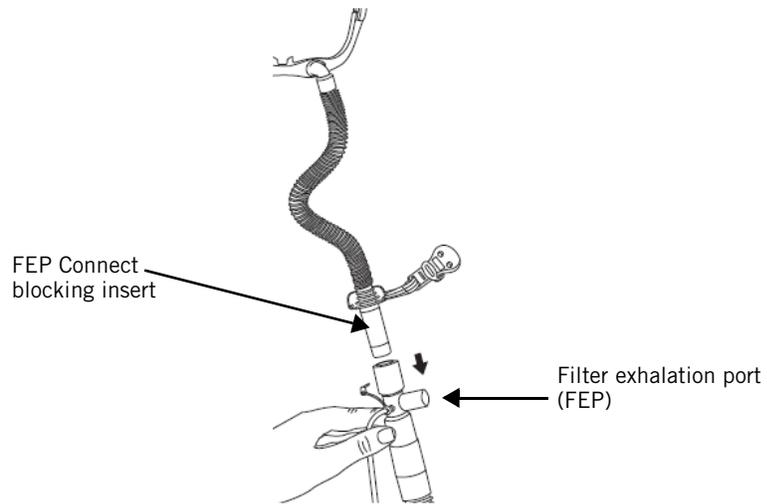

*Figure 7-1: High flow nasal cannula using the FEP Connect*

2. Disconnect the proximal pressure line from the ventilator port.
3. Leave the other end of the proximal pressure line connected to the FEP and anchor the tubing to the breathing circuit using several tubing clips (included with most circuits), as shown in Figure 7-2.

---

**NOTE:** Follow your institutional guidelines for infection control and single-patient use interfaces.

---

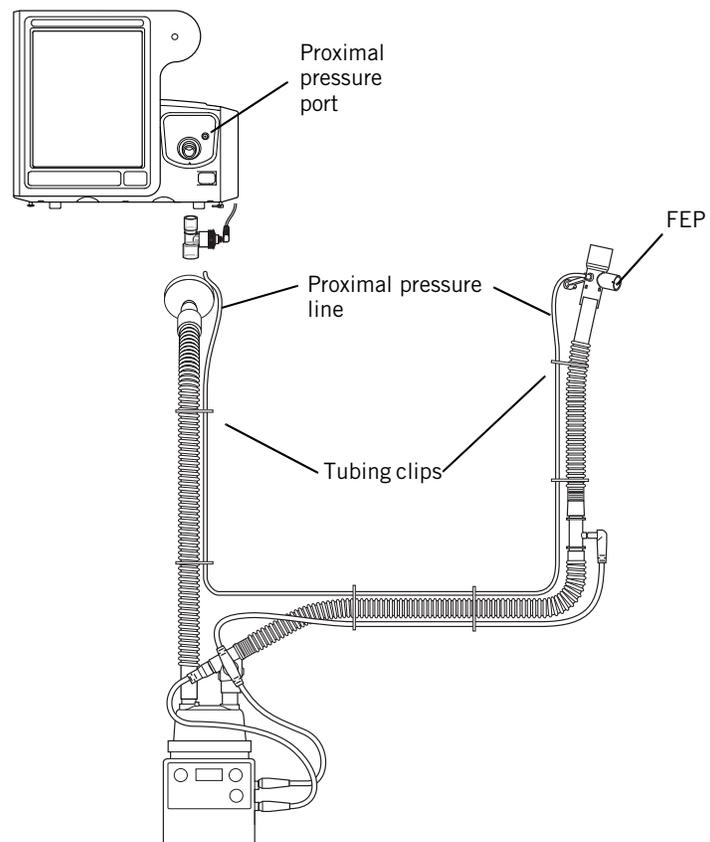


Figure 7-2: Proximal pressure line anchored to circuit with tubing clips

### Changing from ventilation to high flow therapy (HFT)

Follow the instructions in this chapter. *Immediately after starting HFT* disconnect the proximal pressure line from the ventilator.

### Changing from high flow therapy (HFT) to ventilation

Follow the instructions in this chapter. *Immediately before activating ventilation*, reconnect the proximal pressure line to the ventilator.

## High flow therapy

### Connecting directly to a 22 mm circuit

Remove the FEP, and connect the high flow nasal cannula with 22 mm connector directly to the circuit.


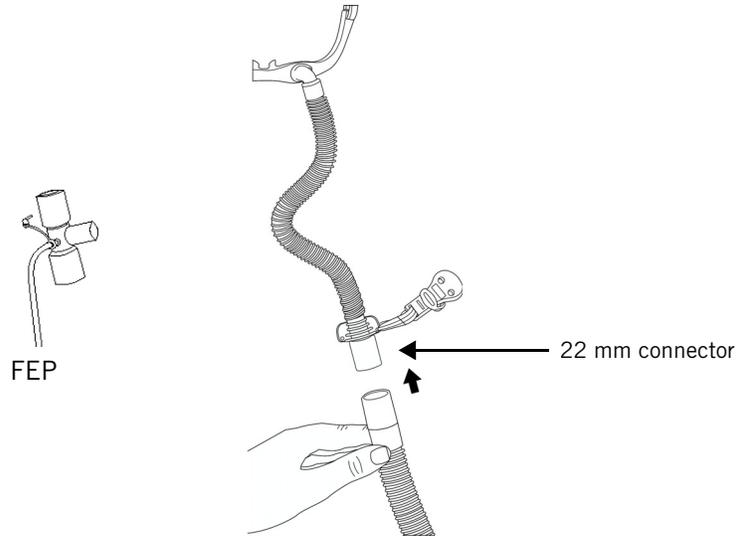


Figure 7-3: High flow nasal cannula, 22 mm connection

## Changing from an NIV mode to high flow therapy


Follow these instructions to use the V60/V60 Plus Ventilator for high flow therapy (HFT).

1. Select **Standby**. The Entering Standby window opens.
2. Remove the patient mask or ET interface to enter Standby.
3. Install a Philips-approved high flow nasal cannula (Figure 7-1 and Figure 7-3 above) or a high flow tracheostomy interface on the patient circuit.
4. Select **HFT**.

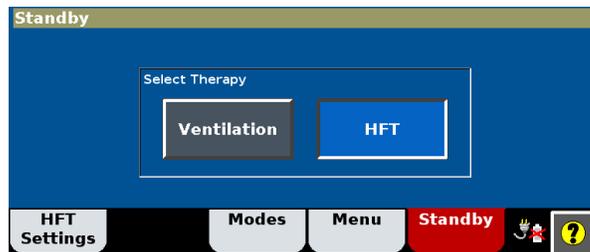


## High flow therapy

- From the Active Mode window, you can adjust Flow and O<sub>2</sub>%.

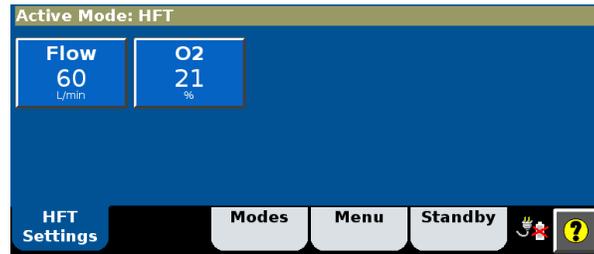

- Press **Start HFT**.

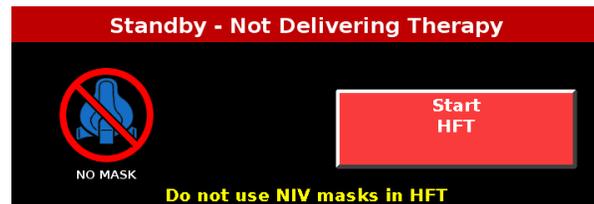

- The High Flow Therapy Active message is displayed during HFT.




- Apply the HFT interface to the patient.
- Note the low priority alarm stating that patient alarms are disabled during HFT. Press alarm reset to confirm this message.



## Viewing and pausing the HFT graph


A flow graph is displayed during high flow therapy. Press the Pause button to view an event.

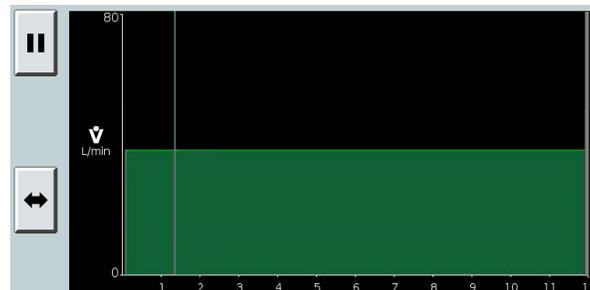


## High flow therapy

### Changing from high flow therapy to an NIV mode


1. Verify that the high flow nasal cannula is removed from the patient and disconnected from patient circuit.
2. Select Standby to open the Standby window.
3. Press the Enter Standby button.

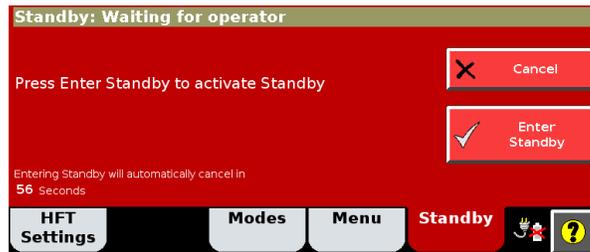

4. In the Select Therapy window, press Ventilation.

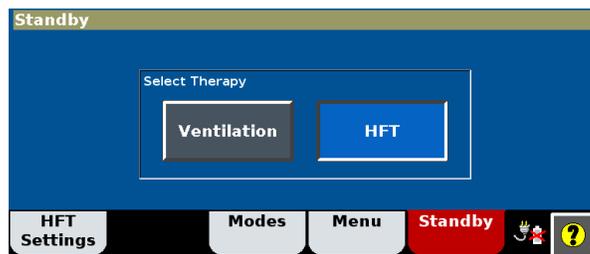

5. Replace the high flow patient interface with a Philips-approved NIV mask.
6. Review patient settings and alarms.
7. Install the appropriate interface on the patient.
8. Verify that the ventilator detects the patient's breath to activate ventilation, or press the Start Mode button.

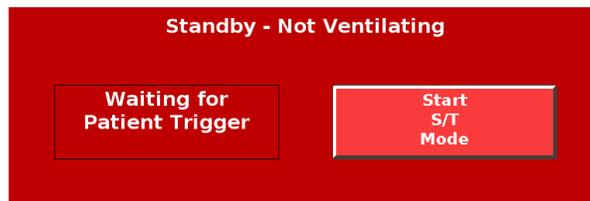

### HFT alarms and messages

Table 7-1 is a list of alarms and other messages displayed by the ventilator, along with descriptions, suggested corrective actions, and other information. The ID (identifier) listed with the priority type is the priority number of the alarm. This priority number determines the order of alarm message display. Unless otherwise indicated, alarms listed.

Table 7-1: HFT Alarm and other messages: summary and troubleshooting

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autoresetable	Silence able
Cannot Reach Target Flow	Displays when HFT (high flow therapy) is active. Indicates that flow target is not achieved.	Check the patient. Check that the high flow nasal cannula size is appropriate for the flow setting. Check that an occlusive interface is NOT in use (a cannula fully sealed within the nares, an NIV mask or direct connection to an ETT/trach). Check for an occlusion, kink or liquid in the patient circuit.	Low (66)	No	Yes	Yes
Patient alarms are disabled during HFT	Displays when HFT (high flow therapy) is active. Patient alarms are not available in this therapy.	Manually reset to confirm and clear the audible alarm.	Low/Information (68)	Yes	No	Yes
Patient Circuit Occluded	Displays when HFT (high flow therapy) is active. Gas flow to the patient is obstructed.	Check the patient. Check that an occlusive interface is NOT in use (a cannula fully sealed within the nares, an NIV mask or direct connect to an ETT/trach). Check for an occlusion, kink or liquid in the patient circuit. If problem persists, provide alternative ventilation.	High (67)	No	Yes	Yes
Check Vent: Proximal Pressure Sensor Range Error	Displays when HFT (high flow therapy) is active. Proximal pressure is out of range.	When using a high flow nasal cannula with FEP Connect, the proximal pressure line should be disconnected from the ventilator port during HFT. If this message is seen when switching the patient from ventilation to HFT, check to make sure the proximal pressure line is disconnected. If the message still occurs, provide alternative ventilation and have the ventilator serviced.	High (12)	Yes	No	No

## High flow therapy

## Chapter 8. Patient monitoring

The ventilator displays numeric patient data in the patient data window and real-time graphics in the waveform window (Figure 8-1). Numeric patient data is updated every breath. Table 8-1 on page 8-2 lists the ventilator's monitored parameters.

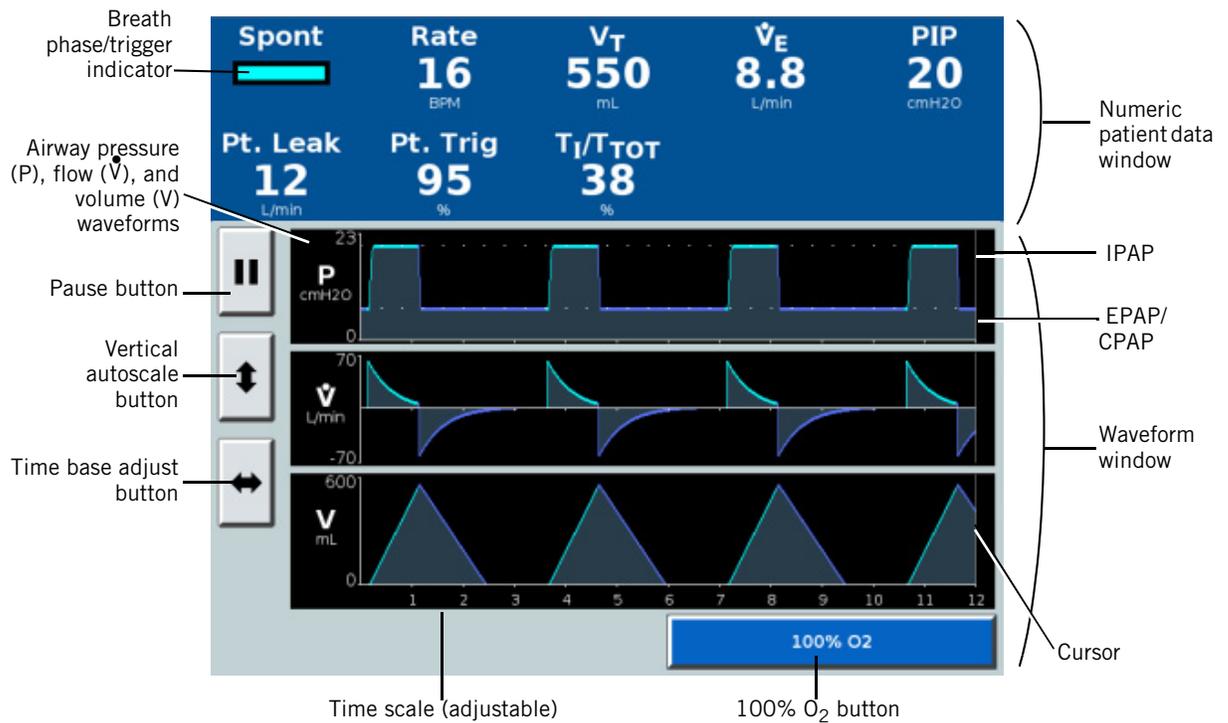


Figure 8-1: Patient data and waveform window

### Display conventions

The following symbols may be displayed in place of numeric values:

- \*\*\* Data is not valid, and/or ventilator is in standby mode or disconnected
- +++ Data is over range
- Data is under range

## Patient monitoring

### Table of monitored parameters

Table 8-1: Monitored parameters

Parameter	Definition
<b>Patient data window</b>	
Breath phase/trigger indicator	<b>Spont</b> (spontaneous): Inspiratory phase, patient-triggered breath (color: turquoise) <b>Timed</b> : Inspiratory phase, ventilator-triggered breath (color: orange) <b>Exhale</b> : Expiratory phase (color: blue)
PIP	Peak inspiratory pressure. The highest patient pressure during the previous breath cycle.
Pt. Leak	Estimated patient leak or unintentional leak. Average during the previous breath cycle. Displayed only after a suitable exhalation port and mask/patient interface are selected.
Pt. Trig	Patient-triggered breaths, as a percentage of total breaths over the last 15 minutes.
Rate	Respiratory rate or total breathing frequency. Moving average over the last 6 breaths (or 15 seconds).
$T_I/T_{TOT}$	Inspiratory duty cycle or inspiration time divided by total cycle time. Moving average over the last 8 breaths.
Tot. Leak	Estimated total leak. Average during the previous breath cycle. Displayed before a suitable exhalation port and mask/patient interface are selected.
$\dot{V}_E$	Estimated minute ventilation. The product of tidal volume (spontaneous and timed) and rate (spontaneous and timed). Moving average over the last 6 breaths.
$V_T$	Estimated exhaled tidal volume. Moving average over the last 6 breaths. It is body temperature pressure saturated (BTPS) compensated.
<b>Waveform window</b>	
<i>P waveform</i>	Airway pressure. Where applicable, dotted lines represent target IPAP and EPAP.
$\dot{V}$ waveform	Estimated patient flow. The total delivered flow minus the leak flow ( <b>Tot. Leak</b> ), where <b>Tot. Leak</b> includes known (intentional) leakage through the exhalation port plus any unintentional leakage in the circuit or at the mask/patient interface.
<i>V waveform</i>	Estimated patient volume. In AVAPS mode, the dotted line represents target volume.

### Scaling the waveform axes

Scale the vertical and horizontal waveform axes with the scale buttons.



The vertical scale button autoscales the Y axes to best fit the current data.



The horizontal (time adjust) button rescales the X axis to show 3, 6, 12, or 24 seconds.

### Freezing and unfreezing waveforms



Freeze waveforms for extended viewing by selecting the pause button to the left of the waveform window.



The cursor makes one complete sweep across the waveform and then displays the pause in progress symbol. The graphic display is then frozen, and the cursor is visible in the middle of the display (Figure 8-2). Reposition the cursor by touching the waveform screen or with the navigation ring (legacy versions). Data values at cursor location for pressure, flow, and volume are displayed in the white boxes.

boxes.



Unfreeze the waveforms with the resume button.



Figure 8-2: Waveform window with frozen screen

## Patient monitoring

## Chapter 9. Alarms, messages, and troubleshooting

---

Alarms and messages on the ventilator alert you to situations that require your attention. The ventilator can also actuate remote alarms. Figure 9-1 on page 2 shows the visual alarm characteristics. Table 9-3 on page 9-7 summarizes the different types of alarm and tells you how to respond to each.

### Responding to alarms

---

**WARNING:** If AC power fails and the backup battery is not installed or is depleted, an audible and visual alarm annunciates for at least 2 minutes. Immediately discontinue ventilator use and secure an alternative means of ventilation. As in most ventilators with passive exhalation ports, when power is lost, sufficient air is not provided through the circuit and exhaled air may be rebreathed.

**NOTE:** If an alarm persists for no apparent reason, discontinue ventilator use and contact Philips.

---

Respond to an alarm as follows:

1. Approach the patient immediately. Secure sufficient and effective ventilation for the patient. You may silence the alarm if possible.
2. Correct the alarm condition, referring to the alarm messages in Table 9-3.

You can modify alarm settings at any time through the **Alarm Settings** tab.

# Alarms, messages, and troubleshooting

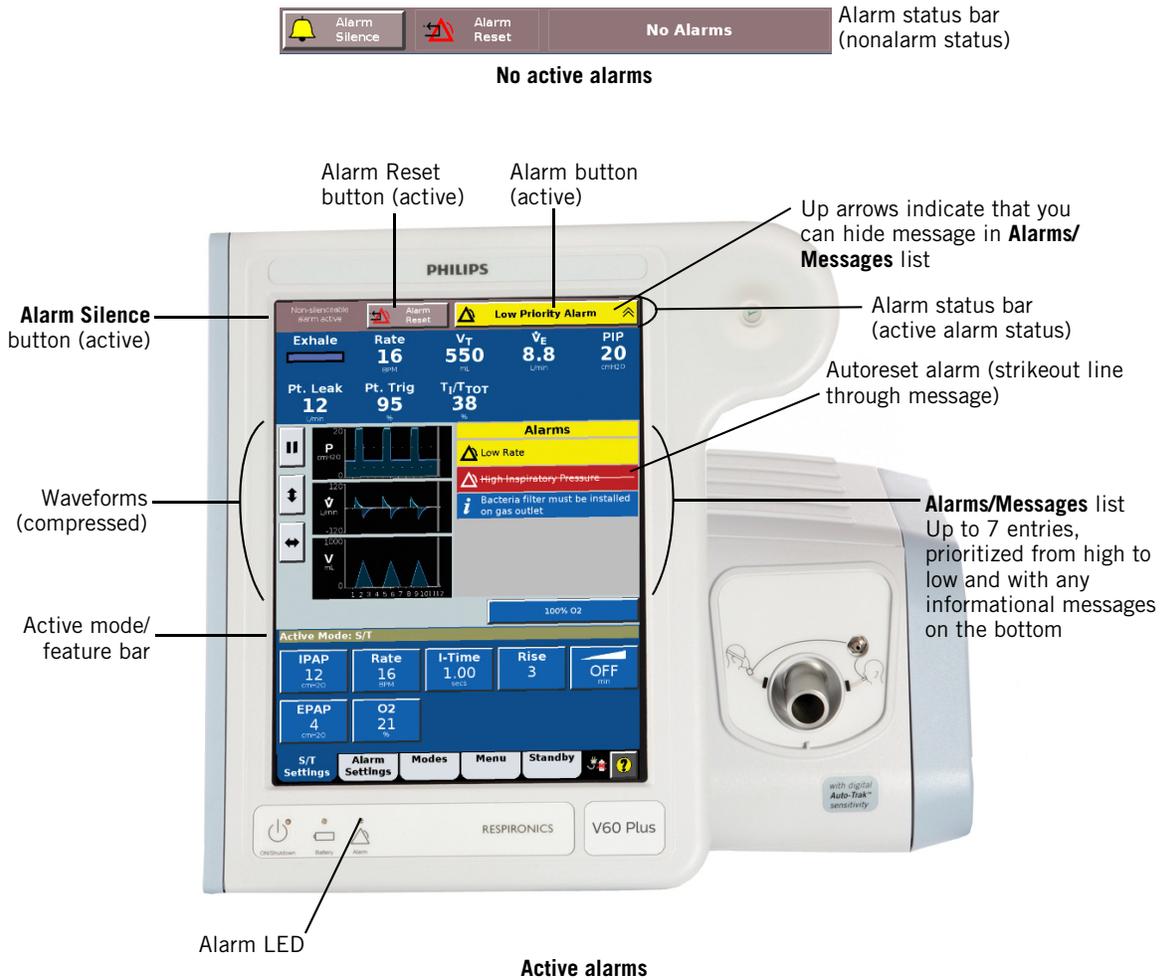


Figure 9-1: Visual alarm indications


## Alarms, messages, and troubleshooting

Table 9-1: Alarm summary

Status	Alarm LED on front panel	Alarm status bar	Alarm message in Alarms list	Audio*	Action required	Remote alarm
No alarms	Off		None	Off	None	Off
Autoreset alarm	Off	Red (high-priority) or yellow (low-priority) 	Background color same as that of active alarm. Message with strike-out text. Alarm icon.			
Informational message	Off	Blue 	Blue background color. Informational icon.		Important information or instructions.	
Low-priority alarm	Off	Yellow 	Yellow background color. Alarm icon.	Intermittent tone at an interval of approximately 20 seconds	Respond promptly. Troubleshoot as per Table 9-3.	
High-priority alarm	Flashes	Alternates black and red 	Red background color. Alarm icon.	Repeating sequence of 5 tones	Respond immediately to ensure patient safety. Troubleshoot as per Table 9-3.	On
High-priority alarm – Check Vent					Respond immediately to ensure patient safety. Do not use equipment that is malfunctioning or that indicates a potential problem until the problem is corrected. Troubleshoot as per Table 9-4.	
High-priority alarm – Vent Inoperative	On continuously	Vent Inoperative screen, including code (Figure 9-2)		Primary alarm (Repeating sequence of 5 tones) or backup alarm (alternating tone for a minimum of 2 minutes)	Continued safe ventilator operation may be in jeopardy. O <sub>2</sub> flow and blower operation are disabled. Immediately secure alternative ventilation for the patient. Troubleshoot as per Table 9-5.	
Loss of power	Off	Blank	Blank		Immediately secure alternative ventilation for the patient.	

\* The volume of the primary alarm is the same for low- and high-priority alarms.

## Alarms, messages, and troubleshooting

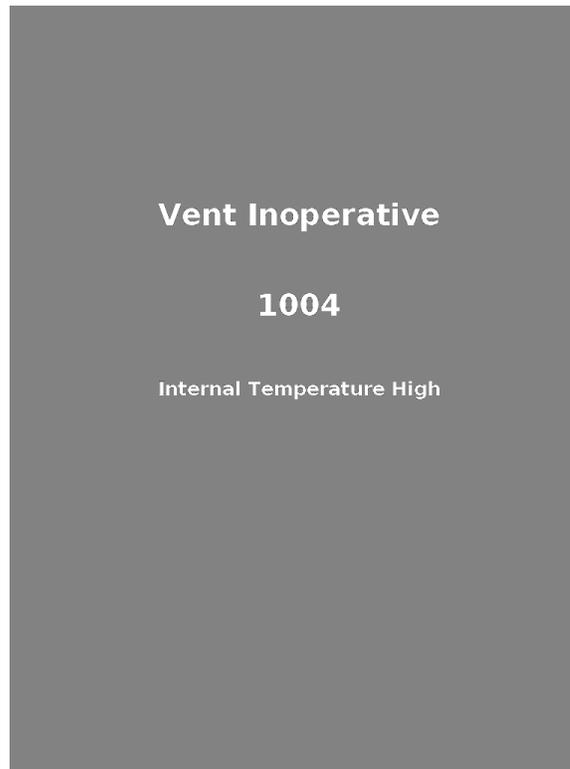

*Figure 9-2: Vent Inoperative screen*

### Setting alarm loudness

You can set the alarm loudness from the **Menu** window (see “Loudness” on page 6-17).

### Silencing alarms

Silence an alarm for 2 minutes by selecting the **Alarm Silence** button.



The button icon is replaced by this one. A timer shows time remaining in the 2-minute alarm silence period.



Select **Alarm Silence** again at any time to reset the counter to 2:00 minutes. During patient maneuvers, you can pre-silence audible alarms as desired.

Some alarms cannot be silenced; these are listed in Table 9-3. When a non-silenceable alarm is annunciated, the following is shown.



### Resetting alarms

Most alarms reset themselves (autoreset) when the alarm triggering condition is removed, but you must manually reset others. Table 9-3 specifies whether an alarm is autoreset.

#### Manually resetting alarms

Manually reset an alarm by selecting **Alarm Reset**.



When an alarm is manually reset, the message is cleared from the **Alarms** list, any other alarm indications are removed, and the alarm silence is terminated.

If the alarm cannot be manually reset, you see the following:



#### Clearing autoreset alarms from the Alarms list

Autoreset alarms are shown with text crossed out in the **Alarms** list.



Clear the message from the **Alarms** list by selecting **Alarm Reset**.

## Alarms, messages, and troubleshooting

### Hiding/displaying alarm messages

To hide an alarm or informational message in the **Alarms** or **Messages** list, touch the flashing alarm indicator button or informational message button when up arrows are present. To display messages, touch the flashing alarm indicator or Informational Message button when down arrows are present. Both active and autoreset alarms and informational messages are displayed and hidden.



### Symptom-based troubleshooting

To troubleshoot a problem with the ventilator, see Table 9-2. If an alarm message is displayed, see also Table 9-3.

Table 9-2: Symptom-based troubleshooting

Symptom	Recommended Action
Ventilation stops, but the ventilator has power. The ventilator may be in standby or in the <b>Vent Inoperative</b> state. When the ventilator is in the <b>Vent Inoperative</b> state, continued safe ventilator operation may be in jeopardy. O <sub>2</sub> flow and blower operation are disabled.	<p>Check the patient.</p> <p>If the ventilator is in standby, reconnect the patient, and ventilation should resume.</p> <p>If the Alarm LED on the front panel is continuously lit, and/or the <b>Vent Inoperative</b> screen is displayed, provide alternative ventilation. Have the ventilator serviced. For details about the specific <b>Vent Inoperative</b> alarm, see Table 9-5 on page 9-16.</p>
The touchscreen is unresponsive or does not respond properly. The ventilator continues to function at the selected settings. Patient settings and patient data continue to be visible and accurate, and alarms continue to be annunciated.	<p>Check the patient.</p> <p>If the patient requires a setting change, provide alternative ventilation. Have the ventilator serviced.</p> <p>NOTE: If you cannot shut off the ventilator when the touchscreen is unresponsive, press the ON/Shutdown key, then the Accept (✓) button on the top-right front of the ventilator.</p>
The ventilator does not switch from battery to AC power.	<p>Check the patient.</p> <p>Make sure the ventilator is connected to AC power. (If the ventilator is connected to AC power and the ventilator is functioning correctly, the ON/Shutdown LED should be on.)</p> <p>If problem persists, provide alternative ventilation. Have the ventilator serviced.</p>
The ventilator does not switch from AC power to battery. A back-up alarm should annunciate.	<p>Check the patient.</p> <p>If problem persists, provide alternative ventilation. Have the ventilator serviced.</p>
The estimated exhaled tidal volume reading (V <sub>T</sub> ) is inaccurate.	<p>Check the patient.</p> <p>Check for large leaks.</p> <p>Make sure ventilator and alarm settings are appropriate.</p> <p>Make sure an approved patient circuit is in use.</p> <p>Adjust the mask to ensure proper fit and adequate leak compensation.</p> <p>If the problem persists, provide alternative ventilation. Have the ventilator serviced.</p>

### Alarms and other messages

Table 9-3 is a list of alarms and other messages displayed by the ventilator, along with descriptions, suggested corrective actions, and other information. The ID (identifier) listed with the priority type is the priority number of the alarm. This priority number determines the order of alarm message display. Unless otherwise indicated, alarms listed as autoresettable are reset when the alarm condition is removed.

Table 9-3: Alarm and other messages: summary and troubleshooting

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autoresettable	Silence able
AVAPS: Target V <sub>T</sub> Exceeded. Min Pressure Too High	AVAPS target pressure is less than Min P setting. The ventilator limits its applied pressure to Min P.	Check the patient. Confirm pressure settings are compatible with target. Evaluate pressure and volume settings.	Information (63)	No	Yes	N/A
AVAPS: Target V <sub>T</sub> Not Achieved. Insufficient Max Pressure	AVAPS target pressure exceeds Max P setting. The ventilator limits applied pressure to Max P.	Check the patient. Confirm pressure settings are compatible with target. Evaluate pressure and volume settings.	Information (62)	No	Yes	N/A
Bacteria filter must be installed onto gas outlet	An inspiratory bacteria filter must be installed on the patient gas outlet port.	Confirm that a bacteria filter is installed. Install a bacteria filter if one is not present.	Information (67)	Yes	N/A	N/A
Cannot Reach Target Flow	Displays when HFT (high flow therapy) is active. Indicates that flow target is not achieved.	Check the patient. Check that the high flow nasal cannula size is appropriate for the flow setting. Check that an occlusive interface is NOT in use (a cannula fully sealed within the nares, an NIV mask or direct connection to an ETT/trach). Check for an occlusion, kink or liquid in the patient circuit.	Low (53)	No	Yes	Yes
Check Vent: <i>description of failure</i>	See Table 9-4 on page 9-12					
High Inspiratory Pressure	Measured inspiratory pressure is greater than the HIP setting, and the ventilator cycles into exhalation. Autoresets after a complete inspiration without the alarm condition.	Check the patient. Confirm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator serviced.	High (45)	Yes	Yes	Yes
High O <sub>2</sub> Supply Pressure	O <sub>2</sub> inlet pressure is greater than 92 psig, so O <sub>2</sub> enrichment ends. Autoresets when O <sub>2</sub> supply pressure falls below 87 psig.	Check the patient. If problem persists, provide alternative ventilation. Have the ventilator serviced.	High (49)	No	Yes	Yes

## Alarms, messages, and troubleshooting

Table 9-3: Alarm and other messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autoresetable	Silence able
High Rate	Measured respiratory rate is greater than the Hi Rate setting. Escalates to a high-priority alarm if the alarm condition persists for more than 60 sec.	Check the patient. Confirm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator serviced.	Low/High (59)	Yes	Yes	Yes
High Tidal Volume	Measured estimated tidal volume is greater than the Hi V <sub>T</sub> setting. Escalates to a high-priority alarm if the alarm condition persists for more than 60 sec.	Check the patient. Check for large leaks. Confirm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator serviced.	Low/High (58)	Yes	Yes	Yes
Low Inspiratory Pressure	Measured inspiratory pressure is less than the LIP setting.	Check the patient. Confirm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator serviced.	High (46)	Yes	Yes	Yes
Low Internal Battery	Battery can provide operating power for only an additional 15 minutes under nominal conditions. Autoresets when ventilator is connected to AC power.	Connect ventilator to AC power. Provide alternative ventilation.	High (43)	No	Yes	No
Low Leak–CO <sub>2</sub> Rebreathing Risk	Estimated volume of exhaled gas returned to the patient is high.	Check the patient, as possibility of CO <sub>2</sub> rebreathing could pose a potential problem. Check the exhalation port for occlusions. Check for appropriate patient interface and exhalation port settings.  If the approved exhalation port is unobstructed, mask and port settings are appropriate, and problem persists, increase the ventilator baseline flow by adding leak or increasing EPAP, if possible.  Switch to an approved nebulizer.	High (42)	Yes	Yes	Yes

## Alarms, messages, and troubleshooting

Table 9-3: Alarm and other messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Low Minute Ventila- tion	Estimated minute ventila- tion is less than the Lo $\dot{V}_E$ setting. Escalates to a high-priority alarm if the alarm condition persists for more than 60 sec.	Check the patient. Con- firm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator ser- viced.	Low (56)	Yes	Yes	Yes
Low O <sub>2</sub> Supply Pres- sure	O <sub>2</sub> inlet pressure is less than 30 psig and deliv- ered O <sub>2</sub> is at least 5% lower than O <sub>2</sub> setting. The ventilator continues to de- liver as much O <sub>2</sub> support as possible, but it ends O <sub>2</sub> support when the O <sub>2</sub> inlet pressure drops to less than 18 psig. Autoresets when O <sub>2</sub> inlet pressure ex- ceeds 23 psig.	Check the patient. Attach to oxygen source with suf- ficient pressure.  If problem persists, pro- vide alternative ventila- tion. Have the ventilator serviced.	High (48)	No	Yes	Yes
Low Rate	A low-priority alarm if the measured respiratory rate is less than the Lo Rate setting, escalating to a high-priority alarm in 60 sec.  A high-priority alarm from the start if: <ul style="list-style-type: none"> <li>• The Lo Rate setting is ≤ 4 BPM and there are no breaths for &gt; 60/Lo Rate setting.</li> <li>• The Lo Rate setting is &gt; 4 BPM and there are no breaths for &gt; 15 sec.</li> </ul>	Check the patient. Con- firm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator ser- viced.	Low/ High (55)	Yes	Yes	Yes
Low Tidal Volume	Estimated tidal volume is less than the Lo V <sub>T</sub> set- ting. Escalates to a high- priority alarm if the alarm condition persists for more than 60 sec.	Check the patient. Con- firm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator ser- viced.	Low (57)	Yes	Yes	Yes
Mask:x, Exh Port:y Use Menu to change	Displays when ventilator is turned on. Displays select- ed mask type and exhala- tion port.	Select mask and port from Menu tab. Message is re- moved when user con- firms selections, or after 5 minutes.	Informa- tion (64)	No	Yes	N/A

## Alarms, messages, and troubleshooting

Table 9-3: Alarm and other messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores- ettable	Silence- able
Oxygen Not Available	Oxygen supply pressure out of range, oxygen device failed, air flow sensor and/or oxygen flow sensor calibration failed, or oxygen inlet pressure sensor calibration failed. The ventilator discontinues oxygen support.	Check the patient. Check if high/low O <sub>2</sub> source is the problem and correct. If problem persists, provide alternative ventilation. Have the ventilator serviced.	High (47)	No	Yes	Yes
Patient alarms are disabled during HFT	Displays when HFT (high flow therapy) is active. Patient alarms are not available in this therapy.	Manually reset to confirm and clear the audible alarm.	Low/ Infor- mation (54)	Yes	No	Yes
Patient Circuit Occluded	Proximal pressure and patient flow are low. Patient circuit occluded.	Check the patient. Check the patient circuit for bulk liquid, crimps, or blocked filter. Confirm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator serviced.	High (40)	Yes	Yes	Yes
Patient Circuit Occluded	Displays when HFT (high flow therapy) is active. Gas flow to the patient is obstructed.	Check the patient. Check that an occlusive interface is NOT in use (a cannula fully sealed within the nares, an NIV mask or direct connect to an ETT/trach). Check for an occlusion, kink or liquid in the patient circuit. If problem persists, provide alternative ventilation.	High (52)	No	Yes	Yes
Patient Disconnect	Patient is no longer connected to the ventilator, either through circuit or mask, or the patient circuit is disconnected from the ventilator and the patient is no longer receiving ventilatory support. Ventilation continues.  NOTE: The Patient Disconnect alarm is triggered when 11 seconds have elapsed.	Check the patient. Reconnect patient circuit. Confirm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator serviced.	High (39)	Yes	Yes	Yes

## Alarms, messages, and troubleshooting

Table 9-3: Alarm and other messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Power has been re-stored	Power is restored follow- ing loss of power. The ven- tilator restarts and continues ventilation in the mode set before power was lost.	Check the patient. Con- firm ventilator and alarm settings are appropriate.	Inform- ation (66)	Yes	N/A	N/A
PPV Maximum Pres- sure	Computed target pressure is greater than the PPV maximum pressure alarm limit. Possible causes are excessive patient inspira- tory effort; a significant change in the leak around the patient interface; or high <b>PPV %</b> , <b>Max E</b> , or <b>Max R</b> setting. Target pressure is limited.  At first, an information message. If condition per- sists for three consecu- tive PPV inspirations, this escalates to a high-priority alarm.	Check the patient. Con- firm ventilator and alarm settings are appropriate. Check for circuit or mask leaks. If problem persists, provide alternative ventila- tion. Have the ventilator serviced.	Inform- ation/ High (51)	Yes	Yes	Yes
PPV Maximum Vol- ume	Estimated delivered pa- tient tidal volume is great- er than the PPV maximum volume alarm limit. Possi- ble causes are excessive patient inspiratory effort; a significant change in the leak around the patient in- terface; or high <b>PPV %</b> , <b>Max E</b> , or <b>Max R</b> setting. Ventilator cycles to exha- lation.  At first, an information message. If condition per- sists for three consecu- tive PPV inspirations, this escalates to a high-priority alarm.	Check the patient. Con- firm ventilator and alarm settings are appropriate. Check for circuit or mask leaks. If problem persists, provide alternative ventila- tion. Have the ventilator serviced.	Inform- ation/ High (50)	Yes	Yes	Yes
Pressure Regulation High	Pressures exceed ventila- tor-defined thresholds for IPAP, EPAP, or both. Ven- tilation continues. Autore- sets when alarm condition removed; otherwise, tran- sitions to the ventilator in- operative state if pressure continues to rise.	Check the patient. Con- firm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator ser- viced.	High (44)	Yes	Yes	Yes

## Alarms, messages, and troubleshooting

Table 9-3: Alarm and other messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Proximal Pressure Line Disconnect	Proximal pressure low for a few seconds. Proximal pressure line is disconnected. Air flow to the patient continues.	Check the patient. Reconnect proximal pressure line. Confirm ventilator and alarm settings are appropriate. If problem persists, provide alternative ventilation. Have the ventilator serviced.	High (41)	Yes	Yes	Yes
Running on Internal Battery	System is powered by the internal battery. Autore-sets when ventilator is connected to AC power.	Connect ventilator to AC power.	Low (60)	Yes	Yes	Yes
Trigger:+x, E-Cycle: +x Use Menu to change	Auto-Trak+ is active and using the displayed settings. This messages is displayed for 5 min after start-up.	Confirm that Auto-Trak+ settings are appropriate.	Infor- mation (65)	Yes	Yes	N/A
Using Default Set- tings	Displayed after power on if setting values are corrupted or not set, or if default values were restored by the user.	Check the patient. Check and adjust settings as re- quired.	Infor- mation (61)	Yes	Yes	N/A
Vent Inoperative x description of failure	See Table 9-5 on page 9-16					

Table 9-4: Check Vent alarm messages: summary and troubleshooting

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Check Vent: 1.8 V Supply Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (21)	Yes	No	No
Check Vent: 3.3 V Supply Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (22)	Yes	No	No
Check Vent: 5 V Sup- ply Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (23)	Yes	No	No
Check Vent: 12 V Supply Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (24)	Yes	No	No
Check Vent: 24 V Supply Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (25)	Yes	No	No

## Alarms, messages, and troubleshooting

Table 9-4: Check Vent alarm messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Check Vent: 35 V Supply Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (26)	Yes	No	No
Check Vent: Air Flow Sensor Calibration Data Error	Flow-related patient data is disabled. O <sub>2</sub> concentration switches to 21% (ventilates with air only). Default volume used in AVAPS mode. Standby disabled. Volume, leak, disconnect, and occlusion alarms compromised.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (13)	Yes	No	No
Check Vent: Alarm LED Failed	Technical failure.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (6)	Yes	No	No
Check Vent: Aux Supply Failed	Backup alarm problem	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (20)	Yes	No	No
Check Vent: Backup Alarm Failed	Backup alarm problem	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (5)	Yes	No	No
Check Vent: Barometer Calibration Data Error	Default barometric pressure of 686.0 mmHg (approximately 900 m/2953 ft above sea level) used in calculations	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (18)	Yes	No	No
Check Vent: Barometer Sensor Range Error	Default barometric pressure of 686.0 mmHg (approximately 900 m/2953 ft above sea level) used in calculations	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (19)	Yes	No	No
Check Vent: Battery Failed	Battery problem	Check the patient. Connect the ventilator to AC. Provide alternative ventilation. Have the ventilator serviced.	High (35)	Yes	No	No
Check Vent: Battery Temperature High	Battery problem	Check the patient. Connect the ventilator to AC. Check for causes of overheating, such as high room temperature, blocked vents, clogged air inlet filter, or nonfunctional fan. Provide alternative ventilation. Have the ventilator serviced.	High (34)	Yes	No	No

## Alarms, messages, and troubleshooting

Table 9-4: Check Vent alarm messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autoresetable	Silenceable
Check Vent: Blower Stalled	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (2)	Yes	No	No
Check Vent: Blower Temperature High	Technical failure	Check the patient. Check for causes of overheating, such as high room temperature, blocked vents, clogged air inlet filter, or nonfunctional fan. Provide alternative ventilation. Have the ventilator serviced.	High (33)	Yes	No	No
Check Vent: Cooling Fan Speed Error	Overheating of ventilator possible	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (36)	Yes	No	No
Check Vent: CPU PCBA ADC Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (29)	Yes	No	No
Check Vent: Data Acquisition PCBA ADC Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (27)	Yes	No	No
Check Vent: Flash File System Error	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (37)	Yes	No	No
Check Vent: Internal Temperature High CPU	Technical failure	Check the patient. Check for causes of overheating, such as high room temperature, blocked vents, clogged air inlet filter, or nonfunctional fan. Provide alternative ventilation. Have the ventilator serviced.	High (30)	Yes	No	No
Check Vent: Internal Temperature High Daq	Technical failure	Check the patient. Check for causes of overheating, such as high room temperature, blocked vents, clogged air inlet filter, or nonfunctional fan. Provide alternative ventilation. Have the ventilator serviced.	High (31)	Yes	No	No
Check Vent: Internal Temperature High Mtr	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (32)	Yes	No	No
Check Vent: Machine Pressure Sensor Auto-Zero Failed	Proximal pressure is not measured. Pressure-related alarms are compromised.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (9)	Yes	No	No

## Alarms, messages, and troubleshooting

Table 9-4: Check Vent alarm messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Check Vent: Machine Pressure Sensor Calibration Data Error	Proximal pressure is not measured. Pressure-related alarms are compromised.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (7)	Yes	No	No
Check Vent: Machine Pressure Sensor Range Error	Proximal pressure is not measured. Pressure-related alarms are compromised.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (11)	Yes	No	No
Check Vent: Motor Control PCBA ADC Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (28)	Yes	No	No
Check Vent: O <sub>2</sub> Flow Sensor Calibration Data Error	Continues to ventilate with air only	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (14)	Yes	No	No
Check Vent: O <sub>2</sub> Pressure Sensor Calibration Data Error	Continues to ventilate with air only	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (15)	Yes	No	No
Check Vent: O <sub>2</sub> Supply Pressure Sensor Range Error	Continues to ventilate with air only	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (17)	Yes	No	No
Check Vent: OVP Circuit Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (38)	Yes	No	No
Check Vent: Oxygen Device Failed	Continues to ventilate with air only	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (16)	Yes	No	No
Check Vent: Primary Alarm Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (4)	Yes	No	No
Check Vent: Program CRC Test Failed	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (1)	Yes	No	No
Check Vent: Proximal Pressure Sensor Auto-Zero Failed	Proximal pressure is not measured. Pressure-related alarms are compromised.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (10)	Yes	No	No
Check Vent: Proximal Pressure Sensor Calibration Data Error	Proximal pressure is not measured. Pressure-related alarms are compromised.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (8)	Yes	No	No

## Alarms, messages, and troubleshooting

Table 9-4: Check Vent alarm messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Check Vent: Proximal Pressure Sensor Range Error	Proximal pressure is not measured. Pressure-related alarms are compromised. Also displays when HFT (high flow therapy) is active and proximal pressure is out of range.	Check the patient. Provide alternative ventilation. Have the ventilator serviced. When using a high flow nasal cannula with FEP Connect, the proximal pressure line should be disconnected from the ventilator port during HFT. If this message is seen when switching the patient from ventilation to HFT, check to make sure the proximal pressure line is disconnected.	High (12)	Yes	No	No
Check Vent: Ventilator Restarted	Technical failure	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (3)	Yes	No	No

Table 9-5: Vent Inoperative alarm messages: summary and troubleshooting

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Vent Inoperative 1000 3.3 V Supply Failed	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (2)	No	No	No
Vent Inoperative 1001 12 V Supply Failed	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (3)	No	No	No
Vent Inoperative 1002 Blower Temperature Too High	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (4)	No	No	No
Vent Inoperative 1003 Internal Temperature High	Technical failure of the CPU PCBA. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (5)	No	No	No
Vent Inoperative 1004 Internal Temperature High	Technical failure of the DAQ PCBA. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (6)	No	No	No

## Alarms, messages, and troubleshooting

Table 9-5: Vent Inoperative alarm messages: summary and troubleshooting (continued)

Message	Description	Corrective Action	Priority type (ID)	Manually resettable	Autores ettable	Silence able
Vent Inoperative 1005 Internal Temperature High	Technical failure of the motor PCBA. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (7)	No	No	No
Vent Inoperative 1006 Data Acquisition PCBA ADC Failed	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (8)	No	No	No
Vent Inoperative 1007 Machine and Proximal Pressure Sensors Failed	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (9)	No	No	No
Vent Inoperative 1008 Machine and Proximal Pressure Sensors Failed	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (10)	No	No	No
Vent Inoperative 1009 Pressure Regulation High	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (11)	No	No	No
Vent Inoperative 100A Data Acquisition PCBA ADC Reference Failed	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (12)	No	No	No
Vent Inoperative 100B Watchdog Test Failed	Technical failure. The ventilator is in the ventilator inoperative state.	Check the patient. Provide alternative ventilation. Have the ventilator serviced.	High (13)	No	No	No

## Alarms, messages, and troubleshooting

## Chapter 10. Care and maintenance

---

---

**WARNING:** To reduce the risk of electric shock, power down the ventilator and disconnect it from AC power before cleaning, disinfecting, or servicing it.

**WARNING:** This product consists of devices that may contain mercury, which must be recycled or disposed of in accordance with local, state, or federal laws. (Within this system, the backlight lamps in the monitor display contain mercury.)

**WARNING:** Before performing maintenance procedures, disconnect the ventilator from the patient, shut it down, and disconnect it from AC power. All operator maintenance must be performed with the patient off the ventilator. Failure to do so can result in electric shock to the patient and operator.

**NOTE:** It is the user's responsibility to comply with the information provided in this chapter.

**NOTE:** Cleaning and disinfection are most effective if soiling is not allowed to dry on a medical device.<sup>1</sup>

**NOTE:** Disinfection is most effective on medical devices that were previously cleaned.<sup>1</sup>

**NOTE:** For all V60/V60 Plus hardware accessories recommended by Philips, follow the cleaning and disinfection guidelines in this chapter. For multi-patient interface and circuit accessories, consult the product instructions for use. For single patient use accessories, no cleaning and disinfection is needed.

---

To ensure the safety and reliability of your ventilator, follow these maintenance procedures along with your own institutional policies for cleaning, disinfecting, and maintaining equipment. All the procedures in this manual are intended to be performed by the operator. For further maintenance, contact your service representative.

---

1. Reprocessing Medical Devices in Health Care Settings: Validation Methods and Labeling Guidance for Industry and Food and Drug Administration Staff, 2015. Food and Drug Administration (FDA)

## Care and maintenance

### Exterior and touchscreen cleaning

---

**CAUTION:** To prevent possible damage to the ventilator, use only those cleaning and disinfecting agents listed in this manual.

**CAUTION:** To prevent possible damage to the ventilator, do not drip or spray any liquids directly onto any surface including the front panel, touchscreen, navigation ring (legacy versions), and Accept button.

**CAUTION:** Never clean or disinfect the touchscreen with an abrasive brush or device, since this will cause irreparable damage.

**CAUTION:** Do not attempt to sterilize or autoclave the ventilator.

**NOTE:** Use of unapproved cleaning and disinfecting agents may cause damage to the enclosure, touchscreen, or parts of the ventilator.

---

#### Approved cleaning agents

The following cleaning agent is acceptable for use on the touchscreen and exterior surfaces of the ventilator:

- Medivators Intercept Detergent, per manufacturer's recommendation at 1/3 oz (10 mL) per gallon of warm tap water.

#### Cleaning instructions

1. Apply cleaning agent to a soft lint-free cloth or use a disposable wipe. The cloth or wipe should be saturated but not dripping.
2. Wipe cleaning agent over the entire exterior surface and touchscreen of the ventilator.
3. Continue wiping until all visible contaminants and soiling are removed.
4. Rinse with a clean, water-dampened cloth and allow to dry completely before reuse.

**Exterior and touchscreen disinfection**

- 
- CAUTION: To prevent possible damage to the ventilator, use only those cleaning and disinfecting agents listed in this manual.
- CAUTION: To prevent possible damage to the ventilator, do not drip or spray any liquids directly onto any surface including the front panel, touchscreen, navigation ring (legacy versions), and Accept button.
- CAUTION: Never clean or disinfect the touchscreen with an abrasive brush or device, since this will cause irreparable damage.
- CAUTION: Do not attempt to sterilize or autoclave the ventilator.
- NOTE: Use of unapproved cleaning and disinfecting agents may cause damage to the enclosure, touchscreen, or parts of the ventilator.
- 

**Approved disinfecting agents**

The following disinfecting agents are acceptable for use on the touchscreen and exterior surfaces of the ventilator:

*Table 10-1: Exterior disinfection*

Disinfectant
Solution of 1 part 5% sodium hypochlorite (bleach) diluted in 9 parts deionized water.
3% hydrogen peroxide

**Disinfection instructions**

1. Ensure that cleaning was done per the procedure in “Exterior and touchscreen cleaning” on page 10-2.
2. Apply disinfecting agent to a soft lint-free cloth or use a disposable wipe. The cloth or wipe should be saturated but not dripping.
3. Wipe disinfecting agent over the entire exterior surface of the ventilator.
4. Allow disinfectant to remain on the surface for the following contact times:
  - bleach solution — 2 minutes
  - hydrogen peroxide — 15 minutes
5. Rinse with a clean cloth dampened with water and allow to dry completely before reuse.

## Care and maintenance

### Bacteria filter, patient circuit, and other accessories

Follow the manufacturer's instructions that accompany the accessory.

---

- WARNING:** To prevent patient or ventilator contamination, inspect and replace the main flow bacteria filter between patients and at regular intervals (or as stated by the manufacturer).
- WARNING:** To prevent possible patient injury, inspect and verify the proper operation of the exhalation port regularly during use.
- CAUTION:** Because some environments cause a quicker collection of lint and dust than others, inspect the filters more often when needed. The air inlet filter should be replaced; the cooling fan filter should be cleaned.
- CAUTION:** To ensure proper system performance, use a Respiration-approved air inlet filter.

**Preventive maintenance**

**WARNING:** Only authorized service personnel should replace parts within the ventilator or perform other service activities. Unauthorized personnel without proper training are at risk of electric shock.

**WARNING:** Turn off the ventilator and disconnect it from the AC mains outlet before you perform decontamination or maintenance procedures. Failure to do so may result in electric shock.

The expected service life of your V60/V60 Plus ventilator is 10 years. Perform preventive maintenance on the ventilator according to the schedule in Table 10-2. You can view the hours of ventilator operation in the Vent Info window (“Vent Info (ventilator information)” on page 6-18). The following subsections provide details for some of these preventive maintenance procedures.

*Table 10-2: Schedule of preventive maintenance*

Frequency	Component	Maintenance
Between patients and per institutional guidelines	Patient circuit	Per manufacturer recommendations.
	Main flow bacteria filter	Replace per institutional guidelines.
Every month	Cooling fan filter	Inspect for occlusions, dust, lint, etc. If discolored or dirty, remove and wash or rinse thoroughly, and let dry completely before reinstalling.
	Air inlet filter	Inspect and replace if needed
Every year	Backup battery	Inspect, test, and replace if needed*
	Ventilator	Preventive maintenance*
As required	Backup battery	A new backup battery should be installed and charged within one year of the date of manufacture identified on the battery and on the shipping box.
Every 5 years	Backup battery	Replace.* Battery replacement is based on the date of manufacture recorded on the battery label. Also viewable in Diagnostic Mode on the system information screen.

\* Must be done by authorized service personnel according to the instructions in the service manual.

## Care and maintenance

### Replacing the air inlet filter

Replace the air inlet filter as follows, referring to Figure 10-1.

1. Power down the ventilator and disconnect it from AC power. Remove ventilator from cart, if applicable.
2. Turn the captive D-ring fastener counter-clockwise one-quarter turn and release. Remove the side panel.
3. Remove the inlet filter by pinching it out of the recess in the bracket.
4. Install a new air filter by tucking it into the recessed area. Replace the side panel, and push in and turn the D-ring fastener one-quarter turn until it locks.



Figure 10-1: Replacing the air inlet filter

### Cleaning or replacing the cooling fan filter

Clean or replace the cooling fan filter as follows, referring to Figure 10-2:

1. Insert a small, flat blade driver tip between the foam filter and the filter retaining cover (Figure 10-2).
2. Gently pry the filter cover from the back of the ventilator. Do not remove the fan retaining pins.
3. Wash or rinse the filter. Let it dry completely before reinstalling.
4. Replace the filter, then snap the filter cover into place.



Figure 10-2: Replacing the cooling fan filter

## Care and maintenance

### Removing and replacing the battery

See “Installing the battery” on page A-4.

## Disposal

---

**WARNING:** This product consists of devices that may contain mercury, which must be recycled or disposed of in accordance with local, state, or federal laws. (Within this system, the backlight lamps in the monitor display contain mercury.)

---

Dispose of all parts removed from the device according to your institution’s protocol. Follow all local, state, and federal regulations with respect to environmental protection, especially when disposing of the electronic device or parts of it (for example, oxygen cell, batteries).

Dispose of this device in accordance with local collections and recycling regulations. For more information, visit [www.philips.com/recycling](http://www.philips.com/recycling).

This device contains a lithium ion battery that cannot be recycled. Please contact customer service for safe disposal instructions of this part.

## Storage between patient use

Follow the steps below when storing the ventilator between patient use:

1. Ensure that the patient circuit is assembled and installed properly as described in “Installing the patient circuit” on page 5-3.
2. Plug the ventilator into an AC outlet and verify that the power source symbol is displayed.
3. Adjust settings to hospital defaults.
4. Check oxygen cylinder fill status (if applicable).
5. Ensure oxygen cylinders are turned off.
6. Ensure the environmental specifications are met. Refer to Table 11-8: “Environmental specifications” on page 11-6.

---

**CAUTION:** Avoid allowing the ventilator battery to become completely discharged. Otherwise, the battery may become over-discharged and require long recharge times of up to 16 hours or more. The over-discharged condition may permanently damage the battery so that it is unable to recharge. To prevent the occurrence of a non-recoverable over-discharged battery, always keep the ventilator connected to an AC outlet while in storage and schedule regular preventive maintenance to ensure battery health.

---

## Service and repairs

For technical service or repair information not included in this chapter, contact Philips.

A V60/V60 Plus service manual is available (order number 989805612651, part number 1049766), and can also be downloaded from [www.philips.com/ifu](http://www.philips.com/ifu). The service manual includes removal and installation procedures, parts lists, and testing and troubleshooting information.

## Repacking and shipping

---

**CAUTION:** To prevent possible damage to the ventilator, always ship it with the original packing material. If the original material is not available, contact Philips to order replacements.

**NOTE:** Transport of lithium ion batteries is strictly controlled by international regulations and laws. Do not ship the battery either in the ventilator or separately by sea or air.

---

Remove the battery from the ventilator before shipping the ventilator. See “Installing the battery” on page A-4 for more information. Ship the battery and ventilator separately in appropriate packaging in conformance with federal, state, and local regulations.

If necessary, Philips will send you a battery return kit that contains a prepaid postal label and instructions for returning the battery if it has failed prematurely (within 5 years of its manufacture date). The order number for the return kit is 989805663471 / part number 1146145.

## Care and maintenance

# Chapter 11. Technical specifications

## Control settings

Table 11-1 lists ventilator control setting ranges, resolutions, and accuracies. Table 11-2 lists the controls active in the different ventilation modes.

Table 11-1: Control setting ranges, resolutions, and accuracies

Parameter	Range	Resolution	Performance Accuracy	Factory default
<b>Mode settings</b>				
Modes	AVAPS, CPAP, S/T, PCV, PPV (optional)	N/A	N/A	S/T
<b>Control settings</b>				
C-Flex	OFF, 1 to 3	1	N/A	2
CPAP	4 to 25 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	± (2 cmH <sub>2</sub> O + 4% of target)	4 cmH <sub>2</sub> O
EPAP	4 to 25 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	± (2 cmH <sub>2</sub> O + 4% of target)	4 cmH <sub>2</sub> O
Flow (High flow therapy, 3.00 software and above, and V60 Plus)	10 to 80 L/min	5 L/min	N/A	35 L/min
IPAP	4 to 40 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	± (2 cmH <sub>2</sub> O + 4% of target)	12 cmH <sub>2</sub> O
I-Time (Inspiratory Time)	0.30 to 3.00 sec	0.05 sec	± 0.03 sec	1.00 sec
Max E	0 to 100 cmH <sub>2</sub> O/L	1 cmH <sub>2</sub> O/L	N/A	15 cmH <sub>2</sub> O/L
Max P (PPV Maximum Pressure Limit)	5 to 40 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	± (2 cmH <sub>2</sub> O + 4% of target)	20 cmH <sub>2</sub> O
Max P (AVAPS Maximum IPAP Pressure)	6 to 40 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	± (2 cmH <sub>2</sub> O + 4% of target)	25 cmH <sub>2</sub> O
Max R	0 to 50 cmH <sub>2</sub> O/L/sec	1 cmH <sub>2</sub> O/L/s	N/A	4 cmH <sub>2</sub> O/L/s
Max V (PPV Maximum Volume Limit)	200 to 3500 mL	5 mL	± 15%	1000 mL
Min P (AVAPS Minimum IPAP Pressure)	5 to 30 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	± (2 cmH <sub>2</sub> O + 4% of target)	10 cmH <sub>2</sub> O
O <sub>2</sub> (Oxygen)	21 to 100%	1%	± 5%	21%
PPV %	0 to 100%	1%	N/A	30%

## Technical specifications

Table 11-1: Control setting ranges, resolutions, and accuracies (continued)

Parameter	Range	Resolution	Performance Accuracy	Factory default
Ramp Time	OFF, 5 to 45 min	5 min	± 1 sec	OFF
Rate (Respiratory Rate)	4 to 60 BPM	1 BPM	± 1 BPM	4 BPM
Rise (Rise Time)	1 to 5	1	N/A	3
V <sub>T</sub> (AVAPS Target Tidal Volume)	200 to 2000 mL BTPS	5 mL	± 15%	500 mL

Table 11-2: Controls active in Respironics V60/V60 Plus ventilation modes

	CPAP	S/T	PCV	AVAPS	PPV
Timing		Rate			Rate*
		I-Time			I-Time*
Baseline pressure	CPAP	EPAP			
Inspiratory pressure		IPAP		Max P	Max P
				Min P	IPAP*
Rise Time		Rise			Rise*
O <sub>2</sub>	O <sub>2</sub>				
Volume				V <sub>T</sub>	Max V
Ramp feature	Ramp Time				
Mode-specific	C-Flex				PPV %
					Max E
					Max R

\* Used in backup only

Patient data

Table 11-3: Patient data ranges, resolutions, and accuracies during ventilation

Parameter	Range	Resolution	Accuracy
<b>Patient data window</b>			
<i>Breath phase/trigger indicator</i>	Spont, Timed, Exhale	Color-coded display: Spont - turquoise, Timed - orange, Exhale - blue	N/A
PIP	0 to 50 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	± 2 cmH <sub>2</sub> O
Pt. Leak	0 to 200 L/min BTPS	1 L/min	N/A
Pt. Trig	0 to 100%	1%	± 10%
Rate	0 to 90 BPM	1 BPM	± 1 BPM
T <sub>I</sub> /T <sub>TOT</sub>	0% to 91%	1%	± 5%
Tot. Leak	0 to 200 L/min BTPS	1 L/min	N/A
V <sub>E</sub>	0 to 99.0 L/min BTPS	0.1 L/min	± 15% or 0.3 L/min (whichever is greater)
V <sub>T</sub>	0 to 3500 mL BTPS	1 mL	± 15% for volumes above 200 mL Note: Accuracy specification was measured with patient circuit PN 582073
<b>Waveform window</b>			
<i>P waveform</i>	0 to 50 cmH <sub>2</sub> O	Time axis: 1 second	N/A
<i>V waveform</i>	-240 to 240 L/min BTPS	Time axis: 1 second	N/A
<i>V waveform</i>	0 to 3500 mL BTPS	Time axis: 1 second	N/A

## Technical specifications

### Alarms

Table 11-4 lists the adjustable alarm ranges and resolutions. Table 9-3 on page 9-7 describes other, nonadjustable alarms.

*Table 11-4: Adjustable alarm ranges and resolutions*

Parameter	Range	Resolution	Factory default
Hi Rate (High Rate Alarm)	5 to 90 BPM	1 BPM	30 BPM
Lo Rate (Low Rate Alarm)	1 to 89 BPM	1 BPM	10 BPM
Hi $V_T$ (High Tidal Volume Alarm)	200 to 3500 mL BTPS	5 mL	2500 mL
Lo $V_T$ (Low Tidal Volume Alarm)	OFF, 5 to 1500 mL BTPS	5 mL	OFF
HIP (High Inspiratory Pressure Alarm)	5 to 50 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	50 cmH <sub>2</sub> O
LIP (Low Inspiratory Pressure Alarm)	OFF, 1 to 40 cmH <sub>2</sub> O	1 cmH <sub>2</sub> O	OFF
Lo $\dot{V}_E$ (Low Minute Ventilation Alarm)	OFF, 0.1 to 99.0 L/min BTPS	0.1 L/min	OFF
LIP T (Low Inspiratory Pressure Delay Time Alarm)	5 to 60 sec	1 sec	20 secs

### Menu window settings

*Table 11-5: Menu window settings and ranges*

Parameter	Range
Brightness	1 to 5
Loudness	1 to 10
Mask/ET Selection	ET/Trach, 1, 2, 3, 4, Other
Exhalation Port Selection	<ul style="list-style-type: none"> <li>• FEP (Philips Respironics Filtered Exhalation Port)</li> <li>• DEP (Philips Respironics Disposable Exhalation Port)</li> <li>• Whisper Swivel (Philips Respironics Whisper Swivel),</li> <li>• PEV (Philips Respironics Plateau Exhalation Valve),</li> <li>• Other (Other Exhalation Port),</li> <li>• None (No circuit exhalation port)</li> </ul>
Screen Lock	Off, On
Auto-Trak+ (optional)	Trigger: Normal, +1 to +7. E-Cycle: -2 to -1, Normal, +1 to +6

**Diagnostic mode functions**

*Table 11-6: Diagnostic mode functions*

<b>Function</b>	<b>Range</b>
Language	English, Nederlands, Français, Deutsch, Italiano, Português, Español, Dansk, Suomi, Norsk, Svenska, Chinese, Japanese, Türkçe
Date/Time	--
Pressure Units	cmH <sub>2</sub> O, hPa
Restore Default Settings	--
Software Options	--
Baud Rate	9,600, 19,200, 115,200
Alarm Volume Escalation *	Enable, Disable (default)
Significant Event Log	
Touch Screen Calibration	

\* Available in Revision 2.30 software and above.

## Technical specifications

### Physical characteristics

Table 11-7: Physical characteristics

Parameter	Specification
Dimensions	<p>(33.7 cm) 13.3 in.</p> <p>(39.4 cm) 15.5 in.</p> <p>(42.9 cm) 16.5 in.</p>
Mass (weight) of the V60/V60 Plus ventilator in its most usual configuration	12 kg (26 lb) with battery
Installed weight (V60/V60 Plus ventilator on stand, including accessories as listed)	V60/V60 Plus Ventilator with backup battery, ventilator stand, O <sub>2</sub> tank holder, two E-cylinders (full), two gauge clusters, oxygen analyzer with mount, humidifier with chamber, water bag and pole, nebulizer, circuit arm with mount, patient circuit with mask, O <sub>2</sub> manifold with hoses, remote alarm cable, and power cord. Weight: 64 kg (142 lb)
Maximum load (Mass/weight of the V60/V60 Plus ventilator stand, including its safe working load)	70.5 kg (155 lb)

### Environmental specifications

Table 11-8: Environmental specifications

Parameter	Specification
Temperature	<i>Operating:</i> 5 to 40 °C (41 to 104 °F) <i>Storage/transport:</i> -20 to 50 °C (-4 to 122 °F)
Relative humidity	<i>Operating:</i> 15 to 95% (noncondensing) <i>Storage/transport:</i> 10 to 95% relative (noncondensing)
Barometric pressure	<i>Operating:</i> 600 mmHg to 765 mmHg (80 kPa to 102 kPa): approximately -61 m to 1951 m (-200 ft to 6400 ft) relative to sea level <i>Storage/transport:</i> 450 mmHg to 765 mmHg (60 kPa to 102 kPa): approximately -61 m to 4267 m (-200 ft to 14000 ft) relative to sea level

Pneumatic specifications

Table 11-9: Pneumatic specifications

Parameter	Specification
High-pressure oxygen supply	<p>Connector: DISS male, DISS female, NIST                      Pressure: 2.76 to 6.00 bar / 276 to 600 kPa / 40 to 87 psig                      Flow: 175 SLPM</p> <p>Connector: SIS                      Pressure: 3.31 to 6.00 bar / 331 to 600 kPa / 48 to 87 psig                      Flow: 175 SLPM</p>
High-pressure oxygen supply (using V60/V60 Plus manifold)	<p>Connector: DISS male, DISS female, NIST                      Pressure: 3.10 to 6.00 bar / 310 to 600 kPa / 45 to 87 psig                      Flow: 175 SLPM</p> <p>Connector: SIS                      Pressure: 3.66 to 6.00 bar / 366 to 600 kPa / 53 to 87 psig                      Flow: 175 SLPM</p>
Air supply	Integrated blower
Inspiratory outlet (to patient port)	Connector: ISO 15 mm female/22 mm male conical
Maximum limited pressure ( $P_{LIMmax}$ )	64 cmH <sub>2</sub> O
Maximum working pressure range ( $P_{Wmax}$ )	5 to 50 cmH <sub>2</sub> O, ensured by <b>High Inspiratory Pressure (HIP)</b> alarm limit
Subatmospheric pressures generated during exhalation	None

## Technical specifications

### Electrical specifications

Table 11-10: Electrical specifications

Parameter	Specification
AC voltage	100 to 240 VAC
AC frequency	50/60 Hz
AC power	300 VA
Battery	PN 1076374: 14.4 V, 11.0 Ah, 163 Wh  Maximum system current draw: 11 A Charge voltage: +16.9 V maximum Operating time: 360 minutes (6 hours) under normal conditions

### Accessory requirements

---

NOTE: Any part that is added to the patient circuit changes the compliance and resistance of the ventilator breathing system. To achieve the ventilator performance specified in this manual, the ventilator breathing system must meet the specifications listed in Table 11-11.

---

Table 11-11: Accessory Requirements

Parameter	Specification
Compliance	For volume accuracy requirements, the patient circuit compliance should be 0.98 mL/cmH <sub>2</sub> O. For all other performance requirements except volume accuracy, the maximum compliance of the circuit can be up to 2.8 mL/cmH <sub>2</sub> O.
Resistance	Maximum resistance of the breathing circuit and attachments: 2.9 cmH <sub>2</sub> O at 60 L/min.
Bacteria filter	Dead Space: 66 ml Bacterial/Viral Filter Efficiency: >99.99% Resistance (@ 0.5 L/s): 0.7 cmH <sub>2</sub> O (hPa)/L/s Male Connector: 15 mm I.D./22 mm O.D. Female Connector: 22 mm I.D.

Alarm-related specifications

Table 11-12: Alarm-related specifications

Parameter	Specification
Delay time from the onset of the alarm condition to the point that the representation of the alarm condition leaves the signal output port	< 10 ms from onset of the alarm condition to the transmission of the signal < 500 s until alarms received from the ventilator are displayed on the IntelliVue patient monitor. This delay is in addition to the alarm detection and processing delays of the external device.
Remote alarm delay time	The time it takes the message to appear on the remote alarm depends on the characteristics of the device.
Maximum time from the alarm condition triggering event to the generation of an alarm signal	All alarms except those listed below: < 10 s <ul style="list-style-type: none"> <li>• Patient disconnect alarm: 11 s</li> <li>• Proximal pressure line disconnect alarm: 15 s</li> <li>• Low Inspiratory Pressure (LIP) alarm: user-configurable from 5 to 60 s</li> <li>• Check vent: Blower temperature high: 10 minutes</li> </ul>
Mean delay, or the sum of mean delays, from the triggering event to the generation of an alarm signal	All alarms except those listed below: < 5 s <ul style="list-style-type: none"> <li>• Cooling fan failure alarm: 10 s</li> <li>• Blower temperature alarm: 10 s</li> <li>• ADC wraparound: 6 s</li> </ul>
Audio alarm loudness*	Highest volume setting: Average sound pressure level is approximately 76 dB(A) Lowest volume setting: Average sound pressure level is approximately 62 dB(A)
High-priority auditory alarm signal sound pressure level range, measured per ISO 3744:2010 ANSI/	Average sound power level is approximately 54 dB(A) measured at the ventilator Average sound pressure level is approximately 46 dB(A) measured 1 m from the ventilator

\* In accordance with 3rd Edition testing methods

Other specifications

Table 11-13: Other specifications

Parameter/Characteristic	Specification
Expected ventilator service life	10 years
Flow delivery	150 L/min at 40 cmH <sub>2</sub> O at 1951 m (6400 ft) altitude (10% degradation in flow at 2286 m (7500 ft))
Flow range	0 to 240 L/min BTPS

## Technical specifications

Table 11-13: Other specifications (continued)

Parameter/Characteristic	Specification
High flow therapy (3.00 software and above, and V60 Plus)	10 to 80 L/min BTPS  NOTE: The maximum deliverable flow rate varies based on the orifice size of the nasal cannula and on the patient's nasal passage resistance.
Pressure range	4 to 40 cmH <sub>2</sub> O
Dynamic pressure regulation	± (2 cmH <sub>2</sub> O + 4% of target)  NOTE: Negative (subatmospheric) pressure settings are not available.
Start-up time	Ready to ventilate 9 seconds after power on
Triggering, cycling, and leak tolerance	As per the Digital Auto-Trak Sensitivity algorithms (see "Auto-Trak Sensitivity" on page 4-3)
Inspiratory and expiratory pressure drop following equipment failure: measured at patient connection, when the recommended breathing system is in use.	< 2.0 cmH <sub>2</sub> O (at 60 LPM) < 1.0 cmH <sub>2</sub> O (at 30 LPM)
Maximum time required for the O <sub>2</sub> concentration in the delivered volume to change from a volume fraction of 21% to 90% using the worst-case ventilator breathing system or using the maximum internal volume ventilator breathing system.	The ventilator adjusts O <sub>2</sub> within one breath. FiO <sub>2</sub> within the gas delivery system and entire breathing circuit adjusts at the following rate: Up to 18 seconds for delivered volume of 500 mL Up to 17 seconds for delivered volume of 200 mL
The 10 s average input flow required by the ventilator for each gas at a pressure of 280 kPa	107 SLPM
The maximum transient input flow averaged for 3 s required by the ventilator for each gas at a pressure of 280 kPa	130 SLPM
Operational acoustics* ISO/FDIS 80601-2-12:2010 201.9.6.2.1.101	Average noise level is less than 45 dB(A) when measured 1 m from the ventilator

\* In accordance with 3rd Edition testing methods

## Appendix A. First-time installation

---

---

**WARNING:** Philips-authorized personnel must install the ventilator. For information about installation, contact your Philips representative.

---

Before putting the ventilator into service for the first time, install it as described in this chapter.

### Unpacking and inspection

Unpack the ventilator and inspect it for damage. Inspect the exterior cabinet of the ventilator for cracks, scratches, or blemishes. Inspect the front panel for scratches or abrasions. Correct and/or report any problems found to Philips before using the ventilator.

Before using the ventilator the first time, we recommend wiping the exterior clean and disinfecting components according to the instructions in Chapter 10.

## First-time installation

### Mounting the ventilator

---


**CAUTION:** To prevent possible damage to the ventilator, always secure it to its stand or securely place it on a flat, stable surface that is free of dirt and debris. Do not use the ventilator adjacent to, or stack it with, other equipment.

**NOTE:** If you mount the ventilator to a stand, make sure the stand is approved by Philips.

---

The ventilator may be mounted to the optional stand or placed on a flat, stable, clean surface. Figure A-2 shows the installed ventilator.

Align the ventilator above the stand as shown in Figure A-1. Make sure the four feet at the base of the ventilator are aligned to slide into the grooves at the top of the stand. Ensure that the locking lever on the back right of the stand locks into place.

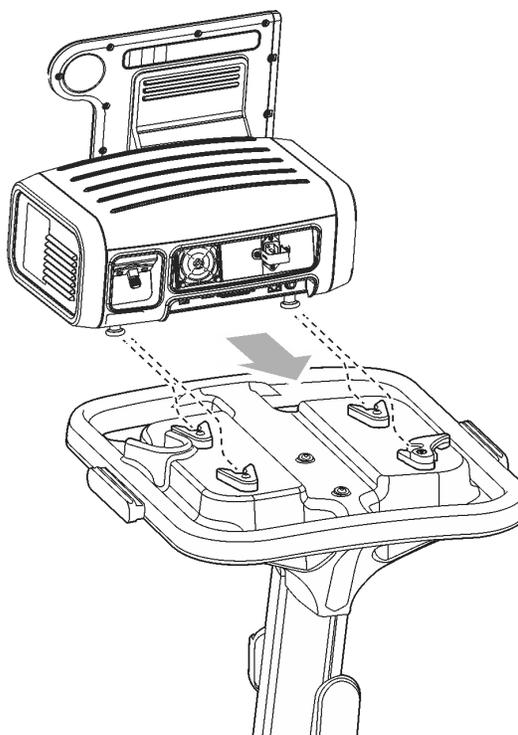

*Figure A-1: Mounting the V60/V60 Plus Ventilator on the stand*

## First-time installation

Figure A-2 shows the installed ventilator. Use the brakes to lock and unlock the wheels as needed. Make sure the wheels are unlocked before moving the ventilator.



*Figure A-2: Respirationics V60/V60 Plus Ventilator on stand*

## First-time installation

### Installing the battery

---

**WARNING:** Installation or replacement of lithium batteries by inadequately trained personnel could result in a hazard.

**WARNING:** To reduce the risk of fire, explosion, leakage, or other hazard, take these precautions with respect to the battery:

- Do not attempt to disassemble, open, drop, crush, bend or deform, insert foreign objects into, puncture, or shred the battery pack; modify or remanufacture it; immerse or expose it to water or other liquids; expose it to fire, excessive heat (including soldering irons); or put it in a microwave oven.
  - Replace the battery only with another battery specified by the manufacturer.
  - Follow all instructions for proper use of the battery.
  - Do not short-circuit the battery or allow metallic or conductive objects to contact the battery connector housing.
  - Use the battery with the Respiroics V60/V60 Plus Ventilator only.
- 

Install the battery as follows (Figure A-3).

1. Shut down and then unplug the ventilator.

---

**NOTE:** Failure to properly shut down the ventilator before battery installation may result in erroneous alarms after power-on.

---

2. Remove the side panel by turning the captive fastener a  $\frac{1}{4}$  turn and releasing.
3. Using a 3-mm hex wrench, remove the battery bracket by removing two screws.
4. Holding the battery so that the vent hole faces up and the Philips logo faces out, thread the battery cable through the battery bracket. Position and place the battery inside the battery compartment. Pinching the end of the battery connector, plug it in so that it locks in place.
5. Reinstall the battery bracket by replacing the two screws. Reinstall the side panel and secure the fastener with a  $\frac{1}{4}$  turn clockwise.
6. Make sure the battery is properly installed by plugging the ventilator into an AC power receptacle and verifying that the yellow Battery (charged) LED on the front panel flashes. The flashing LED indicates the battery is being charged.

7. Attach the BATT label as shown in Figure 3-5 on page 3-9.

---

**WARNING:** Never attempt to disconnect or connect the battery during operation.

**CAUTION:** Following battery installation, if a Check Vent or Vent Inoperative alarm occurs when verifying ventilator operation, discontinue use of the ventilator immediately and contact Philips. The Vent Inoperative alarm occurs if AC power is disconnected and a battery is not installed, or if the battery is fully discharged.

**NOTE:** A new battery must be charged for at least 5 hours before being placed into service. Based on the age and state of the battery, it may take up to 16 hours or more to fully charge the battery.

---

## First-time installation

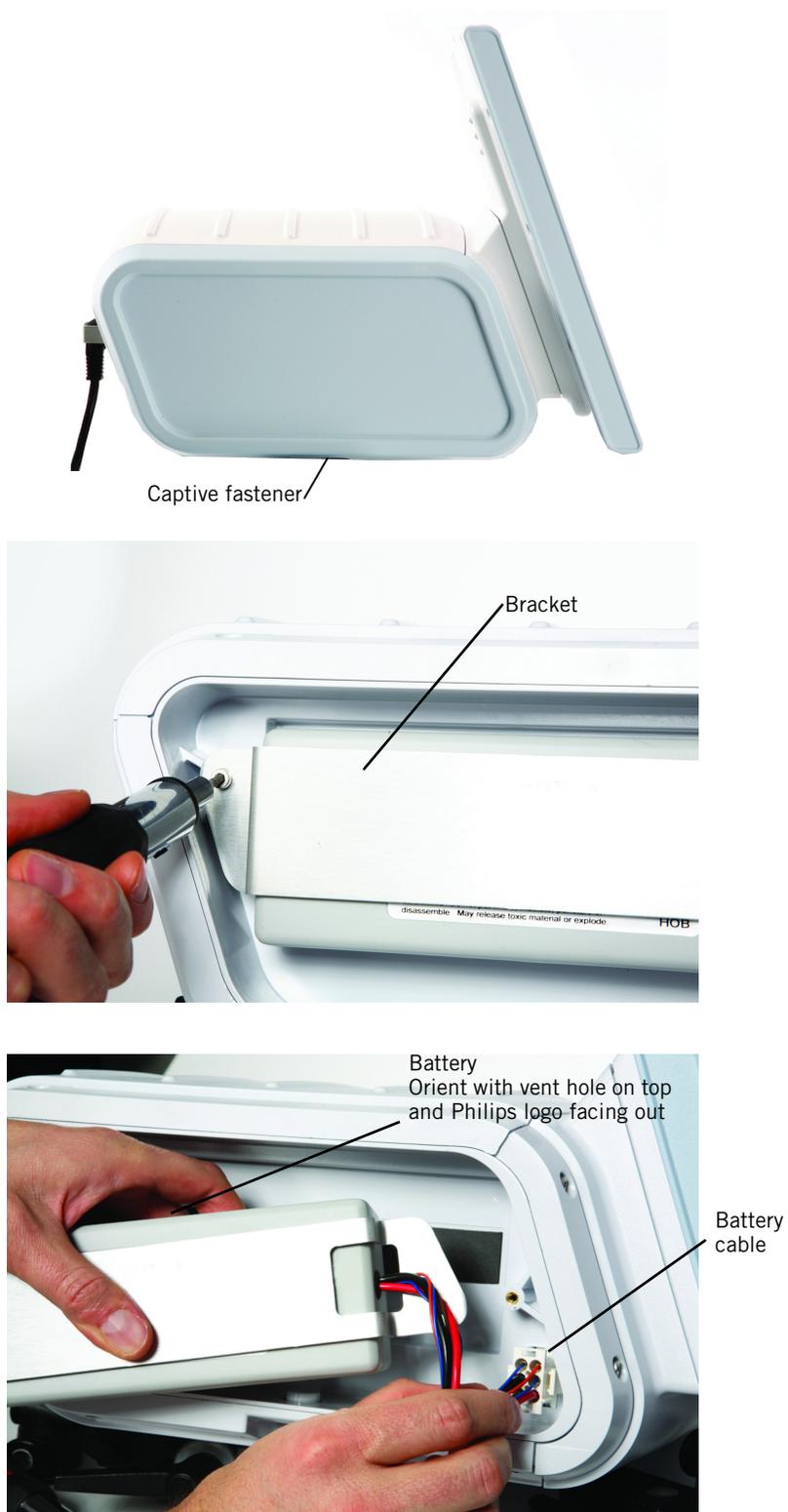


Figure A-3: Installing the battery

### Installing oxygen inlet connector and AC power cord

The Respironics V60/V60 Plus Ventilator destined for Japan, China, and the USA are typically pre-configured. V60/V60 Plus Ventilators shipped to other countries may require installation of the power cord and oxygen inlet connector.

1. Install the oxygen inlet connector as follows (Figure A-4):
  - a. Gently fit connector into the hole provided with flat sides to the left and right.
  - b. Install the oxygen inlet connector retaining plate. Tighten the two screws with a 2.5-mm hex wrench.



Figure A-4: Installing the oxygen inlet connector

## First-time installation

---

**WARNING:** To prevent unintentional disconnection of the power cord, always use the correct, Philips-supplied power cord and lock it into place with the power cord retainer before you switch the ventilator on. The retainer is designed to hold the connector end of the Philips-supplied cord securely in place.

**WARNING:** The V60/V60 Plus Ventilator should not be positioned in a way that makes it difficult to disconnect from mains power if necessary. Disconnect from supply mains by removing the power cord from the wall outlet. The AC mains plug is used as disconnection device.

---

2. Secure the power cord with the power cord retainer (Figure A-5):
  - a. Remove the power cord retainer by removing two screws.
  - b. Connect the power cord that is appropriate to your region into the AC power connector.
  - c. Reinstall the power cord retainer over the power cord, and tighten the screws with a 3.0-mm hex wrench.



Figure A-5: Installing the power cord retainer

### Installing the oxygen manifold kit

If desired, install the oxygen manifold kit as described in the accompanying instructions.

### Verifying ventilator operation and audible alarm

Perform the following steps to verify ventilator and audible alarm operation:

1. Assemble and install a patient circuit. (See Chapter 5 for instructions on installing a patient circuit.)
2. Power on the ventilator and verify that it completes the power-on self-test.
3. Disconnect the proximal pressure airway pressure line from the ventilator connector, and verify that the **Proximal Pressure Line Disconnect** alarm is annunciated (audio, visual, and flashing alarm LED).
4. Reconnect the proximal pressure line, and manually reset the alarm.
5. Turn the ventilator off.
6. Remove the patient circuit.

The ventilator is ready to be set up for use as described in Chapter 5.

### Configuration and screen calibration

After completing the setup activities described in Chapter 5, set or check the ventilator settings for language, units of measure, and time in the diagnostic mode (see Appendix E). Calibrate the screen as required, referring to Appendix E

## First-time installation

## Appendix B. Communications interface

---

- WARNING:** Connect to the ventilator only items that are specified as part of or compatible with the ventilator system. Additional equipment connected to medical electrical equipment must comply with the respective IEC or ISO standards. Furthermore, all configurations shall comply with the requirements for medical electrical systems (see IEC 60601-1-1 or clause 16 of edition 3 of IEC 60601-1, respectively). Anybody connecting additional equipment to medical electrical equipment configures a medical system and is therefore responsible for ensuring that the system complies with the requirements for medical electrical systems. Also be aware that local laws may take priority over the above mentioned requirements. If in doubt, consult Philips.
- WARNING:** The USB port is not currently available for use. DO NOT connect or attempt to power any equipment from the USB port.
- WARNING:** It is the responsibility of the end user to validate the compatibility and use of information transmitted from the ventilator to the device to be connected to the ventilator.
- WARNING:** The data provided through the communications interface is for reference only. Decisions for patient care should be based on the clinician's observations of the patient.
- 

The ventilator provides the following communications interface ports (Figure B-1):

- RS-232 serial port. Through this port the ventilator receives commands from a host computer or bedside monitoring system and responds with fixed-format records. The port is also used for ventilator servicing and software downloading.
- Remote alarm/nurse call port. Used to activate alarms remotely.
- USB and RJ-45 ethernet ports (not currently used)

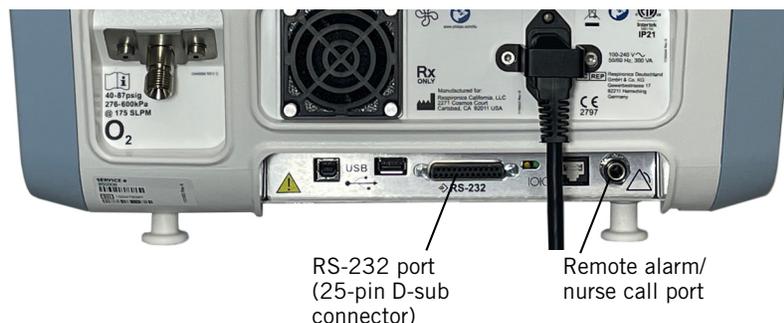


Figure B-1: Location of communications interface ports

## Communications interface

### RS-232 serial port

The 25-pin D-sub RS-232 connector on the rear panel permits the ventilator to export parameters to a patient monitor or a hospital information system.

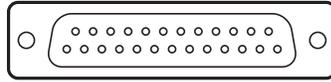


Figure B-2: RS-232 serial connector pinout

#### Supported communication protocols

The ventilator supports the following communication protocols:

- Philips IntelliBridge (VueLink): Use for connecting to Philips monitors (see “Using Philips IntelliBridge or VueLink” on page B-2).
- PVOI (Philips Ventilator Open Interface): Use for interfacing with non-Phillips monitors or devices. Contact your Philips representative to obtain the *PVOI Developer’s Guide*.
- Legacy (VRPT/SNDA): Fixed text-based protocol (not to be used for developing new driver interfaces). For this protocol specification, refer to the *V60 Communications interface VRPT / SNDA Developers Guide* (1147828MC), available at [www.philips.com/IFU](http://www.philips.com/IFU).

## Using Philips IntelliBridge or VueLink

### Using Philips monitors and the IntelliBridge or VueLink Open Interfaces

---

NOTE: Data displayed on the IntelliBridge or VueLink systems is for reference purposes only. Decisions for patient care should not be based solely on the data obtained through the IntelliBridge system.

---

The Respironics V60/V60 Plus Ventilator can communicate with a Philips patient monitor using the IntelliBridge Open Interface or the VueLink Open Interface.\* Figure B-3 shows the required hardware setup. The IntelliBridge Open Interface and the VueLink Open Interface require a ventilator baud rate of 19,200. Check for the correct baud rate in the ventilator diagnostic mode (see “Baud Rate” on page E-10).

\*NOTE: VueLink has been discontinued, but is still supported. Information is included for backwards compatibility only. The IntelliBridge EC10 is the VueLink Interface module successor.

## Communications interface

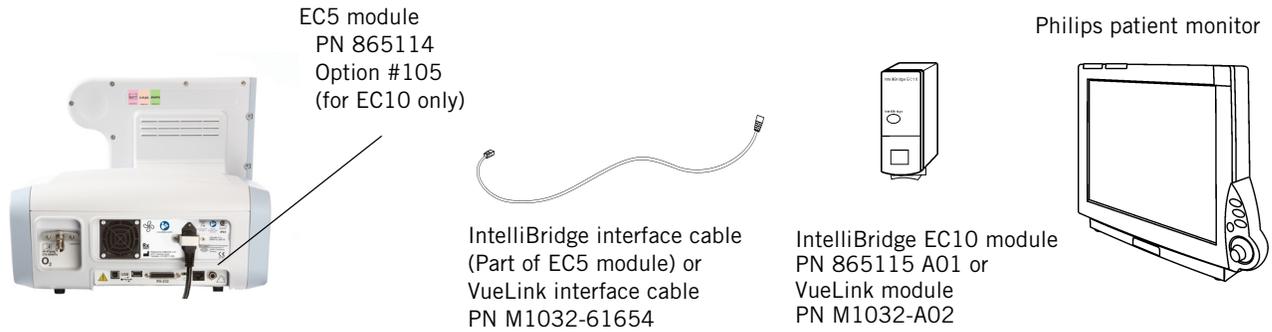


Figure B-3: Connection from ventilator to Philips patient monitor

### Data display

The data from your Respironics V60/V60 Plus Ventilator is displayed in several windows on your Philips monitor. This data may be labeled differently on the monitor than on the ventilator. Refer to Table B-1 to interpret these labels.

For more information, consult the documentation for your IntelliBridge or VueLink module and patient monitor.

Table B-1: Ventilator data displayed on Philips monitor

Monitor label	Ventilator label
<b>Waveform</b>	
AWP	P (airway pressure)
AWF	$\dot{V}_E$ (flow)
AWV	V (volume)
<b>Monitored parameters</b>	
%Bsp:t	Pt. Trigger
Leak	Pt. Leak or Tot. Leak
MINVOL	$\dot{V}_E$
PIP	PIP
RRaw	Rate
Tin/Tt	$T_I/T_{TOT}$
TVexp	$V_T$

## Communications interface

Table B-1: Ventilator data displayed on Philips monitor (continued)

Monitor label	Ventilator label
<b>Modes</b>	
Same as ventilator mode name	All modes except standby
STNDBY	Standby
<b>Control settings</b>	
sEppv	Max E
Not shown	C-Flex
PAVsup	PPV %
sCPAP	CPAP
sEPAP	EPAP
sfgFI	Flow (in HFT)
sFIO <sub>2</sub>	O <sub>2</sub>
sInsTi	I-Time (Inspiratory Time)
sIPAP	IPAP
sPmax	Max P (AVAPS Maximum IPAP Pressure) Max P (PPV Maximum IPAP Pressure)
sPmin	Min P (AVAPS Minimum IPAP Pressure)
sRisTi	Rise (Rise Time)
sRmpTi	Ramp Time
sRppv	Max R
sRRaw	Rate (Respiratory Rate)
sTV	V <sub>T</sub> (AVAPS Target Tidal Volume)
sVmax	Max V (PPV Maximum Volume Limit)
sVMode	Ventilation mode
<b>Alarm messages</b>	
HIGH INSP PRESS	High Inspiratory Pressure
HIGH O <sub>2</sub> SUPPLY	High O <sub>2</sub> Supply Pressure
HIGH RESP RATE	High Rate
HIGH EXH TV	High Tidal Volume
LOW INSP PRESS	Low Inspiratory Pressure
LOW BATTERY	Low Internal Battery
LOW FLOW	Cannot Reach Target Flow
LOW LEAK	Low Leak – CO <sub>2</sub> Rebreathing Risk

## Communications interface

Table B-1: Ventilator data displayed on Philips monitor (continued)

Monitor label	Ventilator label
LOW EXH MV	Low Minute Ventilation
LOW O2 SUPPLY	Low O <sub>2</sub> Supply Pressure
LOW RESP RATE	Low Rate
LOW EXH TV	Low Tidal Volume
NO O2 SUPPLY	Oxygen Not Available
OCCLUSION	Patient Circuit Occluded
PT. DISCONNECT	Patient Disconnect
PPV MAX P	PPV Max P
PPV MAX V	PPV Max V
PRESS REG HIGH	Pressure Regulation High
PROX DISCONNECT	Proximal Pressure Line Disconnect
Vent CHK DEVICE	Check Vent:
VENT ON BATTERY	Running on Internal Battery
Ventilation parameters blanked	Vent Inoperative xxxx

### Remote alarm port

- 
- WARNING:** To prevent possible patient injury due to nonannunciating alarms, verify the operation of any remote alarm device before use.
- WARNING:** To ensure the functionality of the remote alarm, connect only Respirationics-approved cables to the remote alarm port.
- CAUTION:** The remote alarm port is intended to connect only to an SELV (safety extra-low voltage and ungrounded system with basic insulation to ground), in accordance with IEC 60601-1. To prevent damage to the remote alarm, make sure the signal input does not exceed the maximum rating of 24 VAC or 36 VDC at 500 mA with a minimum current of 1 mA.
- NOTE:** Selecting Alarm Silence deactivates the remote alarm.
- 

The remote alarm (nurse call) port allows ventilator alarm conditions to be annunciated at locations away from the ventilator (for example, when the ventilator is in an isolation room). The ventilator sends alarm signals to a remote alarm through the connector at the rear of the ventilator (Figure B-1 on page B-1). Figure B-4 shows the pin assignments for this connector. The connector is a standard ¼-inch, female, audio (ring, tip, sleeve) connector.

The ventilator signals an alarm using either a normally open (NO) or normally closed (NC) relay contact. The de-energized state of the relay represents an

## Communications interface

alarm state (any high-priority alarm) and the energized state represents a non-alarm state. This application requires one of the cables listed in Table B-2.

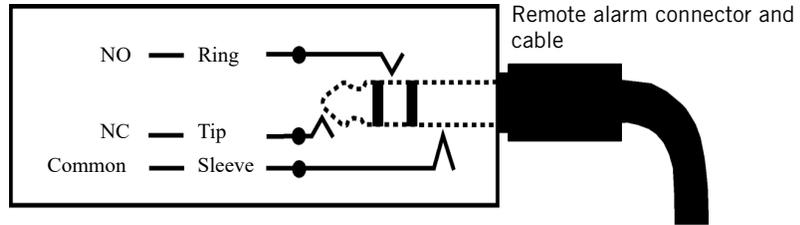


Figure B-4: Remote alarm port

Table B-2: Remote alarm cable kits

Description	System	Part Number
Remote alarm cable kit, alarm state = open	Use on Normally Closed nurse call systems (system expects to see open contacts when ventilator alarms). For ¼" jack.	1003741
Remote alarm cable kit, alarm state = closed	Use on Normally Opened nurse call systems (system expects to see closed contacts when ventilator alarms). For ¼" jack.	1003742
Remote alarm cable kit	Philips Respironics (LifeCare)	1003743

## Appendix C. Parts and accessories

---

This appendix lists parts and accessories supplied by Philips that are compatible with the Respironics V60/V60 Plus Ventilator. All parts and accessories are not available in all markets.

---

**WARNING:** Avoid adding resistive circuit components on the patient side of the proximal pressure line. Such components may defeat the disconnect alarm.

**NOTE:** Resistive components can include but are not limited to HMEs, proximal flow sensors, a filter at the patient connection, or a narrow diameter circuit attached to a mask.

**NOTE:** To ensure the correct performance of the ventilator and the accuracy of patient data, we recommend you use only Respironics-approved accessories with the ventilator.

---

For the most current list of approved accessories plus comprehensive ordering information for compatible parts available from Philips, contact your Philips representative or refer to the V60/V60 Plus ordering and accessories guide (downloadable from [www.philips.com/ifu](http://www.philips.com/ifu)).

## Parts and accessories

### Patient Interfaces

---

NOTE: Pediatric masks intended for patients weighing less than 20 kg (44 lb) are not approved for use with the V60/V60 Plus Ventilator.

---

#### Ventilation interfaces

Compatible interfaces include entrainment elbow (EE) versions from these product lines:

Description	Type
Respironics Contour Deluxe	Nasal
Respironics AP111	
Respironics PerformaTrak	Oro-nasal
Respironics AF541, over-the-nose	
Respironics AF541, under-the-nose	
Respironics AF531	Total
Respironics PerforMax, adult only	

#### HFT interfaces

For use with V60 Plus ventilators and V60 ventilators that have 3.00 software (or higher) and the HFT option installed.

Description
Respironics AC611 high flow nasal cannula
Respironics AC611 high flow nasal cannula with filtered exhalation port (FEP) connector

### O<sub>2</sub> analyzer/monitor

Description	Part number	Order number
Analytical Industries AII-2000M oxygen analyzer/monitor	1128903	989805654621
O <sub>2</sub> sensor, AII-2000M	1130625	989805655111
Tee adapter, O <sub>2</sub> sensor	1129064	989805654731
Tee adapter, O <sub>2</sub> sensor, threaded, amber	1020380	453561509031

## Patient breathing circuits

Description	Quantity	Part number	Order number
<b>Circuits Supporting Noninvasive Ventilation and High-Flow Therapy</b>			
Patient circuit, single-limb, heated, proximal pressure line, single-use, adult (Fisher & Paykel RT139).	10	1020523	989805610851
Patient circuit, single-limb, heated, filtered exhalation port (FEP), humidifier chamber, proximal pressure line, single-use, 22-mm ID, WILMed (Not available in North America)	10	1122059	989805653191
Patient circuit, single-limb, heated, filtered exhalation port (FEP), humidifier chamber, proximal pressure line, single-use, 22-mm ID, Fisher & Paykel RT239	10	1135739	989805658961
<b>Circuits Supporting Noninvasive Ventilation Only</b>			
Patient circuit, single-limb, non-heated, filtered exhalation port (FEP), proximal pressure line, single-use, 22-mm ID	10	1065830 (with inspiratory and exhalation port filters)	989805621311
		1065832 (with inspiratory port filter only)	989805621321
		1069210 (with no filters)	989805634871
Patient circuit, single-limb, non-heated, disposable exhalation port (DEP), proximal pressure line, single-use, 22-mm ID	10	582073	989805609611
Patient circuit, single-limb, non-heated, filtered exhalation port (FEP), water trap, proximal pressure line, single-use, 22-mm ID	10	652002	989805609681

## Humidifiers

Description	Quantity	Part number	Order number
<b>Humidifiers</b>			
Respiratory humidifier, Fisher & Paykel MR850	---	---	Refer to the V60 ordering and accessories guide
Respiratory humidifier, WILMed AIRcon Gen2, 230V (Not available in North America)	---	---	
<b>Chambers</b>			
Humidifier chamber, adult, Fisher & Paykel MR290	10	22104	989805642931
Humidifier chamber, autofill, WILMed (Not available in North America)	30	1121825	989805653111

## Parts and accessories

### Bacteria filters

Description	Quantity	Part number	Order number
Filter inspiratory, 22-mm male x 22-mm female, single-use	10	342077	989805609521
Filter, patient pressure, single-use	1	1002362	453561517101

### Nebulizers

Description	Quantity	Part number	Order number
Aerogen Solo nebulizer (for inline applications)	---	---	Refer to the V60 ordering and accessories guide
NIVO generator (for mask elbow applications)	5	1076302	989805634301

### Operator maintenance parts

Description	Quantity	Part number	Order number
Cooling fan filter	5	1054280	453561507301
Air inlet filter	5	1054279	453561505991

### Other parts

Compatible with all V60/V60 Plus Ventilators:

Description	Part number	Order number
Test lung, hard-sided (low-compliance)	1021671	989805611871
Backup battery	1076374	989805626941

## Appendix D. Regulatory compliance

---

### Electromagnetic compatibility (EMC)

IEC 60601-1-2; 2014, Ed. 4.0	Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances
------------------------------	--

### Electromagnetic compatibility declaration

Medical equipment needs special precautions regarding electromagnetic compatibility (EMC) and needs to be installed and put into service according to the EMC information provided in this document.

- 
- WARNING:** The V60/V60 Plus Ventilator may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the ventilator or shielding the location.
- WARNING:** Use of non-approved accessories, transducers or cables may increase EMC emissions or decrease the EMC immunity performance of the equipment.
- WARNING:** Use portable radio-frequency (RF) communications equipment (including peripherals such as antenna cables and external antennas) no closer than 30 cm (12 in.) to any part of the ventilator system, including cables specified for use with the ventilator. Otherwise, equipment performance can be degraded. If higher immunity test levels than those specified in IEC 60601-1-2:2014, Table 9, are used, the minimum separation distance may be lowered. Lower minimum separation distances shall be calculated using the equation specified in chapter 8.10 of the standard.
- WARNING:** Do not use the ventilator near radio-frequency identification (RFID) or electromagnetic security systems. The ventilator may disrupt the operation of this equipment.
- WARNING:** This equipment is designed to comply with IEC 60601-1-2. This equipment generates, uses, and can radiate radio-frequency energy and, if not installed and used in accordance with instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. Harmful interference to other devices can be determined by turning this equipment on and off. Try to correct the interference using one or more of the following:
- Reorient or relocate the receiving device.
  - Increase the separation between the equipment.

## Regulatory compliance

- Connect the equipment to an outlet on a different circuit from that to which the other devices are connected and contact Philips Service for help.

**WARNING:** Take care when operating the ventilator around other equipment, to avoid reciprocal interference. Potential electromagnetic interference (EMI), electrostatic discharge (ESD), or other interference can occur to the ventilator or other equipment. Interference can interrupt ventilator operation or degrade system performance, which can result in patient injury. Try to minimize this interference by not using other equipment in conjunction with the ventilator. If adjacent or stacked use is necessary, follow the recommendations for placement of the equipment in “Electromagnetic compatibility declaration” on page D-1. Monitor the ventilator and nearby devices to verify normal operation.

**WARNING:** Do not use the ventilator near active high frequency (HF) surgical equipment, medical devices such as X-ray devices and diathermy, or in an RF-shielded room of medical equipment or system for magnetic resonance imaging, where the intensity of electromagnetic (EM) disturbances is high. These devices may degrade the performance of the ventilator.

### Electromagnetic emissions

**NOTE:** The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A).

Guidance and manufacturer's declaration - electromagnetic emissions		
The V60/V60 Plus Ventilator is intended for use in the electromagnetic environment specified below. The user of the V60/V60 Plus Ventilator should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic enforcement - guidance
RF Emissions CIS-PR 11	Group 1	The V60/V60 Plus Ventilator uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF Emissions CIS-PR 11	Class A	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

**Electromagnetic immunity**

<b>Guidance and manufacturer's declaration - electromagnetic immunity</b>			
The V60/V60 Plus Ventilator is intended for use in the electromagnetic environment specified below. The user of the V60/V60 Plus Ventilator should assure that it is used in such an environment.			
<b>Immunity test</b>	<b>IEC 60601-1-2 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines. ±1 kV for input / output lines	±2 kV for power supply lines. ±1 kV for input / output lines	Mains power quality should be that of a typical hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% $U_T$ for 0.5 cycles 0% $U_T$ for 1.0 cycle 70% $U_T$ for 25 cycles (50 Hz)/ 30 cycles (60 Hz) 0% $U_T$ for 250 cycles (50 Hz)/ 300 cycles (60 Hz)	0% $U_T$ for 0.5 cycles 0% $U_T$ for 1.0 cycle 70% $U_T$ for 25 cycles (50 Hz)/ 30 cycles (60 Hz) 0% $U_T$ for 250 cycles (50 Hz)/ 300 cycles (60 Hz)	Mains power quality should be that of a typical hospital environment. If the user of the V60/V60 Plus Ventilator requires continued operation during power mains interruptions, it is recommended that the V60/V60 Plus Ventilator be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical hospital environment.
NOTE: $U_T$ is the AC mains voltage prior to application of the test level.			

## Regulatory compliance

Guidance and manufacturer's declaration - electromagnetic immunity			
The V60/V60 Plus Ventilator is intended for use in the electromagnetic environment specified below. The customer or the user of the V60/V60 Plus Ventilator should assure that it is used in such an environment.			
Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz outside ISM bands <sup>a</sup>	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the V60/V60 Plus Ventilator, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = \left[ \frac{3.5}{V_1} \right] \sqrt{P}$ where $V_1 = 3$ Vrms $d = \left[ \frac{12}{V_2} \right] \sqrt{P}$ where $V_2 = 6$ Vrms
	6 Vrms 150 kHz to 80 MHz in ISM bands <sup>a</sup>	6 Vrms	
Radiated RF IEC 61000-4-3	3 V/m <sup>e</sup> 80 MHz to 2.7 GHz	3 V/m	$d = \left[ \frac{12}{E_1} \right] \sqrt{P}$ 80 MHz to 800 MHz, where $E_1 = 3$ V/m $d = \left[ \frac{23}{E_1} \right] \sqrt{P}$ 800 MHz to 2.7 GHz, where $E_1 = 3$ V/m where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). <sup>b</sup> Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>c</sup> should be less than the compliance level in each frequency range. <sup>d</sup>
	380 MHz to 390 MHz	27 V/m	
	430 MHz to 470 MHz	28 V/m	
	704 MHz to 787 MHz	9 V/m	
	800 MHz to 960 MHz	28 V/m	
	1.700 GHz to 1.990 GHz	28 V/m	
2.400 GHz to 2.570 GHz	28 V/m		
5.100 GHz to 5.800 GHz	9 V/m		
NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a. The ISM (industrial, scientific and medical) bands between 150 kHz and 80 MHz are 6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz. b. The compliance levels in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2.7 GHz are intended to decrease the likelihood that mobile/ portable communications equipment could cause interference if it is inadvertently brought into patient areas. For this reason, an additional factor of 10/3 has been incorporated into the formulae used in calculating the recommended separation distance for transmitters in these frequency ranges. c. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the V60/V60 Plus Ventilator is used exceeds the applicable RF compliance level above, the V60/V60 Plus Ventilator should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the V60/V60 Plus Ventilator. d. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m. e. According to IEC 60601-1-2: 2014			

Recommended separation distances between portable and mobile RF communications equipment and the V60/V60 Plus Ventilator				
Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)			
	150 kHz to 80 MHz outside ISM bands	150 kHz to 80 MHz in ISM bands	80 MHz to 800 MHz	800 MHz to 2.7 GHz
	$d = \left[ \frac{3.5}{V_1} \right] \sqrt{P}$	$d = \left[ \frac{12}{V_2} \right] \sqrt{P}$	$d = \left[ \frac{12}{E_1} \right] \sqrt{P}$	$d = \left[ \frac{23}{E_2} \right] \sqrt{P}$
0.01	0.12	0.20	0.40	0.77
0.1	0.37	0.63	1.26	2.42
1	1.17	2.00	4.00	7.67
10	3.69	6.32	12.65	24.24
100	11.67	20.00	40.00	76.67
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.				
NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.				
NOTE 2: The ISM (industrial, scientific and medical) bands between 150 kHz and 80 MHz are 6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz.				
NOTE 3: An additional factor of 10/3 has been incorporated into the formulae used in calculating the recommended separation distance for transmitters in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2.7 GHz to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas.				
NOTE 4: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.				
NOTE 5: <b>The minimum separation distance for RF communication equipment operating within the following frequency bands is 0.3 m:</b>				
<ul style="list-style-type: none"> <li>• 380 – 390 MHz (TETRA 400)</li> <li>• 430 – 470 MHz (GMRS 460, FRS 460)</li> <li>• 704 – 787 MHz (LTE Band 13, 17)</li> <li>• 800 – 960 MHz (GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5)</li> <li>• 1 700 – 1 990 MHz (GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS)</li> <li>• 2 400 – 2 570 MHz (Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7)</li> <li>• 5 100 – 5 800 MHz (WLAN 802.11 a/n)</li> </ul>				

## Regulatory compliance

### RF immunity

Test specifications for RFID immunity		
The frequencies specified are not yet part of formal regulatory standard IEC 60601-1-2 4.0ED. They are based on 1) a working draft of IEC60601-1-2 4.1ED:2018 Table 11 Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral Standard: Electromagnetic disturbances – Requirements and tests, and 2) AIM Standard 7351731 2.00ED:2016.		
RFID specification	Test Frequency	Test level (RMS)
ISO 14223	134.2 kHz	65 A/m
ISO/IEC 14443-3 (Type A)	13.56 MHz	7.5 A/m
ISO/IEC 14443-4 (Type B)	13.56 MHz	7.5 A/m
ISO/IEC 15693 (ISO 18000-3 Mode 1)	13.56 MHz	5 A/m
ISO 18000-3 Mode 3	13.56 MHz	12 A/m
ISO/IEC 18000-7	433 MHz	3 V/m
ISO/IEC 18000-63 Type C	860-960 MHz	54 V/m
ISO/IEC 18000-4 Mode 1	2.45 GHz	54 V/m

### RFID reader separation distance

The testing specified in this standard applies to medical electrical equipment and systems used near RFID readers. A medical electrical equipment or system that is in conformity with this standard is qualified for use as close as 2.5 cm for some RFID applications and 20 cm for others, based on the maximum amount of radio frequency (RF) output power allowed by the Federal Communications Commission (FCC) and typical use distances. Medical electrical equipment and systems that meet less stringent standards, such as IEC 60601-1-2, could be used safely with RFID readers if the RF output power of the RFID reader is less than the allowed maximum or it can be assured that the separation distance between the RFID Reader and the medical electrical equipment or system will always be much greater than 20 cm.

Guidance on EMC usually includes the parameters of the RF output of the source ( $P$ ), the RF immunity of the medical electrical equipment or system, and the distance between them ( $d$ ). In general, a medical electrical equipment or system should operate safely when it experiences field strength ( $E$ ) that is equal to or lower than its specified RF immunity. In general, the field strength falls off proportional to  $1/d$ , although care must be taken because some environments (where large metallic objects such as HVAC ducts and cabinetry are present) can cause reflections such that field strength in certain close-in ranges can be higher further away from the medical electrical equipment or system that is at closer distances.

## Regulatory compliance

So, in addition to selection of RFID systems and medical electrical equipment and systems, EMC management includes maintaining adequate separation distances between them.

For medical electrical equipment and systems that meet this standard, the minimum necessary separation distances are shown in Annex J of the AIMS standard 7351731, as well as the guidance required by standards such as IEC 60601-1-2.

For medical electrical equipment and systems that do not meet this standard, the minimum necessary separation distances can be determined using the guidance required by standards such as IEC 60601-1-2.

In general, the recommended minimum separation distance ( $d$ ) is of the form:

$$d = \left[ \frac{k}{E} \right] \sqrt{P}$$

where  $E$  is the RF immunity of the medical electrical equipment or system,  $P$  is the maximum rated RF output power of the RFID system, and  $k$  is an antenna efficiency factor. In Edition 3 of IEC 60601-1-2 [23] and earlier editions,  $k$  was assumed to be 3.5 for frequencies less than 800 MHz and 7 at frequencies greater than or equal to 800 MHz. In Edition 4 [24],  $k$  is assumed to be 6 for all frequencies.

### Cables That May Affect IEC 60601-1-2 Compliance

Cables, such as remote alarm cables, longer than 3.05-m may affect the compliance of the ventilator system with the emissions and immunity requirements of IEC 60601-1-2.

### WEEE recycling directive

Waste electrical and electronic equipment (WEEE) recycling directive.

Waste electrical and electronic equipment must not be disposed of as unsorted municipal waste at the end of its expected service life. It must be collected separately and must be disposed of per local regulations. Contact your Philips authorized representative for information concerning the decommissioning of your equipment.



Compliant with the WEEE recycling directive.

If you are subject to the WEEE directive, refer to [www.philips.com/recycling](http://www.philips.com/recycling) for the passport for recycling this product.

Dispose of this device in accordance with local collections and recycling regulations. For more information, visit [www.philips.com/recycling](http://www.philips.com/recycling).

This device contains a lithium ion battery that cannot be recycled. Please contact customer service for safe disposal instructions of this part.

## Regulatory compliance

### Classification

Protection Against Electric Shock	Class I
Degree of Protection Against Electric Shock	Type B
Degree of protection against ingress of particulate matter and water given by the enclosure	IP21 Protected against solid objects $\geq 12.5$ mm in diameter. Protected against vertically falling water drops when enclosure is tilted up to $15^\circ$
Mode of operation	Continuous Operation
Method of sterilization	Not intended to be sterilized
Suitability for use in an O <sub>2</sub> -rich environment	Not suitable

### Safety

IEC 60601-1; 2012, Ed. 3.1	Medical electrical equipment – Part 1: General requirements for basic safety and essential performance
IEC 60601-1-6; 2013, Ed. 3.1	Medical electrical equipment – Part 1-6: General requirements for basic safety and essential performance; collateral standard: usability
IEC 60601-1-8; 2012, Ed. 2.1	Medical electrical equipment – Part 1-8: General requirements for basic safety and essential requirements; collateral standard - alarm systems
IEC 62366-1, 2015, Ed. 1.1	Medical devices - Application of usability engineering to medical devices
ISO 14971; 2007	Medical devices – Application of risk management to medical devices
EN ISO 14971; 2012	Medical devices – Application of risk management to medical devices
ISO 80601-2-12; 2011	Medical electrical equipment – Particular requirements for basic safety and essential performance of critical care ventilators
IEC 60529; 2013, Ed. 2.2	Degrees of protection provided by enclosures (IP Code)
IEC 62304; 2015, Ed. 1.1	Medical device software - Software life cycle processes

### Applied parts

The V60/V60 Plus Ventilator system includes these applied parts:

- Patient interfaces
- Tracheal and endotracheal tubes
- High flow therapy nasal cannula and interface
- Nebulizer T-adapter
- Ventilator breathing system - Type B (inspiratory or expiratory pathways through which gas flows at respiratory pressures and bounded by the ports through which gas enters and the patient-connection port), which can also include the following:
  - Humidifier chamber
  - Patient circuits

### Accessible parts

The V60/V60 Plus Ventilator system includes these accessible parts:

- Filters
- Roll stand
- Humidifier
- O<sub>2</sub> monitor/analyzer
- Nebulizer controller

### Detachable components

The V60/V60 Plus Ventilator system includes these detachable components (see Figure 3-4 and Figure 3-5):

- Left side panel
- Right side panel
- Air inlet filter (under side panel)

### Essential performance

Per ISO/EN 80601-2-12: 2011, Medical electrical equipment Part 2-12: Particular requirements for basic safety and essential performance of critical care ventilators, the ventilator's essential performance requirements are given in "Control settings" on page 11-1, "Alarms and other messages" on page 9-7, and "Table of monitored parameters" on page 8-2. Alarms, including O<sub>2</sub> supply failure alarms and gas failure alarms, are identified in "Alarms and other messages" on page 9-7. AC mains power information is given in "Connecting to AC power" on page 5-2. Battery backup information is given in "About the backup battery" on page 3-10. Gas connection information is given in "Connecting oxygen" on page 5-1 and "Installing oxygen inlet connector and AC power cord" on page A-7.

## Regulatory compliance

## Appendix E. Diagnostic mode

---

In the diagnostic mode you select the language of software display, set the date and time, select pressure units, enable software options, and calibrate the touchscreen.

---

**NOTE:** The diagnostic mode is primarily for use by authorized service personnel to download software and perform other diagnostic procedures.

---

### Entering the diagnostic mode

---

**WARNING:** To prevent possible patient injury, do not enter the diagnostic mode while a patient is connected to the ventilator. Verify that the patient is disconnected before proceeding.

---


Enter the diagnostic mode as follows:

1. Make sure the patient is disconnected and the ventilator is powered off.
2. Press and hold the Accept button on the top-right front of the ventilator and turn on the ventilator by pressing the ON/Shutdown key.

The screen displays “Press  again for Diagnostics or wait for Ventilation.”

## Diagnostic mode

3. Within less than 5 seconds, release and press the Accept button again. The Diagnostics Menu (Figure E-1) is displayed.

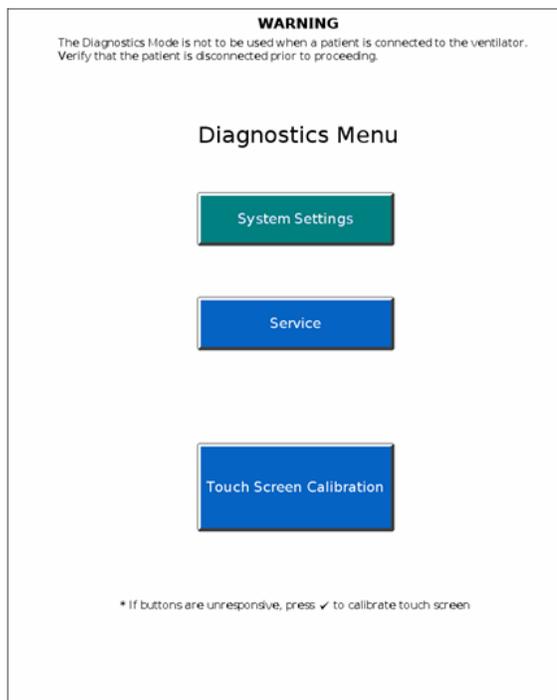

*Figure E-1: Diagnostics Menu*

4. Select the desired function.

## System settings

From the System Settings screen (Figure E-2) you can perform the functions below.

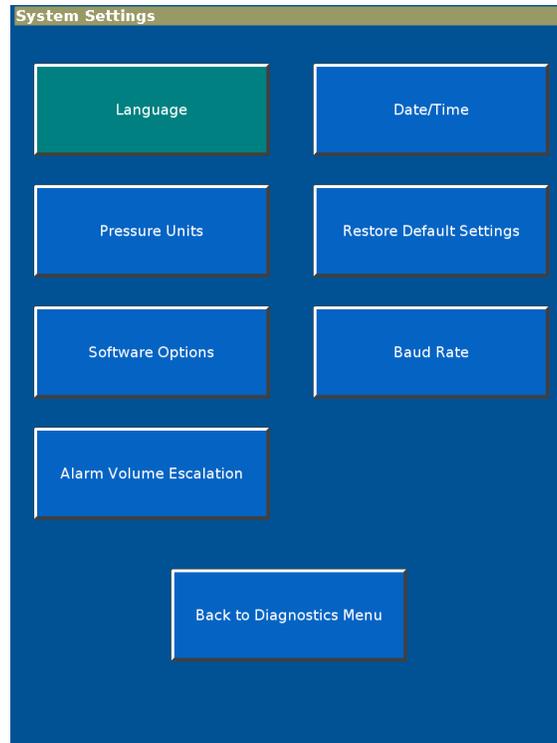


Figure E-2: System Settings screen

## Diagnostic mode

### Language

The Language function lets you set the language of software display.

1. From the System Settings screen, select Language to display the Set Language screen (Figure E-3).

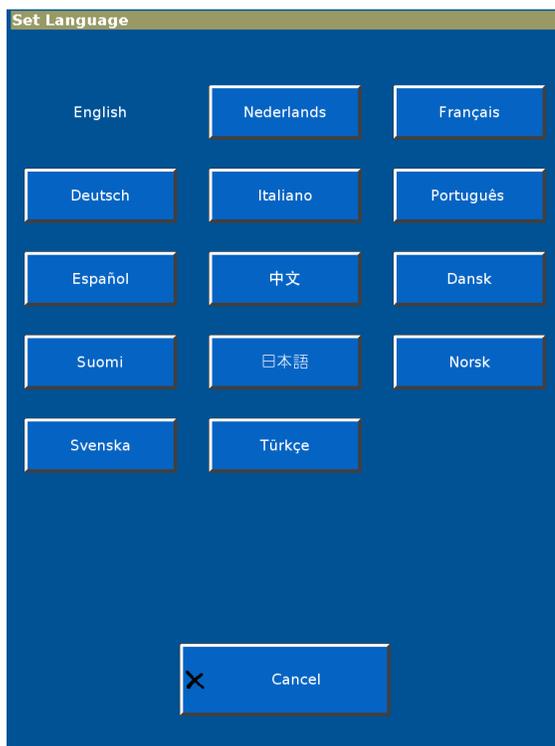


Figure E-3: Set Language screen 1

## Diagnostic mode

2. The active language is shown in white type. Select the new language.
3. A second Set Language screen is displayed (Figure E-4). Select Ventilator Shutdown to apply the change. The change is effective after you restart the ventilator.

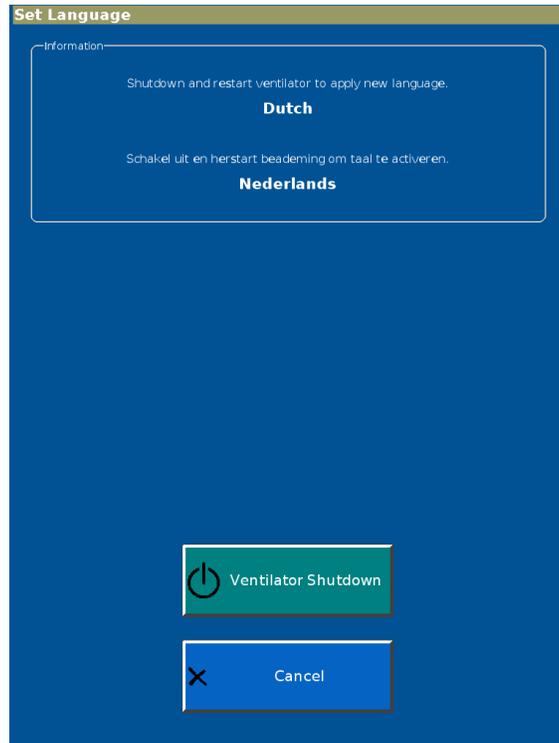


Figure E-4: Set Language screen 2

## Diagnostic mode

### Date/Time

The Date/Time function lets you verify date and time settings.

1. From the System Settings screen, select Date/Time to display the Set Date and Time screen (Figure E-5).

Set Date and Time

Information

Hours Minutes

+ +

15 : 52

- -

Year Month Day

+ + +

2011 - 1 - 29

- - -

✓ Accept

✗ Cancel

Figure E-5: Set Date and Time screen

2. Adjust the date and time with the + and - buttons; then Apply.

### Pressure Units

The Pressure Units function lets you select the unit of measure for pressure display.

1. From the System Settings screen, select Pressure Units to display the Set Pressure Units screen (Figure E-6).

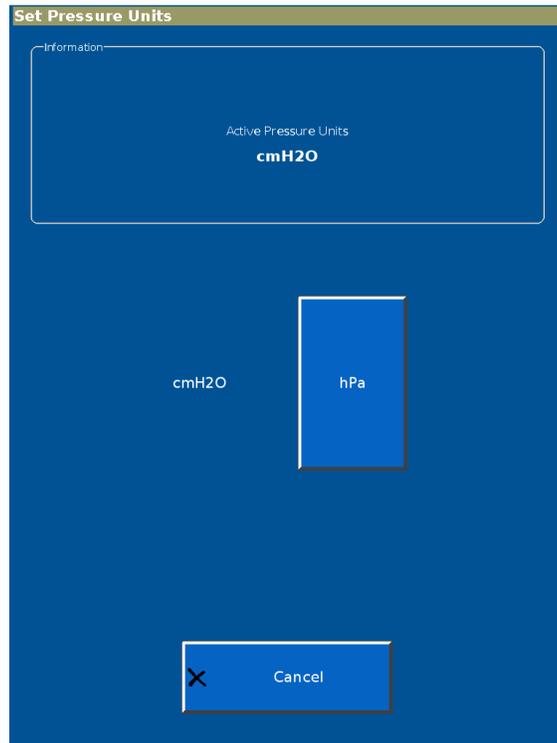


Figure E-6: Set Pressure Units screen


2. The active pressure unit is shown in white type. Select the desired pressure unit. The change is effective after you restart the ventilator.

## Diagnostic mode

### Restore Default Settings

The Restore Default Settings function lets you return ventilator settings to factory defaults. The factory defaults are listed in Chapter 11.

1. From the System Settings screen, select Restore Default Settings to display the Restore Default Settings screen (Figure E-7).

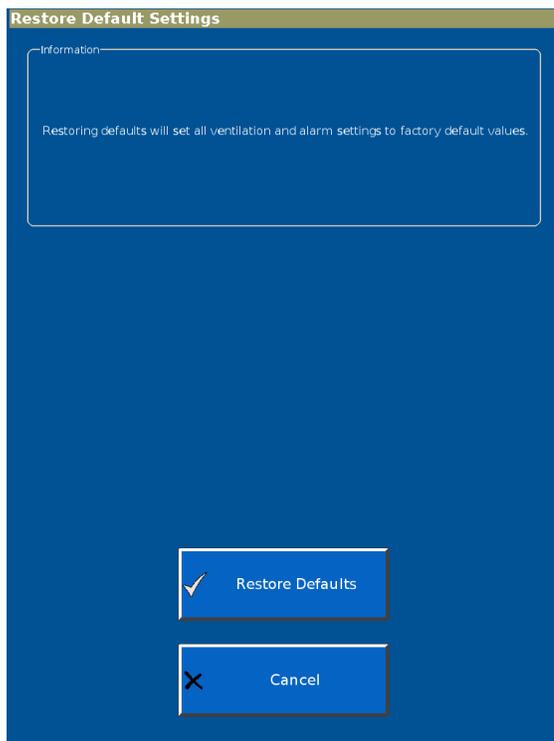

*Figure E-7: Restore Default Settings screen*

2. Select Restore Defaults.

### Software Options


With the Software Options function, you enable a software option using a unique code specific to the option and the ventilator serial number. Options can also be enabled through the Respi-Link remote service program.

---

**NOTE:** Before installing an option, verify that the ventilator serial number matches the serial number shown in the Vent Info window (“Vent Info (ventilator information)”) on page 6-18. If the serial numbers do not match, contact Philips.

---

1. From the System Settings screen, select Software Options to display the Enable Software Options screen (Figure E-8).

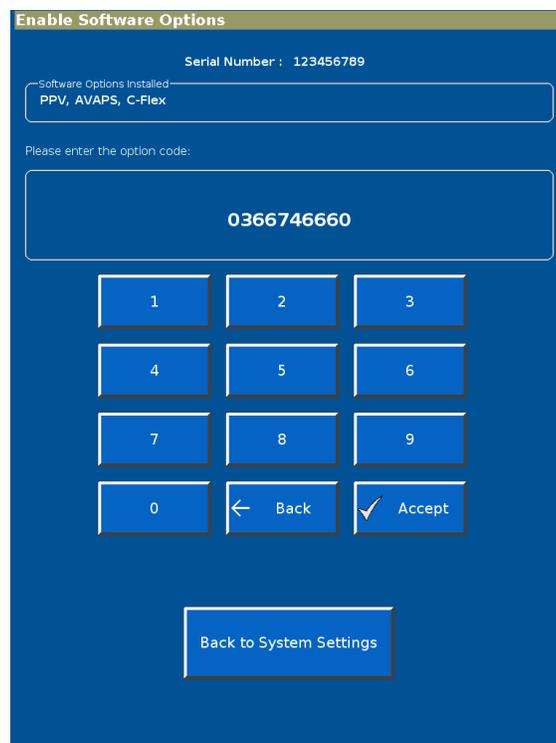

*Figure E-8: Enable Software Options screen*


2. Use the onscreen keypad to enter the code; then select Enter. The screen displays Enabled: followed by the name of the software option.
3. Repeat as needed to enable additional options.
4. Verify that the options are enabled by selecting Back to System Settings, then Back to Diagnostics Menu, then Service. The Vent Info window should show the new options.
5. Attach the option label as shown in Figure 3-5 on page 3-9.

## Diagnostic mode

### Baud Rate

The Baud Rate function lets you set the baud rate for serial communications.

1. From the System Settings screen, select Baud Rate to display the Set Baud Rate for Serial Communications screen (Figure E-9).

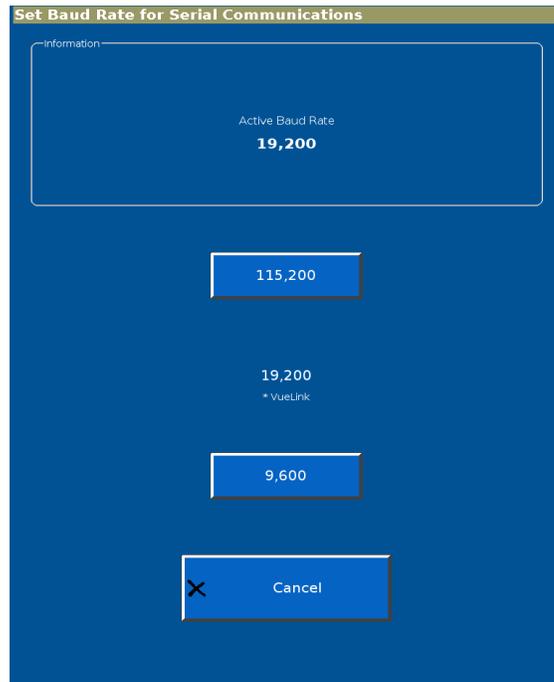

*Figure E-9: Set Baud Rate for Serial Communications screen*

2. The active baud rate is shown in white type. Select the desired baud rate.

### Alarm Volume Escalation

The Alarm Volume Escalation function lets you enable or disable volume escalation<sup>1</sup>. When alarm volume escalation is Enabled and a high priority alarm is not responded to within 40 seconds, the ventilator alarm volume increases to maximum over an 20-second period.

When the Alarm Volume Escalation function is active and a touchscreen or button press is detected, the ventilator automatically returns the alarm volume to the user setting.

1. From the System Settings screen, select Alarm Volume Escalation to display the Set Alarm Volume Escalation screen (Figure E-10).

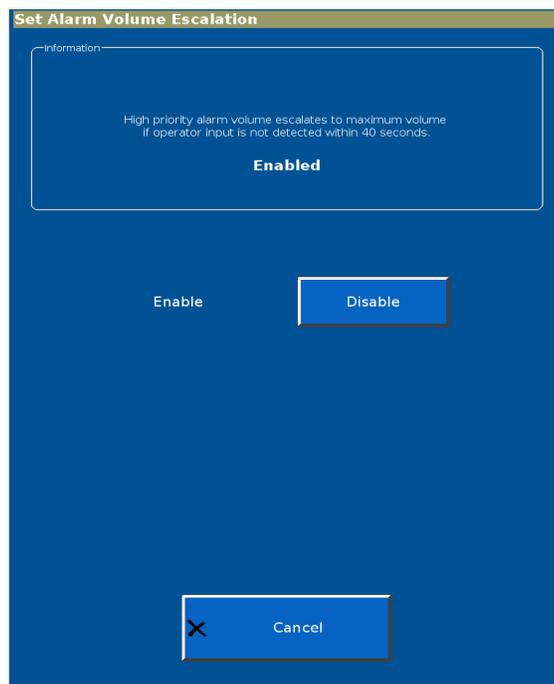


Figure E-10: Set Alarm Volume Escalation

2. The current setting is shown in the Information box at the top of the screen. If Alarm Volume Escalation is currently Disabled, you will see a selectable Enable button. If Alarm Volume Escalation is currently Enabled, you will see a selectable Disable button. Press the button to change the setting.
3. The new setting is applied after the V60/V60 Plus Ventilator is shut down and powered on again.

---

1. Available in Revision 2.30 software and above.

## Diagnostic mode

### Service

---

**NOTE:** All ventilator mode and alarm settings, alarm messages and significant events are automatically logged and retained, even when power is lost.


---

The Service screen lets you view the event log. Other service functions are for use by authorized service personnel.

#### Significant Event Log

The Significant Event Log contains data about clinically relevant ventilator occurrences, including alarms and setting changes. The time, date, and an identifier for event classification are included. A maximum of 2,000 event records are retained. The oldest events are overwritten first to allow recording of new events.

1. From the Service screen, select the Misc tab.

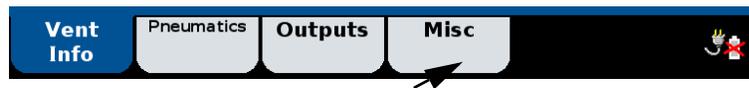

2. The Miscellaneous screen opens (Figure E-11). Select Significant Event Log.

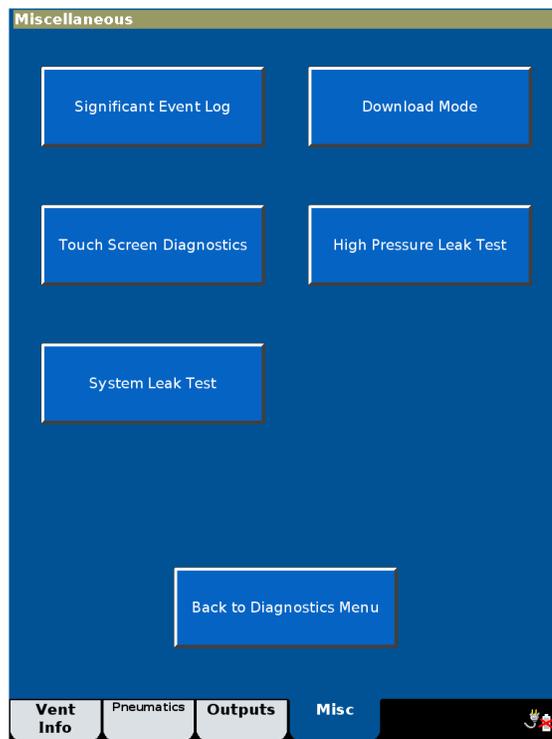


Figure E-11: Miscellaneous screen

3. The Significant Event Log opens (Figure E-12). Use the buttons on right side to navigate through the log.



Figure E-12: Significant Event Log screen

## Diagnostic mode

### Touchscreen calibration


Calibrate the touchscreen X and Y coordinates as follows:

1. From the Diagnostics Menu, select Touch Screen Calibration. The Touch Screen Calibration screen is displayed (Figure E-13).

---


NOTE: If the Touch Screen Calibration button does not respond, press the Accept button on the top-right front of the ventilator to begin.

---

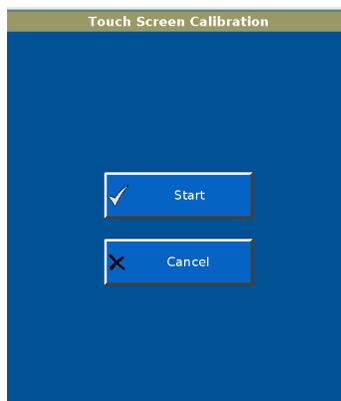

*Figure E-13: Calibrate Touch Screen screen*

2. Follow the steps shown. Press on the middle of each target with a blunt, narrow object.

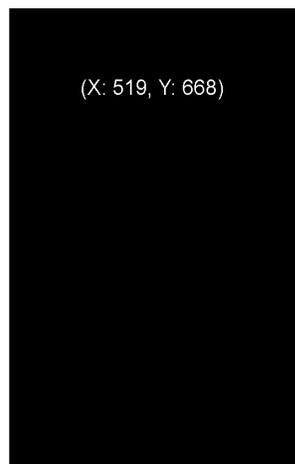

## Diagnostic mode

3. When prompted, touch the screen to exit calibration.

If the calibration is not successful, have the ventilator serviced.

## Exiting the diagnostic mode

Exit the diagnostic mode by turning off ventilator power with the ON/Shutdown key.

## Diagnostic mode

# Glossary

---

**A** Ampere, a unit of current.

**AC** Alternating current.

**Alarm Silence button** Silences alarm sound for 2 minutes.

**Alarm Volume escalation** When enabled, this function becomes active if there is no response to a high priority alarm within 40 seconds. Ventilator alarm volume then increases to its maximum over a 20-second period.

**Auto-Trak+** An optional feature that allows adjustments to trigger and cycle thresholds beyond Auto-Trak Sensitivity settings.

**Auto-Trak Sensitivity** A Respirationics innovation in triggering and cycling that utilizes several different methods to provide enhanced sensitivity in the presence of leaks and changing breathing patterns.

**AVAPS** Average volume-assured pressure support. A ventilation mode in which pressure support is automatically adjusted to maintain the user-defined target tidal volume.

**AVAPS Maximum IPAP Pressure** See Max P.

**AVAPS Minimum IPAP Pressure** See Min P.

**AVAPS Target Tidal Volume** See  $V_T$ .

**Average volume-assured pressure support** See AVAPS.

**Baseline** As in *baseline pressure*. The pressure at end exhalation.

**BPM** Breaths per minute.

**BTPS** Body temperature (98 °F, ambient pressure), 100% saturated (with water vapor).

**C-Flex** A setting in CPAP mode, which enhances traditional CPAP by reducing the pressure at the start of exhalation.

**cmH<sub>2</sub>O** Centimeters of water, a unit of pressure measurement.

**Continuous positive airway pressure** See CPAP.

## Glossary

**CPAP** Continuous positive airway pressure. A ventilation mode that provides a single, continuous level of positive pressure to the patient and a control setting in that mode.

**Cycle** To end inspiration.

**dB(A)** Decibel, a unit of acoustic power.

**DISS** Diameter index safety standard, a standard for high-pressure gas inlet fittings.

**E-Cycle (Expiratory Cycle Sensitivity)** A control setting in Auto-Trak+. It determines the threshold at which the ventilator will transition from inspiration to exhalation.

**Elast.** See Elastance.

**Elastance** The elastic opposition to ventilation or the tendency of the lungs to resist inflation (elastance is the reciprocal of compliance).

**EPAP** Expiratory positive airway pressure. A control setting. The application and maintenance of pressure above atmospheric at the airway throughout the exhalation phase of positive-pressure mechanical ventilation.

**Estimated exhaled tidal volume** See  $V_T$ .

**Estimated minute ventilation** See  $\dot{V}_E$ .

**Estimated patient leak** See Pt. Leak

**Estimated total leak** See Tot.Leak.

**ET** Endotracheal.

**Exhalation Port test** Performed to assess the leak flow rate through the exhalation port.

**Expiratory Cycle** See E-Cycle.

**Expiratory positive airway pressure** See EPAP.

**Flow** Flow rate, a setting in high flow therapy

**HFT** High flow therapy, a feature that provides a constant flow of mixed air and oxygen.

**HIP** High Inspiratory Pressure Alarm, an alarm setting.

**Hi Rate** High Rate Alarm, an alarm setting.

**Hi  $V_T$**  High Tidal Volume Alarm, an alarm setting.

**hPa** Hectopascal, a unit of pressure measurement. 1 hPa is equal to 1 mbar, which is approximately equal to 1 cmH<sub>2</sub>O.

**ID** Inner diameter.

**IEC** International Electrotechnical Commission.

**I:E ratio** Ratio of inspiratory to expiratory time.

**Inop** Inoperative.

**Inspiration:exhalation ratio** See I:E ratio.

**Inspiratory positive airway pressure** See IPAP.

**Inspiratory time** See I-Time.

**Inspiratory duty cycle** See  $T_I/T_{TOT}$ .

**Intentional leakage** “Known,” quantifiable leakage that is a function of the mask.

**IPAP** Inspiratory positive airway pressure. A control setting. The application and maintenance of pressure above atmospheric at the airway throughout the inspiration phase of positive-pressure mechanical ventilation.

**ISO** International Organization for Standardization, a worldwide federation of national standards bodies.

**I-Time** Inspiratory time. The duration of inspiration during mechanical ventilation.

**L** Liter.

**LCD** Liquid crystal display.

**LED** Light-emitting diode.

**Limit** To prevent from exceeding a specified maximum value during a breath.

**LIP** Low Inspiratory Pressure Alarm, an alarm setting.

**Lo Rate** Low Rate Alarm, an alarm setting.

**Lo  $\dot{V}_E$**  Low Minute Ventilation Alarm, an alarm setting.

**Lo  $V_T$**  Low Tidal Volume Alarm, an alarm setting.

**Mandatory breath** A breath for which either the timing or volume is controlled by the ventilator. That is, the machine triggers and/or cycles the breath.

**Max E** Maximum elastance (volume assist). A control setting in PPV.

## Glossary

**Max P** AVAPS Maximum IPAP Pressure. A control setting in AVAPS.

**Max P** Maximum Pressure. See PPV Maximum Pressure Limit.

**Max R** Maximum resistance (flow assist). A control setting in PPV.

**Max V** Maximum Volume. See PPV Maximum Volume Limit.

**Min P** AVAPS Minimum IPAP Pressure. A control setting in AVAPS.

**mL** Milliliter.

**mm** Millimeter.

**NIST** Non-Interchangeable Screw-Threaded. A connector for high-pressure gas inlet fittings.

**Noninvasive** Pertaining to a diagnostic or therapeutic technique that does not require the skin to be broken or a cavity or organ of the body to be entered. Mechanical ventilation via mask, nasal prongs, or mouthpiece.

**O<sub>2</sub>** Oxygen (concentration). A control setting.

**OD** Outer diameter.

**PCV** Pressure-controlled ventilation. A ventilation mode that provides mandatory and spontaneous breaths with a set frequency, pressure, and inspiratory time.

**Peak inspiratory pressure** See PIP.

**Percentage of patient-triggered breaths** See Pt. Trig.

**PIP** Peak inspiratory pressure. The peak pressure for the previous inspiration.

**PPV %** A control setting in PPV. The percent of proportional pressure ventilation supplied by the ventilator.

**PPV** proportional pressure ventilation. A ventilation mode that delivers a pressure-controlled breath in proportion to the patient's effort. The ventilator responds to patient instantaneous efforts, allowing the patient to determine when to start and end a breath, and how flow and pressure change as the patient breathes spontaneously.

**PPV Maximum Pressure Limit (Max P)** A control setting in PPV.

**PPV Maximum Volume Limit (Max V)** A control setting in PPV.

**Pressure-controlled ventilation** See PCV.

**Pressure-supported breath** A patient-triggered, pressure-targeted breath.

**psi** Pounds per square inch.

**psig** Pounds per square inch gauge (above atmospheric pressure).

**Proportional pressure ventilation** see PPV.

**Pt. Leak** The leak resulting from leaks around the mask or from unintentional leaks in the circuit. A monitored parameter shown when the intentional leak is known.

**Pt. Trig** Percentage of patient-triggered breaths. Patient-initiated breaths as a percentage of total breaths during the last 15 minutes.

**Ramp** Can be used to allow the patient to become accustomed to respiratory ventilatory therapy over time. Ramp will allow the pressure to linearly increase over a user-set period.

**Rate (Respiratory Rate)** Respiratory frequency, a control setting and monitored parameter.

**Resist.** See Resistance

**Resistance** The pressure drop across a pneumatic device (i.e., bacteria filter, patient circuit tubing) for a unit of flow when the volume of the device remains constant, i.e., cmH<sub>2</sub>O/mL/sec.

**Respiratory Rate (Rate)** Respiratory frequency, a control setting.

**Rise Time (Rise)** The time required for a pressure-supported or pressure-controlled breath to reach its target pressure, a control setting.

**RS-232** Serial data communications protocol.

**SIS** Sleeve Indexed System (Australia). A connector for high-pressure gas inlet fittings.

**Spont indicator** Denotes patient-initiated breathing.

**Spontaneous breath** A breath for which both the timing and volume are controlled by the patient. That is, the patient both triggers and cycles the breath.

**Spontaneous/timed mode** See S/T mode.

**S/T mode** Spontaneous/timed mode. A pressure support ventilation mode that ensures patients receive a minimum number of breaths per minute if their spontaneous breathing rate drops below the respiratory rate setting.

**Standby** Suspends ventilation and retains current settings when the clinician wants to temporarily disconnect the patient from the ventilator.

## Glossary

**Time Trigger** Initiation of inspiration by the ventilator according to the **Respiratory Rate** setting.

**Timed indicator** Denotes machine-triggered (mandatory) breathing.

**$T_I/T_{TOT}$**  Inspiratory duty cycle. Inspiratory time divided by total cycle time, averaged over 8 breaths, a monitored parameter.

**Tot.Leak** Estimated total leak, both intentional and unintentional. A monitored parameter shown when the mask leak and type of exhalation port are not known.

**Trigger** To begin inspiration.

**Trigger** Trigger Sensitivity, a control setting in Auto-Trak+.

**Trigger Sensitivity** See Trigger.

**Unintentional leakage** Unpredictable leakage that cannot be quantified.

**V** Volt, a unit of electrical potential *or* volume.

**$\dot{V}$**  Flow.

**$\dot{V}_E$**  Estimated minute ventilation. The product of tidal volume (spontaneous and timed) and rate (spontaneous and timed), a monitored parameter.

**$V_T$**  Estimated exhaled tidal volume, a monitored parameter and AVAPS Target Tidal Volume, a control setting in AVAPS mode.

# Index

---

## A

- AC power
  - how to connect to 5-2
  - specifications 11-8
- AC power cord, how to install A-7–A-8
- Accessories, part numbers C-1–C-4
- Air inlet filter. *See* Filter, air inlet
- Alarm LED
  - description 3-7
  - state during alarm conditions 9-3
- Alarm Reset button, function 9-5
- Alarm Silence button, function 9-5
- Alarm tests 5-9–5-10
- Alarm Volume Escalation
  - definition E-11
  - how to enable or disable E-11
  - status display 6-17
- Alarms
  - adjustable, specifications 11-4
  - Check Vent (high-priority), description 9-3
  - Check Vent, troubleshooting table 9-12–9-16
  - high-priority, description 9-3
  - how to clear autoreset alarms from the Alarms list 9-5
  - how to hide/display messages 9-6
  - how to manually reset 9-5
  - how to respond to 9-1
  - how to set 6-12
  - how to silence 9-5
  - loudness, how to adjust 6-17
  - low-priority, description 9-3
  - messages, list 9-7–9-17
  - See also name of specific alarm*
  - settings
    - how to change 6-12
    - list of descriptions and ranges 6-26–6-27
  - troubleshooting table 9-7–9-17
  - Vent Inoperative (high-priority), description 9-3
  - Vent Inoperative, troubleshooting table 9-16–9-17
  - visual alarm indications 9-2
- Alarms and messages** 9-1–9-17
- Assembly, ventilator A-2–A-8
- Auto-Trak sensitivity
  - description 3-3
  - principles of operation 4-3–4-6
- Auto-Trak+ option
  - description 4-6
  - how to change settings 6-18–6-19
  - settings and ranges 11-4
- AVAPS (average volume-assured pressure support) mode
  - precautions 1-1, 1-2
  - principles of operation 4-11–4-12
- AVAPS Maximum IPAP Pressure setting. *See* Max P
- AVAPS Minimum IPAP Pressure setting. *See* Min P
- AVAPS: Target  $V_T$  Exceeded. Min Pressure Too High message 9-7
- AVAPS: Target  $V_T$  Not Achieved. Insufficient Max Pressure

- message 9-7
- Average volume-assured pressure support mode. *See* AVAPS mode

## B

- Backup alarm 3-10
- Backup battery. *See* Battery, backup
- Bacteria filter must be installed onto gas outlet message 9-7
- Baseline pressure, description 4-2
- Batch settings, how to make 6-3, 6-19
- Battery, backup
  - description 3-10–3-11
  - how to install A-4–A-6
  - location 3-9
  - operation time 3-10
  - part number C-4
  - specifications 11-8
- Baud rate
  - for ventilator used with Philips monitor and VueLink Open Interface B-2
  - how to change E-10
- Breath frequency. *See* Rate
- Breath phase/trigger indicator, definition 8-2
- Breathing circuit. *See* Patient circuit
- Brightness, display, how to set 6-17
- Buzzer, backup 3-10

## C

- Calibration, touchscreen E-14
- Cannot Reach Target Flow alarm 7-7, 9-7
- Care and maintenance** 10-1–10-9
- C-Flex setting, definition 6-23
- Check Vent alarm, description 9-3
- Check Vent: 1.8 V Supply Failed alarm 9-12
- Check Vent: 12 V Supply Failed alarm 9-12
- Check Vent: 24 V Supply Failed alarm 9-12
- Check Vent: 3.3 V Supply Failed alarm 9-12
- Check Vent: 35 V Supply Failed alarm 9-13
- Check Vent: 5 V Supply Failed alarm 9-12
- Check Vent: Air Flow Sensor Calibration Data Error alarm 9-13
- Check Vent: Alarm LED Failed alarm 9-13
- Check Vent: Auxiliary Alarm Supply Failed alarm 9-13
- Check Vent: Backup Alarm Failed alarm 9-13
- Check Vent: Barometer Calibration Data Error alarm 9-13
- Check Vent: Barometer Sensor Range Error alarm 9-13
- Check Vent: Battery Failed alarm 9-13
- Check Vent: Battery Temperature High alarm 9-13
- Check Vent: Blower Stalled 9-14
- Check Vent: Blower Temperature High alarm 9-14
- Check Vent: Cooling Fan Speed Error alarm 9-14
- Check Vent: CPU PCBA ADC Failed alarm 9-14
- Check Vent: Data Acquisition PCBA ADC Failed alarm 9-14
- Check Vent: Flash File System Error alarm 9-14
- Check Vent: Internal Temperature High CPU alarm 9-14
- Check Vent: Internal Temperature High Daq alarm 9-14

## Index

Check Vent: Internal Temperature High Mtr alarm 9-14  
Check Vent: Machine Pressure Sensor Auto-Zero Failed alarm 9-14  
Check Vent: Machine Pressure Sensor Calibration Data Error alarm 9-15  
Check Vent: Machine Pressure Sensor Range Error alarm 9-15  
Check Vent: Motor Control PCBA ADC Failed alarm 9-15  
Check Vent: O<sub>2</sub> Flow Sensor Calibration Data Error alarm 9-15  
Check Vent: O<sub>2</sub> Pressure Sensor Calibration Data Error alarm 9-15  
Check Vent: O<sub>2</sub> Supply Pressure Sensor Range Error alarm 9-15  
Check Vent: OVP Circuit Failed alarm 9-15  
Check Vent: Oxygen Device Failed alarm 9-15  
Check Vent: Primary Alarm Failed alarm 9-15  
Check Vent: Program CRC Test Failed alarm 9-15  
Check Vent: Proximal Pressure Sensor Auto-Zero Failed alarm 9-15  
Check Vent: Proximal Pressure Sensor Calibration Data Error alarm 9-15  
Check Vent: Proximal Pressure Sensor Range Error alarm 7-7, 9-16  
Circuit, patient. *See* Patient circuit  
Cleaning  
    ventilator exterior 10-2  
CO<sub>2</sub> rebreathing precautions 1-2, 1-4, 1-7, 1-8, 3-2  
**Communications interface** B-1–B-6  
    using the ventilator with Philips monitors and VueLink Open Interface B-3–B-5  
Connector  
    oxygen inlet  
        how to install A-7  
        location 3-9  
    remote alarm/nurse call, location 3-9  
    RS-232 serial and analog I/O, location 3-9  
Continuous positive airway pressure. *See* CPAP  
Continuous-tone alarm 3-10  
Contraindications 3-2  
Control settings  
    how to change 6-3, 6-4, 6-19  
    how to make batch setting changes 6-19  
    list of descriptions and ranges 6-23–6-26  
    ranges, resolutions, and accuracies 11-1–11-2  
    *See also name of specific setting*  
Cooling fan filter. *See* Filter, cooling fan  
CPAP (continuous positive airway pressure) mode 4-8  
CPAP (continuous positive airway pressure) setting, definition 6-23

## D

Date and time, how to change E-6  
Default factory settings, how to restore E-8  
Definitions F-1–F-6  
Delivered pressure specifications 11-10  
**Diagnostic mode** E-1–E-15  
    how to calibrate touchscreen E-14–E-15  
    how to change baud rate E-10  
    how to change date and time E-6

    how to change language E-4–E-5  
    how to change pressure unit E-7  
    how to enable or disable Alarm Volume Escalation E-11  
    how to enable software options E-9  
    how to enter E-1  
    how to exit E-15  
    how to restore default factory settings E-8  
    how to view event log E-12–E-13  
Dimensions, ventilator 11-6  
Disinfecting  
    ventilator exterior 10-3  
Display, brightness, how to set 6-17  
Disposal 10-8

## E

E-cycle (expiratory cycle Sensitivity) setting, definition 6-23  
Elastance, definition 4-13  
Electrical specifications 11-7, 11-8  
Electromagnetic compatibility (EMC), compliance with standards D-1  
Environmental specifications 11-6  
EPAP (expiratory positive airway pressure) setting, definition 6-23  
Estimated exhaled tidal volume. *See* V<sub>T</sub>  
Estimated minute ventilation. *See* V<sub>E</sub>  
Estimated total leak. *See* Tot.Leak  
Event log, how to view E-12–E-13  
Exhalation port  
    how to select 6-12–6-15  
    part numbers C-2  
Exhalation port test, how to run 6-16–6-17  
Exhale breath phase indicator, definition 8-2  
Expiratory positive airway pressure. *See* EPAP

## F

Fan filter. *See* Filter, cooling fan  
Filter  
    air inlet  
        how to install 10-6  
        maintenance schedule 10-5  
    cooling fan  
        how to clean or replace 10-7  
        location 3-9  
        maintenance schedule 10-5  
        part number C-4  
    main flow (inspiratory) bacteria, part numbers C-4  
**First-time installation** A-1–A-9  
Fitting. *See* Connector  
Flow assist/maximum resistance setting. *See* Max R  
Frequency, breath. *See* Rate

## G

**General information** 3-1–3-12  
**Glossary** F-1–F-6  
Graphical user interface  
    how to calibrate E-14  
    how to use 3-13  
    illustration 3-12  
GUI. *See* Graphical user interface

**H**

- Help feature, description 6-22
- Hi Rate (High Rate Alarm) setting, definition 6-26
- Hi  $V_T$  (High Tidal Volume Alarm) setting, definition 6-26
- High Flow Therapy** 7-1–7-7
- High flow therapy (HFT)
  - alarms and messages 7-6–7-7
  - changing from HFT to NIV 7-6
  - changing from NIV to HFT 7-4–7-5
  - description 3-3
  - flow specifications 11-1
  - HFT flow graph 7-5
  - precautions 1-7–1-8, 7-1
  - principles of operation 4-6–4-7
  - specifications 11-10
- High Inspiratory Pressure alarm 9-7
- High  $O_2$  Supply Pressure alarm 9-7
- High Rate alarm 9-8
- High Tidal Volume alarm 9-8
- High-priority alarm, description 9-3
- HIP (High Inspiratory Pressure Alarm) setting, definition 6-26

**I**

- Informational message, description 9-3
- Inlet filter. *See* Filter, air inlet
- Inspiration, triggering, volume method 4-3
- Inspiratory duty cycle. *See*  $T_I/T_{TOT}$
- Inspiratory positive airway pressure. *See* IPAP
- Inspiratory time setting. *See* I-Time
- Installation
  - AC power cord A-8
  - air inlet filter 10-6
  - cooling fan filter 10-7
  - oxygen inlet connector A-7
- Intake filter. *See* Filter, air inlet
- IntelliBridge
  - using with Philips monitors B-2–B-5
- Intended use, Respironics V60 Ventilator 3-1
- Interface. *See* Patient interface *or* Communications interface
- IPAP (inspiratory positive airway pressure) setting, definition 6-23
- I-Time (inspiratory time) setting, definition 6-24

**L**

- Language, how to change E-4–E-5
- Leak adaptation, principles of operation 4-5–4-6
- Leak, estimated total. *See* Tot.Leak
- LIP (Low Inspiratory Pressure Alarm) setting, definition 6-26
- LIP T (Low Inspiratory Pressure Delay Time) setting, definition 6-27
- Lo Rate (Low Rate Alarm) setting, definition 6-26
- Lo  $V_E$  (Low Minute Ventilation Alarm) setting, definition 6-27
- Lo  $V_T$  (Low Tidal Volume Alarm) setting, definition 6-26
- Locking and unlocking the screen 6-18
- Log, event, how to view E-12–E-13
- Loudness, how to set alarm 6-17

- Low Inspiratory Pressure alarm 9-8
- Low Inspiratory Pressure Alarm setting. *See* LIP
- Low Inspiratory Pressure Delay Time setting. *See* LIP T
- Low Internal Battery alarm 9-8
- Low Leak-- $CO_2$  Rebreathing Risk alarm 9-8
- Low Minute Ventilation alarm 9-9
- Low Minute Ventilation Alarm setting. *See* Lo  $V_E$
- Low  $O_2$  Supply Pressure alarm 9-9
- Low Rate alarm 9-9
- Low Tidal Volume alarm 9-9
- Low-priority alarm, description 9-3

**M**

- Main flow bacteria filter. *See* Filter, main flow bacteria
- Mains power
  - how to connect to 5-2
  - specifications 11-8
- Maintenance. *See* Care and maintenance
- Mask x, Exh Port y message 9-9
- Mask, part numbers C-2
- Max E (maximum elastance/volume assist) setting, definition 6-24
- Max P (AVAPS Maximum IPAP Pressure) setting, definition 6-24
- Max P (PPV Maximum Pressure Limit) setting, definition 6-24
- Max R (maximum resistance/flow assist) setting, definition 6-24
- Max V (PPV Maximum Volume Limit) setting, definition 6-24
- Maximum elastance/volume assist setting. *See* Max E
- Maximum resistance/flow assist setting. *See* Max R
- Menu window, functions 6-17, 6-18
- Message, informational, description 9-3
- Min P (AVAPS Minimum IPAP Pressure) setting, definition 6-25
- Minute ventilation. *See*  $V_E$
- Modes, ventilation. *See* Ventilation modes
- Monitored parameters
  - definitions 8-2
  - ranges, resolutions, and accuracies 11-3
- Mounting the ventilator A-2–A-3

**N**

- Nurse call connector, location 3-9
- Nurse call port B-5–B-6

**O**

- $O_2$  setting, definition 6-25
- Operation 6-1, 6-18
- Operator replacement parts, part numbers C-4
- Options, software, how to enable E-9
- Oxygen
  - conserving during transport 5-11
  - duration of cylinder 5-12
  - how to connect 5-1
  - mixing, principles of operation 4-15
- Oxygen inlet connector
  - how to install A-7
  - location 3-9

## Index

Oxygen Not Available alarm 9-10

### P

**Parts and accessories** C-1–C-4

Patient alarms are disabled during HFT message 7-7, 9-10

Patient circuit

how to install 5-3–5-7

part numbers C-3

specifications 11-8

Patient Circuit Occluded alarm 7-7, 9-10

Patient data

ranges, resolutions, and accuracies 11-3

See *also* Monitored parameters

symbols for invalid or out of range data 8-1

Patient Disconnect alarm 9-10

Patient interface

how to select for leak adjustment purposes 6-12–6-15

See Masks

Patient leak. See Pt. Leak 8-2

**Patient monitoring** 8-1–8-3

Patient-triggered breaths. See Pt. Trig

PCV (pressure-controlled ventilation) mode, principles of operation 4-9

Peak inspiratory pressure. See PIP

PIP (peak inspiratory pressure) monitored parameter, definition 8-2

PM. See Preventive maintenance

Port, exhalation

how to select 6-12–6-15

how to test 6-16–6-17

part numbers C-2

Power cord

how to install A-8

location 3-9

Power cord retainer, location 3-9

Power has been restored message 9-11

Power indicators, illustration 3-11

PPV (proportional pressure ventilation) mode

adjusting settings 6-7

principles of operation 4-13–4-14

setup 6-7–6-11

PPV % setting, definition 6-25

PPV gain setting. See PPV % setting

PPV Max P alarm 9-11

PPV Max V alarm 9-11

PPV Maximum Pressure Limit setting. See Max PPV

PPV Maximum Volume Limit setting. See Max V

Precautions

alarms and messages 1-8

AVAPS 1-1

backup battery 5-8

care and maintenance 1-8–1-9

cleaning, disinfecting, and sterilizing 10-2, 10-3

CO<sub>2</sub> rebreathing 1-2, 1-4, 1-7, 1-8, 3-2

communications interface 1-9–1-10

Diagnostic mode 1-10

first-time installation 1-9

general 1-1–1-4

high flow therapy (HFT) 1-7, 7-1

operation 1-7

packing and shipping 10-9

preparing for ventilation 1-4–1-7

preventive maintenance 10-4

remote alarm B-5

**Preparing for ventilation** 5-1–5-13

Pressure Regulation High alarm 9-11

Pressure unit, how to change E-7

Pressure, baseline, description 4-2

Pressure-controlled ventilation mode. See PCV

Preventive maintenance 10-4–10-8

schedule 10-5

**Principles of operation** 4-1–4-15

Proportional pressure ventilation mode. See PPV mode

Proximal Pressure Line Disconnect alarm 9-12

Pt. Leak (patient leak) monitored parameter 8-2

Pt. Trig (patient-triggered breaths) monitored parameter 8-2

### R

Ramp Time setting, definition 6-25

Rate (Respiratory Rate) setting, definition 6-25

Rate monitored parameter, definition 8-2

**Regulatory compliance** D-1–D-9

Remote alarm cable kits B-6

Remote alarm port B-5–B-6

Remote alarm/nurse call connector, location 3-9

Repacking and shipping 10-9

Resistance, definition 4-13

Respi-Link

description 3-4

location of Ethernet connector 3-9

Respiratory rate. See Rate

Respironics V60 Ventilator

front view 3-7

physical description 3-4–3-9

potential side effects 3-2

rear view 3-9

side view 3-8

system overview 3-3–3-4

Rise (Rise Time) setting, definition 6-25

RS-232 serial port B-2

location B-1

Running on Internal Battery alarm 9-12

### S

S/T (spontaneous/timed) mode, principles of operation 4-10

Safety, compliance with standards D-8

Scalars. See Waveforms

Screen lock 6-18

Settings

default factory, how to restore E-8

See *also* Control settings, Alarm settings, or *name of specific setting*

Shutdown, ventilator 3-15

Software options, how to enable E-9

Specifications. See Technical specifications

Spont (spontaneous) breath type indicator, definition 8-2

Spontaneous/timed mode. See S/T mode

Stand, ventilator

mounting the ventilator to A-2–A-3

Standby 6-20–6-21

Startup

time specifications 11-10

ventilator startup 3-15

**Symbols** 2-1–2-9

System Settings functions, in diagnostic mode E-3

## T

**Technical specifications** 11-1–11-9

alarm settings 11-4

control setting ranges, resolutions, and accuracies 11-1–11-2

delivered pressure 11-10

dimensions 11-6

electrical 11-7, 11-8

environmental 11-6

high flow therapy (HFT) 11-1, 11-10

monitored parameter ranges, resolutions, and accuracies 11-3

patient circuit 11-8

patient data ranges, resolutions, and accuracies 11-3

ventilator startup time 11-10

weight 11-6

Tests

alarm tests 5-9–5-10

exhalation port 6-16–6-17

$T_I/T_{TOT}$  (duty cycle) monitored parameter, definition 8-2, B-3

Tidal volume, estimated exhaled. *See*  $V_T$

Time and date, how to change E-6

Timed breath type indicator, definition 8-2

Tot.Leak (estimated total leak) monitored parameter, definition 8-2

Total leak. *See* Tot.Leak

Touchscreen

how to calibrate E-14

Training information 3-15

Trigger (trigger sensitivity) setting, definition 6-26

Trigger +x, E-Cycle +x message 9-12

Triggering

principles of operation 4-3

volume method 4-3

Troubleshooting alarms 9-7–9-17

## U

Unpacking and inspection A-1

Using Default Settings message 9-12

## V

Vent Info (Ventilator Information) window, description 6-18

Vent Inoperative 1000 3.3 V Supply Failed alarm 9-16

Vent Inoperative 1001 12 V Supply Failed alarm 9-16

Vent Inoperative 1002 Blower Temperature Too High alarm 9-16

Vent Inoperative 1003 Internal Temperature High alarm 9-16

Vent Inoperative 1004 Internal Temperature High alarm 9-16

Vent Inoperative 1005 Internal Temperature High alarm 9-17

Vent Inoperative 1006 Data Acquisition PCBA ADC Failed alarm 9-17

Vent Inoperative 1007 Machine and Proximal Pressure Sen-

sors Failed alarm 9-17

Vent Inoperative 1008 Machine and Proximal Pressure Sensors Failed alarm 9-17

Vent Inoperative 1009 Pressure Regulation High alarm 9-17

Vent Inoperative 100A Data Acquisition PCBA ADC Reference Failed alarm 9-17

Vent Inoperative 100B Watchdog Test Failed alarm 9-17

Vent Inoperative alarm, description 9-3

Ventilation modes

how to change 6-2

principles of operation 4-7–4-14

Ventilator breathing circuit. *See* Patient circuit

Ventilator settings. *See* Control settings *or* Alarm settings

Verifying ventilator operation 5-8

Volume

AVAPS Target Tidal Volume. *See*  $V_T$

estimated exhaled tidal. *See*  $V_T$

waveform. *See* Waveforms

Volume assist/maximum elastance setting. *See* Max E

$V_T$  (AVAPS Target Tidal Volume) setting, definition 6-26

$V_T$  (estimated exhaled tidal volume) monitored parameter, definition 8-2

VueLink Open Interface, using with the ventilator and Philips monitors B-3–B-5

## W

**Warnings, cautions, and notes** 1-1–1-10

Waveforms

description 8-2

how to freeze and unfreeze 8-3

how to scale axes 8-2

WEEE recycling directive, compliance statement D-7

Weight, ventilator 11-6

## Index



[www.philips.com/healthcare](http://www.philips.com/healthcare)  
[healthcare@philips.com](mailto:healthcare@philips.com)



Respironics California, LLC  
2271 Cosmos Court  
Carlsbad, CA 92011 USA



**Australian sponsor**

Philips Electronics Australia Ltd  
65 Epping Road  
North Ryde, NSW Australia 2113

You can find the most current electronic copy of this user manual at:  
[www.philips.com/IFU](http://www.philips.com/IFU)

This manual documents software versions 2.30, 3.00, and 3.10.



© 2009 - 2022 Koninklijke Philips N.V.

All rights are reserved. Reproduction or transmission in whole or in part, in any form or by any means, electronic, mechanical or otherwise, is prohibited without the prior written consent of the copyright owner.

Printed in USA  
1047358 Rev AB \* 2022-10



989805612141