# HEARTSTART FRx

INFANT/CHILD KEY

CLE DE DEFIBRILLATION PEDIATRIQUE

KINDERSCHLÜSSEL

BABY-/KINDERSLEUTEL

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## ENGLISH: About This Edition

The information in this guide applies to the Infant/Child Key 989803139311 for the FRx defibrillator. This information is subject to change. Please contact Philips at [www.philips.com/productdocs](http://www.philips.com/productdocs) for information on revisions.

### Edition History

#### Edition 19

Publication Date: September 2020

Publication Number: 453564918231 (REV II)

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No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Unauthorized copying of this publication may not only infringe copyright but also reduce the ability of Philips Medical Systems to provide accurate and up-to-date information to users and operators alike.

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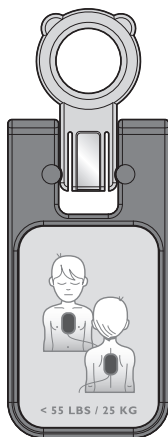
Philips Medical Systems  
22100 Bothell Everett Highway  
Bothell, WA 98021-8431, USA

INTENTIONALLY BLANK  
INTENTIONNELLEMENT LAISSE EN BLANC  
DIESE SEITE BLEIBT FREI.  
OPZETTELIJK ONBEDRUKT GELATEN  
ESTE ESPACIO SE HA DEJADO EN BLANCO DELIBERADAMENTE  
PAGINA LASCIATA INTENZIONALMENTE VUOTA  
PÁGINA DEIXADA EM BRANCO INTENCIONALMENTE  
SIDAN ÄR AVSIKTLIGEN TOM  
SKAL VÆRE TOM  
JÄTETTY TARKOITUKSELLA TYHJÄKSI  
DENNE SIDE ER TILSIGTET TOM  
TĘ STRONĘ UMYŚLNIE POZOSTAWIONO PUSTĄ  
ZÁMĚRNĚ PONECHÁNO PRÁZDNÉ  
СТРАНИЦА ОСТАВЛЕНА ПУСТОЙ НАМЕРЕННО  
**ΣΕΛΙΔΑ ΣΚΟΠΙΜΑ ΚΕΝΗ**  
BOŞ BIRAKILMIŞTIR  
刻意留白  
刻意留空  
このページは空白です  
이 페이지는 공백 페이지입니다  
หน้านี้ไม่มีข้อมูล  
فارغ عن قصد

## REF989803 | 393 | I INFANT/CHILD KEY

*Infant/Child Key, front.*

The Infant/Child Key is for use *only* with the HeartStart FRx Defibrillator to treat an infant or child who is under 55 pounds (25 kg) or 0-8 years old. If you are uncertain of the child's exact weight or age, apply the defibrillator *without* the Infant/Child Key. For instructions on using the FRx Defibrillator, refer to the FRx Defibrillator *Owner's Manual* or *Quick Reference Guide*.

**WARNING:** Most cardiac arrests in children are not caused by heart problems. When responding to cardiac arrest in an infant or child:

- Provide infant/child CPR while a bystander calls EMS and brings the FRx.
  - If no bystander is available, provide 1-2 minutes of CPR before calling EMS and retrieving the FRx.
  - If you witnessed the child's collapse, call EMS immediately and then get the FRx.
- Alternatively, follow your local protocol.


With the Infant/Child Key inserted, the HeartStart FRx will announce "Infant/Child Mode," automatically reduce the defibrillation energy, and provide optional infant/child CPR guidance.

No special defibrillator pads are required to use the HeartStart FRx to treat an infant or child; the HeartStart SMART Pads II provided with the defibrillator are designed for both adult and infant/child use.

**NOTE:** It does not matter whether you insert the Infant/Child Key before or immediately after turning on the FRx. However, the Key should be inserted *before* placing the pads on the patient. If the Infant/Child Key is removed during use, the FRx will announce “Adult Mode.” Any shocks delivered will be adult energy and optional CPR guidance will be for adults.

## TO USE THE KEY

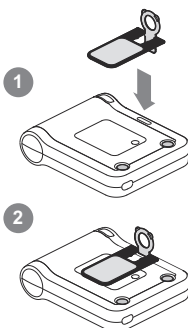Remove the Key from its pocket in the carry case.


*Insert the Key.*

Insert the Key into the slot at the top center of the front panel of the FRx Defibrillator. Insert the Key as shown on the back of the Key. The pink portion of the Key pivots (1) and fits into the slot (2), with the front of the Key lying flat on the surface of the FRx so the pediatric pads placement diagram is visible.

The pads icons on the Key start flashing when the defibrillator is turned on and the Key is inserted.

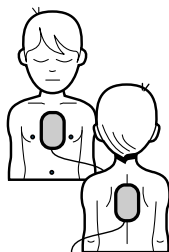**IMPORTANT: Use the illustration on the Infant/Child Key as a guide to correct pads placement.** This placement is different from the standard placement illustrated on the pads for adults and older children.

Place pads.

Place pads, one on the chest and one on the back. *It does not matter which pad is placed on the chest or the back.* Once pads are placed, the pads icons stop flashing. Follow the FRx Defibrillator's voice instructions. It will talk you through each step of using it.

**WARNING:** Do not leave the Infant/Child Key inserted in the FRx Defibrillator when it is not being used in an emergency. This will cause the FRx to start chirping. In addition, the lower energy provided when the Key is inserted is only appropriate for treating patients under 55 pounds (25 kg) or 0-8 years old.

**TRAINING.** The Infant/Child Key can be used with HeartStart Training Pads II to create a training scenario for treating infants and children. CPR for infants and children is different than for adults. It is strongly recommended that anyone who may use the Infant/Child Key be trained in infant/child CPR.

**STORAGE.** Keep the Infant/Child Key in its pocket in the carry case.

**MAINTENANCE.** The Infant/Child Key can be cleaned with a soft cloth dampened in soapy water, chlorine bleach (2 tablespoons per quart or liter of water), an ammonia-based cleaner, or isopropyl alcohol (70% solution).

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Niets uit deze uitgave mag in enige vorm of op enige manier worden verveelvoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
Philips Japan, Ltd.
Philips Medical Systems Nederland B.V.