PHILIPS

Trilogy Evo Trilogy EV300

Device software update guide

Languages

- 1 English
- 5 Español
- 9 Português BR

For Homecare Users / DME Technicians

For homecare users / DME technicians, software for Trilogy Evo, Trilogy Evo O2, and Trilogy Evo Universal can be downloaded from the My Philips for Professionals (MyP4P) website: <u>https://www.my.philips.com/</u>. If any issues are encountered when registering your account or downloading the software, please contact: **respironics.service.operations@philips.com**.

Note: MyP4P is not compatible with Internet Explorer. Access MyP4P using Microsoft Edge, Chrome, or Firefox browsers.

MyP4P

If you have not previously created an account on My Philips for Professionals (MyP4P), you will need to register before downloading software.

- 1. Go to the following link: <u>Registration | Philips</u>
- To create an account, provide your relevant Personal Information and Organization Information. Fill out the required fields and click Save for each of the two sections. If information is completed accurately, green check marks will appear.
- **3.** Click **Submit** to complete the registration request.
- **4.** Once your registration is approved, you will receive an email with instructions to activate your account.
- 5. Create a password. If information is completed accurately, green check marks will appear. Click **Submit**.
- 6. Once the password has been submitted, you can click on the link to MyP4P to choose your SRC groups. This will determine what types of documents you will have access to.
- 7. For specialty, select Sleep Therapy and Respiratory Care. For groups, select the SRC groups.
- 8. Select **Request Access** for each group you need to access. For the Service Software group, open the ULA first and check the box before you can click **Request Access**.
- 9. Enter your account number.
- **10.** A banner will appear at the top confirming access has been requested. When you are approved for the group(s) you signed up for, you will receive a confirmation email.

Upgrade device software using USB

A. Download software from MyP4P

- 1. Log on to <u>MyP4P</u> with your customer account and password.
- 2. Click on the Group Documents tab.
- **3.** Use the search tool and type: Trilogy Evo, Trilogy Evo O2, Trilogy EV300.
- **4.** Using the Filters on the left under Document Type, select **Software update**.
- **5.** Identify the appropriate software version approved for use in your country. Read the description to be sure you are permitted to download a particular software revision.
- 6. Click on the applicable software version update. The file will automatically download.

Note: The file is in a compressed format (.zip).

7. Save the .zip file to a known location.

B. Download software to USB

1. Connect a USB flash drive to the PC.

Note: Minimum USB memory size should be 2 GB or greater.

- **2.** Extract/unzip the software update file. Once completed, an .exe file will be available.
- **3.** Run the .exe file to self-extract the software update file.
- **4.** During the self-extraction, select the USB flash drive location to unzip the file to.
- 5. Winzip will copy the TrilogyEvo.upg file to the USB flash drive. Acknowledge the prompts and close WinZip.

Note: Confirm the USB flash drive does not contain any other files, except the software file, which should be named **TrilogyEvo.upg**.

6. The USB flash drive should now contain the software update file necessary to upgrade the device's software. Remove the USB flash drive from the PC.

C. Upgrade device software

1. Connect device to AC power. Press On/Off (Standby) 🕐 button.

Note: If AC power is not available, you can update the software using battery power.

- 2. Insert the USB flash drive in any of the two Ψ USB ports of your device.
- 3. On the user interface, go to **Options** $\partial_{i}^{j}\mathcal{B}$, then **Data Transfer**.
- **4.** The device should recognize the USB flash drive and display the software version on the "Install Software Update" box. Confirm the intended software version is shown.

Note: If the software version is not shown, then the USB stick was not properly inserted, or no Trilogy Evo software was found.

- 5. Select Install Software Update. Select Yes to confirm.
- 6. Please wait while your device is installing the new software.
- Confirmation the software installation was completed will display. Press OK and the device will reboot.
- **8.** After the device reboots, the software upgrade process is complete. The USB flash drive can be removed.
- **9.** Press **On/Off (Standby) U** button. Confirm the intended software version is shown.

For Hospital Users

For hospital users, software for Trilogy EV300 ventilators can be downloaded from InCenter: <u>https://philips.mizecx.com/login.html</u>

InCenter

If you have not previously created an account in InCenter, you will need to register before downloading software.

- 1. Send an email to **PCCI_CS_OPS@philips.com** that includes the following information:
 - Subject line: Request for access to InCenter Service P&S and Software Downloads for Respiratory Care-Ventilators.
 - Customer name (company/institution)
 - Street address, city, state, postal/zip code, country
 - Telephone number
 - Email address
 - Ventilator serial number (to confirm that the request is from a valid customer)
- 2. The InCenter team will process your request and email you a temporary password.

Upgrade device software using USB

A. Download software from InCenter to PC

- 1. Log on to <u>InCenter</u>. If you are logging in with a temporary password, you must first create a new password.
- 2. From the product tree section, select: Hospital Respiratory Care > Ventilation > Trilogy
- 3. Select the Software tab, and then select Software Downloads.
- **4.** Identify the appropriate software version approved for use in your country. Read the description to be sure you are permitted to download a particular software revision.
- 5. Click Save As and download the file to a known location.

Note: The file is in a compressed format (.zip).

B. Download software to USB

1. Connect a USB flash drive to the PC.

Note: Minimum USB memory size should be 2 GB or greater.

- **2.** Extract/unzip the file. Once completed, an .exe file will be available.
- **3.** Run the .exe file to self-extract the software update file.
- **4.** During the self-extraction, select the USB flash drive location to unzip the file to.
- 5. Winzip will copy the TrilogyEvo.upg file to the USB flash drive. Acknowledge the prompts and close WinZip.

Note: Confirm the USB flash drive does not contain any other files, except the software file, which should be named **TrilogyEvo.upg**.

6. The USB flash drive should now contain the software update file necessary to upgrade the device's software. Remove the USB flash drive from the PC.

C. Upgrade device software

1. Connect device to AC power. Press On/Off (Standby) 🕐 button.

Note: If AC power is not available, you can update the software using battery power.

- Insert the USB flash drive in any of the two USB ports of your device.
- 3. On the user interface, go to **Options** $\partial_{i} \mathcal{B}$, then **Data Transfer**.
- The device should recognize the USB flash drive and display the software version on the "Install Software Update" box. Confirm the intended software version is shown.

Note: If the software version is not shown, then the USB stick was not properly inserted, or no Trilogy Evo software was found.

- 5. Select Install Software Update. Select Yes to confirm.
- 6. Please wait while your device is installing the new software.
- 7. Confirmation the software installation was completed will display. Press **OK** and the device will reboot.
- **8.** After the device reboots, the software upgrade process is complete. The USB flash drive can be removed.
- 9. Press On/Off (Standby) 🕐 button. Confirm the intended software version is shown.

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© 2024 Koninklijke Philips N.V. Reservados todos los derechos.
Respironics Inc. 1001 Murry Ridge Lane Murrysville, PA 15668 USA