Instructions for Use - English

Pico Traditional Nasal Mask

Intended Use

This nasal mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. The mask is for single patient use in the home or multi-patient use in the hospital/institutional environment. The mask is to be used on patients (>66lbs/30kg) for whom CPAP or bi-level therapy has been prescribed.

Notes:


- Exhalation is built into this mask. A separate exhalation device is not required.
- This mask is not made with natural rubber latex or DEHP.

Warnings

- This mask is not suitable for providing life support ventilation.
- Hand wash prior to use. Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.
- Consult a physician or dentist if you encounter tooth, gum, or jaw soreness. Use of a mask may aggravate an existing dental condition.
- Consult a physician if you experience the following symptoms while using the mask or after removing it: Drying of the eyes, eye pain, eye infections, or blurred vision. Consult an ophthalmologist if symptoms persist.
- Do not block or seal off the exhalation ports.
- Do not overtighten the headgear straps. Watch for signs of overtightening, such as excessive redness, sores, or bulging skin around the edges of the mask. Loosen the headgear straps to alleviate problems.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. **Do not block or try to seal the exhalation port.** Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. This warning applies to most models of CPAP systems.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. Explanation of the Warning: When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- At low CPAP or EPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- Attaching an exhalation device requires therapy pressure level adjustment to compensate for increased leak.

Note: Any serious incident that has occurred in relation to this device should be reported to Philips and the competent authority of the Member State in which the user and/or patient is established.

Features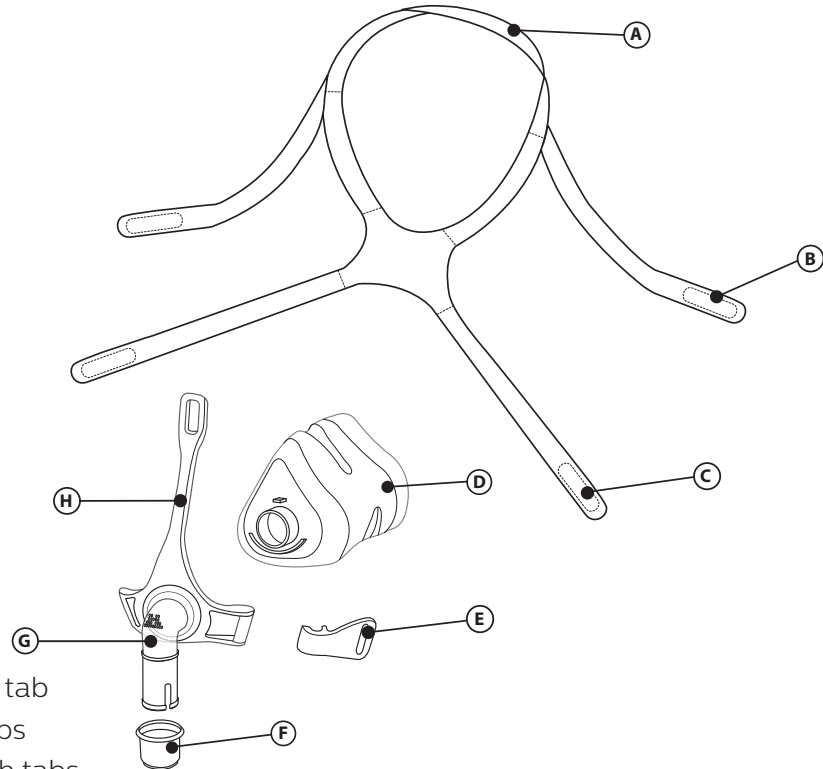

- Ⓐ Headgear crown strap with tab
- Ⓑ Headgear top strap with tabs
- Ⓒ Headgear bottom strap with tabs
- Ⓓ Mask cushion
- Ⓔ Clip
- Ⓕ Tubing quick release
- Ⓖ Elbow with built-in exhalation ports (Do not block)
- Ⓗ Mask frame

Accessories

The following accessories are available for use with this mask. Contact Philips Respironics or your healthcare provider for additional information.

Part #	Description
1006085	Deluxe Chin Strap
1012911	Premium Chin Strap

Before Use Read and Understand the Instructions Completely.

- Hand wash the entire mask ❶.
- Wash your face. Do not use moisturizer/lotion on your hands or face.
- Inspect the mask for damage or wear. Discard and replace any damaged or worn parts.

Cleaning Instructions

Hand wash the entire mask before first use ❶. The non-fabric parts should be hand washed daily. The fabric parts should be hand washed weekly.

1. Hand wash in warm water with a liquid dish soap.

⚠Cautions:

- Do not place the fabric parts into the dryer.
- Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.
- Any deviation from these instructions may impact the performance of the product.

- Inspect the entire mask for damage or wear (cracking, crazing, tears, damage, etc). Discard and replace any parts as necessary.
1. Rinse thoroughly. Air dry completely before use. Make sure the entire mask is dry before use. Lay the fabric parts flat or line dry.

Dishwashing Instructions

In addition to hand washing, the mask may be cleaned in the dishwasher once a week.

⚠ Caution: Use a liquid dish washing detergent only to wash the mask.


1. Remove the headgear. Do not wash the headgear in the dishwasher.
2. Separate the cushion and the frame and wash in the top shelf of the dishwasher.
3. Air dry. Make sure the mask is dry before use.

⚠ Warning: Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any parts as necessary.

Institutional Disinfection

Clean and disinfect the mask daily and between patients when in use in a clinical setting. For multi-patient use in the hospital/institutional environment, use the Disinfection Guide to reprocess the mask between patients.

Achieving the Right Fit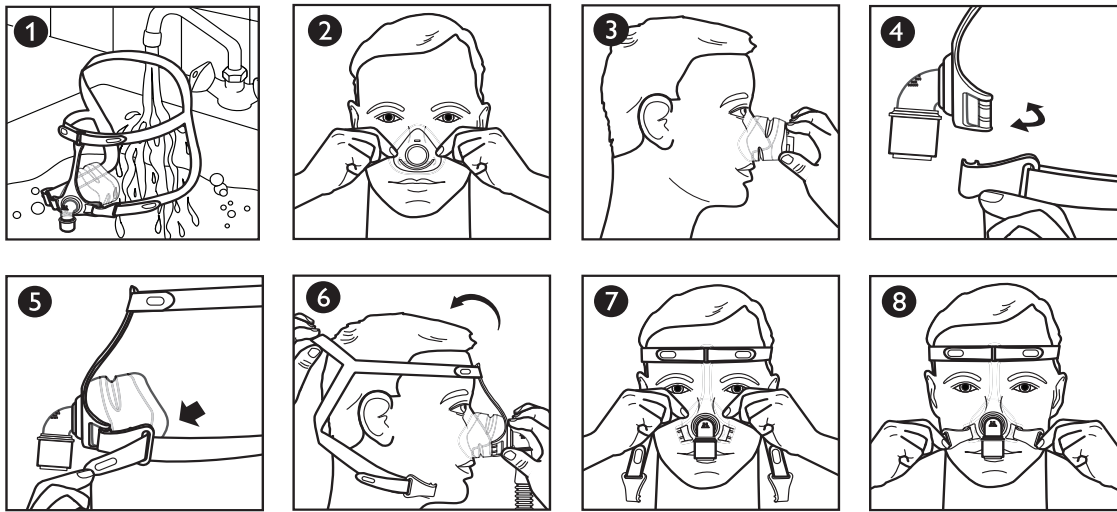

Before Putting on the Mask

1. Sizing: The cushion should fit the width of your nose without blocking the nostrils **2**. The top of the cushion may sit between the bridge and tip of your nose **3**.


Putting on the Mask

2. Clip: To disconnect turn like a key **4**.
3. Headgear tab: To undo peel away from the headgear strap **5**.
4. Headgear: Pull the headgear on over your head **6**. Then, pull the headgear crown strap tab over to lift the straps off of your ears.
5. Cushion: Place the cushion over your nose **7**.
6. Clip: To connect, press onto the mask frame **4**.
7. Headgear tab: Press tab against the headgear strap to reattach **8**.

Adjusting the Mask

8. Headgear: Adjust the headgear crown strap to reduce leaks around your nose and lift the straps off of your ears **9**.
9. Headgear: Adjust the top straps evenly **10**.
10. Headgear: Adjust the bottom straps evenly **11**.

 **Note:** Do not overtighten the headgear.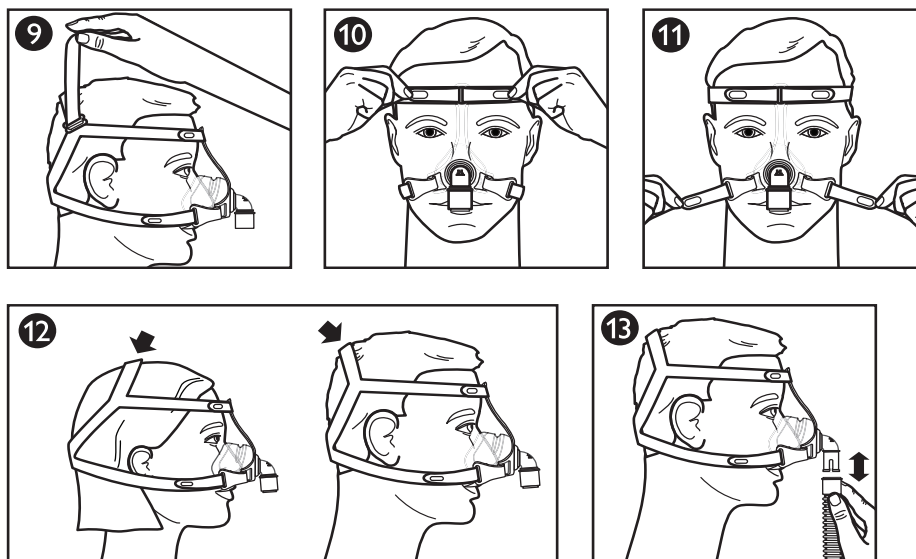

The location of the headgear crown strap may vary depending on head size. Position the strap so the mask sits comfortably 12.

Using the Mask

11. Connect the flexible tubing (included with CPAP or bi-level device) to the quick release swivel 13.
12. Turn the therapy device on. Lie down. Breathe normally.
13. Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments. Some air leaking is normal.

Tubing Quick Release

The elbow is equipped with a tubing quick release for the CPAP tubing. Pull on the tubing quick release and it will disengage from the elbow and come loose with the tubing 15.


* Comfort Tips

- The most common mistake is overtightening the headgear. The headgear should fit loose and comfortable. If your skin bulges around the mask or if red marks appear on your face, loosen the headgear.
- Re-seat the mask by pulling it directly away from your face and gently setting it back into place.

Removing the Mask

Disconnect a bottom clip or undo a headgear tab. Slide the headgear forward over your head.

Disassembly/Assembly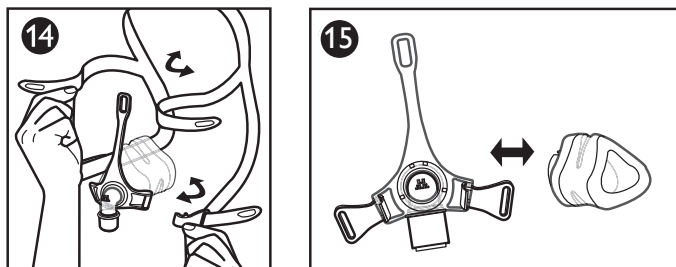

Removing the Headgear 14

Peel away the tabs and pull through the slots in the mask frame or clip.

Removing the Cushion 15

Pull cushion away from the frame.

Attaching the Cushion 15

1. Align the cushion size (S/M, L, XL) mark on the bottom of the cushion to the bottom edge of the frame.
2. Press cushion onto frame until it clicks into place.

Attaching the Headgear 14

1. With the crown strap upward hold the top straps together.

Note: The Philips Respironics logo is on the outside back of the headgear when correctly assembled.

2. Thread the top straps through the top slot in the mask frame.
3. Thread the bottom straps through the bottom slots in the mask frame or clips.
4. Fold the ends back and press the tabs onto the straps.

Philips Respironics System One Resistance Control

This mask when combined with a Philips Respironics System One device, provides optimal resistance compensation. This mask value is X1 and should be set by the provider.


Notes:

- Compare the mask to the device. See the device manual if the values do not match.
- System One is not compatible with masks that use a separate exhalation device.

Specifications

Warning: The technical specifications of the mask are provided for the healthcare professional to determine if it is compatible with the CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function.

Pressure Flow Curve

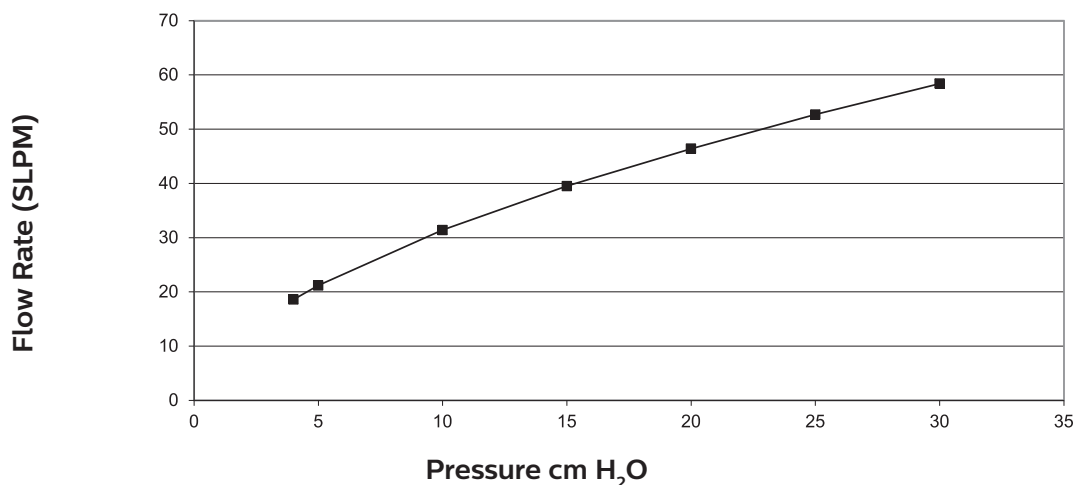Resistance

Drop in Pressure (cm H ₂ O)	at 50 SLPM	at 100 SLPM
All cushion sizes	0.4	1.4

Deadspace

S/M	60 mL
L	72 mL
XL	80 mL

Sound Levels

A-weighted Sound Power Level 27 dBA

A-weighted Sound Pressure Level at 1 m 19 dBA

Useful Life

The useful life of the mask depends on the use conditions and maintenance (cleaning, institutional disinfection, and component replacement) of the device. Inspect the mask parts regularly for damage or wear. Replace components as deemed necessary.

Disposal

Dispose of the mask and headgear annually.

Dispose of this device in accordance with local collections and recycling regulations. For more information, visit www.philips.com/recycling.

Storage Conditions

Temperature: -4° F to 140° F (-20° C to 60° C)

Relative Humidity: 15% to 95%, non-condensing

Symbols Glossary

Symbol	Title and Meaning	Symbol	Title and Meaning
	Warning or Caution		Medical device - Indicates that the item is a medical device.
	Packaging unit - Indicates the number of pieces in the package.		Unique device identifier - Indicates the Unique Device Identifier information.
	Not made with natural rubber latex		Consult instructions for use
	Date of manufacture - Indicates the date when the medical device was manufactured. Country of manufacture - Indicates the country of manufacture of the product. Note: When applied to the label, "CC" is replaced by the country code.		Philips Respironics System One Resistance Control Value - Indicates the therapy device comfort setting.
	Importer - Indicates the entity importing the medical device into the locale.		

Limited Warranty

Respironics, Inc., a Philips company ("Philips Respironics") provides this non-transferable, limited warranty for the Pico Traditional Nasal Mask (including mask frame and cushion) ("Product") to the customer who originally purchased the Product directly from Philips Respironics.

What this warranty covers: Philips Respironics warrants each new Product will be free from defects in materials and workmanship and will perform in accordance with the Product

specifications under normal and proper use and maintenance in accordance with applicable instructions, subject to the exclusions below.

How long does this warranty last: For the product, ninety (90) days from the longer of the date of shipment to the purchaser or date of setup by purchaser for the end user. The warranty period for accessories, replacement parts, and disposables including, but not limited to, chin straps is ninety (90) days from the date of shipment to the original purchaser.

What this warranty does not cover: This warranty does not apply to any software included with the Product as the software warranty is included in the software license. This warranty does not cover damage or injury whether to the Products, personal property, or persons caused by accident, misuse, abuse, Acts of God, water ingress, repair or alteration by anyone other than Philips Respironics or its authorized service center, failure to operate in accordance with the terms of the operating manual and instructions, lack of reasonable care, the discontinuance of a network (e.g. 2G, 3G, etc.) by a carrier (e.g. ATT, Verizon, etc., or other defects not related to material or workmanship. This warranty is not transferable. If Philips Respironics finds that a Product returned for service or the issue raised is not covered under this limited warranty, Philips Respironics may charge an evaluation fee and return shipping.

What Philips Respironics will do: If a Product fails to conform to the warranties set forth above during the applicable warranty period, Philips Respironics will repair or replace the Product or refund the original purchase price, in Philips Respironics sole discretion. Philips Respironics may use new or remanufactured assemblies, components, and parts in repair and new or recertified refurbished devices for replacement. The balance of the original warranty period will apply to any Product or component of a Product repaired or replaced under this warranty.

Warranty Disclaimer; Limitation of Liability: EXCEPT AS SET FORTH IN THIS LIMITED WARRANTY, PHILIPS RESPIRONICS MAKES NO WARRANTIES, EXPRESSED OR IMPLIED, STATUTORY OR OTHERWISE, REGARDING THE PRODUCT OR ITS QUALITY OR PERFORMANCE. PHILIPS RESPIRONICS SPECIFICALLY DISCLAIMS THE IMPLIED WARRANTY OF MERCHANTABILITY AND THE IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT WILL PHILIPS RESPIRONICS MAXIMUM LIABILITY UNDER THESE WARRANTIES EXCEED THE ORIGINAL PURCHASE PRICE OR WILL PHILIPS RESPIRONICS BE LIABLE FOR ANY ECONOMIC LOSS, LOSS OF PROFITS, OVERHEAD, OR SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. Repair, replacement, or return of purchase price by Philips Respironics is the original purchaser's sole and exclusive remedy under this warranty.

This warranty gives you specific legal rights, and you may also have other rights that vary from state to state. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusion and limitations may not apply to you.

How to get warranty support: Patients contact your local authorized Philips Respironics dealer and dealers contact Respironics, Inc. at:

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Limited Warranty for Australia and New Zealand only

Note: For Australian and New Zealand customers this warranty replaces the warranty contained above.

1. Respironics, Inc., a Philips company ("Philips Respironics"), warrants that the Products shall be free from defects of workmanship and materials and will perform in accordance with the Product specifications. 2. This warranty is valid for a period of ninety (90) days from the date of purchase from an authorized Philips Respironics dealer. 3 If the Product is found to contain a defect of workmanship or materials or fails to perform in accordance with the Product specifications during the applicable warranty period, Philips Respironics will repair or replace, at

its option, the defective material or part. 4. The customer is responsible for returning the product to an authorised Philips Respironics dealer, and collecting the product from the authorised Philips Respironics dealer after repair or replacement, at its own cost. Philips Respironics is responsible only for the freight cost of transporting the product between the authorised Philips Respironics dealer and Philips Respironics. Philips Respironics reserves the right to charge an evaluation and postage fee for any returned Product where no problem is found following investigation. 5. This warranty does not cover:

- products purchased outside of Australia or New Zealand;
- damage caused by accident, misuse, abuse, alteration, pest infestation, liquid ingress, and other defects not related to materials or workmanship.

6. The warranty provided by Philips Respironics herein is not transferrable by the Buyer in the event of any sale or transfer of Products purchased by the Buyer from an authorized Philips Respironics dealer. 7. To the extent permitted by law, where the Buyer has the benefit of an implied guarantee under the Australian Consumer Law, but the Product is not of a kind ordinarily acquired for personal, domestic or household use or consumption Philips Respironics' liability shall be limited, at the option of Philips Respironics, to the replacement or repair of the Product or the supply of an equivalent Product. 8. To exercise your rights under this warranty, contact your local authorised Philips Respironics dealer. A list of all authorised dealers is available at the following link: <http://www.philips.com.au/healthcare>. Alternatively, you can make a claim under this warranty by contacting Philips Respironics directly at: Philips Electronics Australia Limited, 65 Epping Road, North Ryde NSW 2113, Australia. Tel: 1300 766 488, Email: repairs-src@philips-easyconnect.com. 9. The following statement is provided to a Buyer who is a "consumer" under the Australian Consumer Law: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the good repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. 10. The following statement is provided to a Buyer who is a "consumer" under the Consumer Guarantees Act 1993, New Zealand: Our goods come with guarantees that cannot be excluded under the Consumer Guarantees Act 1993. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.
65 Epping Road, North Ryde, NSW 2113
Australia.

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