Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

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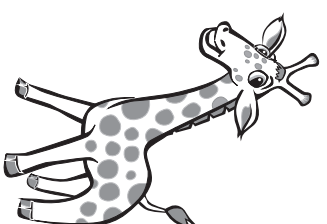
1 Safety Information

Intended Use

This mask is intended to provide an interface for application of non-invasive ventilation to patients. The mask is for single patient use in the home and multi-patient use in the hospital/institutional environment. The mask is to be used on patients 5 kg/11 lbs or greater (≥ 5 kg/11 lbs) for whom non-invasive ventilation has been prescribed. Use of this mask is limited by the indications for use of the compatible therapy device with respect to patient weight.

ⓘ Note: This mask is not made with natural rubber latex, DEHP or BPA.

ⓘ Note: This mask can be used with a variety of therapy devices which may have their own limitations on patient population. If the therapy device itself is limited to patients greater than 5 kg, then the use of this mask does not expand the intended use of that therapy device.

Contraindications

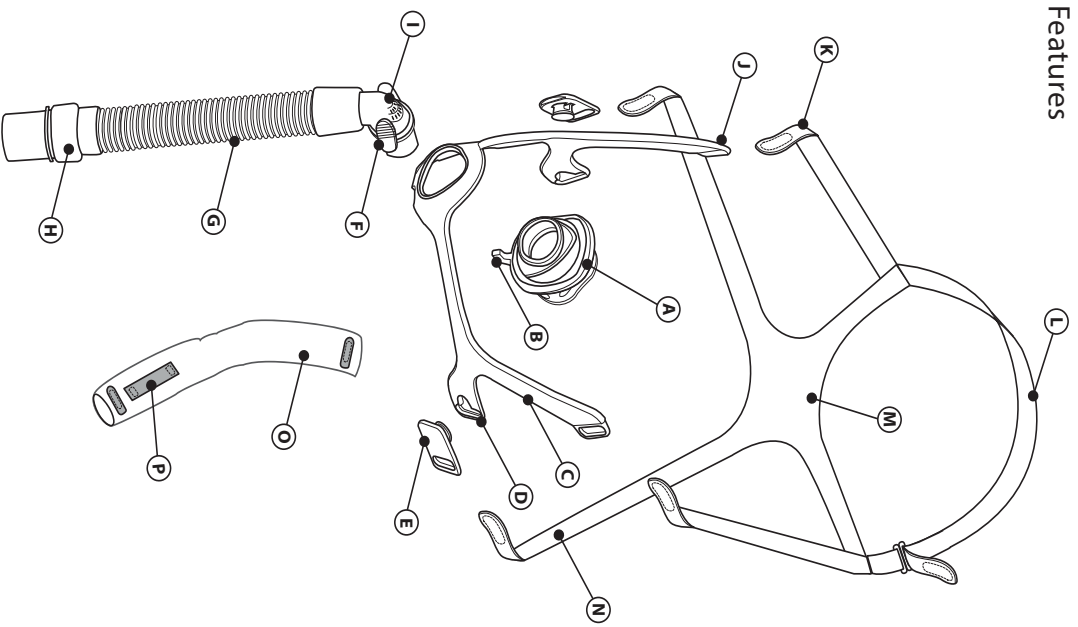
This mask is not intended for patients who are dependent on mechanical ventilation for their life support.

⚠ Warnings

- When using this mask, it is the responsibility of the healthcare professional to ensure proper device set-up in accordance with patient condition, including the use of proper alarms.
- Appropriate patient monitoring and alarms should be used for patients who are unable to remove the mask themselves to assure the mask and tubing remain in place during therapy.
- The use of nasal or full face masks applies forces on the head and may affect cranial facial growth patterns. Regular Healthcare professional assessment is recommended when using this mask.
- Use of this mask while the system is not on and operating may cause rebreathing of exhaled air. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.
- Discontinue use and contact your healthcare professional if any of the following symptoms occur: skin redness, irritation, discomfort, blurred vision, or drying of the eyes.
- Discontinue use and consult your healthcare professional if any of the following symptoms occur: unusual chest discomfort, shortness of breath, severe headache, eye pain or eye infections.
- Discontinue use and consult your healthcare professional if any of the following symptoms occur: tooth, gum, or jaw soreness. Use of a mask may aggravate an existing dental condition.
- The mask material should be away from eyes.
- Do not overtighten the headgear straps. Watch for signs of overtightening, such as excessive redness, sores or bulging skin around the edges of the mask. Loosen the headgear straps to alleviate symptoms.
- Do not block or seal off the exhalation ports.

2 Mask Parts

Features

- (A) Mask cushion assembly
- (B) Leak Correction Dial
- (C) Mask frame
- (D) Frame bottom slot
- (E) Headgear clip
- (F) Quick release tabs
- (G) Tubing assembly
- (H) Swivel
- (I) Elbow with built-in exhalation (do not block)
- (J) Frame top slot
- (K) Headgear top strap
- (L) Headgear crown strap
- (M) Headgear back strap
- (N) Headgear bottom strap
- (O) Tube wrap (optional)
- (P) Tubing management loop

3 Before Use

- Hand wash the entire mask (see **Mask Care** section).
- Wash face. Do not use moisturizer/lotion on hands or face.
- Inspect the mask. Replace the cushion if hardened or torn. Replace any parts that are broken or worn.

4 Mask Care

Cleaning Instructions

Daily: Hand wash the non-fabric parts.

Weekly: Hand wash the fabric parts.

1. Wash in warm water with liquid dishwashing soap.
2. Rinse thoroughly.
3. Air dry completely before use. Lay the fabric parts flat or hang to dry.

⚠ Caution: Do not clean the mask with bleach, alcohol, solutions containing bleach or alcohol, or solutions containing conditioners or moisturizers.

⚠ Caution: Any deviation from these instructions may impact the performance of the product.

Dishwashing Instructions

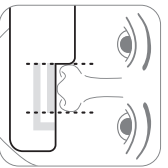
Weekly: Additionally, the non-fabric mask parts may also be cleaned in the dishwasher.

1. Disassemble.
2. Wash the non-fabric parts in the top shelf of the dishwasher with liquid dishwashing detergent.
3. Air dry completely before use.




5 Achieving the Right Fit

Before Putting on the Mask

Sizing: Select the cushion that fits the width of the nose without blocking the nostrils. The mask sizing gauge can be used as a guide by holding the gauge under the nose (see illustration).

Putting on the Mask

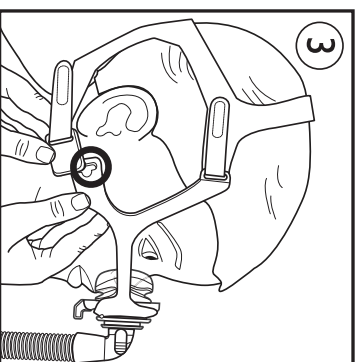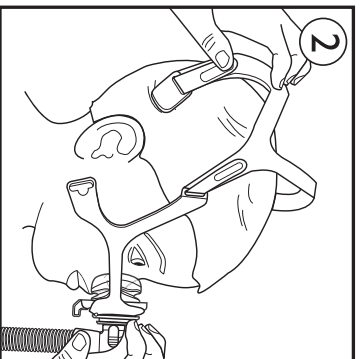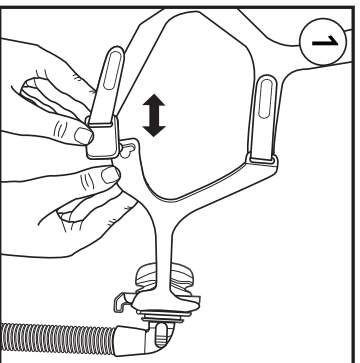1. Headgear clip: Disconnect one or both by twisting the clip outward ①.

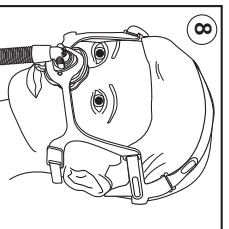
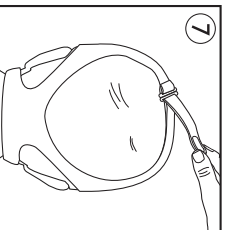
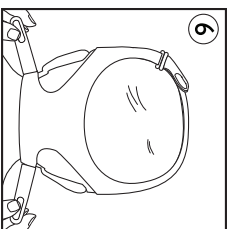
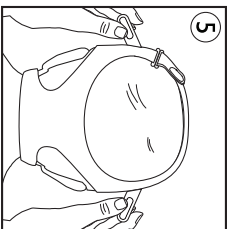
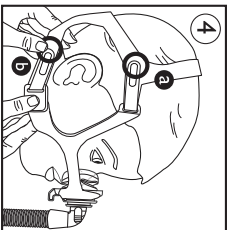
ⓘ **Note:** Use of the headgear clips is optional. You can peel headgear tabs away from headgear straps and pull through the slots in the mask frame.

2. Headgear: Pull headgear on over head ②.

3. Cushion: With the Leak Correction Dial facing downward, place cushion over nose ②. The top of the cushion should sit at the bridge of the nose.

ⓘ **Note:** Whisp adult cushions should not be used with the pediatric mask.

4. Headgear bottom straps or clip: Thread bottom strap through slot in mask frame or press clip against the frame slot to reattach ③.

Adjusting the Mask

1. Headgear: Peel the headgear tabs away from the headgear straps then press tabs against the headgear straps to reattach (4).
- a Connect the top straps to the headgear to achieve a comfortable fit.
 - b Connect the bottom strap to the lower headgear to achieve a comfortable fit.

2. Headgear: Adjust top straps evenly (5).

3. Headgear: Adjust bottom straps evenly (6).

4. Headgear: Adjust crown strap so that the headgear and frame do not touch the ears (7). The mask should rest comfortably against face (8).

If there are excessive air leaks adjust the headgear. For small mask leaks during sleep see **Leak Correction Dial** section.

Note: Do not overtighten the headgear.

Using Tube Management (Optional)

The flexible tubing may be placed over the head (9)(10). Slide the end of the crown strap through the loop on the tube wrap. Then route the end of the crown strap through the slot in the headgear and fasten into place.

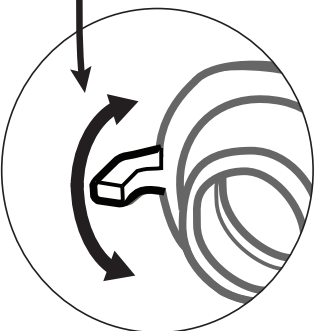
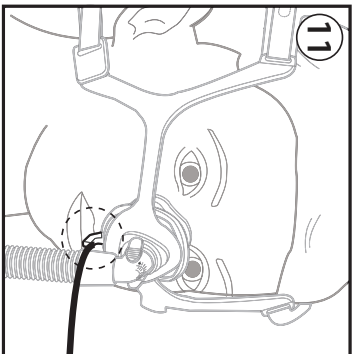
Using the Mask

1. Connect the flexible tubing (included with the CPAP or bi-level device) to the swivel.
2. Turn the therapy device on. Make sure the child is lying down and breathing normally.

Leak Correction Dial



The Leak Correction Dial should point straight downward each time the mask is placed on the child.

If the child experiences small mask leaks while sleeping, move the dial to the left or right (11). If leaks cannot be corrected using the dial, re-seat the mask by pulling it directly away from the face and gently setting it back into place.

Removing the Entire Mask


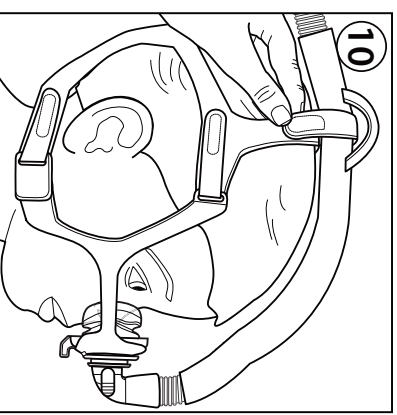
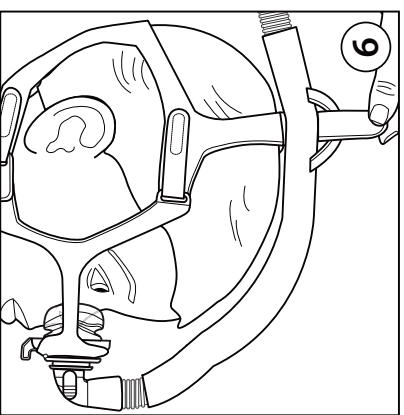
1. Disconnect a bottom headgear clip.
2. Slide the mask upward over the head.

6 Disassembly

Removing the Elbow and Tubing

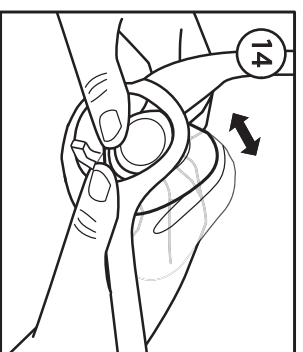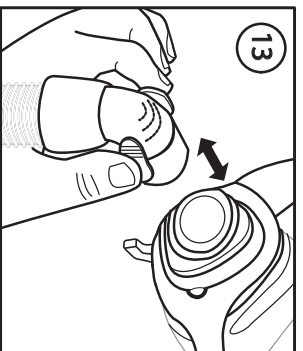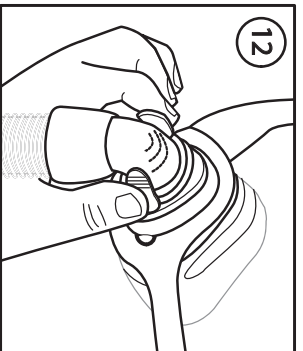Squeeze the quick release tabs and pull the elbow away from the cushion (12)(13).

Removing the Cushion

Push the cushion backwards through the frame (14).

Removing the Headgear

1. Detach the headgear bottom clips from the frame or undo the bottom tabs and pull.
2. Undo the headgear top tabs and pull the straps through the frame.

7 Assembly

Attaching the Cushion

With the Leak Correction Dial pointing downward, push the cushion into the frame towards the front and press into place.

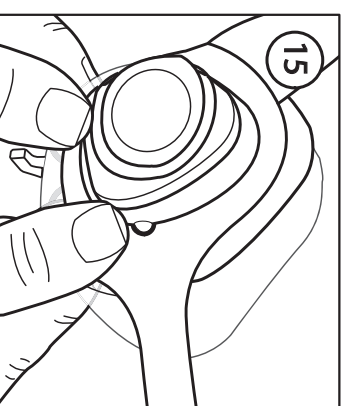
The sides of the cushion will match the alignment cutouts in the mask frame (15).

Attaching the Elbow and Tubing

Push the elbow into the cushion until the quick release tabs click into place.

Attaching the Headgear

1. Hold the crown strap upward.
2. Make sure the back strap that displays "Philips Respironics" is facing outward and is right side up.
3. Slide the headgear top strap into the frame top slots.
4. Connect the headgear clips to the frame bottom slots or slide the headgear bottom strap into the frame bottom slots.

8 Comfort Tips for Caregivers

- If the child has a preference for a certain sleeping position, consider this when choosing placement of the machine.
- The most common mistake is overtightening the headgear. The headgear should fit loosely and comfortably. If the skin bulges around the mask or if red marks appear, loosen the headgear.
- If necessary, adjust the crown strap to move the headgear off of the ears.
- Headgear clips are optional. If you prefer, you can thread the headgear strap directly through the frame slot.
- The quick release elbow allows the mask to be temporarily disconnected from the tubing without the child having to remove the mask. The elbow can be quickly clicked back into place when the child returns to bed.

- Use of the optional tube management feature moves the tube away from the front of the body and reroutes it over the child's head. The quick release feature cannot be used when tube is routed over the top of the head using the tubing management loop on the tube wrap.
- The mask can be removed from the child's head by sliding it upward frame first or headgear first, according to the child's preference. Children may accept the process more easily if the headgear is removed first, pulling the mask away from the face, instead of up over the face frame-first.

9 Health Care Provider & Clinician Information

Philips Respironics System One Resistance Control

Using your mask with a Philips Respironics System One device provides optimal comfort. The provider sets this value (X1) on your device.

Multi-Patient Use

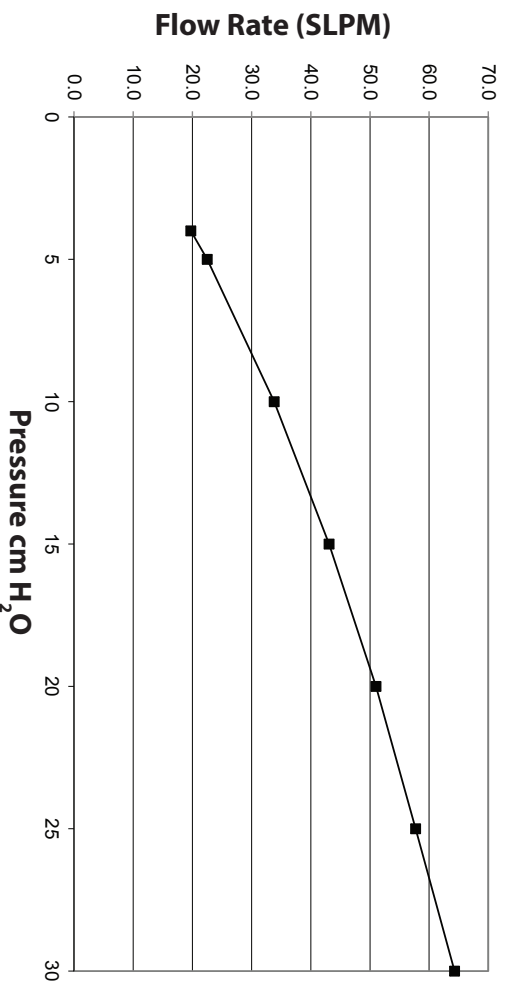
Refer to the Disinfection Guide for Professional Users to reprocess between patients in a clinical setting. Access the latest version of the Disinfection Guide at www.healthcare.philips.com or by contacting customer service at +1-724-387-4000 or Respironics Deutschland at +49 8152 93060.

Specifications

The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your therapy device, and that alerts and alarms function properly.

⚠ Warning: Use outside of these specifications may result in ineffective therapy. The pressure flow curve shown below is an approximation of expected performance. Exact measurements may vary.

Pressure Flow Curve

Resistance

Drop in Pressure at:

50 SLPM

100 SLPM

All Sizes 0.7 cm H₂O

2.8 cm H₂O

Disposal

Dispose of in accordance with local regulations.

Deadspace

SCS 18.9 mL

SCM 24.3 mL

SCL 35.9 mL

Storage Conditions

Temperature: -4° F to 140° F (-20° C to 60° C)

Relative Humidity: 15% to 95%, non-condensing

Sound Levels

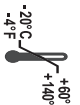
A-weighted Sound Power Level

29.8 dBA

A-weighted Sound Pressure Level at 1m

22.1 dBA

10 Symbols

	Manufacturer		Humidity Limits
	Philips Respironics System One Resistance Control Value		Temperature Limits
	Not Made with Natural Rubber Latex		

11 Limited Warranty

Respironics, Inc. warrants that its mask systems (including mask frame and cushion) (the "Product") shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period"). If the Product fails under normal conditions of use during the Warranty Period and the Product is returned to Respironics within the Warranty Period, Respironics will replace the Product. This warranty is nontransferable and only applies to the original owner of the Product. The foregoing replacement remedy will be the sole remedy for breach of the foregoing warranty.

This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship.

This warranty does not apply to any Product that may have been repaired or altered by anyone other than Respironics. Respironics disclaims all liability for economic loss, loss of profits, overhead, or indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the Product. Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN ADDITION, ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED. SOME JURISDICTIONS DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS UNDER LAWS APPLICABLE IN YOUR SPECIFIC JURISDICTION.

To exercise your rights under this limited warranty, contact your local authorized Respironics, Inc. dealer or Respironics, Inc. at 1001 Murry Ridge Lane, Murrysville, Pennsylvania 15668, USA.

Limited Warranty for Australia and New Zealand only

Note: For Australian and New Zealand customers this warranty replaces the warranty contained above.

Respironics, Inc, a Philips Healthcare company warrants that the Products shall be free from defects of workmanship and materials and will perform in accordance with the Product specifications. 2. This warranty is valid for a period of 30 days from the date of purchase from an authorised Respironics dealer. 3. If the Product is found to contain a defect of workmanship or materials or fails to perform in accordance with the Product specifications during the applicable warranty period, Respironics will repair or replace, at its option, the defective material or part. 4. The customer is responsible for returning the product to an authorised Philips Respironics dealer, and collecting the product from the authorised Philips Respironics dealer after repair or replacement, at its own cost. Philips Respironics is responsible only for the freight cost of transporting the product between the authorised Philips Respironics dealer and Respironics. Respironics reserves the right to charge an evaluation and postage fee for any returned Product as to which no problem is found following investigation. 5. This warranty does not cover damage caused by accident, misuse, abuse, alteration, and other defects not related to materials or workmanship. 6. The warranty provided by Respironics herein is not transferable by the Buyer in the event of any sale or transfer of Products purchased by the Buyer from an authorised Respironics dealer. 7. To the extent permitted by law, where the Buyer has the benefit of an implied guarantee under the Australian Consumer Law, but the Product is not of a kind ordinarily acquired for personal, domestic or household use or consumption Respironics' liability shall be limited, at the option of Respironics, to the replacement or repair of the Product or the supply of an equivalent Product. 8. To exercise your rights under this warranty, contact your local authorised Philips Respironics dealer. A list of all authorised dealers is available at the following link: http://www.healthcarephilips.com/au_en/homehealth/distributors_index.wpd.

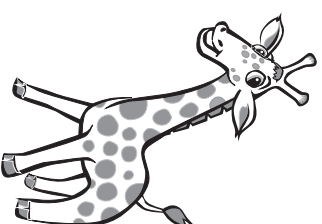
Alternatively, you can make a claim under this warranty by contacting Respironics directly at: Philips Electronics Australia Limited, 65 Epping Road, North Ryde NSW 2113, Australia. Tel: 1300 766 488, Email: prcontact@philips.com. 9. The following statement is provided to a Buyer who is a "consumer" under the Australian Consumer Law: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or

damage. You are also entitled to have the good repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. 10. The following statement is provided to a Buyer who is a "consumer" under the Consumer Guarantees Act 1993, New Zealand: Our goods come with guarantees that cannot be excluded under the Consumer Guarantees Act 1993. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

AUSTRALIAN SPONSOR DETAILS:
Philips Electronics Australia Ltd.
65 Epping Road, North Ryde, NSW 2113
Australia

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©2016 Koninklijke Philips N.V. All rights reserved.