PHILIPS

Children's Medical Ventures

Heel Snuggler

Infant Heel Warmer

REF 99047

A disposable heel warmer is for use when circulation needs to be stimulated in the infant heel in order for blood sampling to occur. When the Heel Snuggler is activated and maintained in a 75° F environment, the temperature of the heel warmer rises from 75° F (24° C) to a maximum of 117° F (47° C) then gradually diminishes over time.

Warnings

- Do not store at temperatures above 75° F (24° C) for one hour prior to use.
- Do not pre-warm the heel warmer before activating. At temperatures above normal room temperature, the heel warmer may exceed a safe temperature.
- Do not puncture bag. If contents contact skin or eyes, flush with water. Do not swallow.
- Do not apply the heel warmer to injured skin.
- Use carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation.
- Use before expiration date to prevent risk of burn.

NOTE: Contents are non-toxic sodium acetate (food grade) and water. All contents are non-toxic.

Symbols

| | | | |
|---|--|---|-------------------------|
|  | Consult Instructions for Use |  | Single Patient Use Only |
|  | European Declaration of Conformity |  | Use by |
|  | Caution! US federal law restricts this device to sale by or on the order of a physician. | | |

Instructions for Use

1. Locate activator disc and grasp with thumb and forefingers of both hands.
2. Flex (bend) activator disc.
3. Knead solution in the bag to soften and activate throughout.
4. To attach, place heel at arrows. Put long section under the foot and short section behind the ankle.
5. Once Heel Snuggler is in place, remove the paper strip that covers the adhesive foot strap. Place around the foot, carefully placing the sticky part of the strap to the bag. Be careful not to adhere to the skin. Repeat with ankle strap.
6. If the procedure has not been started within 10 minutes, Heel Snuggler should be discarded and area should be assessed. If area has cooled, rewarm with a new Heel Snuggler.
7. Dispose of appropriately.

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1059500
HM 4/23/09

PHILIPS

Children's Medical Ventures Heel Snuggler

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© 2009 Koninklijke Philips Electronics N.V.