
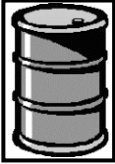




PHILIPS HEALTHCARE	Product Recycling Passport	Page 1 of 5
-------------------------------	-----------------------------------	-------------

Product name:	Brilliance CT Bariatric Patient Support	
Identification code(s)	4535-673-91361	
Total weight (in Kg)	395	
Producer/ Manufacturer	Name company:	Philips Medical Systems
	Address:	595 Miner Road Highland Heights, Ohio
	Zip code:	44143
	Country:	United States of America

Printed copies are uncontrolled

Recycle Info	Items:	Location
Special attention 	Removal of Motor: Couch will drop when motor is removed!	Figure 1 (4)
	Green Safety Block: When removed, couch may collapse!	Figure 1 (3)
	Giant Spring: Potential energy hazard!	Figure 1 (1)
Fluids / Gases 	Items:	Location
	Pressure springs: Must drill 2mm hole to relieve pressure before disposing of springs	Figure 3 (5)
Batteries 	Type:	Location
To be Removed		
Hazardous 	Substances:	Location
	All Printed Circuit Boards	Figure 1 (2)
To be Removed		

Show locations of materials mentioned on the previous sheet. (Picture information)

Printed copies are uncontrolled


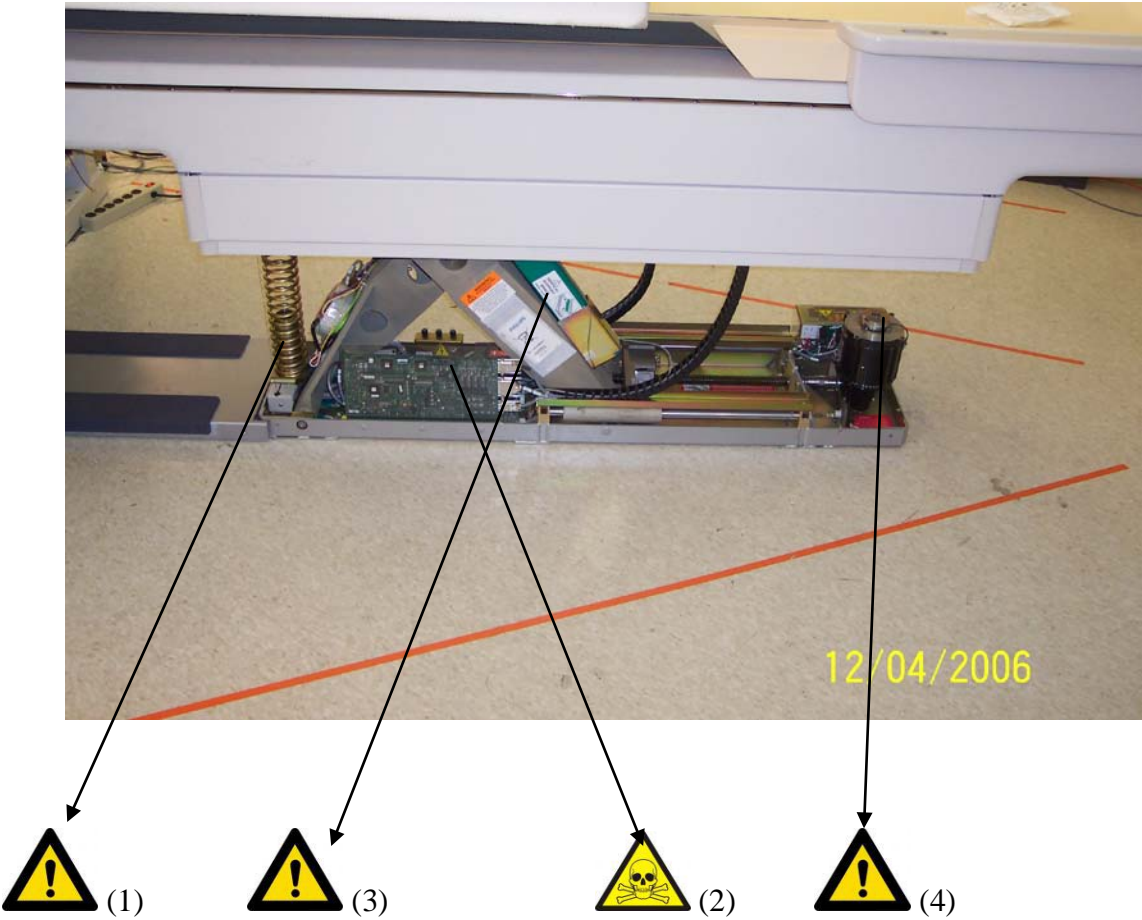


Figure 1: Side View of Rear Subframe

Printed copies are uncontrolled



Figure 2: Underneath Rear of Sub-frame

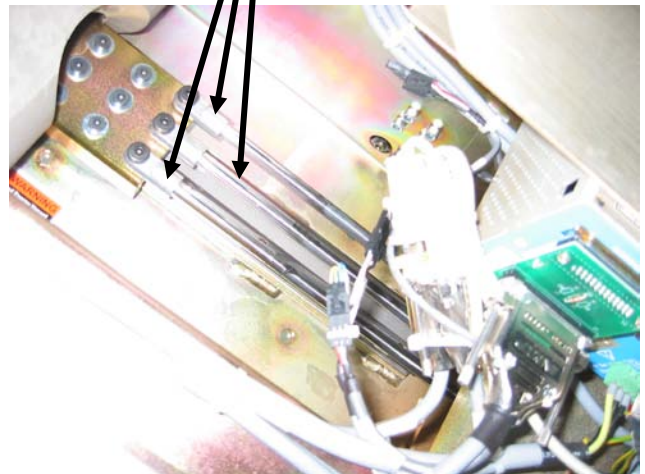

Printed copies are uncontrolled

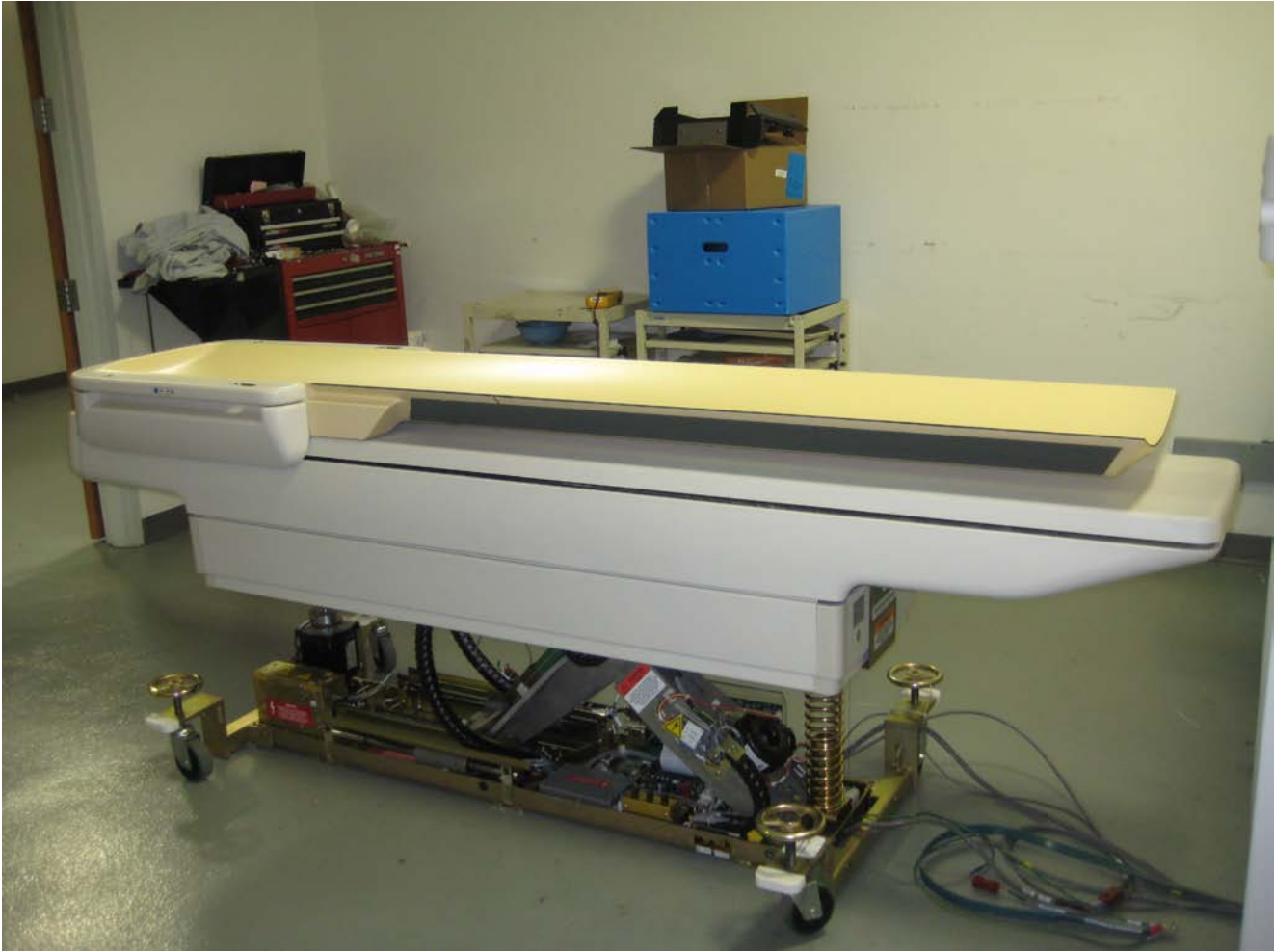


Figure 3: Full view of Patient Support

Title: Recycle Passport Brilliance CT Bariatric Patient Support
Recycling Passport Number: 453567444201 Rev: C

PHILIPS HEALTHCARE	Product Recycling Passport	Page 5 of 5
-------------------------------	-----------------------------------	-------------

Recycling Passport Revision History

Rev	Description	Date
A	CO 2015923 Initial Release of Bariatric Couch WEEE Passport	2 May 2007
B	CO 2019848 - UPDATED GAS SHOCK LOCATION	24 February 2009
C	CO 2020759 - Removed superfluous footer, added titles to pictures, added whole view of patient support and corrected Rev A & B description to be "CO 2019848"	9 June 2009

Printed copies are uncontrolled